

Customer Service Bulletin

THERMOMETRY

Welch Allyn, Inc.® 4341 State Street Road Skaneateles Falls, New York 13153-0220 USA

Bulletin Type: AS NEEDED

Manufacturer: OEM

Tel: 1-800-535-6663

Fax: 1-315-685-3361 www.WelchAllyn.com

Date: 2013-04-25 SAP DIR #: 80018503

Subject: CSB-BRAUN PRO4000 BATTERY PACK UPDATE

Distribution: Welch Allyn Global Technical Support, Product, Field & Customer Service

Teams.

WA Customers, ASP's, and Distributors

All Welch Allyn products that utilize the following Braun Pro 4000 part Product(s) Referenced:

numbers:

P/N	Description
04000-200	BRAUN PRO 4000 NA
04000-300	BRAUN PRO 4000 + Base STA NA
04000-500	BRAUN PRO 4000 + Base STA MN
04000-600	BRAUN PRO 4000 MN
53020-0000	RECHARGEABLE BATTERY PACK FOR PRO4000

This Customer Service Bulletin provides detail on an updated battery pack design Summary:

that reduces brown outs that lead to a lack of data transfer from the Braun PRO

4000 thermometer to a Vital Signs Device.

Issue: The Braun thermometer does not transfer a measurement taken to the Connex Vital

Signs Monitor, Connex Integrated Wall System, or Spot LXi upon docking.

Customers experiencing communication transfer problems should replace installed rechargeable battery packs with the new design in addition to following periodic cleaning instructions provided in Customer Service Bulletin # 20012578A as proactive device maintenance. Device maintenance should be completed at regular

intervals such as when thermometer calibration is being tested.


The combination of the new design and periodic electrical contact cleaning

will yield the best communication performance.

THIS INFORMATION IS THE PROPERTY OF WELCH ALLYN, INC. AND AS SUCH SHALL NOT BE REPRODUCED, COPIED, OR USED AS A BASIS FOR THE MANUFACTURE OR SALE OF EQUIPMENT OR DEVICES WITHOUT THE EXPRESS WRITTEN PERMISSION OF WELCH ALLYN, INC.

Action:

1) Customers experiencing communication transfer problems should follow the cleaning process in CSB#20012578A and visually inspect the rechargeable battery pack for determination of whether the new battery pack design is in use. The following images can be used to determine the battery pack version in use.

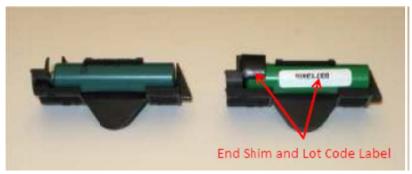

Old Battery Pack

New Battery Pack



Old Battery Pack

New Battery Pack



Old Battery Pack

New Battery Pack

2) If the old battery pack design is in use and the customer is experiencing problems, they should contact Welch Allyn Technical Support for assistance. Welch Allyn customer service can contact Category Management for the product in use for approval to replace the old design with the new design at no charge under warranty. Battery pack 53020-0000 is the replacement part. The battery pack warranty period matches the warranty period of the device it was purchased with. If a customer is experiencing issues and the related device is out of warranty, a one year grace period will be allowed for warranty coverage of the battery pack.

THIS INFORMATION IS THE PROPERTY OF WELCH ALLYN, INC. AND AS SUCH SHALL NOT BE REPRODUCED, COPIED, OR USED AS A BASIS FOR THE MANUFACTURE OR SALE OF EQUIPMENT OR DEVICES WITHOUT THE EXPRESS WRITTEN PERMISSION OF WELCH ALLYN, INC.

Notes:

- 1. Contact the Welch Allyn Complaints Department to initiate or process a medical device complaint resulting from this or other issues.
- 2. Drawings, illustrations, and part numbers in this document are for reference purposes only and subject to change.

End of Bulletin

Revision History							
Version	Description	Change #	Init	Release Date	Appr		
A	Initial Bulletin	D*	ARS	D*	D*		
D * SEE SAP DIR FOR CHANGE NUMBER, APPROVER NAME AND DATE OF APPROVAL							

THIS INFORMATION IS THE PROPERTY OF WELCH ALLYN, INC. AND AS SUCH SHALL NOT BE REPRODUCED, COPIED, OR USED AS A BASIS FOR THE MANUFACTURE OR SALE OF EQUIPMENT OR DEVICES WITHOUT THE EXPRESS WRITTEN PERMISSION OF WELCH ALLYN, INC.