




## Bracket Repositioning

1. Put the back cushion (D) in the up position (see figure 1 on page 2).

**Figure 1. Mount Bracket Repositioning**

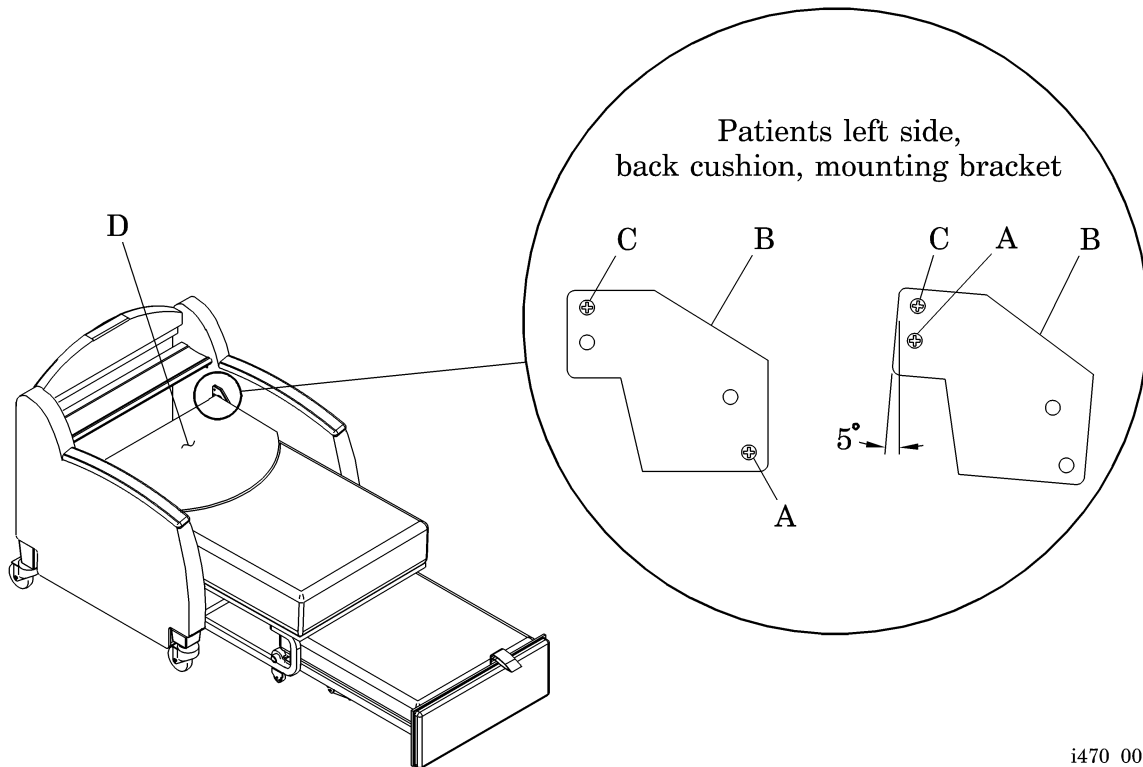

i470\_001

2. Using the phillips head screwdriver, remove the screw (A) on the left and right, back cushion, mounting brackets (B).
3. Fold the back cushion (D) to the down position.
4. Using the phillips head screwdriver, loosen the screw (C) on the left and right, back cushion, mounting bracket (B) two turns.
5. Rotate the patient's left-side, back cushion, mounting bracket (B) approximately 5° clockwise.
6. Rotate the patient's right-side, back cushion, mounting bracket (B) approximately 5° counterclockwise.
7. Using the scribe, locate the screw inserts located in the left and right chair arms.

**NOTE:**

The inserts should be in the same location as the bracket after the rotating.

8. Once the insert has been located, remove any upholstery over the screw hole of the screw insert with the knife.

**NOTE:**

On some chairs, there will not be any upholstery needing to be cut away.

9. If needed, align the mounting hole in the back cushion, mounting bracket (B) with the insert.
10. Install the screw (A), removed in step 3, through the bracket and into the insert.
11. Tighten the screw (A) and screw (C) on the left and right, back cushion, mounting bracket (B).
12. Reach between the back cushion and seat cushion to ensure the back cushion is resting on the seat platform.
  - If the back cushion is not resting on the seat cushion rest, contact Hill-Rom Technical Support at (800) 445-3720.
  - If the repositioning of the bracket allows the back cushion to rest on the seat cushion rest, fold the chair up into the normal sitting position.
13. On chairs with bare wood showing around the repositioned mounting brackets, pull any excess upholstery as close to the mount bracket as possible and staple in place with the staple gun.

**NOTES:**

---