

Coseal Surgical Sealant Preparation and Application Instructions

Baxter


Preparation



Open pouches and transfer the **Coseal** Sealant components into the sterile field.



In sterile field, attach powder syringe to transfer port on syringe housing. Ensure purple syringe clip attached to syringe housing is rotated away from the free plunger.

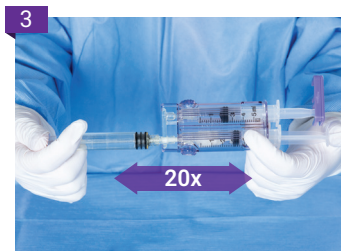

Transfer the liquid into the powder syringe by forcefully depressing the free plunger on the syringe housing. Mix contents back and forth, at least 20 times, until all powder is completely dissolved. End with entire contents in the syringe housing.



Disengage the powder syringe and transfer port together by depressing levers on the side of syringe housing.




Holding the syringe tips up, level the plungers and rotate the syringe clip to connect the other plunger.


Application



Hold the assembly upright and expel all the air from both syringes.

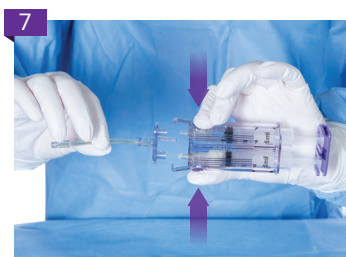

Attach applicator tip. **DO NOT PRIME** the applicator tip.
Use **Coseal** Sealant within 2 hours of preparation.

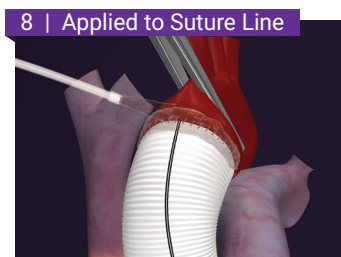

The application pattern depends on the rate and force with which the plunger is depressed. As the force increases, the application pattern will become broader.



Allow at least 60 seconds after application, before touching or laying any object(s) on top of the **Coseal** Sealant.

For spray application, please see instructions for **EASYSpray** Regulator and Spray Set (open application) or **DUPLOSPRAY** Regulator and Spray Applicator Set (laparoscopic application).

The information presented here is based on the **Coseal** Surgical Sealant Instructions for Use.
Baxter, Coseal, DuploSpray and EasySpray are trademarks of Baxter International Inc. or its subsidiaries.
BXU003503 (v1.0) 03/2026

Please consult the device Instructions for Use for full details.