

Instructions for Use


















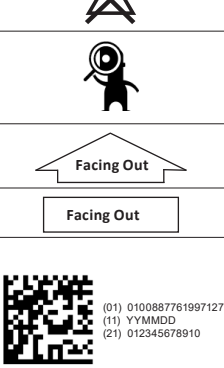
Liftsheet XL is intended to use for lifting and transferring patients in a horizontal position. It is only intended to use in combination with the FlexoStretch™ accessory Prod. No. 3156057, and UltraStretch™ accessory, Prod. No. 3156058. Liftsheet XL has 2 variants, Regular and Tall. Max load 400 kg (880 lbs).

IMPORTANT!

Lifting and transferring a patient always involves a certain level of risk. Read the instructions for use for both the patient lift and the FlexoStretch accessory or the UltraStretch accessory before use. It is important to completely understand the contents of the instructions for use. The equipment should only be used by trained personnel. Exercise care and caution during use. As a caregiver, you are always responsible for the patient's safety. If something is unclear, contact the manufacturer or supplier.

Symbol Description

These symbols can be found in this document and/or on the product.

Symbol	Description
	Warning; this situation requires extra care and attention.
	CE mark
	Legal manufacturer
	Date of manufacturing
	Read instructions for use before use
	Product Identifier
	Serial Number
	Medical device
	Do not dry clean
	Do not iron
	Do not tumble dry
	Washing Symbol is always in combination with recommended temperature in °C and °F.
	Latex Free
	Do not bleach
	Periodic inspection
	Direction upwards and symbol on outside of product.
	Symbol on outside of product.
	GS1 Data Matrix Barcode that may contain following information. (01) Global Trade Item Number (11) Production Date (21) Serial Number

Safety Instructions

Product Changes

Liko's products undergo continuous development, which is why we reserve the right to make product changes without prior notice. Contact your Hill-Rom representative for advice and information about product upgrades.

Design and Quality by Liko in Sweden

The management system for both manufacturing and development of the product is certified in accordance with ISO9001 and its equivalent for the medical device industry, ISO13485. The management system is also certified in accordance with the environmental standard ISO14001.

Notice to Users and/or Patients in EU

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Care and Maintenance

Check the product before each use. Check the following points with regard to wear and damage:

- Fabric
- Seams
- Loops

⚠ Do not use damaged lifting accessories.

If anything is unclear, please contact the manufacturer or supplier.

Washing instructions

Please see the lifting accessory's product label and the attached appendix "Care and Maintenance of Liko Slings".

Periodic Inspection

The product must be inspected at least once every 6 months. More frequent inspections might be required if the product is used or washed more often than normal. For protocols please contact your Hill-Rom representative.

Expected Life Time

The product has an expected life time of 5 years. The expected life time is based on 10 uses per day and washing once per week, according to washing instructions. Hill-Rom recommends the product be used only during its expected life time.

Recycling Instructions


The entire LiftSheet should be sorted as Combustible waste.

Hill-Rom evaluates and provides guidance to its users on the safe handling and disposal of its devices to aid in the prevention of injury, including, but not limited to: cuts, punctures of the skin, abrasions, and any required cleaning and disinfection of the medical device after use and prior to its disposal. Customers should adhere to all federal, state, regional, and/or local laws and regulations as it pertains to the safe disposal of medical devices and accessories.

If in doubt, the user of the device shall first contact Hill-Rom Technical Support for guidance on safe disposal protocols.

Overview Liko™ LiftSheet™ XL

Name	Prod. No.	Material	Length (a)	Max. Load
LiftSheet XL, Regular	3684105	Polyester	165 cm (65 in)	400 kg (880 lbs).
LiftSheet XL, Tall	3684106	Polyester	185 cm (73 in)	400 kg (880 lbs).

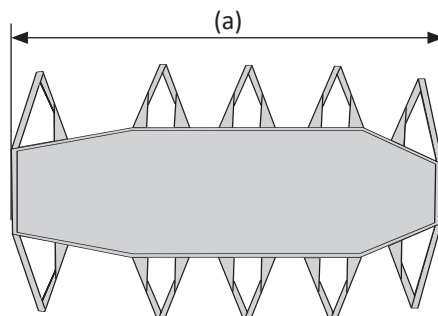

www.hillrom.com

Liko AB
Nedre vägen 100
975 92 Luleå, Sweden
+46 (0)920 474700

Liko AB is a subsidiary of Hill-Rom Holdings Inc.

Enhancing outcomes for
patients and their caregivers:

Hill-Rom