Welch Allyn® 1500 Patient Monitor



Quick reference guide	2
Kurzanleitung	4
Guía rápida	6
Guia de referência rápida	8
Podsumowanie informacji	10
Carte de référence rapide	12
Manuale operativo breve	14
Краткое руководство пользователя	16
Korte gebruiksaanwijzing	18
Kortfattet bruksanvisning	20
Snabb guide	22
快速参考指南	24

Connection and power up

The safety section in the directions for use must be read and understood before using the monitor.

1. Connect the power cable to the input on the rear of the monitor (a).
2. Connect the potential equalization (ground) cable to the central potential equalization lug (b).
3. Press the **On/Off** button (confirmed by a beep).
4. Confirm the **New Patient** dialogue by selecting **Yes** or **No**. The patient data, patient mode, and other settings are selected and edited in the Setup menu (see next page).
5. Check the settings.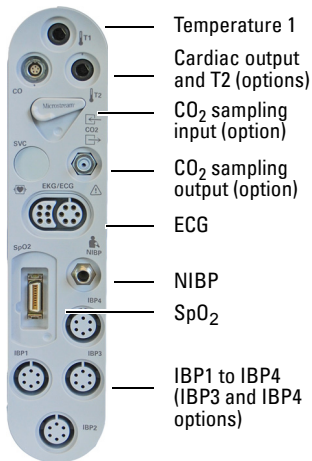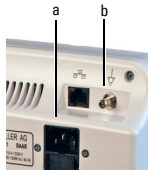

Parameter settings

1. Select the parameter by turning the trim knob. A white box appears around the selected parameter.
2. Press the trim knob to display the settings for the selected parameter.
3. Select and confirm settings by pressing the trim knob.

Function buttons

Print

Prints out three waveforms and the parameter measurements (the printer is an option).

NIBP measurement interval

Sets the interval for non-invasive blood pressure measurement or turns off the interval measurement.

Standby / Discontinue / Discharge

Interrupts or discontinues monitoring.

Trend

Displays trend data and defines trend intervals.

On/Off

Press to switch on. Press and hold for 4 seconds to switch off.

Alarm silence

Silence / resume an audible alarm. The silence duration is defined in Setup > Administrator.

NIBP measurement

Starts or stops an NIBP measurement.

Setup

Displays the Setup menu. The Setup menu includes alarm suspend and alarm overview, speaker and HR/PR volume, patient information and patient mode selection (adult, pediatric, neonatal), and the administrator settings. Menu items are selected by turning and pressing the trim knob.

Home

Pressing this button returns to the monitoring screen. Any open dialogue screen is closed and settings saved.

Trim knob

The trim knob is used for navigation, value selection and value change.

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© 2012
Manufactured by: SCHILLER AG
Manufactured for: Welch Allyn Protocol, Inc.