

Centrella™ Smart+ Bed

Model CNT355 Skills Checklist

Staff Signature: _____

Trainer's Signature: _____

Date: _____



Perform the following.

COMPLETED

Frame Features

| | |
|---|--|
| 1. State the maximum patient weight limit . | |
| 2. Unplug bed's power cord . Identify the one (1) audible and two (2) visual cues when the bed is unplugged from power. | |
| 3. Demonstrate use of battery backup to raise head. Identify controls available on battery backup. Identify % battery 'charged' indicator for bed frame. | |
| 4. Lock head control and unlock retractable foot feature and shorten the bed to shortest position. State all controls that can be locked out. | |
| 5. Explain how bed indicates the communication cable/call light is not connected. There are two (2) audible and two (2) visual indicators. | |
| 6. Lower the head and intermediate siderails on one side of the bed. Raise into a locked position. | |
| 7. Identify one (1) audible and two (2) visual cues that the bed is in the lowest position . | |
| 8. Elevate the lower legs/foot-of-bed so it is parallel to the floor. Explain when this may be useful. Slightly lower feet and then level the legs. | |
| 9. Using CPR control , position bed for CPR. Explain what happens to the mattress. | |
| 10. Remove headboard for emergency access to the head of the bed. | |
| 11. Identify two (2) head-of-bed indicators on the bed. | |
| 12. Identify the built-in siderail line managers . | |
| 13. Identify the hip indicator label for patient placement in the bed. Explain why this is important. | |
| 14. Point to the three (3) areas for attaching restraints on one side of the bed. | |
| 15. Identify the foley/drainage bag holder on one side of the bed. | |
| 16. Explain the foley position limit . Demonstrate how to turn it on and how to recognize it is active. | |
| 17. Explain the difference between when the bed sounds a single beep and a triple beep . | |
| 18. Locate the Boost® control . Explain why to use it and what happens when Boost is activated. | |
| 19. Explain what the three (3) lighted SafeView®+ icons indicate (at the foot-end of the bed and on the floor). Why are there blinking lights? Correct the situation to stop the lights from blinking. | |
| 20. Explain purpose of lighted siderail grip . What turns it on? Explain meaning when light is blue? Amber? Flashing Amber? | |
| 21. Locate and explain purpose of the night light . What activates the brightening of the night light? | |
| 22. Locate these patient accessible siderail features : USB charge port, siderail storage for devices, urinal holder, and head-of-bed angle indicator | |

Centrella™ Smart+ Bed

Model CNT355 Skills Checklist

Perform the following.

COMPLETED

23. Explain the **patient remote controls**: nurse call, bed exit 'on' reminder, stand assist, comfort adjust (mattress) controls. Hang patient remote on all three (3) siderail locations.

24. Demonstrate using the **IntelliDrive® Powered Transport (optional) system** by moving the bed forward and backward. Demonstrate how to move bed side-to-side as if to straighten in a room or up against a stretcher. State % of battery charge.

25. Identify the three (3) indicators for "**brake not set**" when bed is plugged into power.

GCI® Touchscreen

26. Define the three-levels of **bed exit** sensitivity.

27. Set the **bed exit** sensitivity to alert before a patient gets out of the bed. State the four (4) ways you know bed exit is ON (patient in bed).

28. Pre-emptively **SILENCE** the **bed exit alert**. Explain how to add time for Silence period and meaning of **triple beep**.

29. Resume the **bed exit alert**. Identify two (2) ways to do this.

30. Explain the difference between the **scale** "new patient zero" and "same patient re-zero."

31. **Zero scale** for "new patient." Identify three (3) screens you can check to see if the scale was zeroed.

32. **Weigh patient**. Demonstrate how to display weight in lbs, then save weight in kgs. Recall weight after saving.

33. Set the <30° **head-of-bed limit**. Turn off function.

34. Locate the **stand assist** control on the GCI® Touchscreen.

35. Demonstrate putting the bed into the **chair position**. Explain meaning of **triple beep**.

36. Level the bed from the chair position using the **bed flat** control.

37. Locate the **settings** screen. Turn OFF the SafeView® alert lights.

38. Turn off the "Function Not Available" **voice alert**. What Voice Alert cannot be turned off?

39. Locate on the **help screens** and **service wrench** on the GCI® Touchscreen.

Centrella™ MAX Surface Features

40. Discuss the **appropriate patient use** for this product.

41. Discuss how **Focused Advanced Microclimate® Technology** works.

42. Discuss how **weight-based pressure redistribution** works.

43. Discuss **automatic surface adjustments** to help provide support and stability for side sitting and to prevent bottoming out.

44. Discuss how the **surface adjusts with the frame articulation** to help prevent the patient from bottoming out.

45. Discuss the **quiet operation and minimized surface movement** to help improve patient experience.

46. Discuss how the **sloped heel section** can help off load pressure in the vulnerable heel area.

47. Use **Turn Assist** to turn a patient to the RIGHT side. Demonstrate how you stop a turn mid-turn.

Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale or rental of its products.

©2017 Hill-Rom Services, Inc. ALL RIGHTS RESERVED.
203224 rev 2 18-SEP-2017 ENG - US