

Compella™ Bariatric Bed

Skills Checklist



Staff Signature: _____

Trainer's Signature: _____

Date: _____

Perform the following.

COMPLETED

Bed Operations

1. Locate power cords and cord wraps
2. Locate push handles and demonstrate how to stow
3. Demonstrate patient helper trapeze movement and removal
4. Locate equipment sockets and line managers
5. Locate drainage bag holders and patient restraints
6. Locate the no equipment zone
7. Locate nightlight
8. Demonstrate how to activate CPR for both frame and surface

Caregiver Controls

9. Demonstrate operation of siderails
10. Locate and demonstrate use of patient controls
11. Demonstrate use of caregiver controls and basic bed articulations
12. Locate hip indicator for proper patient positioning
13. Demonstrate tilt and reverse tilt
14. Demonstrate bed flat positioning
15. Demonstrate one button chair positioning
16. Demonstrate how to adjust bed length using FlexAfoot™ feature
17. Demonstrate how to activate battery backup
18. Demonstrate how to expand and contract bed using powered controls
19. Demonstrate how to expand and contract bed using manual adjustments
20. Demonstrate use of lockout controls
21. Demonstrate how to zero the scale

Compella™ Bariatric Bed

Skills Checklist

Perform the following.

COMPLETED

22. Demonstrate how to weigh a patient	
23. Demonstrate how to use the bed exit alert	
24. Demonstrate how to use the bed exit silence feature	
25. Explain use of the raise bed control and when to use to remove bed from transport mode	
26. Locate head-of-bed angle indicators	
27. Demonstrate how to activate the 30° head-of-bed alert	
28. Demonstrate the brake and steer mechanism and brake not set alert	
Transport (Optional)	
29. Demonstrate use of IntelliDrive® XL powered transport including transport control pod	
30. Demonstrate how to prepare bed for transport	
31. Demonstrate what to do with bed after transport	
Standard Surface Functions	
32. Demonstrate how to navigate air supply controls	
33. Locate on/off switch, power cord attachment, hose attachments and communication cable	
Optional Low Air Loss Surface Functions	
34. Demonstrate patient setup	
35. Demonstrate Seat Deflate mode	
36. Demonstrate Max Inflate mode	
37. Demonstrate Turn Assist	
38. Demonstrate Continuous Lateral Rotation Therapy	

Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale or rental of its products.

©2015 Hill-Rom Services, Inc. ALL RIGHTS RESERVED.
191353 rev 1 10-MAR-2015 ENG – US

For further information about this product or a service, please contact your local Hill-Rom representative or visit our webpage:

USA 800-445-3730
Canada 800-267-2337

www.hill-rom.com

Enhancing outcomes for patients and their caregivers:

Hill-Rom