



Staff Signature: \_\_\_\_\_

Trainer's Signature: \_\_\_\_\_

Date: \_\_\_\_\_



**BED OPERATIONS**

COMPLETED

1. Demonstrate how to activate CPR.	
2. Locate power cords and cord wraps.	
3. Demonstrate patient helper trapeze movement and removal.	
4. Locate equipment sockets and line managers.	
5. Locate drainage bag holders and patient restraint attachments.	
6. Demonstrate operation of siderails.	
7. Locate hip indicator for proper patient positioning.	
8. Locate head-of-bed angle indicators.	
9. Locate and demonstrate use of patient controls.	
10. Demonstrate use of caregiver controls and basic bed articulations.	
11. Demonstrate use of lockout controls.	
12. Demonstrate the brake and steer mechanism and brake not set alert.	
13. Locate the equipment bar.	
14. Demonstrate the insertion and removal of the foot cushion.	



**CAREGIVER CONTROLS**

COMPLETED

15. Demonstrate how to zero the scale.	
16. Demonstrate how to weigh a patient.	
17. Demonstrate how to use the bed exit alert.	
18. Demonstrate how to use the bed exit silence feature.	
19. Demonstrate how to activate the 30° head-of-bed alert.	
20. Demonstrate how to adjust patient comfort and bead temperature.	

**PATIENT POSITIONING**

21. Demonstrate surface transfer.	
22. Demonstrate side transfer.	
23. Demonstrate turning. <ul style="list-style-type: none"><li>■ Turning for side lying with a pillow.</li><li>■ Turning for back, wound, and perineal care.</li><li>■ Bed pan insertion.</li></ul>	

For more information, please contact your Hillrom sales representative at 1-800-445-3730 or order direct at [rental.hillrom.com](http://rental.hillrom.com).

**hillrom.com**

Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale or rental of its products.