## Installation instructions

---

Montageanweisung

ACROBAT  
*Swing*

**Stativgerät**  
**Tripod-mounted device**

**Table of contents** **Page**

1	Mounting the rollers .....	7
2	Sticking on the color foil (special accessories)...	7
3	Mounting the tripod tubes .....	8
4	Inserting the plastic semi-circular rings .....	8
5	Installing the spring arm .....	9
6	Mounting the end device .....	11
7	Setting the spring force .....	12
8	Change the fuses .....	14
9	Technical data .....	16

**Packing list, Tripod base**

- 1 Tripod base

Accessories supplied for assembly:

- 2 anti-static rollers with brakes
- 2 rollers without brakes

**Packing list, Tripod tube and spring arm**

- 1 tripod tube
- 1 spring arm

Accessories required for assembly:

- 1 color foil (special accessories)
- 2 plastic half rings and 2 screws (not for version with cover, pos. 1 on page 14)
- 1 plastic sleeve
- 1 securing segment
- 1 disk
- 1 securing ring
- 1 retaining ring - pliers

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## Instructions for safe assembly

**Dear user,**

The equipment may only be assembled by a hospital technician or a comparably qualified person.

**The following persons shall be considered as qualified personnel:**

- Persons who underwent special professional training in the field of medicine or medical engineering,
- Persons who can assess their work and recognise the potential hazards involved on the basis of their professional experience and instruction in safety-relevant regulations.
- In States where the performance of tasks in the medical or medical engineering sector are subject to certification, qualified personnel must have obtained the corresponding certificate.

**Dear installer,**

Please read these installation instructions very carefully and follow the safety instructions and requirements of these installation instructions.

If there are any peculiar problems that have not been treated in sufficient detail in these installation instructions, please contact your supplier for your own safety.

Changes to the product are not permitted.

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## Instructions for safe assembly

### Intended purpose:

- The Acrobat appliances have been designed for the carrying and positioning of medical examination lamps and flat screens and also for the power supply of the devices. Depending on the transformer version, e.g. 13,2V, 13,5V, 14V, 24V (without transformer, 230V), 50/60Hz are available (not optionally adjustable).
- The appliance is suitable for continuous operation.
- The maximum load indicated on the rating plate of the individual appliance version must not be exceeded (see Chapter 6 "Mounting the end device").

### Incorrect use

- The maximum loading capacity of the appliance and its components as specified in "Chapter 9, Technical Data" must not be exceeded.

### Contraindications

- The ACROBAT Swing pendant system must not be used close to strong magnetic fields.
- No BF or CF application parts in accordance with IEC 60601-1 may be directly connected to the pendant system.

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Deutsch

## Instructions for safe assembly

Supplementary symbols used in the safety instructions:

**Electric shock hazard:**

Warns of electric shock which can lead to severe injury or even death.

**Pendant system crashing:**

Warns of the risk of the pendant system suddenly crashing because the max. loading capacity has been exceeded.

**Sudden release of spring arm:**

Warns that the spring arm may jump up suddenly while dismantling the end device.

Graphic symbols on the appliance and/or on the packaging:

**Observe the Operating Instructions:**

Read these Operating Instructions carefully prior to initial operation of the pendant system. This ensures that you benefit from all the advantages of the pendant system and prevents any risk of injury or damage.

**Observe the maximum loading capacity:**

Warns of exceeding the maximum loading capacity (payload) on the pendant system of the adaption or the end device (e.g. flat screen, OR lamp, etc.).

**CE mark:**

Ondal declares that the products comply with the relevant regulations set forth in the applicable European Directives.

**Recognised cUL component:**

This component has been recognised by Underwriters Laboratories Inc. Representative samples of this product have been reviewed by UL and comply with the applicable requirements.

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### Mounting / dismantling:

#### CAUTION - Electrical shock:

To prevent the risk of electric shock, the appliance must be connected to a supply network with a protective conductor.

- The appliance must be connected in such a way that it can be disconnected from the mains at all poles and at the same time.

Atmospheric pressure:

Indicates the permissible atmospheric pressure values in a range from 500 hPa to 1060 hPa for transport and storage.

Relative humidity:

Indicates the permissible humidity values in a range from 10% to 75% for transport and storage.

Ambient temperature:

Indicates the permissible ambient temperature values in a range from -25 °C to 70 °C for transport and storage.

### Ambient conditions for storage and transport

The following storage conditions apply for storage times of up to 15 weeks:

Ambient temperature: -25 °C to 70 °C;

Relative humidity: 10 % to 75 %;

Atmospheric pressure: 500 hPa to 1060 hPa.

Store only in indoor rooms; after this time, the values specified for the ambient conditions for operation apply.

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## 1 Mounting the rollers Fig. 1

### NOTE – Versions:

For the 6.5kg tripod base version, no earthing cables are required.

**WARNING - Static charge**  
If the PA-cable is not installed, the tripod base equipment may get statically charged and the charge may get transferred to the patient. Install the PA-cable.

### CAUTION:

The braked rollers must always be installed diagonally (see Fig. 1) in order to prevent tilting/slipping!

1. Guide two anti-static rollers with brakes (1) through the ring cable lug of the PA cable (2) and insert completely in the tripod base (3).

### CAUTION:

The tripod base should be used without additional weight only for end devices that weigh 1-2 kg.

2. Completely insert the two rollers without brakes.
3. Check for firm seating of the rollers.

## 2 Sticking on the color foil Fig. 2

1. Clean eye-shaped recess.
2. Pull out the adhesive foil (1), place the color foil in the eye-shaped recess and press it uniformly.
3. Pull off the clear protective foil (2).

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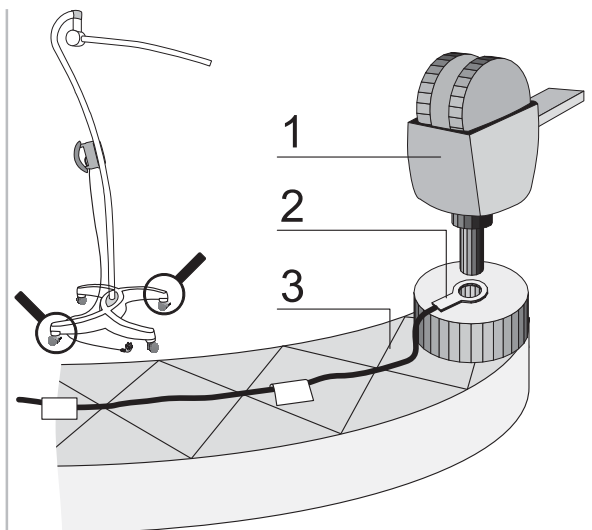
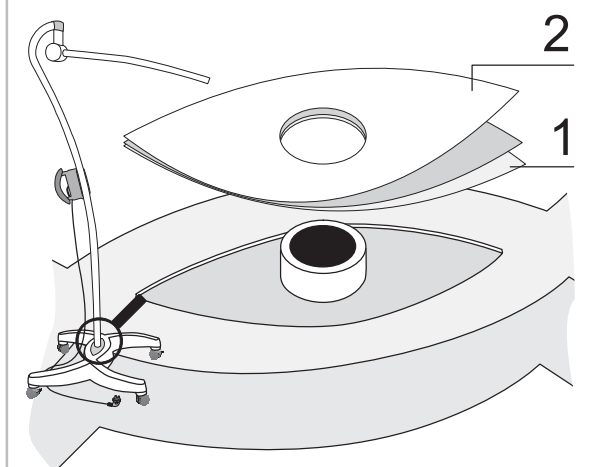
### 3 Mount tripod tube Fig. 3

**!** CAUTION - Damage to the equipment:

If the nose fastening is not inserted properly, and then the tripod tube has not been bolted, the tripod can fall.

Engage the nose fastening and tighten the screw with a spring washer.

1. Unscrew the screw with the spring washer (1) from the tripod tube (2).
2. Insert the tripod tube (2) in such a way that the recess in the tube (3) is seated in the nose of the tripod base (4) and cannot be rotated any more.
3. Check for secure seating.
4. Screw in the screw with the spring washer (1) and tighten it.

### 4 Mounting the plastic semicircular rings Fig. 4

**!** CAUTION - Damage to the equipment:

Without the half rings inserted, the tripod will tip over.

Insert the semi-circular rings, screw them and check secure fitting.

1. Press the plastic half rings (1) into the tripod base (2).
2. Turn in the cross-slotted screw (3) and check whether it is tight.

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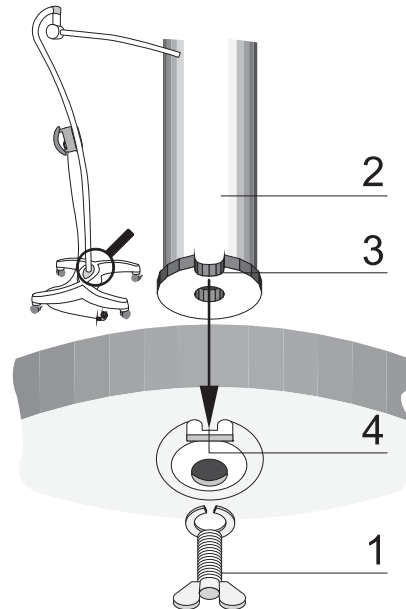
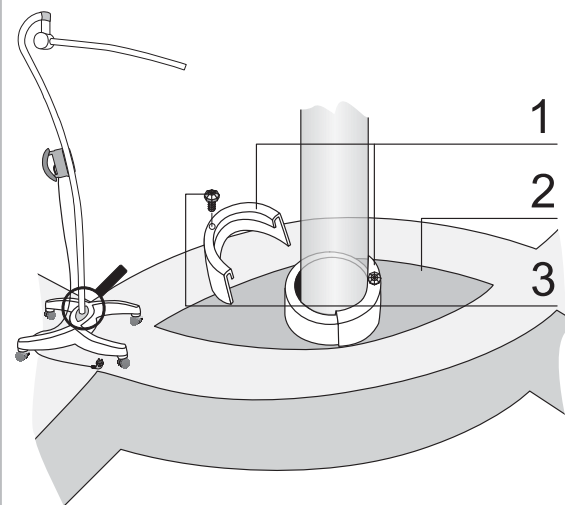
## 5 Federarm montieren

Figure / Abbildung 5

### 5.1 Align the spring arm with the tripod base Fig. 5

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#### NOTE:

The spring force of the spring arm and the tripod base is designed for load torque maximum 39 Nm. The weight category of the spring arm can be seen on the **name plate (1)**.

The weighted tripod base can be recognized from the additional weight (metal plate) on the underside.

#### CAUTION - Damage to the equipment:

The tripod can tip over, when the tripod base is without additional weight and the load torque is greater than 18 Nm.

The spring arm for load torque greater than 18 Nm should only be installed with a weighted tripod base.

1. Determine the weight category on the **name plate (1)**.
2. Mount the spring arm according to the chapter 5.2, „Assembly“.

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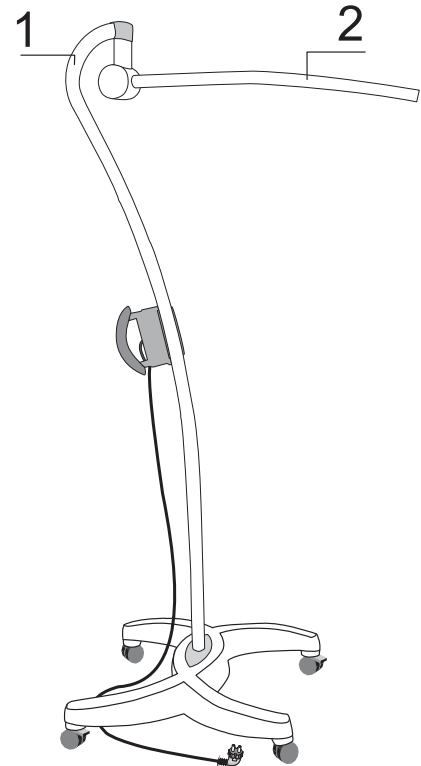
## Figure / Abbildung 6

### 5.2 Assembly Fig. 6

1. Unscrew the **cross-slotted screw (1)**, pull the **cover (2)** forward and move it upward to take it out.

 **CAUTION - Damage to the equipment:**

Without the disk installed, the retaining ring gets loosened and the spring arm falls out of the joint.

- Always install the disk.

2. Insert the **spring arm pin (3)**, place the **disk (4)** and secure them with the **retaining ring (5)**.
3. The **retaining ring (5)** must fit into the pin groove. This must be checked.
4. Make the **electrical connection (6)**.
5. Put on the **cover (2)** and screw it fast with the **cross-slotted screw (1)**.

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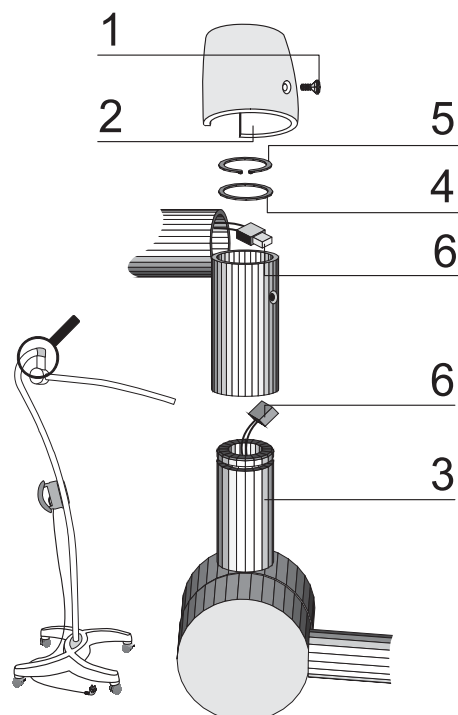
Figure / Abbildung

## 6 Mounting the end-device Fig. 7

1. Pull out the plug and secure it from being inserted again.
2. Take out the protective cover from the spring arm opening.
3. Push the **plastic sleeve (1)** on the spring arm in such a way that the **slots (2)** get covered.

 **WARNIG** - Danger of injury:

The spring arm, which is pressed downwards, can jump backwards and result in injuries. During the installation of the end-device, nobody should be present within swiveling range of the spring arm.

4. Push in the **end-device (3)** and the **securing segment (4)** completely into the **slots (2)**, so that the securing segment is guided in the **groove (5)**.
5. Rotate the **plastic sleeve (1)** through 180 degrees and turn in the **brake screw (5)** till the **end-device (3)** is braked.
6. Check the secure seating of the **end-device (3)**.

 **CAUTION** - Damage to the equipment:

After the end-device has been installed, perform tilt test in accordance with DIN EN 60601-1.

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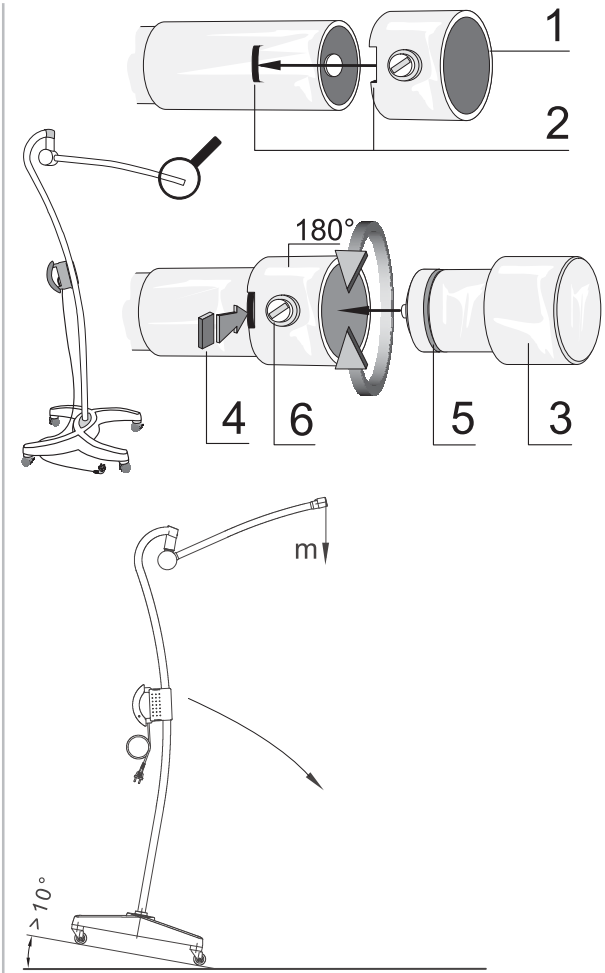
## 7 Adjusting the spring force Fig. 8

Like every technical component, springs are subjected to natural wear. Thus, the spring force may give way and reduce after long operation, and has to be re-adjusted.

**Adjust the spring force in such a way that, the spring arm with end device comes to a standstill in every desired position.**

**! CAUTION - Destruction of the spring arm:**

The spring tension is adjusted in the upper end position.

1. Remove the left-hand **joint cover half (2)** from the spring arm (when looking from the end device). To do this, gently push the joint cover half out of the groove in the spring arm joint using a small slotted screwdriver.
2. Move the end device to the upper end position.
3. Insert the slotted screwdriver into the **drill hole (1)** and adjust the spring tension.
4. Mount the joint cover and make sure that it snaps into place.

**! CAUTION - Destruction of the spring arm:**

Driving in the brake screw too deeply destroys the spring arm. Tighten the brake screw with care while continually checking the braking force.

**If the spring arm drops - the spring force is too low.**

- The adjusting screw must be rotated to the left (in the counter-clockwise direction).

**If the spring arm goes upward - the spring force is too high:**

- the adjusting screw must be rotated to the right (in the clockwise direction).

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Deutsch

## 8 Change the fuses Fig. 9

### **NOTE - Built-in overload protection for versions with toroidal transformer:**

To protect the transformer and the end device, the fuses are installed.

### **WARNING - Electrical shock:**

For all maintenance work, switch off the equipment, pull out the main plug and secure it from being switched on again.

The technical specification for the fuses can be found on page 14 of this document and on the label inside the cover.

### **CAUTION - Damage to the Equipment**

Apply only the specified fuses.

Change the fuses corresponding to the following steps:

#### **Open the cover**

1. Loosening the **screw (3)**, don't unscrew completely.
2. Push up **clamp ring (2)** and **cover (1)** and secure them.

#### **Change fuse**

3. Change broken **fuse (4)**.

#### **Close the cover**

4. Push down **clamp ring (2)** and **cover (1)**.
5. Tighten **screw (3)**.

### **NOTE - To replace the connecting cable:**

The connecting cable may only be replaced by an installer authorised by Ondal.

Circuit diagrams and more detailed information on the components replaced are available from Ondal on request.

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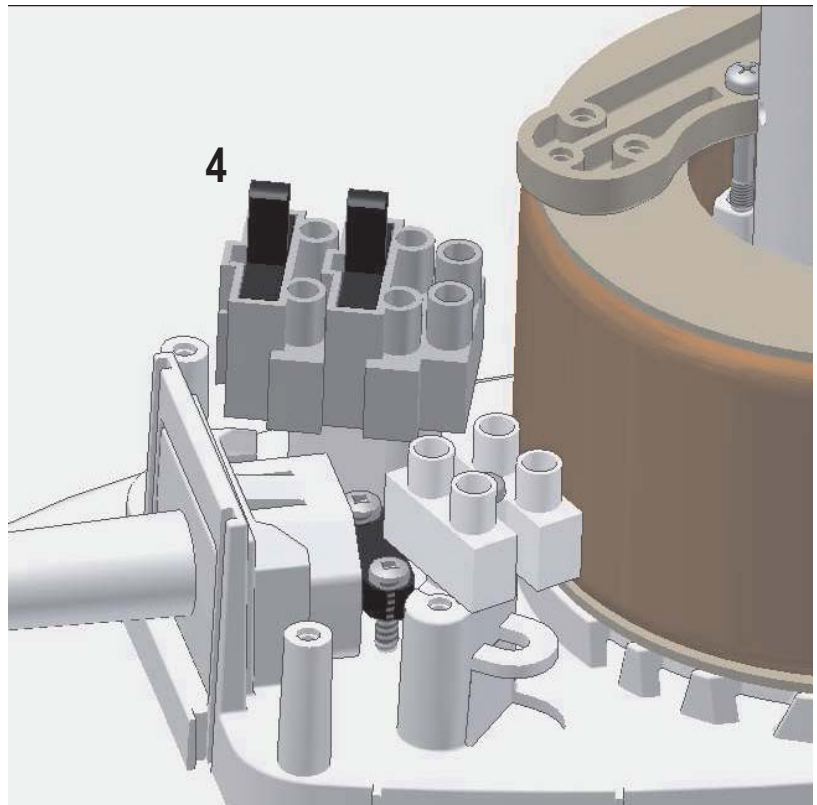
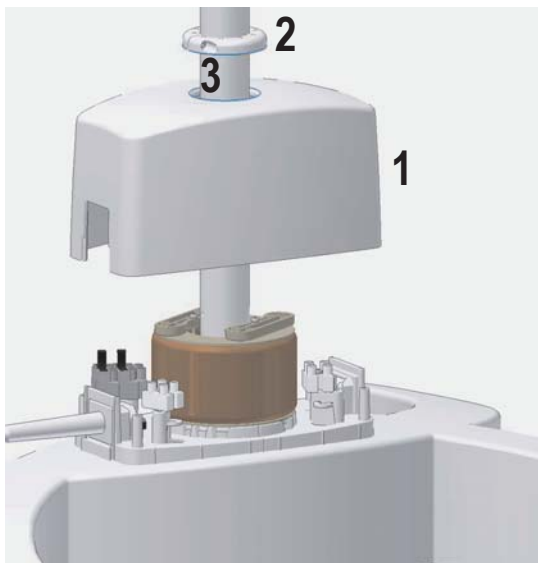
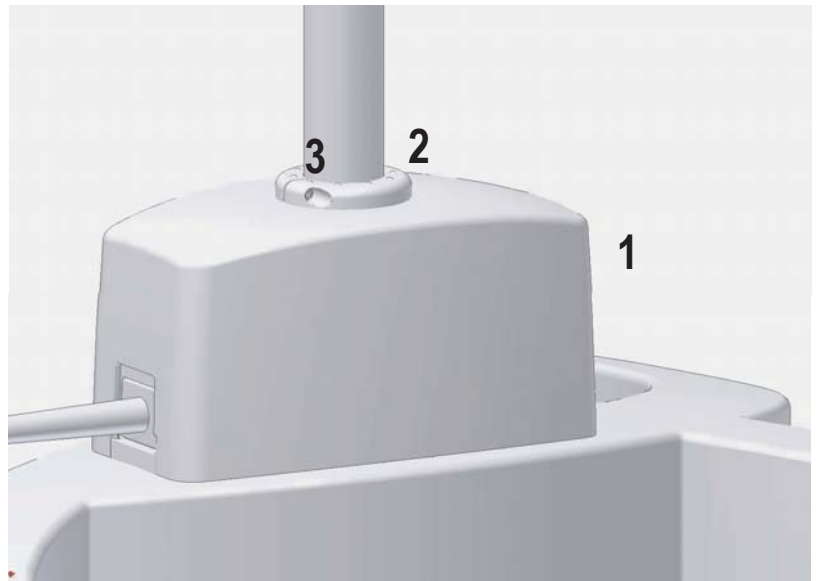
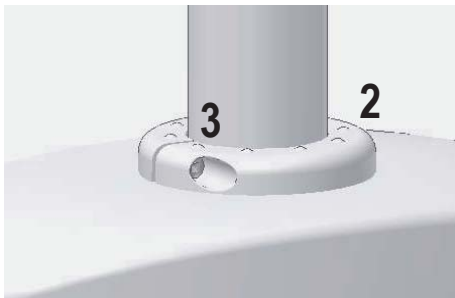
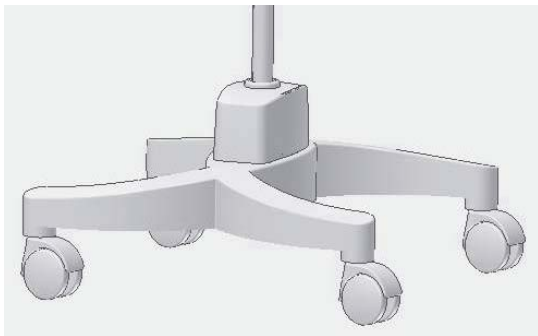
Deutsch

**9 Technical data**

Electrical data	Ceiling-unit	Wall-unit	Stativ-unit
Rated voltage, depending on model	120 V / 230 V	120 V / 230 V	120 V / 230 V
Maximum power*	90 W	80 W	90 W
Nominal frequency	50 / 60 Hz	50 / 60 Hz	50 / 60 Hz
Output current at 230 V *	bis zu 10 A	bis zu 10 A	bis zu 10 A
Typ of fuses: 230 V: T 10 L 250 V prim. 115V/sec. 14V/60VA: T1,0A/250V prim. 230V/sec. 14V/60VA: T0,5A/250V prim. 103V-127V/sec. 13,2V-14V/60VA: T1,25A/250V prim. 213V-247V/sec. 13,2V-14V/60VA: T0,63A/250V prim. 230V/sec. 13,5V/60VA: T0,5A/250V prim. 115V/sec. 24V/90VA: T1,6A/250V prim. 230V/sec. 24V/90VA: T0,8A/250V * depending on version			
Weight in kg	Ceiling-unit	Wall-unit	Stativ-unit
Spring arm (standard)	1,8 kg	1,8 kg	1,8 kg
Extension arm	1,9 kg	1,9 kg	---
Ceiling mounting	3,0 kg - 7,9 kg	---	---
Wall bearing	---	1,8 kg	---
Stativ			3,5 kg
Stativ base			6,5 kg
Stativ base (weighted)			12,0 kg 15,0 kg 18,0 kg
Classification according to EN 60601-1			
Device of protection class I			
IP-Classification according to IEC-60529			IP20
Suitable for continuous operation			
Classification according to guidelines 93/42/EWG, appendix IX			
Class I			
EU-Conformance			
The Acrobat Swing equipment conforms to the specifications of the guideline 93/42/EWG (medical products guideline). The Acrobat Swing equipment is only intended for connecting medical examination lamps, operation lamps and flat-screens with CE-markings and - je nach Ausführung - a maximum of 57 Nm. self-weight. The person operating the equipment must submit a declaration as per article 12 of guideline 93/42/EWG (medical products guidelines). Another conformance evaluation must be carried out for other end devices.			

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