



Hillrom™

Life2000® Ventilation System

STANDALONE CONFIGURATION QUICK START GUIDE



Step One: Charge Ventilator

What You'll
Need for
This Step



Life2000® Ventilator



Battery Charger



AC Power Cord

● Connect
Battery Charger
to Ventilator

● Connect
AC Power Cord
to Battery Charger

● Plug
AC Power Cord
into outlet




The word "UP" indicates the top side.



Battery charger indicator may be red or green – this does not indicate the battery is charging.

Step Two: Prepare the Tank

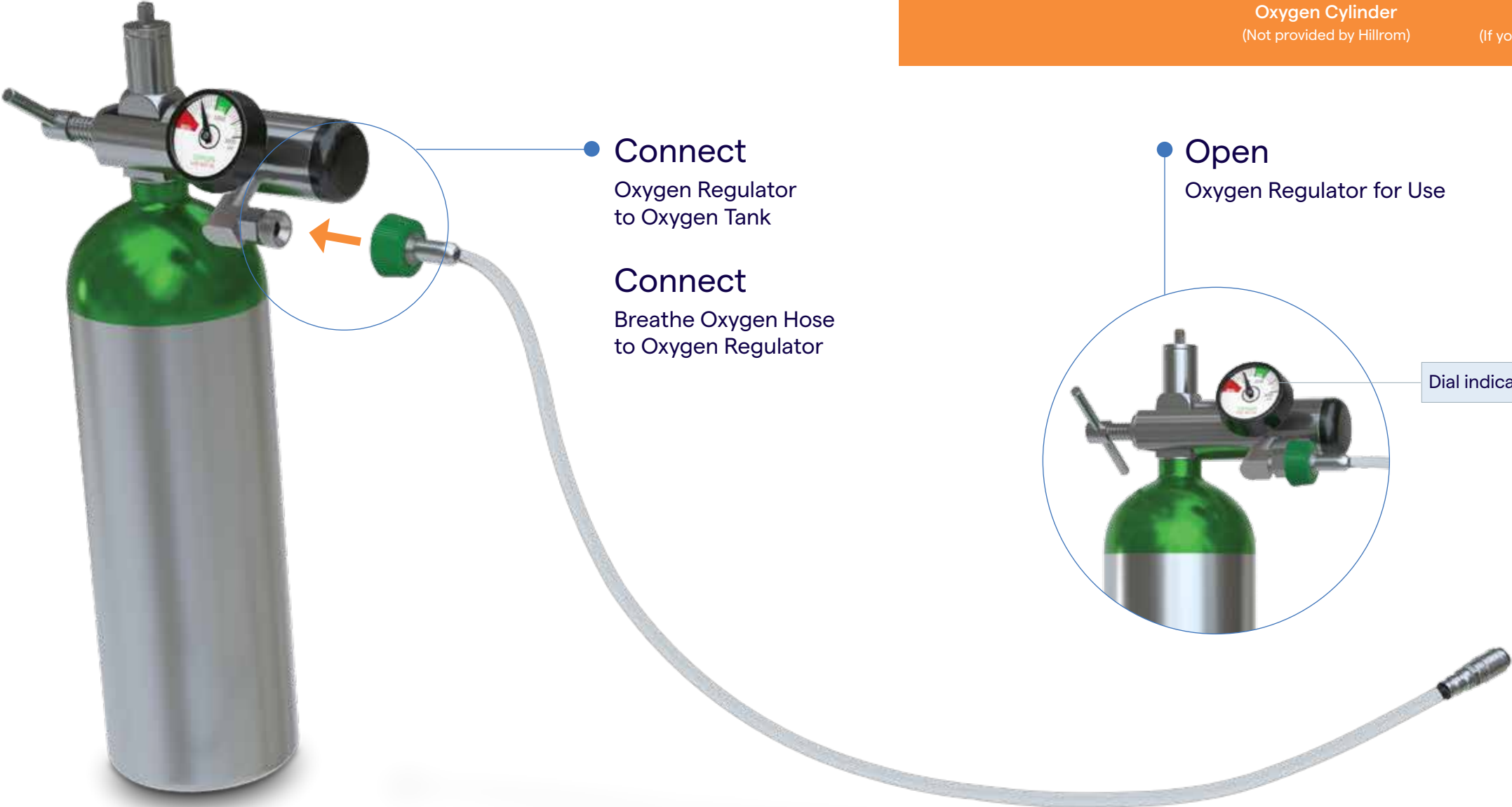

What You'll Need for This Step



Oxygen Cylinder
(Not provided by Hillrom)



Oxygen Regulator
(If you don't have one, Hillrom can provide it)



Breathe
Oxygen Hose

Step Three: Disconnect CombO₂ Hose



What You'll
Need for
This Step



or



Life2000[®] Compressor




CombO₂[®] Hose



Breathe
Pillows
Interface



Life2000
Ventilator

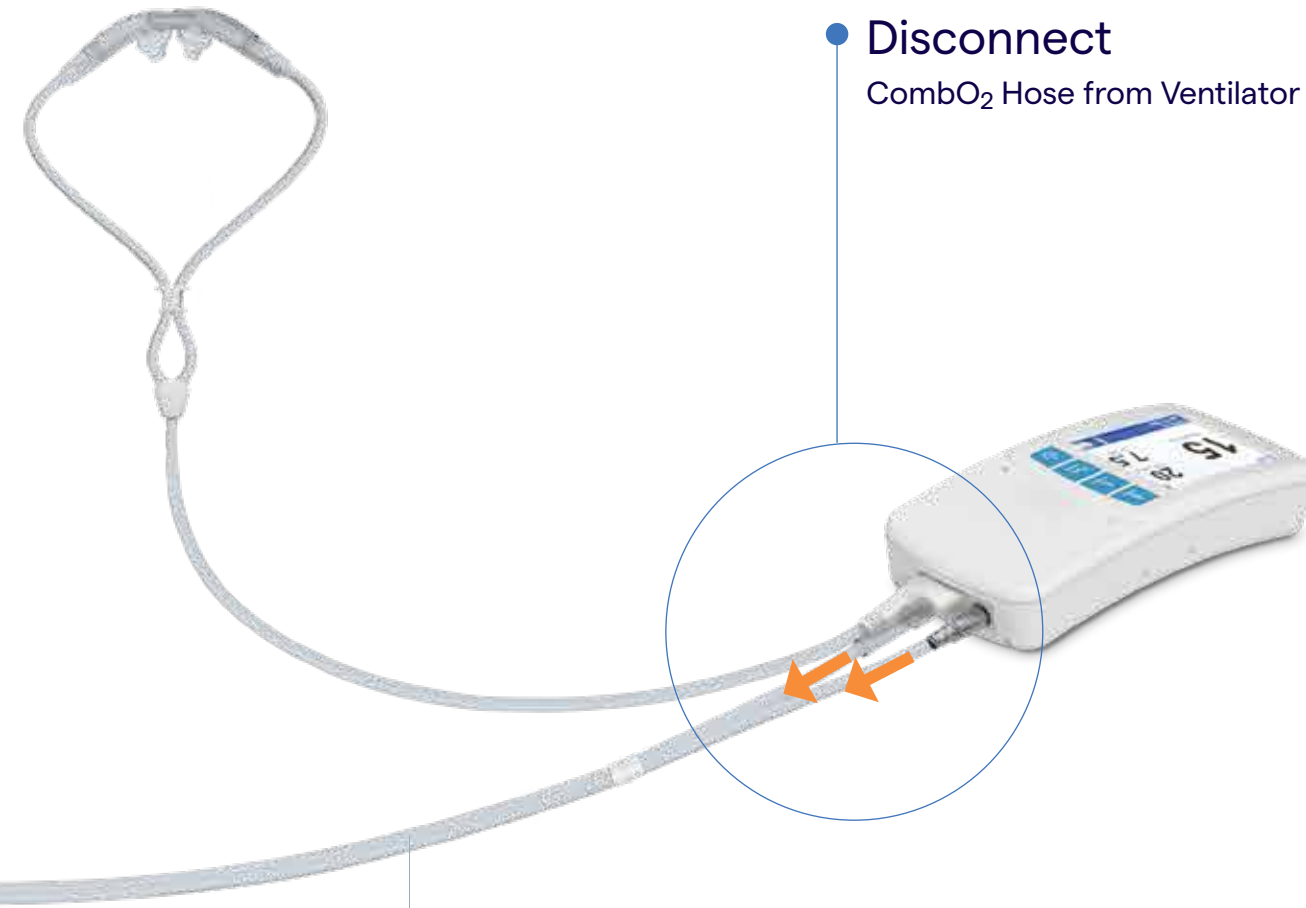

● Disconnect
CombO₂ Hose from Ventilator

Note: When the CombO₂ hose is disconnected, a Low PIP alarm will sound. This is expected and will stop when you connect the Breathe Oxygen Hose.

Step Four: Connect Breathe Oxygen Hose



What You'll
Need for
This Step



Oxygen Cylinder
(Not provided by Hillrom)



Breathe
Oxygen Hose



Breathe
Pillows
Interface



Life2000
Ventilator

● **Connect**
Breathe Oxygen Hose to Ventilator

Note: Breathe
Pillows Interface port
remains unused in this
configuration.



Follow-up Timeline

The Hillrom™ assessment and follow-up program for the Life2000® Ventilation System has been specifically designed to support patients as they learn to how to incorporate ventilation in their lives and take it on the move. The focus of the program is to educate, motivate and support patients.





Hillrom™

For more information about products or services, please contact your Hillrom sales representative or Hillrom customer service at 1-800-426-4224.

[hillrom.com](https://www.hillrom.com)

Life2000® and CombO₂® are registered trademarks of Breathe Technologies, Inc., a Hill-Rom company. Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale or rental of its products.

© 2021 Hill-Rom Services PTE Ltd. ALL RIGHTS RESERVED. APR183402 rev 1 03-MAY-2021 ENG-US