



Hillrom™

Life2000® Ventilation System

EXTENDED RANGE CONFIGURATION QUICK START GUIDE



Step One: Connect Compressor

What You'll
Need for
This Step



Life2000® Compressor



External Power Supply



AC Power Cord

● **Connect**
External Power Supply
to Compressor

Power Supply will click into place.
Lock Tab will be on top.

● **Connect**
AC Power Cord
to External Power Supply

● **Plug**
AC Power Cord
into outlet

Power is connected properly when Green Indicator Light on External Power Supply is lit



Step Two: Connect Ventilator



What You'll Need for This Step



or



Life2000[®] Compressor



CombO₂[®] Hose



Breathe Pillows Interface



Life2000 Ventilator



Step Three: Change Oxygen Connection

What You'll Need for This Step




Oxygen Concentrator
(Not provided by Hillrom)



CombO₂® Hose



Breathe Pillows Interface

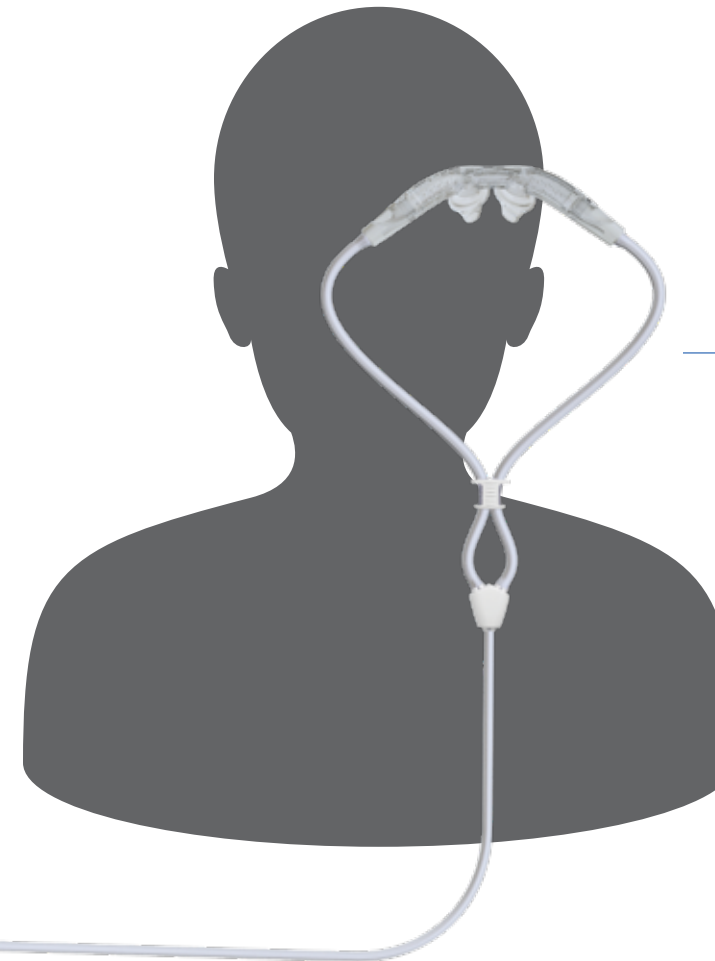

- Remove Current oxygen cannula from face
- Put On Breathe Pillows Interface



Hillrom™

For more information about products or services, please contact your Hillrom sales representative or Hillrom customer service at 1-800-426-4224.

hillrom.com/en/products/life2000-system

Life2000® and CombO₂® are registered trademarks of Breathe Technologies, Inc., a Hill-Rom company. Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale or rental of its products.

© 2021 Hill-Rom Services PTE Ltd. ALL RIGHTS RESERVED. APR186301 rev 1 26-JAN-2021 ENG – US