



Golvo 8000 / 8008

Quick Reference Guide

English

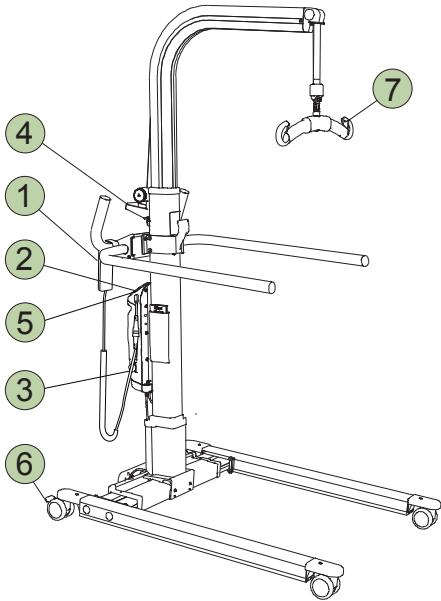


1 Handcontrol

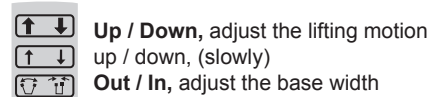

Read the instruction guide for more information.

2 Operation Panel

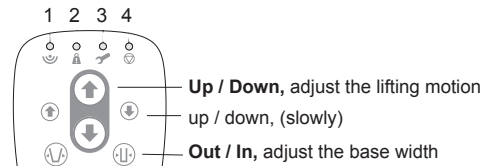


Indicator lamps (LDS)

1. Hand control active
2. Overload
3. Service indication
4. Expected Life Time (ELT)


Read the instruction guide for more information.

3 Emergency Stop

Activate:

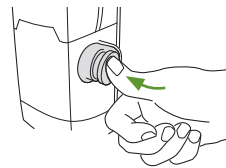


Reset:

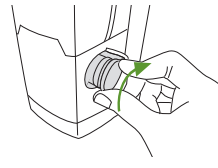


4 Mechanical Emergency Lowering

Activate:

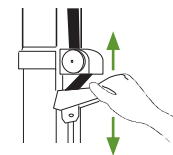


Resetting the lifting range:

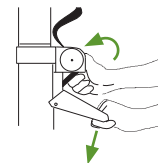


Read the instruction guide for more information.


5 Electrical Emergency Lowering/Raising

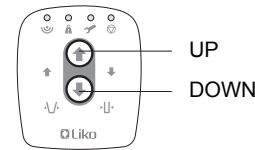


6 Locking the Wheels

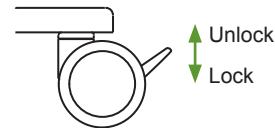

7 Latches

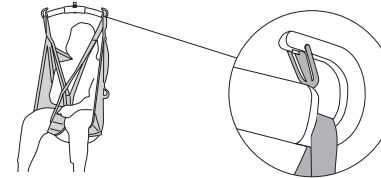


Avoid Unbalanced Load!

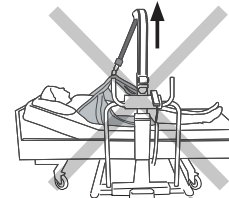

△ An unbalanced load increases the risk of the lift tilting over!

Read the instruction guide for more information.

Charging the Batteries

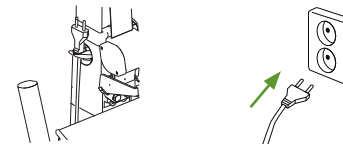

- Charge the lift regularly after use.
- The lift does not function during charging.
- **The lift cannot be charged with the emergency stop engaged.**

Ergonomics

- Never lift manually – let the lift do the lifting.
- Take it easy. Work calmly and methodically.
- Don't strain your back! Work in a walking stance with your back straight.
- Use the bed's raising and lowering functions.
- Reduce the lever effect. Work close to the patient and the lift.

Cleaning

When necessary, clean the lift with a moist cloth, using warm water or disinfectant.

Simple Troubleshooting

Check to ensure that:

- the emergency stop is not pressed in, see ③
- the battery of the lift is charged
- Check that the charger cable is not connected to an electric outlet.

Other Quick Reference Guides

Quick reference guides for Liko's lifts and slings are available for downloading at www.liko.com.

© Copyright Liko AB



Intertek



A Hill-Rom Company

www.liko.com

Manufacturer:

Liko AB
 SE-975 92 Luleå
 Sweden
info@liko.se

7EN140503-01

900003197