



Staff Name (PRINT): _____

Trainer's Signature: _____



INITIALS

1. Demonstrate and verbalize the available bed features to aid in transferring patients.		
2. Verbalize weight capacity of the lift, and demonstrate use of hand controls and safety features—including emergency operations.		
3. Verbalize hospital policy for assessing and determining patient dependency and process for communicating transfer needs (equipment and slings).		
4. Demonstrate seated sling application (Sling Application Golden Rules).		
5. Demonstrate prelift safety check of sling straps. Straps should be secured properly in the sling bar hooks. Safety latches should be in place. Sling straps should not be twisted.		
6. Demonstrate seated transfer from the bed to the chair/toilet/stretchers (seated).		
7. Demonstrate proper lifting height, ie, raising the patient to a minimal height to clear the bed/chair surface.		
8. Demonstrate seated sling removal with minimal manual lifting or skin shear.		
9. Demonstrate turning the patient from side to side and boosting up in bed using the repositioning sheet.*		
10. Demonstrate supine lateral transfer to the stretcher.*		
11. Demonstrate ambulation with LiftPants™ or MasterVest™ slings.*		
12. Demonstrate transfer from the floor.		
13. Demonstrate the use of the MultiStrap™ for limb holding, turning in bed, and foley catheter insertion.*		
14. Demonstrate the use of MultiStrap™ for proning.		
15. Verbalize the reason for never leaving the patient unattended during transfers.		
16. Verbalize the importance of returning the hand control to the charger after each use (if not equipped with in-rail charging).		

*if applicable

I have read and understand the associated Lift and Sling Instruction Guidelines.

Signature: _____

Date: _____

hillrom.com

Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale or rental of its products.