

This Professional PenLite is for general, NON-CONTACT illumination only.

WARNING: Do not use for transillumination as burns could result. This instrument is not to be used in the presence of flammable anesthetics.

PARTS OF THE PROFESSIONAL PENLITE:



CHANGING THE LAMP:

1. Unscrew lamp/tip assembly.
Discard. Replace with Welch
Allyn #07600 lamp/tip assembly.

CHANGING BATTERIES:

1. Unscrew lamp/tip assembly.
And remove old batteries.
2. Insert new AAA batteries
(may be inserted either way).

3. Replace lamp/tip assembly.
- For best light output and
battery life use alkaline
batteries.

WARRANTY:

Welch Allyn guarantees the PenLite, with the exception of lamps and batteries against all manufacturing defects. Welch Allyn will repair or replace free of charge any parts of its own manufacture proven to be defective through causes other than misuse, neglect, damage in shipment or normal wear.

WelchAllyn®

Welch Allyn, Inc.
4341 State St. Rd.
PO Box 220
Skaneateles Falls, NY 13153-0220
(315) 685-4560



European regulatory Manager,
Welch Allyn Ltd.
Navan Business Park
Dublin Road,
Navan, County Meath,
Republic of Ireland.
Tel. +353 46 67775
Fax. +353 46 27128

No. 766111