

Hill-Rom®

Progressa™ Bed System with Pulmonary Surface Skills Checklist

Staff Signature: _____

Trainer's Signature: _____

Date: _____



Perform the following:

COMPLETED

Bed Operations

1. Demonstrate the FlexAfoot™ feature for proper patient positioning	
2. Demonstrate how to weigh a patient	
3. Demonstrate how to zero the scale	
4. Demonstrate how to adjust a patient's weight	
5. Demonstrate how to switch between lbs and kgs	
6. Demonstrate where to view patient therapy history	
7. Demonstrate how to erase patient history information	
8. Demonstrate how to set the bed exit alarm	
9. Demonstrate how to activate the 30° head-of-bed alarm	
10. Demonstrate how to set the RemindMe® alarms	
11. Demonstrate how to use the "Help" function	
12. Demonstrate FullChair® position (only on chair egress frame design)	
13. Demonstrate Chair Egress with Stand Assist from foot end of the bed (only on chair egress frame design)	
14. Demonstrate CPR function and state what will happen to the surface once activated	
15. Demonstrate Trendelenburg and Reverse Trendelenburg	
16. Locate drainage bag holders	
17. Demonstrate the brake and steer mechanism	
18. Demonstrate the Boost® function	
19. Demonstrate use of siderail lockout controls	
20. Discuss appropriate use of the line managers	
21. Demonstrate use of the IntelliDrive® powered transport system (optional)	
22. Demonstrate use of caregiver pendant	
23. Demonstrate use of foot controls	

Progressa™ Bed System with Pulmonary Surface Skills Checklist

Perform the following:

COMPLETED

24. Demonstrate how to unlock Graphical Caregiver Interface (GCI)® controls	
25. Demonstrate patient egress from side of bed	
26. Demonstrate BMI calculator	
Pulmonary Surface Operations	
27. Demonstrate proper patient body alignment for CLRT/Percussion and Vibration therapies	
28. Initiate rotation therapy via the touch screen	
29. Identify bed positions that will stop rotation and sound an alarm	
30. Initiate Percussion and Vibration therapies via the touch screen	
31. Demonstrate Turn Assist	
32. Demonstrate Opti-Rest	
33. Demonstrate how to activate seat deflate	
34. Demonstrate use of the x-ray cassette sleeve	
35. Demonstrate comfort adjust	
36. Demonstrate sleep mode	

Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale or rental of its products.

©2013 Hill-Rom Services, Inc. ALL RIGHTS RESERVED.

ORDER NUMBER 183022 rev 1 E 09/03/2013 ENG – US

Zinc Number US-AC-0813-0349a(1)b

www.hill-rom.com

USA 800-445-3730
 CANADA 800-267-2337

Enhancing outcomes for patients and their caregivers:

