



Staff Name (PRINT): \_\_\_\_\_

Trainer's Signature: \_\_\_\_\_



INITIALS

1. Demonstrate and verbalize the available bed features to aid in transferring patients.		
2. Verbalize weight capacity of lift, and demonstrate use of hand controls and safety features.		
3. Verbalize hospital policy for assessing and determining patient dependency and process for communicating transfer needs (equipment and slings).		
4. Demonstrate process to apply Sabina lift vest.		
5. Demonstrate proper placement of shin pad (below patella) and feet on footplate.		
6. Demonstrate prelift safety check of sling straps. Straps should be secured properly in the sling bar hooks. Safety latches should be in place. Sling straps should not be twisted.		
7. Verbalize best patient hand and body (leaning back) placement during lifting and standing procedure.		
8. Demonstrate transfer from the bed to the chair/toilet/stretchers.		
9. Verbalize reason for never leaving the patient unattended during transfers.		
10. Demonstrate vest removal with minimal manual lifting or skin shear. Verbalize the reason for plugging in the lift when not in use.		

I have read and understand the associated Lift and Sling Instruction Guidelines.

Signature: \_\_\_\_\_

Date: \_\_\_\_\_