

# TotalCare® Bariatric Plus Bed with TotalCare® Bariatric Plus Surface

## Skills Checklist



Staff Signature: \_\_\_\_\_

Trainer's Signature: \_\_\_\_\_

Date: \_\_\_\_\_

Perform the following.

COMPLETED

### Bed Operations

1. Demonstrate the FlexAfoot™ feature for proper patient positioning.	
2. Demonstrate how to weigh a patient.	
3. Demonstrate how to zero the scale.	
4. Demonstrate how to adjust a patient's weight.	
5. Demonstrate how to switch between lbs and kgs.	
6. Demonstrate where to view patient therapy history.	
7. Demonstrate how to erase patient history information.	
8. Demonstrate how to set the bed exit alarm.	
9. Demonstrate how to activate the 30° head-of-bed alarm.	
10. Demonstrate how to set the RemindMe® alarms.	
11. Demonstrate how to use the "Help" function.	
12. Demonstrate preliminary tilt table position.	
13. Demonstrate FullChair® position.	
14. Demonstrate Chair Egress with Stand Assist from both the side and foot end of the bed.	
15. Demonstrate CPR and state what will happen to the surface once activated.	
16. Demonstrate Trendelenburg and Reverse Trendelenburg.	
17. Locate drainage bag holders.	
18. Demonstrate the brake and steer mechanism.	
19. Demonstrate use of the manual hydraulic pedal for non-powered feature activation.	
20. Demonstrate the Boost® function.	
21. Demonstrate use of siderail lockout controls.	
22. Discuss appropriate use of the line managers.	
23. Demonstrate use of the IntelliDrive® powered transport (optional).	

# TotalCare® Bariatric Plus Bed with TotalCare® Bariatric Plus Surface Skills Checklist

Perform the following.

COMPLETED

<b>Optional TotalCare® Bariatric Plus Surface Operations</b>	
24. Demonstrate proper patient body alignment for CLRT/Percussion and Vibration.	
25. Initiate Continuous Lateral Rotation Therapy via the Graphical Caregiver Interface (GCI)® screen.	
26. Identify bed positions that will stop rotation and sound an alarm.	
27. Initiate Percussion and Vibration via the Graphical Caregiver Interface (GCI)® screen.	
28. Demonstrate Turn Assist.	
29. Demonstrate Opti-Rest.	
30. Discuss patient positions that can be used during Percussion and Vibration.	
31. Demonstrate how to activate seat deflate.	
32. Demonstrate use of the x-ray cassette sleeve.	
33. Identify if low air loss, rotation or Percussion and Vibration modules are active.	
34. Demonstrate insertion and removal of the rotation, low air loss and Percussion and Vibration modules.	

Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale or rental of its products.

©2015 Hill-Rom Services, Inc. ALL RIGHTS RESERVED.  
170048 rev 4 25-NOV-2015 ENG – US

For further information about this product or a service, please contact your local Hill-Rom representative or visit our webpage:

**USA 800-445-3730**  
**Canada 800-267-2337**

[www.hill-rom.com](http://www.hill-rom.com)

Enhancing outcomes for patients and their caregivers:

