

TotalCare® Connect Bed with TotalCare SpO₂RT® Surface and TotalCare SpO₂RT® 2 Surface Skills Checklist

Staff Signature: _____

Trainer's Signature: _____

Date: _____



Perform the following.

COMPLETED

Bed Operations

	COMPLETED
1. Demonstrate the FlexAfoot™ feature for proper patient positioning	
2. Demonstrate how to weigh a patient	
3. Demonstrate how to zero the scale	
4. Demonstrate how to manually adjust a patient's weight	
5. Demonstrate how to switch between lbs and kgs	
6. Demonstrate where to view patient therapy history	
7. Demonstrate how to erase patient history information	
8. Demonstrate how to set the bed exit alarm	
9. Demonstrate how to activate the 30° head-of-bed alarm	
10. Demonstrate how to set the RemindMe® alarms	
11. Demonstrate how to use the "Help" function	
12. Demonstrate preliminary tilt table position	
13. Demonstrate FullChair® position	
14. Demonstrate Chair Egress with Stand Assist from both the side and foot end of the bed	
15. Demonstrate CPR and state what will happen to the surface once activated	
16. Demonstrate Trendelenburg and Reverse Trendelenburg	
17. Locate drainage bag holders	
18. Demonstrate the brake and steer mechanism	
19. Demonstrate use of the manual hydraulic pedal for non-powered feature activation	
20. Demonstrate the Boost® function	
21. Demonstrate use of siderail lockout controls	
22. Discuss appropriate use of the line managers	
23. Demonstrate use of the IntelliDrive® powered transport (optional)	

TotalCare® Connect Bed with TotalCare SpO₂RT® Surface and TotalCare SpO₂RT® 2 Surface Skills Checklist

Perform the following.

COMPLETED

Optional TotalCare SpO₂RT® and TotalCare SpO₂RT® 2 Surface Operations	
24. Demonstrate proper patient body alignment for CLRT/Percussion and Vibration therapies	
25. Initiate rotation therapy via the touch screen	
26. Identify bed positions that will stop rotation and sound an alarm	
27. Initiate Percussion and Vibration via the Graphical Caregiver Interface (GCI)® screen	
28. Demonstrate Turn Assist	
29. Demonstrate Opti-Rest	
30. Discuss patient positions that can be used during Percussion and Vibration	
31. Demonstrate how to activate seat deflate	
32. Demonstrate use of the x-ray cassette sleeve	
33. Demonstrate comfort adjust (TotalCare SpO ₂ RT® 2 surface only)	
34. Demonstrate sleep mode (TotalCare SpO ₂ RT® 2 surface only)	
35. Identify if low air loss, rotation or Percussion and Vibration modules are active	
36. Demonstrate insertion and removal of the rotation, low air loss and Percussion and Vibration modules (TotalCare SpO ₂ RT® surface only)	

Hill-Rom reserves the right to make changes without notice in design, specifications and models. The only warranty Hill-Rom makes is the express written warranty extended on the sale or rental of its products.

©2014 Hill-Rom Services, Inc. ALL RIGHTS RESERVED.
174416 rev 3 09-DEC-2014 ENG – US

For further information about this product or a service,
please contact your local Hill-Rom representative or
visit our webpage:

USA 800-445-3730
Canada 800-267-2337

www.hill-rom.com

Enhancing outcomes for
patients and their caregivers:

Hill-Rom