


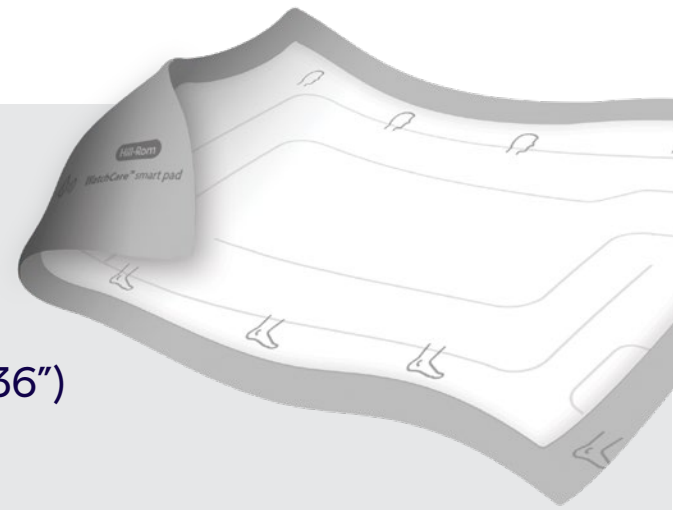

USING WATCHCARE® SMART PADS WITH THE WATCHCARE INCONTINENCE MANAGEMENT SYSTEM


Delivers discreet, timely notification of incontinence and other moisture events for improved patient satisfaction and enhanced skin safety.

- Soft, comfortable topsheet
- Highly absorbent
- Extra-Large (30" x 36")
- Moisture vapor permeable



PLACING THE WATCHCARE SMART PAD

- 1 Place smart pad in middle of bed utilizing the hip indicators on the bed side rail for proper placement. Ensure the head icon on the smart pad is positioned at top and the foot icon at bottom.
- 2 A confirmatory "chirp" will confirm smart pad is detected within 10 seconds.
- 3 Light at end of the bed will turn green, indicating the smart pad is detected.

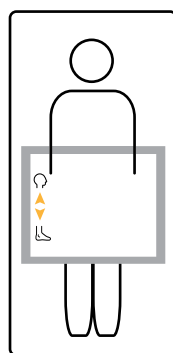

REPLACING THE WATCHCARE SMART PAD

- 1 A wet pad is indicated by a flashing amber light. (Alerts will also be sent to the nurse call light in the hallway and mobile devices as available at your facility.)
- 2 Remove the smart pad and dispose per hospital policy at least 2-3 feet away from the bed.
- 3 Replace the smart pad.



UNDERSTANDING THE WATCHCARE SMART PAD INDICATOR LIGHT


No Light

System is not plugged into power



White Light

System is on but not detecting a smart pad

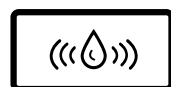

Green Light

System is on and a smart pad is in place



Flashing Amber Light

Smart pad is wet and needs to be changed



If system indicates a white light while a smart pad is on the bed, the system is unable to detect moisture. Check to ensure smart pad is placed in correct position with head icon toward head of bed and foot icon towards foot of bed.

SCAN HERE FOR THE WATCHCARE "HOW TO" VIDEO

