WelchAllyn®

7114x Universal Charger
Chargeur universel 7114x
Cargador universal 7114x
Carregador Universal 7114x
7114x Universal-Ladegerät
7114x Caricatore universale
7114x Universalladdare
Universele lader 7114x
7114x Universaloplader
7114x Universal Charger
7114x Yleislaturi
7114x Universallader
Ładowarka uniwersalna 7114x

Handle and Diagnostic Head sold separately.

English

Material 728486 DIR 80011862 Ver. E

The 7114x Universal Charger is intended to charge Welch Allyn rechargeable handles. Welch Allyn remains committed to providing innovative, quality diagnostic products for healthcare professionals. Call us at (800) 535-6663 to request a full line product catalog or visit us at our Web Site, www.WelchAllyn.com.

WARNINGS

 Attention: Consult user's manual for additional information.

 Green power light indicates that main power is connected to the appliance.

 Only authorized Welch Allyn Service Centers can perform service.


 DANGER: Risk of explosion if used in the presence of flammable anesthetics.

 Use of any accessories or materials not indicated in the user's manual can degrade the minimum safety of the equipment.

 This product complies with current required standards for electro-magnetic interference and should not present problems to other equipment nor is it affected by other devices. As a precaution, avoid using this device in close proximity to other equipment.

For information about electromagnetic compatibility (EMC), see the Welch Allyn website: www.welchallyn.com/emc-dskchrg. A printed copy of the Emissions and immunity information can be ordered from Welch Allyn for delivery within 7 calendar days.

Approvals

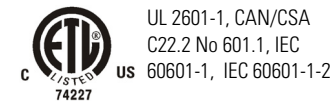The CE mark on this product indicates that it has been tested to and conforms with the provisions noted within the 93/42/EEC Medical Device Directive.

Authorized European Representative Address:
European Regulatory Manager
Welch Allyn Ltd.,
Navan Business Park
Dublin Road, Navan,
County Meath, Republic of Ireland.
Tel.: +353 46 90 67700 / Fax: +353 46 90 67756

REPAIRS

Please call the Technical Service Department for a Return Authorization number prior to shipping the Universal Charger back to Welch Allyn for repair. For repairs in North America, send unit to:

United States:

Welch Allyn, Inc.
Technical Service Department
4341 State Street Road
Skaneateles Falls, NY 13153
Telephone: 1-800-535-6663
FAX: 1-315-685-4653

(1)

Customers outside of North America should contact their local Welch Allyn distributor for the nearest authorized service center.

OPERATION

The Universal Charger is immediately ready for use when the power cord is plugged into any grounded outlet. When illuminated, the center green power light indicates that the Charger is plugged in and ready to charge either one or two rechargeable handles.

1	Power Light	When illuminated, the center green LED indicates the Power Cord is plugged in and is supplying main power to the Universal Charger.
2	Charge Indicator Lights	When illuminated, the side yellow LEDs indicate that a handle is sensed and is charging.
3	Charging Wells	Charge the following handle models: 71500 2.5v NiCad battery handle; 71670 3.5v NiCad battery handle; 71900/11 3.5v Lithium Ion battery handle; 72840 2.5v NiCad PocketScope™ handle (with 71249 adapter)
4	Power Cord	Plugs into any 100-240V, 50-60 Hz grounded outlet.

To Charge Handles

Place handle(s) into well(s). The yellow charge indicator LED directly under the handle well will illuminate, indicating the handle is sensed and charging. To fully charge a completely discharged battery handle, leave the handle in the charging well overnight. If you are charging your battery handle(s) for the first time, follow the charging instructions provided with your new battery handle. Once charged, placing a handle in a well after each use will top off the battery charge, maintaining the charge level of the battery. Leaving the battery handles in the charging wells overnight does not cause the handles to overcharge.

NiCad Battery Handles

Turn NiCad battery handles off before inserting the handle into the well.

Lithium Ion Handles

Once fully charged, the 71900/11 Lithium Ion handle will cause the yellow indicator LED to turn off. If desired, test the function of the charging well by placing a functioning instrument head on the handle in question, turn the handle on, and return the handle to the Universal Charger well. If the instrument lamp goes off, the handle was fully charged and the Universal Charger well is functioning properly (the yellow LED may light momentarily).

CLEANING

Periodically wipe the exterior of the Universal Charger with a damp cloth (soap and water). Use a dry cloth to clean the contacts inside the charging wells and the rechargeable battery handle bottoms. The Universal Charger is not intended for sterilization. Do **NOT** autoclave the Universal Charger, instruments, or handles.

ACCESSORIES

71249 PocketScope™ Rechargeable Handle Adapter for the Universal Charger (required to charge Welch Allyn PocketScope handles in the Charger).

71310 Matching Specula Tray for convenient storage of otoscope specula, currettes or swabs. Place the Specula Tray on a flat surface or mount to a wall.

71420 Wall Bracket for mounting Universal Charger to wall - saves valuable countertop space.

SPECIFICATIONS

Model No. 7114x Series

Power Cord: #18 AWG, 3 wire grounded cord, 6 feet long medical grade detachable with IEC60320 inlet.
Input: 100-240V, 50-60 Hz, 100mA max, Class I, Continuous Operation
Output: 2.4V - 4.0V DC 80mA max.
Weight: Approximately 660 grams (not including the rechargeable handles & cord)

(2)

Dimensions: Charger requires only 7.25" x 3.5" desk space (not including power cord)
IPX0 Not protected against the ingress of water.
Fuses (F1, F2): T100mA
Operating: 60°F (15°C) to 95°F (35°C), 75% R.H. Max, 500 hPa - 1060 hPa Altitude
Transport/Storage: -4°F (-20°C) to 120°F (49°C), 95% R.H. Max, 500 hPa - 1060 hPa Altitude

WARRANTY

Welch Allyn guarantees that this Universal Charger is free of any manufacturing defects. Welch Allyn will repair or replace, free of charge, any parts proven to be defective through causes other than misuse, neglect, damage in shipment and normal wear and tear. Welch Allyn warrants the 7114X Universal Charger will perform to original specifications for a one year period from the original date of purchase.

This product contains no hazardous materials. Its disposal will not contaminate or harm the environment, or present any risk to individuals disposing of the product. As a precaution, it is recommended that you contact your local disposal and/or recycling authority for information regarding the disposal of the equipment.

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