

Propaq® CS Vital Signs Monitor



Directions for Use

Models 242, 244, 246
Software version 3.7X

WelchAllyn®

Advancing Frontline Care™

© 2012 Welch Allyn. All rights are reserved. To support the intended use of the product described in this publication, the purchaser of the product is permitted to copy this publication, for internal distribution only, from the media provided by Welch Allyn.

Welch Allyn assumes no responsibility for any injury to anyone, or for any illegal or improper use of the product, that may result from failure to use this product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual.

Welch Allyn, Propaq, Acuity, FlexNet, and Smartcuf are registered trademarks of Welch Allyn.

SET and Masimo are registered trademarks of Masimo Corporation. Possession or purchase of a Masimo SpO₂-equipped monitor does not convey any express or implied license to use the device with unauthorized sensors or cables which would, alone or in combination with this device, fall within the scope of one or more of the patents relating to this device.

Nellcor and Oximax are registered trademarks of Nellcor Puritan Bennett.

Software in this product is copyright by Welch Allyn or its vendors. All rights are reserved. The software is protected by United States of America copyright laws and international treaty provisions applicable worldwide. Under such laws, the licensee is entitled to use the copy of the software incorporated with this instrument as intended in the operation of the product in which it is embedded. The software may not be copied, decompiled, reverse-engineered, disassembled or otherwise reduced to human-perceivable form. This is not a sale of the software or any copy of the software; all right, title and ownership of the software remain with Welch Allyn or its vendors.

For information about any Welch Allyn product, call Welch Allyn Technical Support:

USA	+ 1 315 685 4560 800 535 6663	Australia	+ 61 2 9638 3000
Canada	800 561 8797	China	+ 86 216 327 9631
European Call Center	+ 35 3 46 906 7790	France	+ 33 1 55 69 58 49
Germany	+ 49 695 098 5132	Japan	+81 42 703 6084
Latin America	+ 1 305 669 9003	Netherlands	+ 31 202 061 360
Singapore	+ 65 6419 8100	South Africa	+ 27 11 777 7555
United Kingdom	+ 44 20 7365 6780	Sweden	+ 46 8 58 53 65 51

DIR 80017878 Ver A



Welch Allyn Protocol, Inc.
8500 SW Creekside Place
Beaverton, OR 97008-7107 USA



Regulatory Affairs Representative
Welch Allyn Limited
Navan Business Park
Dublin Road
Navan, County Meath
Republic of Ireland

www.welchallyn.com



Contents

1 - General information	1
Intended use	1
Symbols	2
General warnings and cautions	3
Controls and connectors	7
Display	10
Menus	11
Learn Propaq CS operation with Inservice Mode	13
2 - Setup	15
Prepare the Propaq CS monitor for a new patient	15
Set patient alarms and alarm limits	17
Change the current Patient Mode	18
Change Powerup Patient Mode or store customized settings	19
3 - Monitoring	21
Perform ECG/RESP monitoring	21
Using the Propaq CS monitor with pacemaker patients	27
Perform Invasive Blood Pressure (IBP) monitoring	28
Take a Noninvasive Blood-Pressure (NIBP) reading	32
Perform temperature monitoring	37
Perform SpO ₂ monitoring	38
Perform Mainstream CO ₂ monitoring	43
Perform sidestream CO ₂ monitoring	45
Set up the CO ₂ display and alarm limits	47
4 - Alarms & Alerts	51
Description of alarm and alert tone patterns	51
Silence an active alarm or alert tone for 90 seconds	51
Inhibit alarm and alert tones for four minutes: 4 MIN SUSPND	53
Inhibit alarm and alert tones indefinitely: ALL ALARMS	54
Summary of alarm and alert keys and Acuity Central Station messages	55
Customize alarm limits based on patient's current vital signs	56
Alarm holdoffs	57
Connect nurse call option	57
Troubleshooting equipment alert messages	58
5 - Printing & trends	67
Print patient data	67
Display or print trends	68
Set printer options and automatic printing	70
Print OxyCRG	71
6 - Acuity	73
Understanding the Propaq CS, Acuity, and FlexNet network	73
Hardwired (Serial) connection to Acuity	77
Wireless connection to Acuity	79

- 7 - Maintenance 83**
 - Avoid electrostatic discharge 83
 - Connect the AC power adapter to recharge the battery. 83
 - Replace monitor input power fuse 86
 - Install printer paper 87
 - Inspect and clean the monitor and accessories 88
 - Service interval recommendations 89
 - Recycling monitor components 89
 - Extended storage precautions 90
 - Change the Wireless Propaq CS network name 90

- 8 - Reference 93**
 - Set the time and date 93
 - Change the date format, ECG filter, and units 94
 - Factory default settings 95
 - Specifications 97

- 9 - Compliance 117**

- Index 123**

1

General information

Intended use

The Propaq CS monitor is intended to be used by skilled clinicians for multiparameter vital signs monitoring of neonatal, pediatric, and adult patients in health care facility bedside applications. It is also intended for intra-facility transport.

- The ECG channel is intended for five-lead or three-lead ECG monitoring.
- The Respiration (RESP) channel is intended to detect the rate or absence of respiratory effort, deriving the signal by measuring the ac impedance between selected terminals of ECG electrodes.
- The Invasive Pressure (IBP) channel is intended for measuring arterial, venous, and intracranial pressures (and umbilical artery and vein pressures for neonates) using invasive transducers.
- The Noninvasive Blood Pressure (NIBP) channel is intended for indirectly measuring arterial pressures using an inflatable cuff. If ECG is also monitored, the Propaq CS Smartcuf™ software algorithm automatically synchronizes the NIBP measurement process to the occurrences of the R-wave, increasing accuracy in cases of extreme artifact and diminished pulses. The operator may disable or enable the Smartcuf algorithm in the NIBP Menu.
- The Temperature (TEMP) channel is intended to measure temperature using an attachable probe.
- The Pulse Oximetry (SpO₂) channel is intended to noninvasively measure oxygen saturation of arteriolar hemoglobin at a peripheral measurement site.
- The Capnography (CO₂) channel is intended to noninvasively measure the following vital signs or events: End-tidal CO₂ (ETCO₂), Inspired CO₂ (INCO₂), Breath Rate, and Apnea.
- Propaq CS monitors that do not include CO₂ or printer options are able to withstand light rain exposure over short periods of time (uniform distribution of approximately 1 mm of water/ minute for 10 minutes or less).

This guide was written for clinicians. Although this guide may describe some monitoring techniques, Welch Allyn® expects that you are a trained clinician who knows how to take and interpret a patient's vital signs. This monitor has been designed as a quality monitor; however, inherent limitations require that good clinical judgment always prevail.

Symbols

-  **WARNING** Indicates conditions that could lead to illness, injury, or death.
-  **Caution** In this manual, indicates conditions that could damage equipment or other property.
-  **Caution** On the product, means "Consult the accompanying documentation."

The following symbols may appear on the Propaq CS monitor or accessories. They are defined by the International Electrotechnical Commission, IEC 878 and IEC 417A.

	Off (Standby)		Two-way communication port
	On		Input port
	For continued fire protection, use only the specified fuse		Output port
	Direct current		Alternating current
	Non-ionizing electromagnetic radiation. This device contains an approved WLAN module frequency 2402 to 2480 MHz		Non-ionizing electromagnetic radiation. This device contains an approved WLAN module frequency 5150 to 5825 MHz
	Fuse		Recycle the monitor and battery separately from other waste. Refer to www.welchallyn.com/weee for collection point and additional information.
	Direct current or alternating current		Separate batteries from other disposables for recycling.
	Caution: Refer to Directions For Use and accompanying documentation		Enclosure Protection Drip proof: Classification IPX1 per EN 60529
	See the accompanying manual		Urgent alarm notification (output to Nurse Call system)
	Battery charging when green indicator illuminated		Temperature sensor input
	Patient connections are Type CF, isolated for direct cardiac application, and protected against defibrillation		Transformer meets requirements of a short-circuit-proof safety-isolating power transformer
	Patient connections are Type BF, and protected against defibrillation		For indoor use only (on power adapter only)
	Protected during defibrillation		Stacking limit by number
	This way up		Temperature limits
	Fragile		Humidity limit
	Keep away from rain		Altitude limit
	Patient connections are Type B		The device has met all essential requirements of European Medical Device Directive 93/42/EEC for a Class 1 product

 The CE Mark and Notified Body Registration Number signify the device has met all essential requirements of European Medical Device Directive 93/42/EEC




The Canadian Standards Association has evaluated this device according to CSA 601-1 and Underwriters Laboratory Standard UL 2601-1.

 Restrictions for use of wireless device in Europe. European Communities Class 2 radio equipment



This device has been tested and certified by the Canadian Standards Association International to comply with applicable U.S. and Canadian medical safety standards.

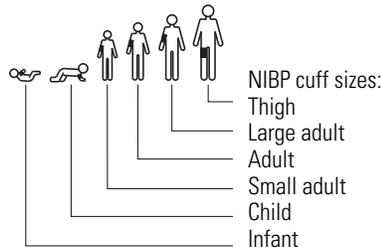

Apply the NIBP cuff as shown.



Apply the NIBP cuff as shown.



Single-use only (not reusable).

FCC ID: PGUWA11A07
IC: 4168a-WA11A07
This device complies with FCC and Industry Canada requirements for international radiators (802.11 wireless)

General warnings and cautions

Familiarize yourself with all warnings and cautions before using the Propaq CS monitor. In addition to the following, other warnings and cautions appear throughout this manual.



WARNING Safe interconnection between the Propaq CS monitor and other devices must comply with applicable medical systems safety standards such as IEC 60101-1-1. Within certain governmental jurisdictions, all interconnected accessory equipment must be labeled by an approved testing laboratory. After interconnection with accessory equipment, risk (leakage) current and grounding requirements must be maintained.

WARNING This monitor is to be operated by qualified personnel only. The operator of this monitor should read this entire manual and all accessory directions for use before operating the monitor.

WARNING Before you use a Propaq CS monitor on a new patient, always turn off the monitor for a few seconds, then turn it on again. This clears the prior patient's trend values, alarm limit settings, and NIBP cuff inflation target.

WARNING Always check the patient mode when monitoring a new patient. The patient mode determines default alarm limits, maximum cuff inflation pressure, and internal algorithm settings.

WARNING The monitor may not meet its performance specifications if stored or used outside the specified temperature and humidity ranges.

WARNING Place the Propaq monitor and accessories in locations where they cannot harm the patient if they fall from their shelf or mount. Lift the monitor only by its handle; do not lift it by any attached cables.



WARNING Do not connect more than one patient to a monitor. Do not connect more than one monitor to a patient.

WARNING Inspect the power adapter cord periodically for fraying or other damage, and replace the adapter as needed. Do not operate the apparatus from ac power with a damaged power adapter cord or plug.

WARNING Make frequent electrical and visual checks on cables, sensors, and electrode wires. All cables, sensors, and electrode wires must be inspected, properly maintained, and in proper working order to allow the equipment to function properly and protect patient safety.

WARNING As with all medical equipment, carefully route the patient cabling to reduce the possibility of patient entanglement or strangulation.

WARNING Avoid electrosurgery burns at monitoring sites by ensuring proper connection of the electrosurgery return circuit so that the return paths cannot be made through monitoring electrodes and probes.

WARNING During defibrillation, keep the discharge paddles away from ECG and other electrodes, as well as other conductive parts in contact with the patient. Avoid contact with any accessories connected to the monitor's left side panel.

WARNING To ensure patient safety, the conductive parts of the ECG electrodes (including associated connectors) and other patient-applied parts should not contact other conductive parts, including earth ground, at any time.

WARNING Do not operate this product in the presence of flammable anesthetics or other flammable substance in combination with air, oxygen-enriched environments, or nitrous oxide; explosion can result.

WARNING Electronic equipment that emits strong electromagnetic or radio frequency signals can cause electrical interference with ECG monitor operation. This interference may distort the displayed or recorded ECG signal, thereby preventing accurate rhythm analysis. Avoid operating this device near equipment of this type.

WARNING When using a power adapter with this monitor, be sure to connect the power adapter to a three-wire, grounded, hospital-grade receptacle. Do not under any circumstances attempt to remove the grounding conductor from the power plug of the power adapter. Do not plug the power adapter into an extension cord. If there is any doubt about the integrity of the protective earth ground of the receptacle for the power adapter, do not plug in the power adapter; operate the monitor only on battery power. Contact your biomedical engineering department for assistance in identifying the proper power receptacle and making appropriate power connections.

WARNING To help protect against electrical shock due to leakage current, use only accessories approved by Welch Allyn. Visit www.welchallyn.com. The use of any other accessories can result in inaccurate patient data, can damage the equipment, and can void your product warranty.

WARNING Always use accessories according to the standards of your facility and according to the manufacturer's directions for use.

WARNING Use of accessories not recommended by Welch Allyn may result in increased electromagnetic emissions or decreased electromagnetic immunity of the equipment.



WARNING If a product has been dropped or severely abused, send it to a qualified service person to confirm proper operation and acceptable risk (leakage) current values.

WARNING Some or all NIBP safety functions are disabled in the NIBP TEST screen in the Service Menu. Do not attempt to conduct NIBP TEST when the cuff is attached to a patient.

WARNING Do not use the Propaq CS monitor in a Magnetic Resonance Imaging (MRI) suite or a hyperbaric chamber. Such use can cause fire or explosion resulting in patient injury and monitor damage.

WARNING Impedance pneumography and CO₂ monitoring may not operate properly when used in conjunction with high-frequency jet ventilation or high-frequency oscillatory ventilation.

WARNING This monitor should only be repaired by qualified service personnel. The operator should not attempt to open the monitor case or perform any maintenance on the monitor except for procedures explicitly described in this manual that can be performed by operators such as inspection and cleaning.

WARNING When taking NIBP measurements, periodically observe the patient's limb to make sure that the circulation is not impaired for a prolonged period of time. Also make sure the blood pressure cuff is properly placed according to the *Propaq CS Directions for Use*. Be especially careful when using the short-term automatic mode (TURBOCUFF). Prolonged impairment of circulation or improper cuff placement can cause contusions.

WARNING The range of values measured by the monitoring parameters is provided in the Specifications section of the *Propaq CS Directions for Use*. Operation of the monitor outside the range of specified values is not recommended and may cause inaccurate results.

WARNING Electronic equipment that emits very strong electromagnetic or radio frequency signals can cause electrical interference with monitor operation, including causing the monitor to turn off power. Avoid operating this monitor near such equipment. For guidance about electromagnetic emissions and the recommended separation distance between the monitor and such equipment, refer to the specifications section of this manual.

WARNING Military radars are allocated as primary users in the bandwidths between 5.25 - 5.35 GHz and 5.47 to 5.725 GHz. In the event a radar signature is detected, the Access Point moves to a new channel, which can temporarily interrupt patient monitoring. If the device (wireless) is operated near a military radar, the radar could cause damage to the device.

WARNING This product contains no user-serviceable components. Any unauthorized changes to the product invalidate Welch Allyn's warranty and also invalidate all applicable regulatory certifications and approvals.

WARNING Do not use the pulse oximeter as a replacement or substitute for ECG-based arrhythmia analysis.



WARNING The bedside patient monitor is the primary alarming source for the patient and a central station is a backup alarm source. The central station is only as reliable as its network and should be relied on only as a backup alarming device.

WARNING The leading cause of patient death or serious injury reported with the use of patient monitoring equipment is failure to respond to alarms notifying the user of an adverse change in patient condition. If you are relying on visual alarm notifications, maintain a clear line of sight and remain within 4 meters of the monitor or the central station. If you are relying on audio alarm notifications, make sure that you can hear audio alarms from where you are. Set the volume as needed considering the environment and ambient noise levels. Verify that the alarm is audible to a clinician working at the maximum distance from the monitor or central station.

WARNING Auto-reboots occur and wireless connection is occasionally disrupted. During this period, the bedside monitors continue to provide their primary alarming functions. Auto-reboots occur infrequently, due to poor environmental conditions. Failure to perform preventative maintenance can increase the frequency of occurrence.

WARNING False alarms may occur in some situations. You must understand and address the cause of the false alarms whenever possible to eliminate the possibility of repeated false alarms and alarm fatigue, which might result in a failure to respond to an actual alarm situation.



Caution Changes or modifications not expressly approved by Welch Allyn could void the purchaser's authority to operate the equipment.


Caution Do not autoclave the Propaq CS monitor. Autoclave accessories only if the manufacturer's instructions clearly approve it. Many accessories can be severely damaged by autoclaving.

Caution Federal USA law restricts this device to sale, distribution, or use by or on the order of a licensed medical practitioner.

It is possible for the monitor to detect a problem that prevents the monitor from operating properly. If this occurs, the monitor displays an error message and error number. Report such errors to Welch Allyn.

The Propaq CS monitor should be serviced only by a Welch Allyn service technician while under warranty. *Propaq CS Service Manual* (810-2708-XX) is available from Welch Allyn to assist the biomedical engineer during post-warranty period service.

Controls and connectors

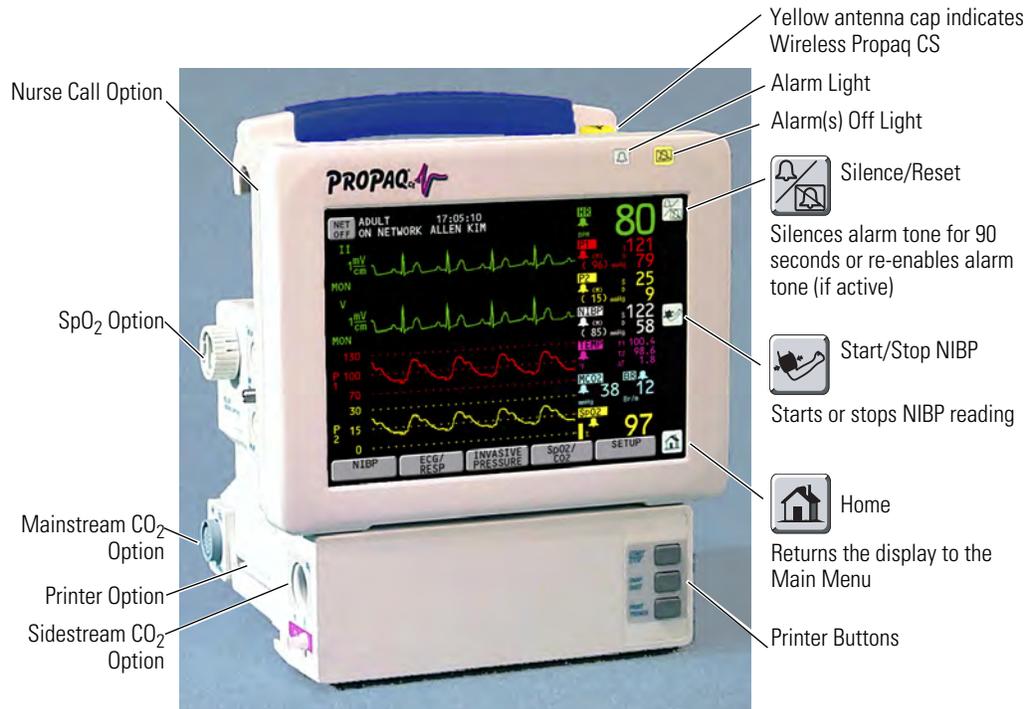

Touch-screen controls


The front panel touch-screen provides five softkeys along the bottom and three icon-labeled keys along the right side. An Acuity® **NET OFF** key is displayed in the upper left corner if the monitor is connected to an Acuity system. These keys allow control of all monitoring and setup functions.

Note Avoid pressing more than one touch-screen key at a time. Touching more than one key area at a time can cause the touch-screen to misinterpret the command and respond to the wrong key.




Caution Do not touch the screen with a sharp object such as a pen or pencil. Sharp objects can damage the touch-screen. Use your finger to press the touch-screen keys.

System control and connectors (right side panel)

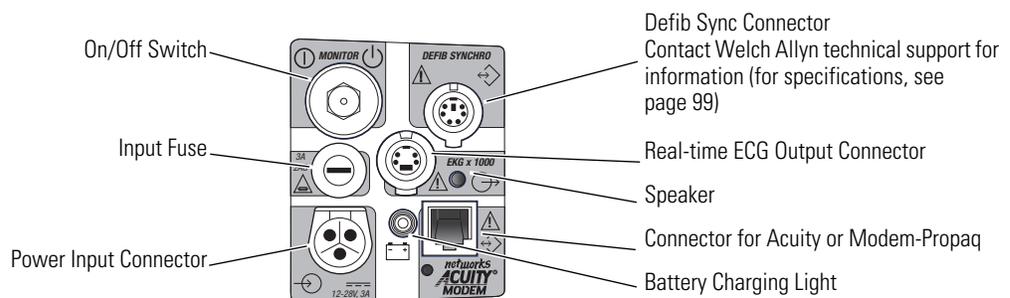

Patient connectors (left side panel)

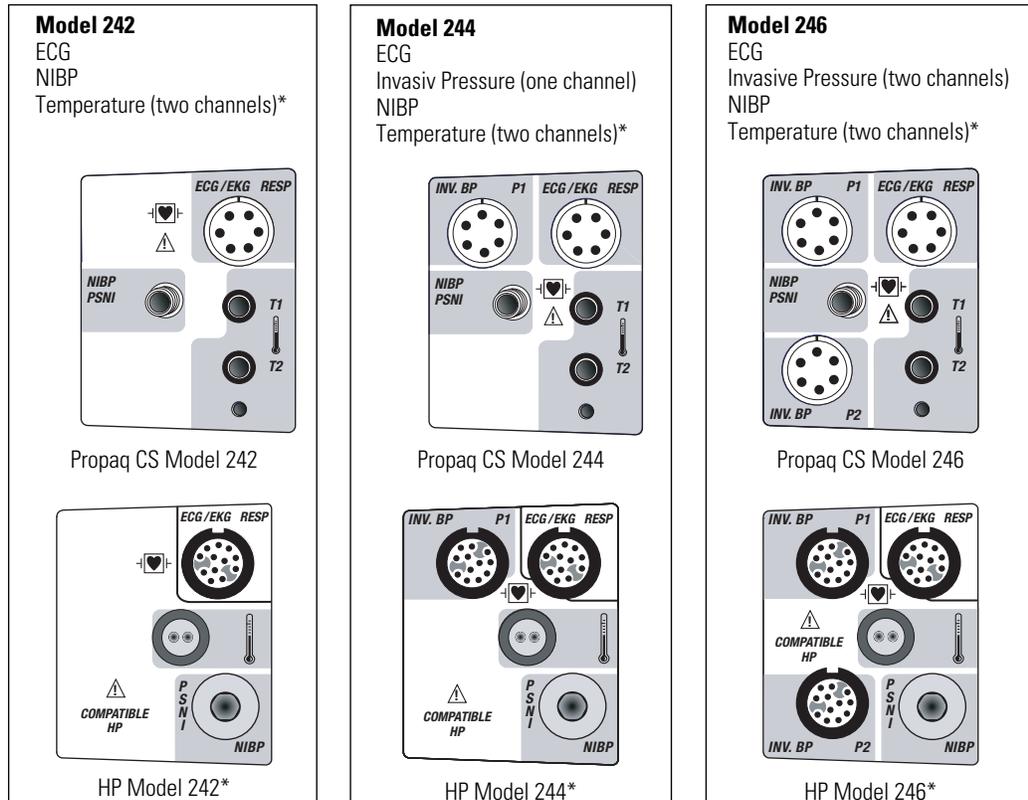


*The HP (Hewlett-Packard) side panels provide only one temperature connector.

Option connectors




Mainstream CO₂ Connector

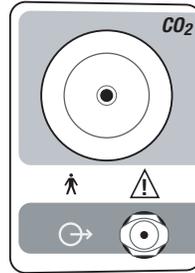


Sidestream CO₂ Connector




Nurse Call Connector

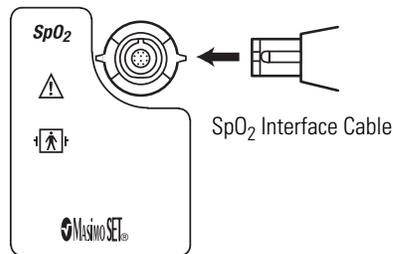

Masimo SpO₂ Connector
(motion-tolerant)

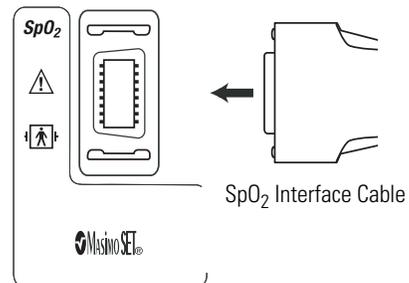


Masimo SpO₂ Connector
(motion-tolerant)

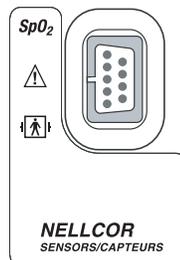


Nellcor SpO₂ Connector

Display

You can select up to four waveforms to be shown on the Propaq CS monitor. When only one waveform is selected, a trend window automatically appears beneath the waveform.

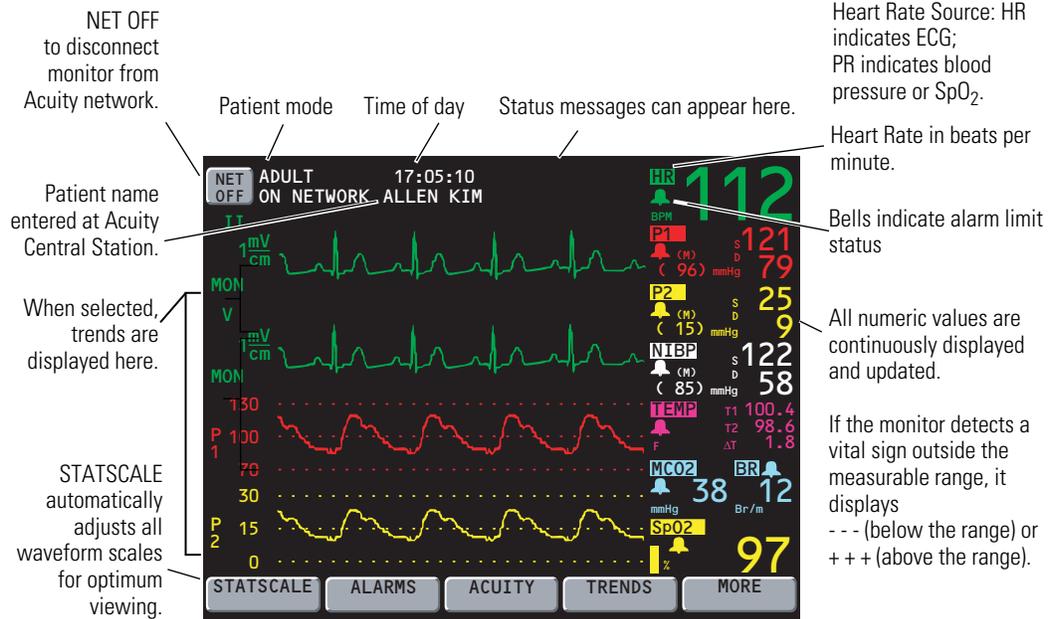


While changing monitor settings, a status window may appear below the waveform:

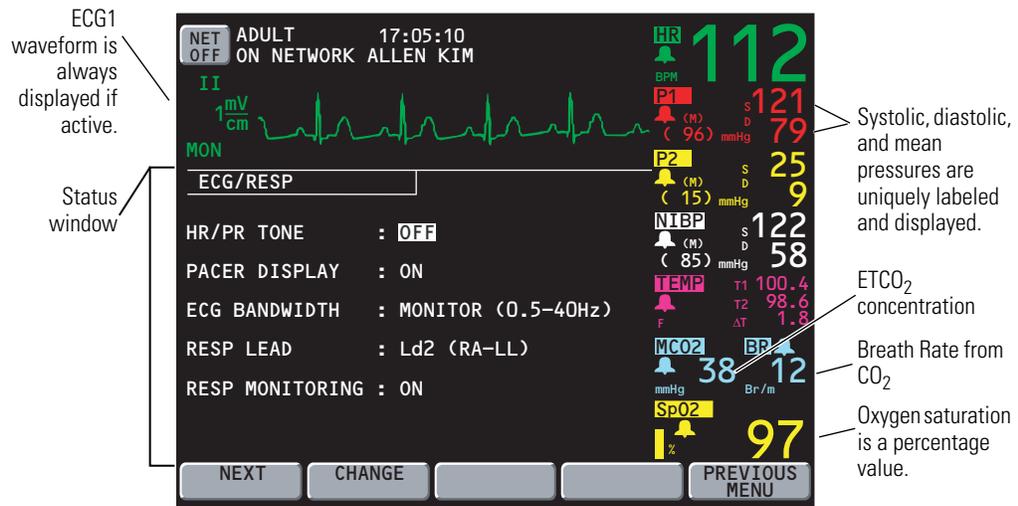


Menus

Menus for some patient vital signs are displayed only if the option is included in your Propaq CS monitor.

Main Menu

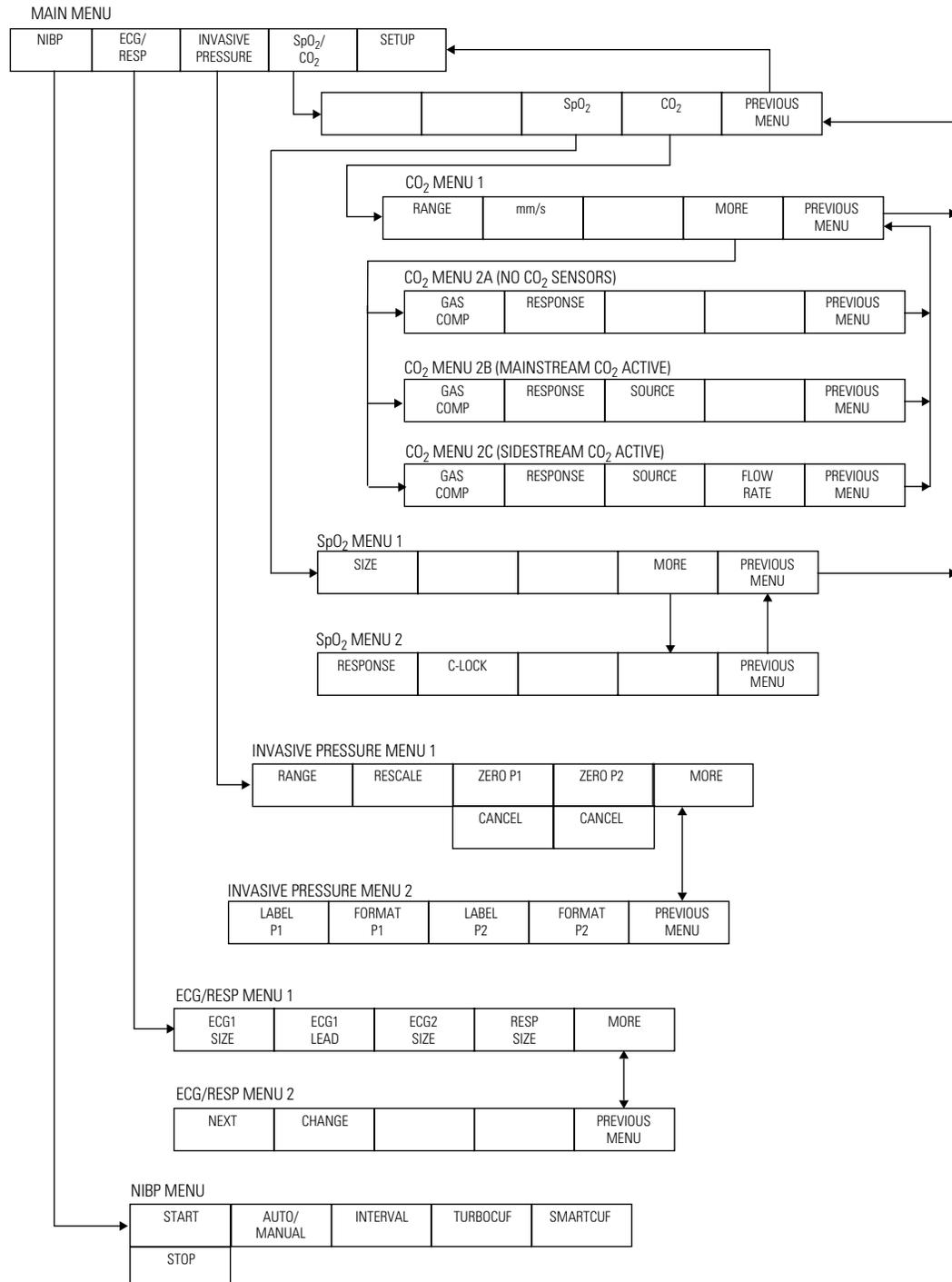

Setup Menus

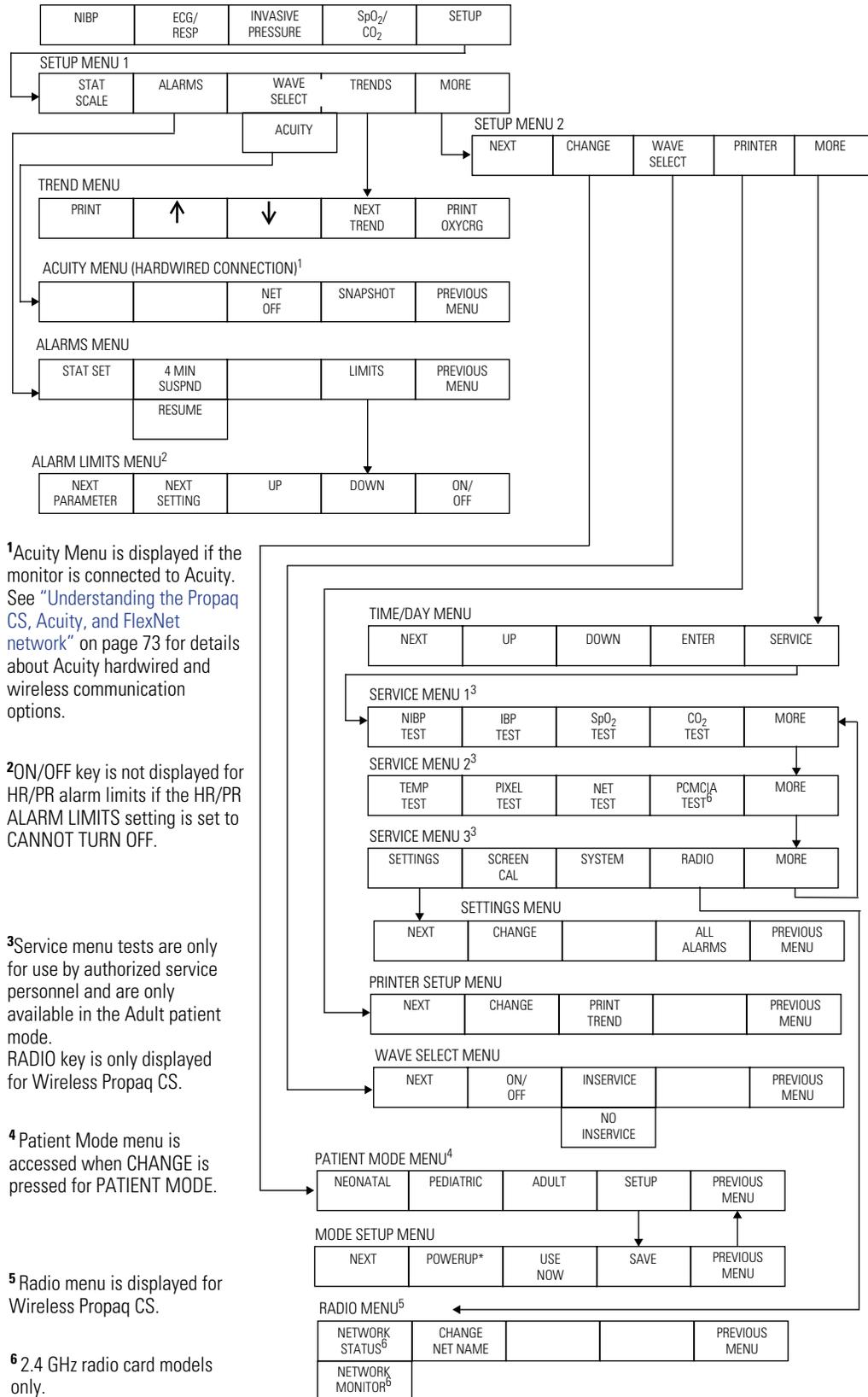

¹Acuity Menu is displayed if the monitor is connected to Acuity. See “Understanding the Propaq CS, Acuity, and FlexNet network” on page 73 for details about Acuity hardwired and wireless communication options.

²ON/OFF key is not displayed for HR/PR alarm limits if the HR/PR ALARM LIMITS setting is set to CANNOT TURN OFF.

³Service menu tests are only for use by authorized service personnel and are only available in the Adult patient mode. RADIO key is only displayed for Wireless Propaq CS.

⁴Patient Mode menu is accessed when CHANGE is pressed for PATIENT MODE.

⁵Radio menu is displayed for Wireless Propaq CS.

⁶2.4 GHz radio card models only.

Learn Propaq CS operation with Inservice Mode

You can practice using the Propaq CS monitor with the in-service mode of operation.

The in-service mode cannot be activated while you are monitoring a patient. In in-service mode, the monitor display and all printouts include the message SIMULATING or SIMULATED DATA.

To practice with your Propaq CS monitor:

1. Disconnect all patient cables connected to the monitor. You can leave the NIBP cuff connected to the monitor so you can take NIBP measurements.
2. To temporarily remove customized alarm settings, press **Home, SETUP, MORE, CHANGE, SETUP, USE NOW, YES**.
3. To put the Propaq monitor in (non-alarming) Inservice Mode 1, press **Home, SETUP, WAVE SELECT, INSERVICE**.
4. To set the Propaq monitor in (alarming) Inservice Mode 2, press **Home, SETUP, WAVE SELECT, INSERVICE**.

Within 5 seconds, the monitor alarms because the “patient’s” vital signs fall outside the alarm limit ranges.

5. Confirm the following:

The monitor is sounding an alarm

A vital sign numeric is flashing

The small red light in the top right corner of the monitor is flashing

The SILENCE and LIMITS keys are showing on the screen

6. To silence the alarm tone for 90 seconds, press **Silence/Reset** or **SILENCE**.

Visual alarm indications remain, and the alarm tone resets after the 90-second silence period.

7. To bring the Propaq monitor back into the (non-alarming) Inservice Mode 1, press **INSERVICE**.

The “patient’s” vital sign readings will return to acceptable levels within the alarm limit range.

8. While in the inservice mode, you can press any monitor keys (except the **AUTO/MANUAL** key in the NIBP Menu) to change a function setting. For example, you can change ECG and RESP waveform sizes, set alarm limits, or set custom settings.

You can also apply the NIBP cuff to yourself and take NIBP measurements.

9. Turn off the Propaq monitor.

Any previously set custom patient mode settings are restored when you turn it on again.

Under these conditions, if the monitor fails to generate visual or auditory alarm indications, carefully repeat the above steps. If the monitor is still unresponsive, remove it from circulation and take it to your facility’s biomedical service department for evaluation.

Note The inservice mode is not available if the monitor detects that a sensor has been connected (except for an NIBP cuff) or the NIBP Automatic Mode has been selected. If the monitor is in inservice mode and you connect a sensor (except for an NIBP cuff) or press the NIBP AUTO/MANUAL key, the monitor will turn off power to exit the inservice mode, and then turn on in the normal operating mode. The pacemaker signal indicators are not displayed in the inservice mode.

2

Setup

Prepare the Propaq CS monitor for a new patient



WARNING Before you use a Propaq CS monitor on a new patient, always turn it off for a few seconds and then turn it on again. This clears the prior patient's trend values, alarm limit settings, and NIBP cuff inflation target.

1. Press the gray recessed **MONITOR** button on the right side of the monitor to turn the monitor off (if it is on). Press it again to turn the monitor on. The monitor displays the powerup screen for about 10 seconds, then displays the Main Menu. The monitor is in the powerup patient mode with the associated settings.
2. Confirm that the monitor emits a tone. If the monitor has SpO₂, listen for two tones and confirm both speakers are working.

Note Check the battery voltage level on the powerup screen (or check it on the Time/Day window: **Home, SETUP, MORE, MORE**). If the battery voltage is 7.4V or less or a low battery message is displayed, connect the monitor to an ac power adapter to recharge the battery (page 20). Connecting the adapter does not interrupt patient monitoring.



3. Confirm the monitor is in the correct patient mode according to the patient's age. If the patient mode is not correct, from the Main Menu press **SETUP, MORE, CHANGE** to access the Patient Mode window:



4. Based on the patient's age, press **NEONATAL, PEDIATRIC, or ADULT**. When the confirmation window appears, press **YES** to confirm your selection.

3. Press **NEXT PARAMETER** to highlight the parameter you want to change, then press **NEXT SETTING** to highlight the limit you want to change.
4. Press **UP**, **DOWN**, or **ON/OFF** to change the limits.
The apnea alarm cannot be turned off at any time.
5. After setting the desired limits, press **Home** to return to the Main Menu.



Change the current Patient Mode

1. To change the current patient mode, from the Main Menu press **SETUP, MORE, CHANGE** to access the Patient Mode window:



2. Based on the patient's age, press **NEONATAL**, **PEDIATRIC**, or **ADULT**. When the confirmation window appears, press **YES** to confirm your selection.

Whenever you change the patient mode, the alarm limit settings, maximum NIBP cuff inflation pressures, and internal computations are automatically changed to the defaults for that patient mode. See page 19 for information about preset Factory patient modes or programmable Custom patient modes.

- Note** If you change the patient mode, the CO₂ alarm limits in the new mode might vary slightly from the originally-programmed CO₂ alarm limits for the new mode. Check the CO₂ alarm limits.

Change Powerup Patient Mode or store customized settings

The Propaq CS monitor has standard, preset, default powerup settings and alarm limits for each patient mode: Adult, Pediatric, and Neonatal. These are “Factory Patient Mode” settings (listed on “[Factory default settings](#)” on page 95).

You can also choose to customize and store programmable powerup settings and alarm limits for each patient mode. These are “Custom Patient Mode” settings.

The instructions below describe how to change the powerup patient mode and how to select and store new Custom Patient Mode settings.

Note When you change patient modes, you also change the alarm limits associated with the new patient mode.

Change the Powerup Patient Mode

1. From the Main Menu, press **SETUP, MORE, CHANGE, SETUP** to access the Mode Setup window.

Asterisk indicates which patient mode is currently selected for powerup.

* = PATIENT MODE ON POWERUP

2. Press **NEXT** to highlight the desired Factory or Custom powerup mode, then press **POWERUP*** and **YES**. The new powerup selection is marked by the asterisk.

Changing the powerup mode does not affect the patient mode currently used.

Customize Patient Mode settings

1. From the Main Menu, press **SETUP, MORE, CHANGE, SETUP** to access the Mode Setup window.
2. The patient mode you want to reprogram (ADULT, PED, or NEO) must be currently selected. To make sure it is currently selected, press **NEXT** as needed to highlight the desired mode, then press **USE NOW** and **YES**.
3. Press **Home** to exit the Mode Setup window, then use other menus and keys to set the monitor settings and alarm limits as desired.



Home

A convenient way to access settings and alarm limits for all functions without connecting cables is to select the in-service mode (disconnect all patient cables, turn the monitor power off and then on, then press **SETUP, WAVE SELECT, INSERVICE** from the Main Menu).



WARNING If any alarms are set to OFF and you select SAVE to store settings for a Custom patient mode, those alarms will be OFF when the monitor powers up in that Custom patient mode or that Custom patient mode is selected. Consider carefully before setting Custom patient mode powerup alarms to OFF.

4. Re-enter the Mode Setup window, press **NEXT** as needed to highlight the desired Custom mode, then press **SAVE** and **YES**.

If the in-service mode is used, turn off the monitor to exit the in-service mode.

3

Monitoring

Perform ECG/RESP monitoring



WARNING Impedance pneumography detects respiratory effort via changes in chest volume; therefore, impedance pneumography can be used to detect central apnea. However, apnea episodes with continued respiratory effort, such as obstructive apnea and mixed apnea, may go undetected. Also, artifact due to patient motion, apnea mattress shaking, or electrocautery use may cause apnea episodes to go undetected. Always monitor and set alarms for SpO₂ when using impedance pneumography to monitor respiratory function.

WARNING The Propaq CS monitor automatically rejects cardiovascular artifact. This function is dependent upon accurate ECG R-wave detection. Therefore, always select the ECG lead with the most prominent QRS complex when monitoring respiration via impedance pneumography.

WARNING Don't place the Propaq CS monitor with RESP in close proximity to another respiration monitor because the RESP measurement frequencies may interfere with one another.

WARNING Because pacemaker pulses in some instances may be falsely counted as breaths, impedance pneumography is not recommended for use on paced patients.

WARNING Motion artifact can cause incorrect breath rate or heart rate readings. Minimize patient motion whenever possible.

WARNING If a disconnected lead is in too close proximity to other electrical devices, it may cause false heart rate, a failure to detect apnea, or a failure to display a Lead Fail message.

WARNING The Propaq CS monitor does not provide arrhythmia analysis. Therefore, arrhythmias are not analyzed and may cause the monitor to display inaccurate heart rates.

WARNING The Propaq CS monitor shows + + + for HR numerics between 301 and 350 beats per minute. Above 350 beats per minute, it may display incorrectly low heart rates, due to intermittent picking of R-waves.



WARNING High-intensity radio frequency (RF) energy from external sources, such as an improperly connected electrosurgical unit, can induce heat into electrodes and cables which can cause burns on the patient. Reading errors and damage to equipment may also result. This hazard can be reduced by (1) avoiding the use of small ECG electrodes, (2) selecting ECG electrode attachment points remote from the surgical site and from the electrosurgical return electrode, (3) using electrosurgical return electrodes with the largest practical contact area, and (4) assuring proper application of the electrosurgical return electrode to the patient.

WARNING Verify patient mode. Incorrect patient mode may result in inaccurate heart rates and inappropriate alarm settings.

WARNING To help prevent injury, use the provided garment clips to route the ECG cables away from the patient's head.

WARNING Use of ECG cables with loose or faulty detachable lead wires may cause erratic behavior of the ECG waveform, SpO₂ (C-LOCK), and NIBP (Smartcuf) due to intermittent ECG lead wire connections.

WARNING Use only ECG safety cables that are designed so that they cannot accidentally be plugged into an ac mains outlet or make contact with other hazardous electrical potentials including earth ground. To prevent damage during defibrillation, don't use ECG cables without 1 k Ω series resistors.

WARNING Before you use a Propaq CS monitor on a new patient, always turn it off for a few seconds, then turn it on again. This clears the prior patient's trend values and alarm limit settings.



Caution To protect the Propaq CS monitor from damage during defibrillation, for accurate ECG information, and for protection against noise and other interference, use only ECG electrodes and cables specified or supplied by Welch Allyn (these cables have the required current-limiting resistors). Follow recommended application procedures.

- Impedance pneumography (RESP) is not recommended for use with high frequency ventilation.
- Since RESP is derived from the same leads as the ECG channel, the Propaq CS monitor determines which signals are cardiovascular artifact and which signals are a result of respiratory effort. If the breath rate is within five percent of the heart rate or a multiple or sub-multiple of the heart rate, the monitor may ignore breaths and trigger an apnea alarm.
- When monitoring RESP it is highly recommended that you use SpO₂ monitoring as a backup monitoring method.
- It is recommended that Propaq CS monitor respiratory measurements not be made in close proximity to electrically radiated equipment. When the Propaq CS monitor is operated per EN 60601-1-1 (radiated RF immunity 3 V/m), field strengths above 2 V/m may cause erroneous measurements at various frequencies.
- The Propaq CS monitor counts as "breaths" respiratory efforts that are larger than two times background cardiovascular artifact.
- Even though the Propaq CS monitor contains fully isolated patient-connected circuitry, it has not been specially designed for direct application on a patient's heart.

- Use only with accessories approved by Welch Allyn. Visit www.welchallyn.com. The use of any other accessories can result in inaccurate patient data, can damage the equipment, and can void your product warranty.
- Severe artifact and interference (such as defibrillation interference) can cause the waveform to move off the display for a few seconds before it is restored.

Prepare for ECG/RESP monitoring

1. Inspect the ECG cable and replace it if it shows signs of wear, breakage, or fraying.
2. Select the appropriate patient mode. To change patient modes, from the Main Menu press **SETUP, MORE, CHANGE**, then the desired patient mode (**NEONATAL, PEDIATRIC, or ADULT**) and then **YES**.

3. Select electrode sites on the patient.

Choose flat areas; avoid fatty areas and major muscles.

4. Shave or clip hair from electrode sites, thoroughly clean skin, and lightly rub dry.


You may use soap and water, isopropyl alcohol or special skin preparation pads. To avoid allergic reactions to electrodes, refer to the electrode manufacturer's directions.


5. If you are using pre-gelled electrodes, make sure the electrode date is not expired and the gel is intact and not dried out. For best results, use only silver/silver chloride electrode.

If you are using non-gelled electrodes, apply a 1/4- to 1/2-inch mound of gel over the electrode contact area.

For best product performance and measurement accuracy, do not use stainless steel needle electrodes, squeeze bulb electrodes, or electrodes with dissimilar metals. Do not use electrodes from more than one manufacturer on the same patient.

6. Attach lead wires to the electrodes before applying them to the patient. Apply the electrodes to the patient as shown.

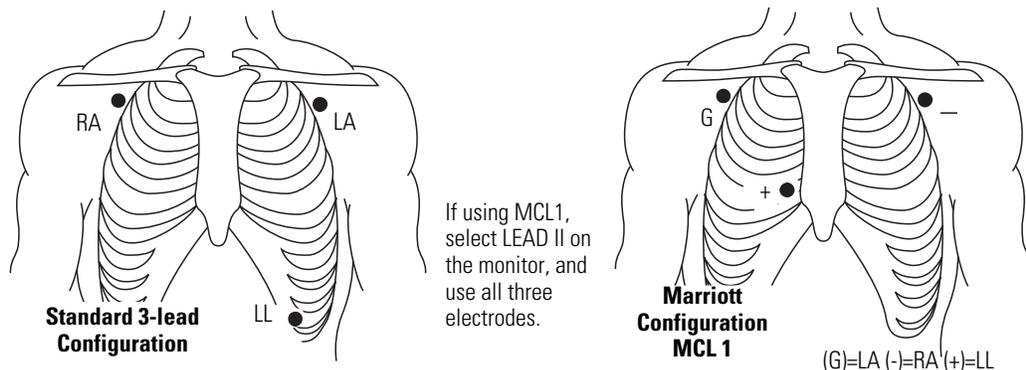

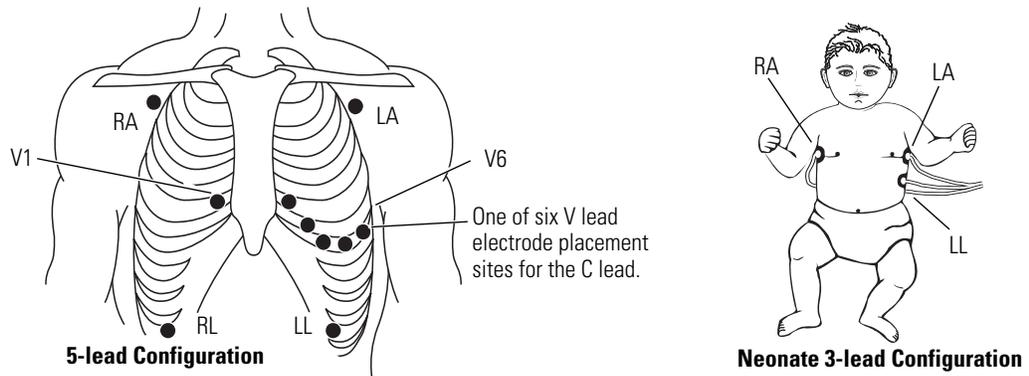


7. Plug the ECG cable into the ECG connector on the monitor's left side panel.
8. Support the ECG cable so it does not stress the electrode wires, ECG cable connectors, or electrodes.
9. If an electrosurgical unit will be used, place the ECG cable and electrode wires as far as possible from the surgical site and from the electrosurgical return electrode and its cables. This minimizes interference.

Although the ECG channel contains electrosurgical interference suppression (ESIS) circuitry, noise artifact may be displayed on the ECG trace while an electrosurgical device is in use. Choose electrode placement to minimize interference.

10. Look for an ECG waveform and heart rate on the monitor. Depending on how your monitor is programmed, a beep tone may occur with each detected QRS.

If there is no waveform, check the electrodes, wires, cable, and the monitor for a possible misconnection or lead fault.

11. To set up the ECG/RESP display, from the Main Menu press **ECG/RESP** to display the first ECG/RESP menu:

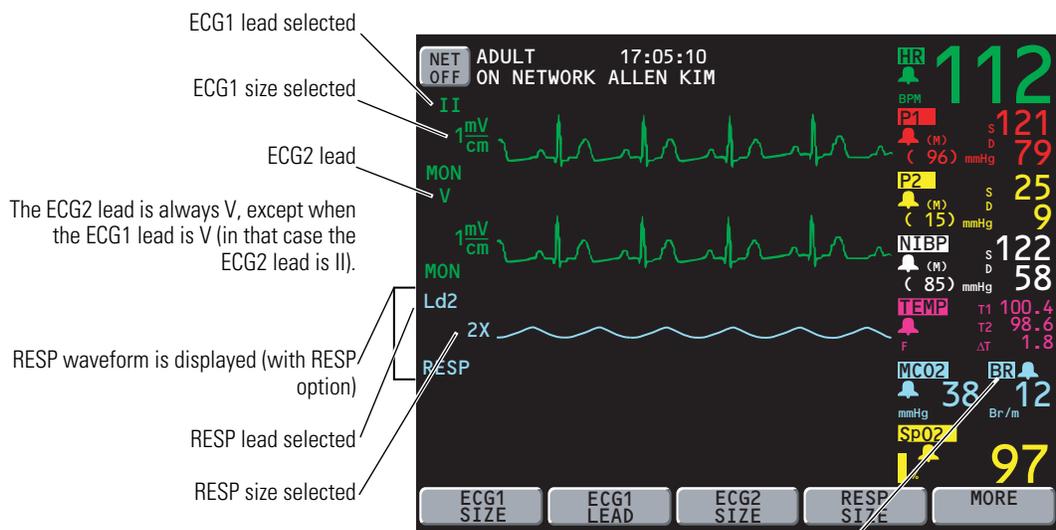

BR indicates respiration rate is derived from active CO₂ channel.
RR indicates CO₂ is not monitored and respiration rate is derived from ECG/RESP.

12. Press buttons as desired to adjust the display:

- ECG1 SIZE Selects the ECG1 waveform size: 4, 2, 1, 0.5, or 0.2 mV/cm.
- ECG1 LEAD Selects the ECG1 lead: I, II, III, aVR, aVL, aVF, or V.
 Selections aVR, aVL, aVF, and V are only available with a 5-lead ECG cable.
 The ECG2 lead is always V, except when the ECG1 lead is V (in that case the ECG2 lead is II).
- ECG2 SIZE Selects the ECG2 waveform size: 4, 2, 1, 0.5, or 0.2 mV/cm.
- RESP SIZE Selects the RESP waveform size: 1x, 2x, 4x, 8x, or 16x.
 The QRS detector sensitivity threshold is not affected by changing the ECG display size. Likewise, the RESP breath detector threshold is not affected by changing the RESP display size.

13. Press **MORE** to display the second ECG/RESP menu and status window:



14. Press **NEXT** and **CHANGE** as desired to adjust the display.

- HR/PR TONE Sets heart tone loudness to LOW, MEDIUM, HIGH, or OFF.
 If SpO₂ is monitored, tone pitch varies with the SpO₂ value.
- PACER DISPLAY Turns on and off the pacer indicator in the ECG waveform.
 If the patient has a pacemaker, you may want to turn on the pacer indicator (see page 27).
- ECG BANDWIDTH Selects the bandwidth for displayed and printed data.
 MONITOR is 0.5 Hz to 40 Hz (Adult mode) or 0.5 Hz to 120 Hz (Pediatric and Neonatal mode).
 Monitor Mode filters out extraneous noise and artifact to provide a more stable display.
 EXTENDED is 0.05 Hz to 40 Hz (Adult mode) or 0.05 Hz to 120 Hz (Pediatric and Neonatal mode).
 Extended Mode is a higher-resolution setting that allow more detailed analysis.

Always use Extended Mode when observing ST segment morphology on the display or printer. Although Monitor Mode is useful to minimize baseline wander due to artifact, ST segments can be distorted in Monitor Mode. This can potentially cause underestimation of ST elevation and overestimation of ST depression. Although the monitor does not have automated ST segment monitoring, ST segments may be accurately displayed and printed in Extended Mode.

RESP LEAD

Selects the RESP lead: Ld1 (RA-LA) or Ld2 (RA-LL). RESP lead selection is independent of ECG lead selection.

Choose the RESP lead that gives you the best signal. If neither signal is adequate, experiment with nonstandard electrode placement such as placing the RA and LA electrodes on the respective mid-axillary lines just above the level of the nipples.

RESP MONITORING Turns RESP on or off.

15. Set alarms according to your facility's standards.

Use the ECG filter to display a better waveform

If the ECG waveform appears unclear or distorted, make sure the monitor ECG filter is properly set to reduce interference from your facility's ac power frequency. To check the filter:

1. Press **SETUP, MORE, MORE, SERVICE, YES** to access the Service Menu.
2. Press **MORE, MORE, SETTINGS** to display the Settings Menu.
3. If the **FILTER** setting does not match your ac power frequency (60 or 50 Hz), press **NEXT** to highlight **FILTER**, then press **CHANGE** to change settings.

Contact a qualified service person if you have questions.

Using the Propaq CS monitor with pacemaker patients




WARNING Pacemaker signals can differ from one pacemaker to the next. The Association for Advancement of Medical Instrumentation (AAMI) cautions that “in some devices, rate meters may continue to count the pacemaker rate during occurrences of cardiac arrest or some arrhythmias. Do not rely entirely upon rate meter alarms. All pacemaker patients should be kept under close or constant observation.”

WARNING The presence of much pacer-like noise can cause the displayed heart rate to be erratic even though the ECG trace may look undistorted with the pacer indicator off. To help avoid this noise problem, use fresh ECG electrodes and make sure the ECG cable lead wires make good connections.

If the patient being monitored has a pacemaker, the Propaq CS monitor detects and can indicate the occurrence of pacemaker signals.

1. To access the monitor pacer indicator, from the Main Menu press **ECG/RESP, MORE**, and then **NEXT** to select the PACER DISPLAY.
2. Press **CHANGE** to set the PACER DISPLAY either ON or OFF.

When ON, the monitor displays (and prints on printouts) vertical dashed lines to indicate each time a pacemaker signal is detected. (If the pacemaker signal is strong enough, the monitor also displays it as a waveform “spike.”)

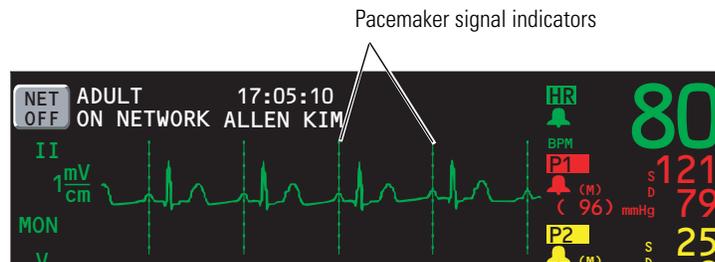

When OFF, the vertical lines are not displayed (or printed), but the pacemaker signal waveform spike is still displayed if strong enough.

Note Pacemaker pulses are not counted as heartbeats as defined by the Pacer Pulse Rejection specifications (“[ECG specifications](#)” on page 97).

Noise on the ECG signal may be detected as pacer signals, causing the pacer indicator to appear on the display. If you don't need to indicate pacemaker signals, turn off the pacemaker indicator for a better ECG waveform display.

Perform Invasive Blood Pressure (IBP) monitoring



WARNING If electrocautery is used, always avoid using any transducer with a conductive (metal) case that is electrically connected to its cable shield. Using a conductive transducer case with such a shield connection risks high-frequency burns at the ECG electrodes if the transducer case becomes earth grounded.


WARNING Although complete disconnections of invasive pressure transducers will be detected by the normal alarm functions, partial disconnection will not be detected, nor will the use of some incompatible transducers. The user must exercise reasonable measures to ensure that approved transducers are used and that pressure transducers are connected properly.

WARNING Before you use a Propaq CS monitor on a new patient, always turn it off for a few seconds, then turn it on again. This clears the prior patient's trend values and alarm limit settings.

WARNING For best product performance and measurement accuracy, use only accessories approved by Welch Allyn. Visit www.welchallyn.com. The use of any other accessories can result in inaccurate patient data, can damage the equipment, and can void your product warranty.

WARNING Always use accessories according to the standards of your facility and according to the manufacturer's directions for use. Do not use light-sensitive disposable transducers.

1. Inspect the transducer cable and transducer dome for wear, breakage, or fraying. Replace any worn or broken accessory.
2. Set up the transducer according to your hospital's procedures. Always refer to the transducer manufacturer's Directions for Use. If the transducer is a disposable unit with separate cable, connect the transducer to the transducer cable.
3. Plug the transducer (or transducer cable) into an invasive pressure connector on the monitor left side panel.

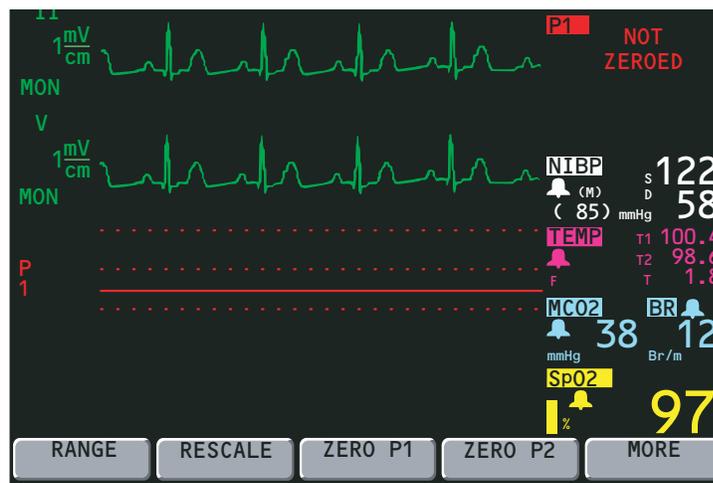

The message NOT ZEROED immediately appears in the IBP numerics window for the invasive pressure channel being used.

4. To zero the transducer, open the transducer's stopcock to atmospheric air. Wait a few seconds for the transducer to settle.

Before zeroing, make sure the transducer cable is properly connected to the monitor and the transducer is open to atmospheric air and positioned at the same level as the patient's heart. The monitor will not zero the transducer if the pressure waveform is pulsatile, there is too much signal noise, or the transducer's offset is too great.

5. If the ZERO menu is not displayed, from the Main Menu press **INVASIVE PRESSURE**, and then **ZERO P1** (or **ZERO P2**). The word ZEROING appears in the numerics window during zeroing.

If you want to cancel the zeroing process, press **CANCEL**.

6. Wait for a brief tone to sound and the word ZEROED to appear in the blood pressure numerics window.
7. Close the transducer's stopcock. The monitor displays the pressure scale and numerics.
8. If the transducer will not zero, the monitor displays the words ZERO REJECTED in the numerics window. Press **CANCEL** and try zeroing again beginning at step 4. The monitor does not display numerics or scales until an acceptable zero reference is established.

You can rezero an IBP transducer at any time after you again open the transducer stopcock to atmospheric air. If the transducer has already produced pressure readings, rezeroing provides a new zero reference for the monitor.


If the zero value is not accepted, the monitor continues to use the previous zero reference and displays numerics and waveforms based on that value.

If the transducer still does not zero, try another transducer or another cable.




WARNING If you press ZERO after an invasive pressure channel has been successfully zeroed and the channel is currently monitoring a pressure waveform, the message ZERO REJECTED will display in the IBP numerics window. This message continues to display in place of the valid invasive pressure numerics until you press Home, INVASIVE PRESSURE, and then CANCEL in the IBP menu. If an IBP alarm occurs while ZERO REJECTED is displayed in place of IBP numerics, the IBP numerics will not flash to indicate invasive pressure is in alarm.

9. To set up the IBP display, from the Main Menu press **INVASIVE PRESSURE** to display the first IBP menu:

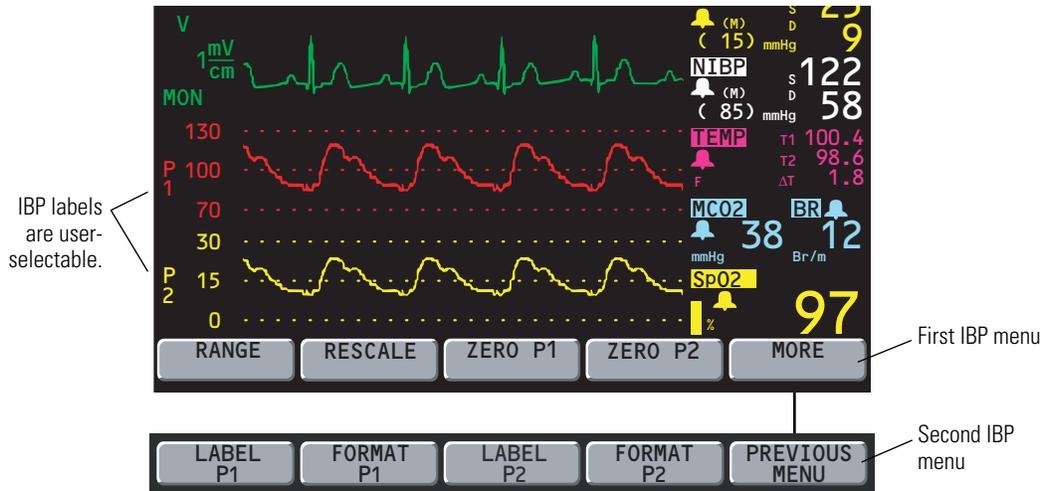

- To display all invasive pressure waveforms on one scale (when two IBP channels are active), press **RANGE** to select the Range Mode.

Range Mode displays both waveforms on one scale.

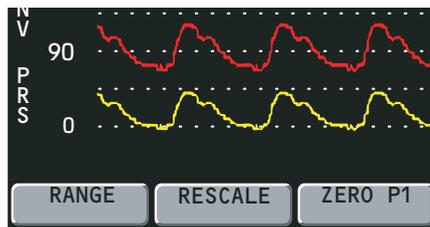


Press **RANGE** again to select another scale. Five scales are available:

- 300/150/0
- 180/90/0
- 120/60/0
- 60/30/0
- 30/15/0

Choose the scale carefully to make sure both waveforms are displayed (if monitored).


- To display each invasive waveform on its own scale, press **RESCALE** to select the Rescale Mode.

Rescale Mode displays each waveform on its own scale.

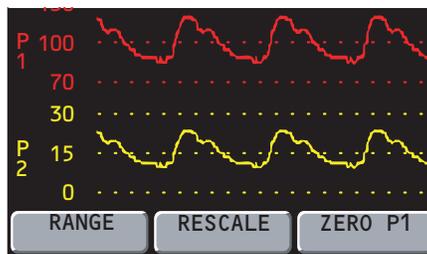

Whenever you press **RESCALE**, the monitor automatically adjusts the scale for the best appearance based on the highest and lowest pressure levels.

12. To change the displayed waveform label, press **MORE** to access the second IBP menu, then press **LABEL P1** (or **LABEL P2**).

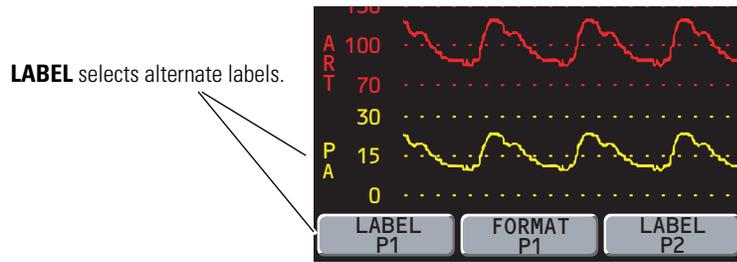

Selectable labels (and display colors) are:

P1	red	default label
P2	yellow	default label
ART	red	arterial
PA	yellow	pulmonary artery
CVP	blue	central venous pressure
ICP	white	intracranial pressure
UA	red	umbilical artery (NEO mode only)
UV	blue	umbilical vein (NEO mode only)

13. To change the format of the IBP numerics, from the second IBP menu press **FORMAT**.



To restore the first format, press **FORMAT** again.



14. Set alarms according to your facility's standards.

Take a Noninvasive Blood-Pressure (NIBP) reading



WARNING Periodically observe the patient's limb to make sure that the circulation is not impaired for a prolonged period of time. Also make sure the cuff is properly placed according to the following instructions. Prolonged impairment of circulation or improper cuff placement can cause bruising.

WARNING The Propaq CS monitor should never be used to monitor NIBP on one patient while simultaneously monitoring ECG on another patient.

WARNING If a noninvasive blood pressure measurement is suspect, repeat the measurement. If you are still uncertain about the reading, use another method.

WARNING Do not attempt to take NIBP pressures on patients during cardiopulmonary bypass.

WARNING When monitoring NIBP, match the monitor patient mode to the NIBP cuff. For neonates, set the monitor to Neonatal Mode unless the circumference of the limb is too large for the cuff. In that case, use the Pediatric Mode. Be aware, however, that the maximum cuff inflation limits are based on the patient mode, not the cuff; the maximum cuff inflation limits for Pediatric Mode are greater than for Neonate Mode (see "[NIBP specifications](#)" on page 101 for values).

WARNING Before you use a Propaq CS monitor on a new patient, always turn it off for a few seconds, then turn it on again. This clears the prior patient's NIBP cuff inflation target, trend values, and alarm limit settings.

At powerup, the Propaq CS monitor has an NIBP default inflation pressure (cuff inflation target) based on the patient mode (see "[NIBP specifications](#)" on page 101 for the values). After each NIBP measurement, the monitor adjusts the target inflation pressure to optimize the next NIBP measurement. To avoid possible patient discomfort, turn the monitor off and then on between different patients to reset the cuff inflation target to the default value.

NIBP measurements can be adversely affected by poorly fitting cuffs or improper cuff placement. Be sure to select the appropriate cuff and apply the cuff properly according to the directions in this manual.

Note NIBP measurements are affected by normal physiological pressure variations from reading to reading.

Improve NIBP accuracy with Smartcuf™

NIBP measurements can be adversely affected by many factors such as cardiac arrhythmias, sudden changes in blood pressure, body motions such as convulsions or shivering, bumping the cuff, vibration, vehicle motion, or weak pulses.

The patented Smartcuf software filtering technology greatly increases NIBP measurement accuracy in the presence of motion artifact or diminished pulses. Smartcuf synchronizes the NIBP reading with the R-wave of the patient's ECG to eliminate noise created by external stimuli such as patient motion or vibration. The monitor must perform ECG monitoring while using Smartcuf.

To enable the Smartcuf filter:

- Connect the ECG leads to the patient and perform ECG monitoring during NIBP.
- From the Main Menu, press **NIBP** to display the NIBP Menu (shown on page 34) and set Smartcuf to ON.

If artifact is so severe while Smartcuf is enabled that it affects the accuracy of an NIBP measurement, that measurement is marked with a special symbol on the display and on printouts:

This symbol indicates the NIBP reading was taken in the presence of high artifact while monitoring ECG with Smartcuf enabled. Artifact can affect accuracy. To help reduce artifact, see step 6 on page 35.

TIME	HR/PR	SpO2	S	D (M)	RR/BR
HH:MM	BPM	%	mmHg		Br/M
10:01	80	97	122	58 (85)	12
9:46	80	97	126	62 (86)	12
9:31	80	97	127	62 (86)	12
9:20	80	97	134	66 (89)	12
9:08	80	97	124	57 (86)	12
8:52	80	97	127	58 (88)	12
8:47	80	97	124	56 (85)	12
8:41	80	97	126	57 (84)	12
8:35	80	97	127	58 (85)	12
8:30	80	97	132	61 (89)	12

Numeric Display

Printout

Trend Display

There may be some situations where it is desirable to disable Smartcuf. This may include situations with very extreme motion artifact, certain types of arrhythmias, or other situations where it is not possible to obtain a good ECG signal. NIBP measurements can still be performed when Smartcuf is disabled.

To disable Smartcuf, from the Main Menu press **NIBP** to display the NIBP Menu and set Smartcuf to OFF.

Take the NIBP reading

1. Select a cuff and hose appropriate for the patient. Select cuff size based on limb circumference.




WARNING Use only accessories approved by Welch Allyn. Visit www.welchallyn.com. The use of any other accessories can result in inaccurate patient data, can damage the equipment, and can void your product warranty.

	Neonate Mode	Pediatric Mode	Adult Mode
Typical hoses	Neonate/Infant	Adult	Adult
Typical cuffs	Neonate #1 to #5 (disposable); newborn, infant (reusable)	Newborn, infant, small child, child, small adult	Child, small adult, adult, large adult, thigh
Recommended limb circumference	up to 15 cm	7.7 to 25 cm	greater than 15 cm

Note Be sure the patient mode selected is appropriate for the cuff you are using. For instructions about changing the patient mode, see ["Change the current Patient Mode" on page 18](#). Be aware that changing patient modes will cancel an NIBP reading in progress.

2. Squeeze as much air from the cuff as you can before placing it on the patient.
3. Place the cuff on the limb.

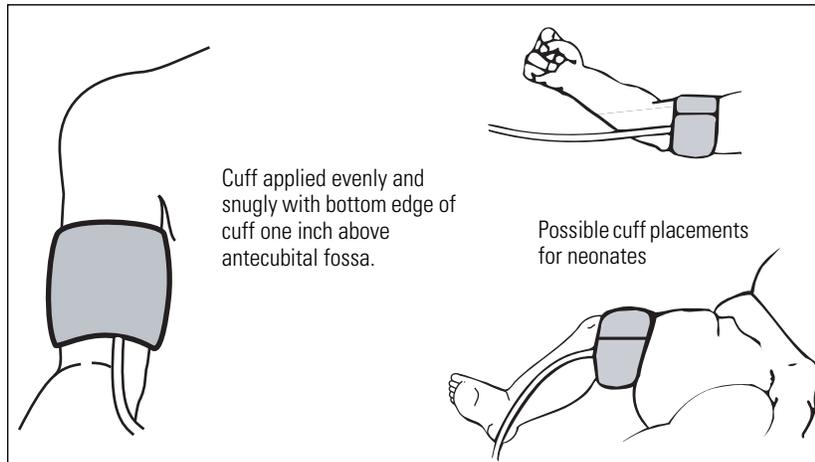

If possible, place it at the same level as the heart. If above the heart, add 1.9 mmHg to the NIBP measurement for every inch above the heart. If below the heart, subtract 1.9 mmHg for every inch.

The cuff should be snug, but not uncomfortable. The hose must not be kinked or pinched.

Make sure the cuff tubing is centered over the brachial artery.

If SpO₂ will also be monitored, place the NIBP cuff on a different limb than the SpO₂ sensor to help reduce unnecessary SpO₂ alarms.

4. Screw the hose connector onto the NIBP connector on the monitor's left side.
5. From the Main Menu, press **NIBP** to display the NIBP Menu:

The last measurement

After 16 minutes, the NIBP display changes as shown below for 44 minutes.

This symbol indicates that the NIBP reading was taken in the presence of high artifact while monitoring ECG with Smartcuf enabled. Artifact can affect accuracy. To help reduce artifact, see step 6 on page 35.

Measurement mode

Time remaining in Turbocuf Mode or Auto Mode interval

Smartcuf status

Manometer Bar: Systolic, diastolic, and mean values are indicated as small triangles.

START is displayed when no measurement is occurring.

START/STOP Starts and stops NIBP measurements. During the measurement, you can press **STOP** (or the **Start/Stop NIBP** key at the right side of the screen) to stop the measurement and vent the cuff.

If the Propaq CS monitor does not recognize a valid NIBP reading, it automatically attempts another measurement while displaying a retry message. The monitor attempts up to two retries (depending on patient mode and settings).

AUTO/MANUAL Switches between Automatic and Manual Mode. In Automatic Mode, the monitor automatically takes measurements at the selected interval.

INTERVAL Selects the measurement interval for Automatic Mode NIBP measurements: 1, 2, 3, 5, 10, 15, 30, or 60 minutes.

For intervals 5, 10, 15, 30, or 60, measurements occur at corresponding intervals past the hour. For example, if 5 is selected at 10:47:20, the measurements occur at 10:50, 10:55, 11:00, etc.

For intervals 1, 2, or 3 minutes, measurements begin 1, 2, or 3 minutes after the interval is set. For example, if 1 is selected at 10:47:20, the next measurement starts at 10:48:20.

TURBOCUF Automatically starts NIBP measurements and takes as many as possible within five minutes. To stop the Turbocuf Mode, press **STOP** or the **Start/Stop NIBP** key.

After you stop the Turbocuf Mode or the monitor completes the five-minute Turbocuf cycle, the monitor returns to the previous NIBP mode (Automatic or Manual).

SMARTCUF Enables or disables the Smartcuf motion artifact filter. NIBP measurements can still be taken when Smartcuf is off. Artifact may interfere with the accuracy of NIBP measurements with Smartcuf off.

- If motion artifact such as shivering, coughing, or other motion interferes with NIBP readings, do the following:

Position the patient's limb away from the body so the applied cuff is not in contact with the patient's body or any other object such as a bed rail. Try to keep the cuff at the same level as the heart.

Make sure the Smartcuf filter is ON. Make sure ECG leads are properly connected to the patient and perform ECG monitoring during NIBP. (ECG monitoring is required for Smartcuf.)

7. Set alarm limits according to your facility's standards.

Note When the SEARCH message appears in an NIBP TREND display or printout, it indicates that the monitor was not able to complete an NIBP measurement during that time period.

Perform temperature monitoring

1. Place the temperature probe on the patient.



WARNING Application and use of metal-jacketed temperature probes that come in contact with conductive objects or clinical personnel during electrocautery may cause burns at the patient-probe/electrode contact points. Do not touch conductive temperature sensors during defibrillation or cautery.

WARNING Use only accessories approved by Welch Allyn. Visit www.welchallyn.com. The use of any other accessories can result in inaccurate patient data, can damage the equipment, and can void your product warranty.

2. Plug the probe cable into one of the temperature connectors on the monitor side panel. Within a few seconds, the monitor displays the temperature:



If you connect a second temperature probe, the monitor displays the temperature for T1, T2, and ΔT .

3. To set alarm limits:
 - a. From the Main Menu press **SETUP, ALARMS, LIMITS** to access the Alarms Limits Menu.
 - b. Press **NEXT PARAMETER** as needed to highlight the desired temperature parameter.
 - c. Use **NEXT SETTING, UP, DOWN**, and **ON/OFF** to set the alarm limits according to your facility's standards.
4. To change the temperature units (°C or °F):
 - a. From the Main Menu press **SETUP, MORE, MORE, SERVICE, YES** (to access the Service Menu), **MORE, MORE, SETTINGS**.
 - b. Use **NEXT** and **CHANGE** to change the temperature units.

Changing units does not clear temperature trends.

Perform SpO₂ monitoring



WARNING Oxygen saturation measurements using pulse oximetry are highly dependent on proper placement of the sensor and patient conditions. Patient conditions such as shivering and smoke inhalation may result in erroneous oxygen saturation readings. If pulse oximetry measurements are suspect, verify the reading using another clinically accepted measurement method, such as arterial blood gas measurements on a co-oximeter.

WARNING Tissue damage can be caused by incorrect application or use of a sensor (e.g., wrapping the sensor too tightly, applying supplemental tape, failing to periodically inspect the sensor site, leaving a sensor on too long in one place). Refer to the Directions for Use provided with each sensor for specific instructions on application and use, and for description, warnings, cautions, and specifications.

WARNING Sensors exposed to ambient light while not applied to a patient can exhibit semi-normal saturation readings. Be sure the sensor is securely placed on the patient and check its application often to ensure accurate readings.

WARNING Before you use a Propaq CS monitor on a new patient, always turn off the monitor for a few seconds, then turn it on again. This clears the prior patient's trend values, alarm limit settings, and NIBP cuff inflation target.

WARNING The pulse oximetry channel must NOT be used as an apnea monitor.

WARNING Inaccurate measurements may be caused by venous pulsations.

WARNING The pulse oximetry option can be used during defibrillation, but the readings may be inaccurate for a short time.

WARNING A very sudden and substantial change in pulse rate can result in erroneous pulse rate readings. Be sure to validate the patient data and patient condition before intervention or change in patient care.

WARNING Interfering Substances: Carboxyhemoglobin may erroneously increase readings; the level of increase is approximately equal to the amount of carboxyhemoglobin present. Methemoglobin may also cause erroneous readings. Dyes, or any substances containing dyes, that change usual arterial pigmentation may cause erroneous readings.

Each SpO₂ sensor is designed for application to a specific site on the patient within a certain size range. To obtain optimal performance, use an appropriate sensor and apply it as described in the sensor's directions for use.

If excessive ambient light is present, cover the sensor site with opaque material to block the light. Failure to do so may result in inaccurate measurements. Light sources that can affect performance include surgical lights (especially those with a xenon light source), bilirubin lamps, fluorescent lights, infrared heating lamps, and direct sunlight.

If NIBP will be monitored while using SpO₂, place the NIBP cuff on a different limb than the SpO₂ sensor to help reduce unnecessary SpO₂ alarms. For optimal measurements, avoid placing the SpO₂ sensor on the same limb as an arterial catheter or intravascular line.

Loss of pulse signal can occur if the sensor is too tight, there is excessive ambient light, an NIBP cuff is inflated on the same limb as the sensor, there is arterial occlusion proximal

to the sensor, the patient is in cardiac arrest or shock, or the patient has hypotension, severe vasoconstriction, severe anemia, or hypothermia.

Perform SpO₂ monitoring with Masimo option

1. Attach the sensor to the patient according to the sensor manufacturer's instructions, observing all warnings and cautions.



WARNING Use only accessories approved by Welch Allyn. Visit www.welchallyn.com. The use of any other accessories can result in inaccurate patient data, can damage the equipment, and can void your product warranty.


2. Inspect the Masimo SpO₂ cable. Replace it if it shows any signs of wear, breakage, or fraying. Plug the sensor into the cable and plug the cable into the Propaq monitor.

The monitor displays STANDBY in the SpO₂ numeric window until it measures and displays the SpO₂ value.

As oxygen saturation increases and decreases, the pitch of the heart tone rises and falls.

The monitor self-calibrates the SpO₂ channel whenever the monitor is first turned on or a sensor is first connected to the SpO₂ channel.

3. From the Main Menu, press **SpO₂** (or **SpO₂/CO₂** and then **SpO₂**) to display the SpO₂ menu similar to the following:

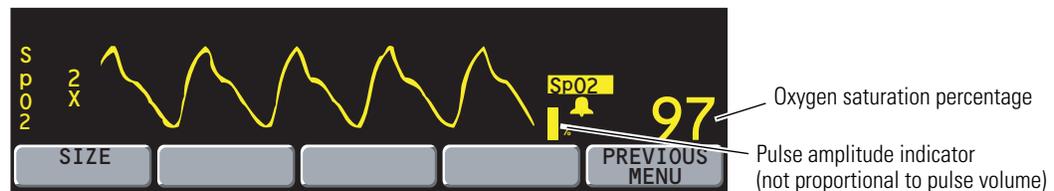

4. Press **SIZE** to adjust the waveform size for best viewing (1x, 2x, 4x, or 8x).
5. Adjust the placement of the sensor until a good SpO₂ waveform is displayed. A waveform with artifact may cause erroneous oxygen saturation readings.
6. Set alarm limits according to your hospital's standards.

Note To help minimize false alarms, the Propaq monitor briefly delays or "holds off" triggering both audible and visual alarms for limit violations for SpO₂% and Pulse Rate for 10 seconds. After the alarm hold-off period begins, if the monitor detects that the patient's vital sign has returned to acceptable limits, the monitor cancels the alarm hold-off. The next time a vital sign limit is violated, the monitor starts a new hold-off period.

The "averaging time" for SpO₂ measurements is fixed at eight seconds.

7. If patient movement interferes with measurements, consider the following possible solutions:
 - Be sure the sensor is secure and properly applied.
 - Use a new sensor with fresh adhesive backing.
 - Select a different type of sensor.
 - Move the sensor to a less active site.

Perform SpO₂ monitoring with Nellcor option

1. Attach the sensor to the patient according to the sensor manufacturer's instructions, observing all warnings and cautions.



WARNING Use only accessories approved by Welch Allyn. Visit www.welchallyn.com. The use of any other accessories can result in inaccurate patient data, can damage the equipment, and can void your product warranty.

2. When using a Nellcor SpO₂ sensor extension cable, inspect the cable before use. Replace it if it shows any signs of wear, breakage, or fraying. Plug the sensor into the cable and plug the cable into the Propaq monitor, or plug the sensor directly into the monitor.
3. If the monitor SpO₂ receptacle has a locking ring, lock the connector in place by turning the locking ring clockwise until it stops. For other connectors, make sure the plug is all the way in.




Caution If you see the error message DEFECTIVE SpO₂ SENSOR, either the sensor is not compatible with the monitor or the sensor is not working properly. Visit www.welchallyn.com to be sure the sensor is compatible. If compatibility is not a problem, try another sensor.

Note The monitor displays STANDBY in the SpO₂ numeric window until it measures and displays the SpO₂ value. As oxygen saturation increases and decreases, the pitch of the heart tone rises and falls.

The Nellcor SpO₂ option periodically performs an internal adjustment which causes the SpO₂ waveform to appear flat for a brief period.

4. From the Main Menu, press **SpO₂** (or **SpO₂/CO₂** and then **SpO₂**) to display the first SpO₂ menu similar to the following:

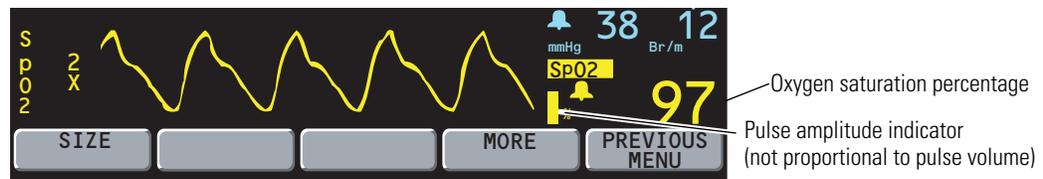

5. Press **SIZE** to adjust the waveform size for best viewing (1x, 2x, 4x, or 8x).

At high magnification (4x, 8x), some waveforms may appear truncated. To view these waveforms, reduce the size until the complete waveform appears.

6. Adjust the placement of the sensor until a good SpO₂ waveform is displayed. A waveform with artifact may cause erroneous oxygen saturation readings.
7. Press **MORE** to display the second SpO₂ menu:



8. Press **RESPONSE** to select the appropriate time required to measure SpO₂:

Response	(Seconds)	Indications for use
NORMAL	5-7	Use for relatively stable patients.
FAST	2-3	Use when patient movement is minimal.
SLOW	10-15	Use when patients exhibiting movement are preventing accurate measurement at NORMAL setting.

9. If the C-LOCK function is desired, press **C-LOCK** to set it to ON.

C-LOCK synchronizes the pulse oximeter's systole determination to the R-wave to reduce the effects artifact may have on SpO₂ measurements. Under some conditions you may find more stable SpO₂ readings with C-LOCK set to ON. SYNC appears next to the waveform when synchronization to the ECG has been obtained. Synchronization takes a few seconds to establish the first time. If C-LOCK is on and the HR source is SpO₂, the heart rate source is automatically changed to ECG. An ECG signal must be present or C-LOCK does not activate.

If you get false SpO₂ alarms with patients with low perfusion states or multiple arrhythmias, try turning off C-LOCK.

10. Set alarm limits according to your hospital's standards.

Note To help minimize false alarms, the Propaq monitor briefly delays or "holds off" triggering both audible and visual alarms for limit violations for SpO₂% and Pulse Rate for 10 seconds. After the alarm hold-off period begins, if the monitor detects that the patient's vital sign has returned to acceptable limits, the monitor cancels the alarm hold-off. The next time a vital sign limit is violated, the monitor starts a new hold-off period.

11. If patient movement interferes with measurements, consider the following possible solutions:

- Be sure the sensor is secure and properly applied.
- Use a new sensor with fresh adhesive backing.
- Select a different type of sensor.
- Move the sensor to a less active site.

Perform SpO₂ “spot-check” monitoring

The SpO₂ Standby Mode allows you to remove the SpO₂ sensor from a patient without having to disable all alarms or disconnect the SpO₂ sensor cable from the Propaq CS monitor. You can therefore perform intermittent or “spot-check” SpO₂ monitoring.

1. While monitoring SpO₂, remove the SpO₂ sensor from the patient, but leave it connected to the monitor. When the monitor detects the lack of a pulsatile waveform, it sounds a patient alarm and displays this menu:



2. Press **STANDBY** to place SpO₂ into the Standby Mode.

The monitor suspends the SpO₂ alarm tone indefinitely and displays **STANDBY** in place of SpO₂ numerics. SpO₂ remains in the Standby Mode until the SpO₂ sensor is reapplied to a patient. Other vital sign monitoring is not restricted. By contrast, if you press **SILENCE** instead of **STANDBY**, the monitor temporarily silences all alarm tones; however, the alarm tone resumes after 90 seconds if the SpO₂ sensor is still disconnected from the patient—see [“Silence an active alarm or alert tone for 90 seconds”](#) on page 51.

3. To resume SpO₂ monitoring, reapply the SpO₂ sensor to a patient.

The monitor exits the Standby Mode and resumes SpO₂ monitoring.

Note The message STBY on the SpO₂ trend display and trend printouts indicates the monitor was in the SpO₂ Standby Mode.

Perform Mainstream CO₂ monitoring

The capnography (CO₂) option measures End-tidal CO₂ (ETCO₂), Inspired CO₂ (INCO₂), Breath Rate, and Apnea. Patients using mainstream CO₂ must either be intubated or breathing through a tight-fitting face mask connected to a breathing system such as an anesthesia circle system. The mainstream CO₂ option requires the SpO₂ option.



WARNING Avoid exposing older mainstream CO₂ sensors to non-patient sources of CO₂ such as vehicle engine exhaust or smoke. Exposure to these CO₂ sources can temporarily trap CO₂ within the monitor or mainstream CO₂ sensor housing, even when monitor power is off. This can temporarily cause an erroneous elevated CO₂ measurement baseline until the trapped CO₂ leaks out and the baseline returns to zero (which can require as long as 3-24 hours).

WARNING Do not attempt to verify operation of the CO₂ sensor by blowing through it directly. Always blow through an attached airway adapter. Otherwise, a small amount of CO₂ from your breath may enter the CO₂ sensor housing and cause a small shift in the measured CO₂ values. It may take 3 to 24 hours for the sensor to return to proper calibration.

WARNING Do not clean and/or reuse a single-patient-use airway adapter. When a single-patient-use airway adapter becomes occluded, replace it.



WARNING For best product performance and measurement accuracy, use only accessories approved by Welch Allyn. Visit www.welchallyn.com. The use of any other accessories can result in inaccurate patient data, can damage the equipment, and can void your product warranty.

WARNING Always use accessories according to the standards of your facility and according to the manufacturer's directions for use.

WARNING Only sensors recommended by Welch Allyn provide calibrated waveforms and numerics.

Note The mainstream CO₂ operating temperature range is 10° to 40° C. This is different than the range of 0° to 40° C for other Propaq CS monitor functions. CO₂ monitoring outside the specified range can cause inaccurate CO₂ measurements.

1. Select the appropriate airway adapter.
2. Connect the adapter, ventilator circuit, and CO₂ sensor according to the manufacturer's instructions.

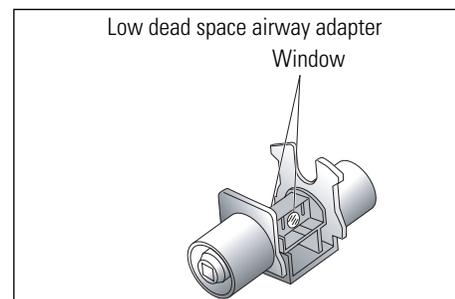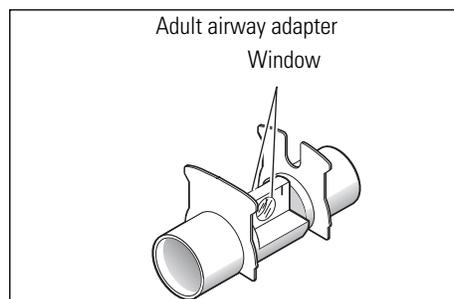



WARNING Before using an airway adapter, always look through the window lumen and inspect the adapter for inadvertently lodged obstructions and for window integrity.

WARNING If the sensor does not easily slide onto the adapter, do not attempt to force these components together. They fit together in only one way. Take care not to damage the glass window.

WARNING After attaching the sensor to the adapter, check for proper placement. Check the sensor and adapter periodically during monitoring to make sure they are properly connected and the adapter is not clogged by obstructions or debris.

WARNING When attaching the airway adapter, position the adapter so the sensor is on top to avoid fluid collection in the sensor airway slot. Any concentration of fluids here can cause inaccurate CO₂ readings.

WARNING When connecting the adapter and sensor to the ventilator circuit, do not use the adapter and sensor as a wrench to twist the adapter into the ventilator circuit. Such action could damage the adapter and sensor.

WARNING Always check to make sure there are no leaks in the breathing circuit. Check all of the connections.

3. Plug in the CO₂ sensor cable to the mainstream CO₂ connector on the monitor left side panel.



WARNING When disconnecting the CO₂ sensor from the tracheal or endotracheal tube, check the sensor to determine how hot it is. If it is too hot for patient comfort, do not allow it to come into contact with the patient.

4. See page 47 and set up the CO₂ display and alarm limits.

Note When disconnecting the airway adapter from the ventilator circuit, always detach the CO₂ sensor from the airway adapter before removing the airway adapter from the ventilator circuit.

Perform sidestream CO₂ monitoring

The capnography (CO₂) option measures End-tidal CO₂ (ETCO₂), Inspired CO₂ (INCO₂), Breath Rate, and Apnea. Patients using sidestream CO₂ can either be intubated or non-intubated using a CO₂ Sampling cannula or a combination CO₂ Sampling/Oxygen Delivery nasal cannula. The sidestream CO₂ option requires the SpO₂ option.



WARNING Do not use sidestream CO₂ if flammable anesthetic gases are in use.

WARNING If the sidestream CO₂ option is connected to a ventilatory circuit, be sure to adjust appropriate ventilator or anesthesia system settings to compensate for the sampling flow volume (90 or 175 ml/min) that is aspirated from the ventilatory circuit by the sidestream CO₂ option.


WARNING Avoid exposing a Propaq CS monitor with the sidestream CO₂ option to non-patient sources of CO₂ such as vehicle engine exhaust or smoke. When such exposure is possible, avoid opening the printer door. Exposure to these CO₂ sources can temporarily trap CO₂ within the monitor, even when monitor power is off. This can temporarily cause an erroneous elevated CO₂ measurement baseline until the trapped CO₂ leaks out and the baseline returns to zero (which can require as long as 24 hours).

WARNING For best product performance and measurement accuracy, use only accessories approved by Welch Allyn. Visit www.welchallyn.com. The use of any other accessories can result in inaccurate patient data, can damage the equipment, and can void your product warranty.

WARNING Always use accessories according to the standards of your facility and according to the manufacturer's directions for use.

Note The sidestream CO₂ operating temperature range is 5° to 40°C. This is different than the range of 0° to 40°C for other Propaq CS monitor functions. CO₂ monitoring outside the specified range can cause inaccurate CO₂ measurements.


- When monitoring a small child with a rapid respiratory rate, mainstream CO₂ can provide a more accurate representation of the expired CO₂ waveform than sidestream CO₂.
 - Breath rates greater than 50 breaths per minute may reduce the reported ETCO₂ values. Select the 175 ml/min flow rate to minimize errors at higher breath rates.
 - The 175 ml/min flow rate is recommended for intubated adult patients.
1. Firmly insert the sidestream CO₂ watertrap adapter into the sidestream CO₂ connector on the monitor left side panel.

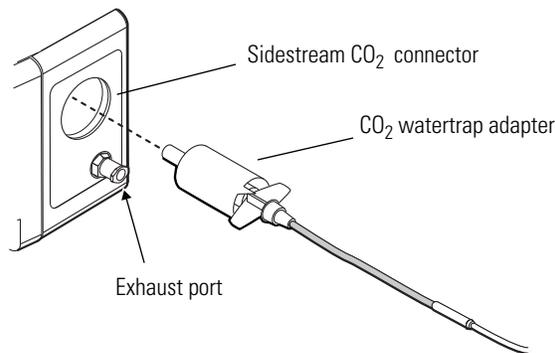

WARNING - Do not connect sample line or patient input to exhaust port.

2. See page 47 and set up the CO₂ display and alarm limits, and then continue this procedure with step 3.
3. **A. For a non-intubated patient**, position the cannula on the patient according to the manufacturer's instructions.


WARNING - Do not connect sample line or patient input to exhaust port

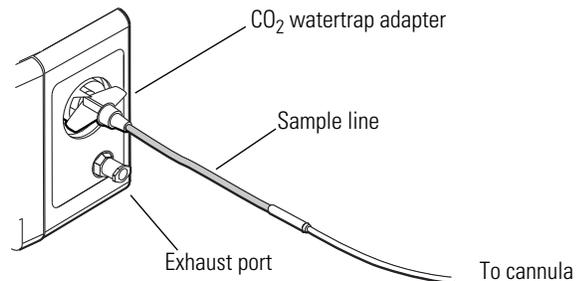

WARNING The cannula is disposable and should only be used for a single patient. Do not reuse the cannula for another patient.

WARNING If oxygen is being delivered while using sidestream CO₂, be sure to use a CO₂ Sampling and O₂ Delivery Cannula. Using a different type of cannula could obstruct oxygen delivery.

- B. For an intubated patient**, connect the gas sampling elbow and elbow connector into the patient's breathing circuit according to the manufacturer's instructions.

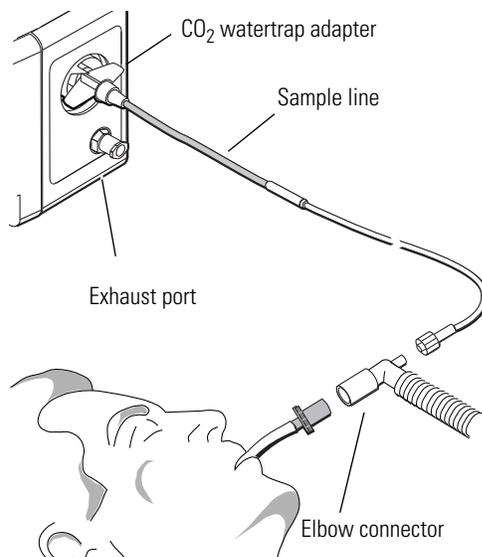

WARNING - Do not connect sample line or patient input to exhaust port

Note If you are using a watertrap and sample line instead of the one-piece CO₂ gas sampling system, perform steps 1 to 3 above with the following changes: attach the watertrap to the monitor, then attach the sample line to the watertrap.

WARNING  The exhaust port for sidestream CO₂ is an output for the expired gases from the patient and any connected breathing apparatus. The exhaust port is intended only for connection to gas collection equipment such as gas scavenger devices (the device should comply with ISO 8835-3:1997 E). Do not allow any other connection to the exhaust port.

WARNING If using a watertrap, the watertrap is disposable and should only be used for a single patient. Do not reuse the watertrap for another patient.

Sidestream CO₂ accuracy decreases if additional tubing is connected to the sample line. Avoid connecting additional tubing to the standard sample line.

Set up the CO₂ display and alarm limits

Note After you connect a mainstream CO₂ sensor or sidestream CO₂ watertrap, the Propaq CS monitor displays the waveform briefly without a scale. It displays WARM UP (for mainstream) or START UP (for sidestream) in the CO₂ numerics window. After about 30 seconds, the monitor displays the CO₂ measurement and waveform range.


CO₂ monitoring is typically displayed as shown:

Measured CO₂ levels are normally displayed as both a waveform and an ETCO₂ numeric value.


MCO₂ indicates mainstream CO₂ is active.

SCO₂ is displayed when sidestream CO₂ is active.

CO₂ is displayed if a mainstream sensor or sidestream watertrap is installed but not active.

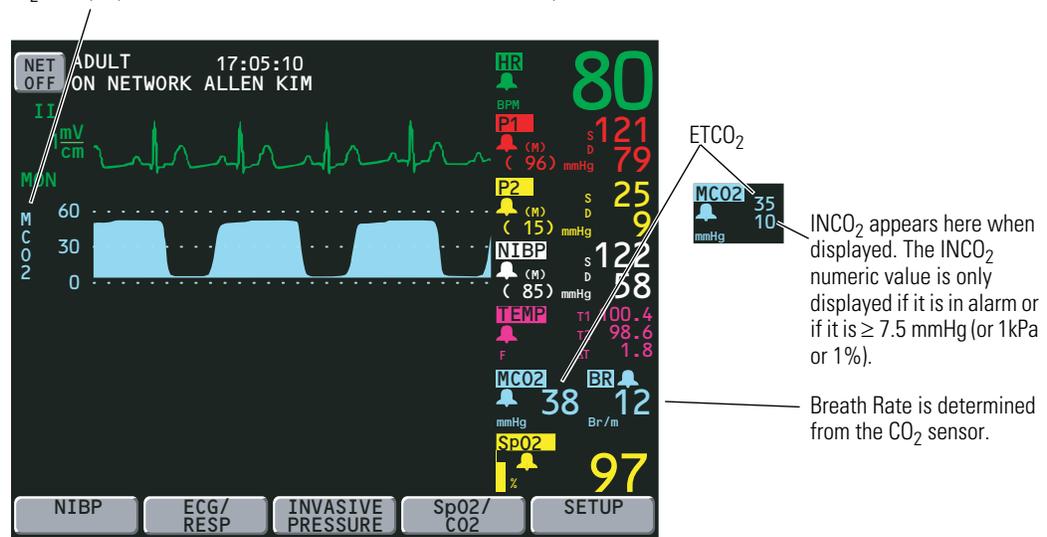

- To adjust the display, from the Main Menu press **SpO₂/CO₂, CO₂** to access the first CO₂ menu:

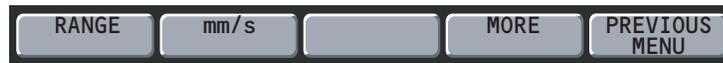

- Press **RANGE** to select the CO₂ waveform scale or range.

mmHg	0-100	0-60 (default)	0-30
kPa	0-14	0-8	0-4
%	0-14	0-8	0-4


To change CO₂ units (mmHg, kPa, or %) see “Change the date format, ECG filter, and units” on page 94.

Note If an inspired value is displayed indicating patient rebreathing (non-zero INCO₂), check the patient breathing circuit for proper function. For mainstream CO₂, also remove the sensor from the patient’s airway, hold it away from any source of breath, and confirm INCO₂ begins to go down to the baseline value. If the Propaq CS monitor continues to display inspired values, return the mainstream CO₂ sensor to Welch Allyn for service.

- Press **mm/s** to set the display sweep speed for CO₂ and RESP (3.13, 6.25, or 12.5 mm/sec). The default is 6.25.

To view the sweep speed setting, press **MORE** to access the CO₂ status window.

- Press **MORE** to access the second CO₂ menu and status window:

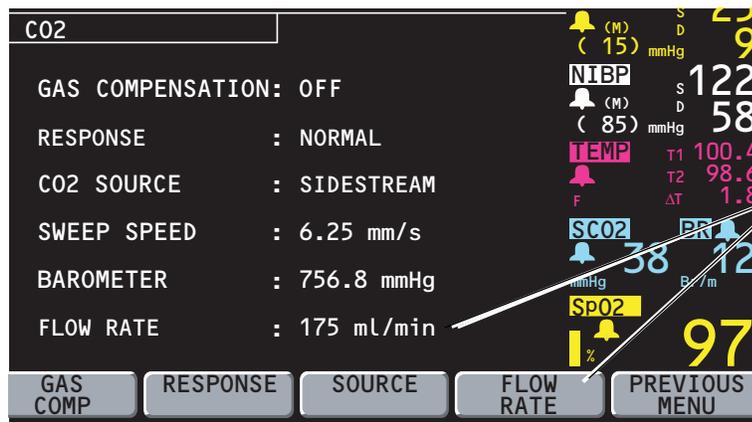

FLOW RATE is displayed only for sidestream CO₂ (not mainstream CO₂).

- If O₂ or N₂O is being administered to the patient, press **GAS COMP** to set the proper gas compensation (for specifications, see “Capnography (CO₂) specifications” on page 105). If no gas is being administered, choose OFF (the default).

Note If ETCO₂ is displayed as + + +, have a biomedical technician check the CO₂ calibration against a known reference gas. If the sensor calibration is not accurate, return it to Welch Allyn for service.

- Press **RESPONSE** to set CO₂ measurement response time (NORMAL, FAST, or SLOW).

FAST is recommended where a sudden step change in ETCO_2 is of concern, such as that induced by an air embolus in certain neurosurgical procedures. SLOW is recommended to help reduce ETCO_2 false alarms when breath morphology varies considerably from one breath to the next. The default is NORMAL.

7. Press **SOURCE** to change between mainstream CO_2 and sidestream CO_2 monitoring (if both options are installed), or to disable CO_2 monitoring.

Choosing OFF allows you to disable CO_2 monitoring without removing the watertrap or sensor. When CO_2 is off, OFF is displayed for CO_2 numerics.

8. For sidestream CO_2 , press **FLOW RATE** to set the sampling flow rate (90 or 175 ml/min).

You can change the flow rate while sidestream CO_2 is active.

9. To set alarm limits, from the Main Menu press **SETUP, ALARMS, LIMITS**. Then set alarm limits for RR/BR, ETCO_2 and INCO_2 .

INCO_2 has an upper alarm limit setting but no lower alarm limit setting.



WARNING For patient safety, it is recommended that the Breath Rate alarm limits always be turned on and set appropriately.

10. Set the Apnea Delay limit (the maximum time allowed between two consecutive breaths before an Apnea alarm occurs) in the Alarm Limits window.

After the first breath has been detected, the Apnea Delay limit setting is automatically turned on for as long as the CO_2 channel is active.

4

Alarms & Alerts

Description of alarm and alert tone patterns


- Patient alarm tone pattern: one second on, two seconds off
- Apnea alarm tone pattern: one second on, one second off (fastest)
- Equipment alert tone pattern: one second on, four seconds off (slowest)

Note Propaq monitors connected to an Acuity Central Monitoring System can sound distinct tone patterns for certain alarms, such as life-threatening Arrhythmia and ST alarms. For further information regarding Acuity System alarm tones, please consult the *Acuity System Directions for Use*.

Silence an active alarm or alert tone for 90 seconds

You can silence the tone of a patient alarm or equipment alert for a period of 90 seconds.

Silence an alarm or alert tone for 90 seconds

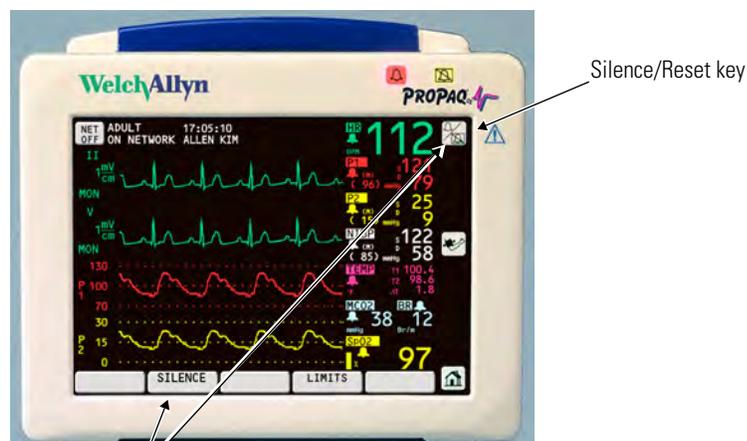

Press either key to silence the alarm or alert tone for 90 seconds.

1. Check the patient and provide appropriate care.
2. Press **Silence/Reset** or the **SILENCE** key.
3. After caring for the patient, make sure the appropriate alarm limits are set.

Re-enable an alarm or alert tone before the 90-second silence period has elapsed

1. Press **Silence/Reset**.

Indications during a 90-second silence period

At the Propaq CS monitor	At the Acuity Central Station
During the silence period, the usual <i>visual</i> alarm and alert indications are displayed.	
If a new alarm or alert occurs, the new alarm or alert tone interrupts the silence period.	Only life-threatening arrhythmia and apnea alarm tones interrupt the suspend period.
To re-enable an alarm or alert tone before the 90-seconds has elapsed, press Silence/Reset .	To resume an alarm or alert tone before the 90-seconds has elapsed, press RESUME .
If an equipment alert occurs during the silence period, you can acknowledge (dismiss) <i>all</i> indications of the alert by pressing any key, as usual.	
If the original alarm or alert condition still exists after 90 seconds, the alarm or alert tone sounds again.	

Inhibit alarm and alert tones for four minutes: 4 MIN SUSPND

As you perform patient care, there might be occasions when you want to suspend potential or current alarm and alert tones for a period of four minutes.

Initiate a four minute suspend period

1. Press **Home**, **SETUP**, **ALARMS**, **4 MIN SUSPND**.

Resume alarm and alert tone capability before the four minute period has elapsed

1. Press **Silence/Reset**.

Indications during a four-minute suspend period

At the Propaq CS monitor	At the Acuity Central Station
If an alarm or alert occurs during the suspend period, the usual <i>visual</i> alarm and alert indications are displayed.	
<i>Propaq monitors connected to Acuity Central Station:</i> Apnea or life-threatening arrhythmia alarm tones interrupt the suspend period.	Apnea or life-threatening arrhythmia alarm tones interrupt the suspend period.
<i>Standalone Propaq monitors:</i> Apnea alarms <i>do not</i> interrupt the suspend period, and arrhythmias are not detected.	
To resume alarm and alert tone capability before the 4 minutes has elapsed, press Silence/Reset .	To resume alarm and alert tone capability before the 4 minutes has elapsed, press RESUME .
If an equipment alert occurs during the suspend period, you can acknowledge (dismiss) <i>all</i> indications of the alert by pressing any key, as usual.	

Inhibit alarm and alert tones indefinitely: ALL ALARMS

Note This feature requires access to the Service Menu. The Service Menu features are not intended for use during ordinary, routine operation.

You can suspend potential or current alarm and alert tones for an indefinite period of time. The tones are disabled until someone re-enables them.

Indefinitely suspend all alarm and alert tones

1. Press **Home, SETUP, MORE, MORE, SERVICE, YES, MORE, MORE, SETTINGS, ALL ALARMS, YES.**



WARNING Whenever audible alarm tones are disabled, make sure the patient is closely observed.

Resume all alarm and alert tone capability

1. Press **Silence/Reset.**

Indications during an all alarms suspended period

At the Propaq CS Monitor	At the Acuity Central Station
If an alarm or alert occurs during the suspend period, the usual <i>visual</i> alarm and alert indications are displayed.	
<i>Propaq monitors connected to Acuity Central Station:</i> Apnea or life-threatening arrhythmia alarm tones interrupt the suspend period.	Apnea or life-threatening arrhythmia alarm tones interrupt the suspend period.
<i>Standalone Propaq monitors:</i> Apnea alarms <i>do not</i> interrupt the suspend period, and arrhythmias are not detected.	
To resume alarm and alert tone capability, press Silence/Reset.	To resume alarm and alert tone capability, press RESUME.
If an equipment alert occurs during the suspend period, you can acknowledge (dismiss) <i>all</i> indications of the alert by pressing any key, as usual.	

Summary of alarm and alert keys and Acuity Central Station messages

The following table summarizes Silence and Suspend behaviors and Acuity Central Station messages for different combinations of current and previous Propaq monitor and Acuity software versions.

Note When your facility uses both older and newer Propaq monitors and Acuity software versions, be aware that messages vary at the Acuity Central Station. Some Acuity Central Station messages do not identify specific alarm/alert tone silence or suspension states.

However, visual alarm indications still continue as usual at the Acuity Central Station and at the Propaq monitors.

Propaq monitor version	Propaq monitor key	Result of Propaq monitor key press	At Acuity Central Station, result of Propaq monitor key press	
			Acuity 6.30 and prior	Acuity 6.31 and later
Propaq CS 3.6X	SILENCE or Silence/Reset	Silences active alarm and alert tones for 90 seconds at the Propaq monitor and Acuity Central Station. All new alarms and alerts interrupt the silence.	Alarms Suspended Message Only life-threatening arrhythmia and apnea alarms interrupt the silence.	Alarms Suspended Message Only life-threatening arrhythmia and apnea alarms interrupt the silence.
	4 MIN SUSPND	Inhibits alarm and alert tones for 4 minutes at the Propaq monitor and Acuity Central Station. ^a	Alarms Suspended Message	Nurse Suspend 4 min Message
	ALL ALARMS	Indefinitely inhibits alarm and alert tones at the Propaq monitor and Acuity Central Station, until someone resumes tone capability. ^a	Alarms Suspended Message	Nurse Suspend Always Message
Propaq CS 3.5X	SUSPEND or Silence/Reset	Suspends (for 90 seconds) the capability for patient alarm and equipment alert tones at the Propaq monitor and Acuity Central Station. ^a	Alarms Suspended Message	Alarms Suspended Message
	ALL ALARMS	Turns off all patient vital sign alarm limits at the Propaq monitor and Acuity Central Station. No visual or auditory alarm indications occur until someone resumes the limits. ^a	Some Alarms Off, Set Alarm Limits Message	Some Alarms Off, Set Alarm Limits Message

a. If connected to the Acuity Central Station, life-threatening arrhythmias and apnea alarms interrupt the suspension.

Customize alarm limits based on patient's current vital signs

To quickly set all alarm limits, from the Main Menu press **SETUP, ALARMS, STAT SET**. The monitor turns on all alarms and calculates new alarm limits based on the patient's current vital sign values. Make sure that the new limits are appropriate for the patient.

STAT SET limit calculations^a

Vital sign	If the patient's vital sign value is	Then the calculated new lower limit is	Then the calculated new upper limit is
Heart Rate	HR ≤ 99 100 - 250 HR ≥ 251	HR x 0.8 HR - 20 Unchanged	HR x 1.2 HR + 20 250
Pulse Rate	PR ≤ 99 PR ≥ 100	PR x 0.8 PR - 20	PR x 1.2 PR + 20
Invasive Pressure	Inv Prs ≤ 25 26 - 99 Inv Prs ≥ 100	Inv. Pressure - 5 Inv. Pressure x 0.8 Inv. Pressure - 20	Inv. Pressure + 5 Inv. Pressure x 1.2 Inv. Pressure + 20
NIBP	NIBP ≤ 25 26 - 99 NIBP ≥ 100	NIBP - 5 NIBP x 0.8 NIBP - 20	NIBP + 5 NIBP x 1.2 NIBP + 20
Respiration Rate/Breath Rate	RR/BR ≤ 25 26 - 99 RR/BR ≥ 100	RR/BR - 5 RR/BR x 0.8 RR/BR - 20	RR/BR + 5 RR/BR x 1.2 RR/BR + 20
Temperature	Temp ≥ 0°C	Temp - 0.5	Temp + 0.5
SpO ₂	SpO ₂ ≥ 0%	SpO ₂ - 5 (min. limit 50%)	100% (adult and pediatric mode) SpO ₂ + 5 (neonate mode)
ETCO ₂	ETCO ₂ ≥ 0 mmHg ETCO ₂ ≥ 2.0 (% or kPa)	ETCO ₂ - 5 mmHg (min. 15 mmHg) ETCO ₂ - 0.7 (% or kPa) (min 2.0% or 2.0 kPa)	ETCO ₂ + 10 mmHg ETCO ₂ + 1.4 (% or kPa)
INCO ₂	INCO ₂ ≥ 0 mmHg INCO ₂ ≥ 0 (% or kPa)	Not affected by STAT SET	INCO ₂ + 5 mmHg INCO ₂ + 0.7 (% or kPa)
Apnea Delay	Not affected by STAT SET		

a. New alarm limits calculated by STAT SET cannot be outside the allowable alarm limit range. If a new limit is calculated to be above or below the allowable alarm limit range, it defaults to the maximum or minimum alarm limit allowed for that vital sign.



WARNING If a patient's vital sign value falls outside of the upper or lower alarm range limit, STAT SET turns off the alarm and the alarm limit except for the following:

- The lower alarm limits for SpO₂ and ETCO₂ are not turned off by STAT SET.
- If HR/PR ALARM LIMITS in the Settings window is set to CANNOT TURN OFF, STAT SET affects HR/PR alarm limits as follows:

HR/PR PATIENT VALUE	DISPLAY	UPPER LIMIT	LOWER LIMIT
Ovrange	+++	Maximum	Unchanged
Underrange	---	Unchanged	Minimum
Indeterminate	???	Unchanged	Unchanged

Alarm holdoffs

To help minimize false alarms, the monitor briefly delays or “holds off” triggering alarms for limit violations for HR/PR, SpO₂, and RR/BR. After the alarm holdoff period begins, if the monitor detects that the patient’s vital sign has returned to acceptable limits, the monitor cancels the alarm holdoff. The next time a vital sign limit is violated, the monitor starts a new holdoff period.

Vital Sign	Alarm Holdoff Time Period
HR/PR	3 seconds (except NIBP PR)
SpO ₂	10 seconds
RR/BR	5 seconds

Connect nurse call option

The Propaq CS monitor can be connected to a Nurse Call system through a customized cable that connects to the left-side Nurse Call connector. When connected, the monitor immediately notifies the Nurse Call system whenever a patient alarm occurs.



To connect the monitor to the Nurse Call system, you need a cable (008-0634-XX) that has been customized for your Nurse Call system. If you do not have this cable, contact your biomedical engineering department for assistance. For specifications, see [“Nurse Call specifications”](#) on page 108.

Note When an apnea alarm or patient alarm occurs, pressing the **Silence/Reset Alarm** key or **SILENCE** silences the alarm tone and Nurse Call alarm for 90 seconds. However, the visual indicators on the monitor are *not* suspended during this time. Even though the Nurse Call option allows remote alarm indication, it does not replace appropriate bedside surveillance by trained clinicians.

Troubleshooting equipment alert messages

Simultaneous equipment alerts

MULTIPLE. If multiple equipment alerts occur simultaneously, the Propaq monitor displays this message. In this situation, the source of the alert may not be displayed (resulting in the message MULTIPLE). Look for blank numeric or waveform areas on the display, or status messages at the top of the display to identify the source(s) of multiple alerts.

Note The monitor can also display MULTIPLE LEAD FAIL if, for example, there is a problem with an ECG lead and RESP lead at the same time. In this case, the monitor will alternately display ECG FAULT and RESP FAULT. Check all ECG and RESP leads for proper connection and operation.

ECG messages

ECG FAULT. (This message is displayed in the status message area at the top of the display.) A problem occurred with ECG. If the problem is caused by a defective or disconnected lead, the monitor typically displays an additional message that indicates which lead failed (such as LL LEAD FAILED). Check the ECG lead for proper connection and operation.

ECG LEAD CHANGED. The Propaq CS monitor has automatically changed an ECG lead due to a lead wire or electrode problem.

LEAD FAIL: REPLACE ELECTRODES. The cable may not be properly connected to the electrodes or the electrodes may have failed. Check for proper connection; replace electrodes if needed.

MULTIPLE LEAD FAIL. The monitor displays this message if multiple ECG leads fail. Check all ECG leads for proper connection and operation.

RESP messages

RESP FAULT. (This message is displayed in the status message area at the top of the display.) A problem occurred with RESP. The monitor typically displays an additional message indicating the cause of the problem, such as a problem with a RESP lead. Check for proper connection; replace electrodes if needed.

LEAD FAIL. One or more electrodes are making very poor or no contact. Check for proper connection; replace electrodes if needed.

INAPPROPRIATE ECG CABLE. ECG cable appears not to contain 1 kW current limiting resistors. These resistors are required for RESP operation and to protect the monitor from damage during defibrillation. Replace cable with proper type.

NOISY SIGNAL, CHECK ELECTRODES. Electrodes are making poor contact and may be dried out. Replace electrodes.

IBP messages

TRANSDUCER NOT DETECTED. The transducer connection is broken.

TRANSDUCER SHORT CIRCUIT. This message appears when the Propaq CS monitor senses a short in the transducer. The transducer should be replaced.

INCOMPATIBLE TRANSDUCER. Visit www.welchallyn.com to confirm you are using a compatible transducer.

NIBP messages

If an error number (ERR# x) is listed in an NIBP trend printout or display, it indicates that the corresponding NIBP equipment alert occurred.

AIR LEAK, CHECK HOSE (ERR# 1). The Propaq CS monitor could not properly inflate cuff. Check the hose and cuff for obvious leaks, such as the O-rings in the hose connections.

CUFF NOT DETECTED (ERR# 2). During cuff inflation the detected pressure did not sufficiently rise. Check that the cuff connection is tight and take the measurement again.

KINKED HOSE, CHECK HOSE (ERR# 3). The Propaq CS monitor could not properly inflate cuff. Check for a kinked hose between the monitor and the patient.

OVERPRESSURE CONDITION (ERR# 4). The pressure in the cuff exceeded the acceptable limits for patient mode. Check the hose and try taking another measurement.

WEAK PULSES, CAN'T FIND SYS/DIA (ERR# 5). There are not enough pulses to determine the systolic or diastolic pressures, but a mean pressure is available. Try reapplying the cuff after squeezing as much air from it as you can.

ARTIFACT, CAN'T FIND SYS/DIA (ERR# 6). The systolic or diastolic pressures are unreliable due to artifact, but a mean pressure is available. May be caused by patient motion.

NO PULSES DETECTED (ERR# 7). The cuff may not be properly applied to the patient, or the patient may not have detectable pulses due to shock or arrhythmias.



WARNING The Propaq CS monitor cannot differentiate between physiologic and cuff application causes of the NO PULSES DETECTED message. Always evaluate the patient for presence of life threatening conditions whenever this message occurs.

CONNECT ECG TO REDUCE NIBP ARTIFACT (ERR# 8). NIBP artifact prevents a valid reading. Connect ECG electrodes to improve NIBP measurements.

NO VALID BLOOD PRESSURE FOUND (ERR# 9). This message can occur due to motion artifact, the Propaq CS monitor being set in the wrong patient mode, or the wrong hose or cuff being used in relation to the patient mode.

CALIBRATING, PLEASE WAIT (ERR# 10). The Propaq CS monitor periodically recalibrates the NIBP channel to ensure it can properly make NIBP determinations. Normal monitor operation continues while the NIBP channel is calibrating. If the NIBP channel has not updated its calibration in 15 minutes, the channel will briefly deactivate until a new calibration has occurred.

LOW BATTERY, NIBP DISABLED (ERR# 11). The battery lacks sufficient voltage to be able to operate the NIBP channel. Connect the Propaq CS monitor to the ac power adapter.

SERVICE REQUIRED, NIBP DISABLED (ERR# 12). Have the monitor serviced.

CUFF TOO LARGE FOR PATIENT MODE (ERR# 13). The monitor detects a cuff too large for the current patient mode. First, verify the patient mode. If the patient mode is correct, confirm the cuff size is correct and make sure the cuff fits snugly. If this alert occurs in Neonatal Mode, change the patient mode to Pediatric Mode and check the alarm limits. If the alert occurs in Pediatric Mode, change to Adult Mode and check the alarm limits. Note that different pressures and retries are used for each mode as stated in “[NIBP specifications](#)” on page 101.

KINKED OR NEONATE HOSE (ERR# 14). This message occurs when a hose is kinked or when a neonate hose is detected in the adult patient mode. Check the hose or the patient mode selection.

ARTIFACT PRESENT, MINIMIZE ARTIFACT (ERR# 15). The monitor has detected too much artifact to allow accurate readings. Take steps to reduce artifact. Position the patient’s limb away from the body so the applied cuff is not in contact with the patient’s body or any other object such as a bed rail. If the Smartcuff motion artifact filter is on, make sure that the ECG leads are properly connected to perform ECG monitoring during NIBP. If the Smartcuff motion artifact filter is off, consider turning it on (and connect ECG if not already connected).

The following messages can appear in the NIBP status window.

CALIBRATING. The NIBP channel is running an internal calibration.

DISABLED, LOW BATT. See LOW BATTERY, NIBP DISABLED above.

NIBP DISABLED, SERVICE REQUIRED. See SERVICE REQUIRED, NIBP DISABLED above.

RETRY. Since the Propaq CS monitor did not receive a valid NIBP reading, it will automatically attempt to take another reading.

The following NIBP status message looks similar to an equipment alert, although it does not indicate a malfunction and does not cause an alert tone.

NIBP IN PROGRESS, PLEASE WAIT, FILTERING ARTIFACT. Noise or artifact such as vehicle motion is causing a delay while measuring NIBP. To remove the message, press any key below the screen. To cancel the NIBP measurement, press the **Start/Stop NIBP** key at the right of the screen.

The following message can be displayed if the monitor detects a system error.

REMOVE CUFF FROM PATIENT. See “[Troubleshooting system error messages](#)” on page 65.

Temperature messages

PROBE NOT DETECTED. This message occurs when the Propaq CS monitor has successfully measured temperature and a probe is then disconnected. Reconnect the probe or acknowledge the equipment alert by pressing any menu key.

PROBE SHORT. Verify that the probe is properly inserted in the left side panel. If so, replace probe.

CALIBRATION ERROR, TEMP DISABLED. This message appears when the Propaq CS monitor has detected that it cannot accurately measure the temperature. The monitor should be serviced.

Malfunction of the temperature probes may result in inaccurate readings. Confirm suspect readings.

SpO₂ messages

SpO₂ messages can appear in the equipment alert window or in the SpO₂ numeric window.

DEFECTIVE SPO2 SENSOR. If you see the error message DEFECTIVE SpO₂ SENSOR, either the sensor is not compatible with the monitor or the sensor is not working properly. Visit www.welchallyn.com to be sure the sensor is compatible. If compatibility is not a problem, try another sensor.

NO SENSOR DETECTED. Indicates an SpO₂ sensor has been disconnected from the monitor after being plugged in for more than a few seconds.

SEARCH: During this search time, the SpO₂ channel tries to detect blood pulsing through the measurement site. After the measurement has been established, the oxygen saturation value is displayed in the numeric window.

STANDBY is displayed in the numeric window when the SpO₂ sensor is disconnected from the patient, an alarm occurs, and you press the **STANDBY** key. STANDBY is also displayed if you first plug the SpO₂ sensor cable into the monitor connector before attaching the SpO₂ sensor to the patient.

Mainstream CO₂ messages

Messages for the mainstream CO₂ option can appear in the equipment alert window and in numeric zones. If a sensor is damaged, contact Welch Allyn Technical Services Department for information on sensor service options.

ALTIMETER FAILURE - RANGE. The Propaq CS monitor is operating at an altitude outside the mainstream CO₂ option's operating altitude range of -2,000 to 15,000 feet. Returning the monitor to within this range automatically cancels this message and restores operation.

ALTIMETER FAILURE - RATE. The altimeter has detected that the ambient pressure is changing at a rate greater than 100 mmHg/minute. When the rate of change is back within the 100 mmHg/minute range, disconnect and reconnect the CO₂ sensor to the Propaq CS monitor.

DEGRADED WAVEFORM - CHECK ADAPTER (UNCAL appears in the numerics area). The mainstream CO₂ adapter is obstructed or the CO₂ sensor has failed. The CO₂

waveform is displayed without range values. Replace the adapter or replace the sensor.

LACK OF WAVE - CHECK ADAPTER, SENSOR. Either the airway adapter is obstructed or the CO₂ sensor has failed. Replace the airway adapter if it is obstructed. The sensor must be unplugged and plugged in again.

LOW BATTERY - HEATER DISABLED (UNCAL appears in the numerics area). The monitor's battery voltage is too low. The CO₂ waveform is displayed without range values. To continue operation, supply ac power to the monitor.

NO MAINSTREAM SENSOR DETECTED (SRCH appears in the numerics area). The mainstream CO₂ sensor has been disconnected from the Propaq CS monitor after providing CO₂ values. Disconnect and reconnect the sensor to the monitor if necessary.

NON-PROTOCOL SENSOR (UNCAL appears in the numerics area). A CO₂ sensor has been connected that does not match Welch Allyn's specifications. The CO₂ waveform is displayed without range values. Replace the sensor with a Welch Allyn CO₂ sensor.

SENSOR FAILURE - CALIBRATION ERROR. A sensor is defective or out of calibration and disabled. Replace the sensor.

SENSOR FAILURE - EEPROM. The sensor has failed. Replace the sensor.

SENSOR FAILURE - HEATER. The sensor's temperature control circuit or the monitor's CO₂ circuitry has failed. Try replacing the sensor. If the message reappears, have the monitor serviced.

SENSOR FAILURE - MOTOR DRIVE. The sensor's motor drive (in the sensor head) has failed. Replace the sensor.

SENSOR TEMPERATURE TOO HIGH. The sensor's temperature is too high. The sensor's ambient operating range is 10° to 46° C. When the ambient temperature returns to this range, this message is automatically removed and operation is restored.

The following messages can appear in the numerics display area.

OFF. No CO₂ source is selected.

SRCH. The sensor is preparing for a measurement.

UNCAL. The monitor has detected a problem such as a lack of calibration, an obstruction, or a low battery.

WARM UP. The sensor heater is warming up. Wait 20 to 30 seconds for the sensor to heat. Values should appear in the numerics area when the sensor is sufficiently warm.

Sidestream CO₂ messages

ALTIMETER FAILURE - RANGE. The Propaq CS monitor is operating at an altitude outside the sidestream CO₂ option's operating altitude range of -2,000 to 15,000 feet. Returning the monitor to within this range automatically cancels this message and restores operation.

ALTIMETER FAILURE - RATE. The altimeter has detected that the ambient pressure is changing at a rate greater than 100 mmHg/minute. When the rate of change is back within the 100 mmHg/minute range, disconnect and reconnect the CO₂ sensor to the monitor.

ALTIMETER NOT CALIBRATED - EEPROM. The sidestream CO₂ option has not been calibrated. Refer the Propaq CS monitor to a biomedical engineer for calibration.

AMBIENT TEMPERATURE TOO HIGH. The sensor temperature is too high. The sidestream CO₂ option is disabled until the ambient temperature is within the operating range specifications.

AMBIENT TEMPERATURE TOO LOW. The sensor temperature is too low. The sidestream CO₂ option is disabled until the ambient temperature is within the operating range specifications.

CALIBRATION ERROR - SERVICE REQUIRED. Send the Propaq CS monitor to a biomedical engineer for service.

DEGRADED WAVEFORM - SERVICE REQUIRED. Send the Propaq CS monitor to a biomedical engineer for service.

LACK OF WAVEFORM - SERVICE REQUIRED. Send the Propaq CS monitor to a biomedical engineer for service.

MOTOR FAILURE - SERVICE REQUIRED. The sensor hardware has failed. Send the Propaq CS monitor to a biomedical engineer for service.

NO WATERTRAP DETECTED. There is no sidestream CO₂ watertrap or CO₂ watertrap adapter installed. Install a watertrap or CO₂ watertrap adapter.

OCCLUSION - CHECK EXHAUST PORT/TUBING. Blockage has been detected on the pneumatic exhaust port. Check the exhaust port and related tubing for occlusions. Make sure that the sampling line and any inputs to the patient breathing apparatus are not connected to the exhaust port.

OCCLUSION - CHECK WATERTRAP/TUBING. Blockage has been detected on the sidestream CO₂ input. Check the watertrap, sample line, and any connected tubing for occlusion.

PUMP FAILURE, SERVICE REQUIRED. The pump is not able to maintain the target flow rate. Send the Propaq CS monitor to a biomedical engineer for service.

SIDESTREAM STICK EEPROM FAILURE. Send the Propaq CS monitor to a biomedical engineer for service.

SSP BOARD EEPROM FAILURE. Send the Propaq CS monitor to a biomedical engineer for service.

The following messages can appear in the numerics display area.

OFF. No CO₂ source is selected.

SRCH. The sensor is preparing for a measurement.

START UP. sidestream CO₂ has been activated and is preparing for operation. This typically requires 30 seconds at room temperature.

UNCAL. The monitor has detected a problem such as a lack of calibration, an obstruction, or a low battery.

Network alert messages with Acuity

CHECK ACUITY/SERIAL CONNECTION. The Propaq CS detects a problem in the hardwired (serial) communication with Acuity. Check the Acuity network cable to be sure it is plugged into the side panel and the bedside Acuity jack. If the cable is damaged, replace the cable. If the cable appears undamaged and the Acuity system is operating normally, contact biomedical engineering for assistance.

CHECK ACUITY/NETWORK CONNECTION. The Wireless Propaq CS detects a problem with the wireless communication with Acuity. The monitor might be out of range of the network, or there might be a problem involving the monitor radio card, the access point, or the Acuity system. If the problem persists, contact biomedical engineering for assistance.

EQUIPMENT ALERT. The Propaq CS detected a download failure or an unauthorized download was attempted. Refer to the *Propaq CS Vital Signs Monitor Service Manual* (part number 810-2692-00) for more information.

Program alert message

PROGRAM FAULT, SETTINGS LOST, TIME/DAY RESET. At powerup, the monitor cannot recall the programmed Custom patient mode settings and current time and date. This can occur if the battery is drained or after new software has been installed.

If this occurs, the monitor provides a special sequence of display windows to help you regain use of your monitor as quickly as possible. Do the following:

1. Connect an ac power adapter to recharge the battery (if the battery is drained).
2. Press any key below the equipment alert screen to acknowledge the alert. The monitor displays the Mode Setup window (shown on ["Change the Powerup Patient Mode"](#) on page 19).
3. Press these keys to select one of the Factory patient modes for use:
 - Factory Adult mode **POWERUP*, YES.**
 - Factory Pediatric mode **NEXT, POWERUP*, YES.**
 - Factory Neonatal mode **NEXT, NEXT, POWERUP*, YES.**

After you press **YES**, the monitor displays the Time/Day window.

4. Press **NEXT, UP**, and **DOWN** as needed to set the time and date. Then press **ENTER** to store the new time and date.
5. Turn off the monitor, then turn it on again so the settings will take effect.

The monitor is ready for use. To store customized patient mode settings, refer to ["Customize Patient Mode settings"](#) on page 20.

If you follow these steps and the equipment alert reappears at powerup, the monitor may need to be serviced and the battery replaced. Contact a qualified service person.

Note These display screens are only displayed in this order if the PROGRAM FAULT equipment alert occurs.

Printer alert messages

CHECK DOOR. The door on the bottom of the printer is open. Close the door to remove this message.

LOW BATTERY, PRINTER DISABLED. The monitor's battery voltage is too low to support printing. Connect the ac power adapter to recharge the battery (see ["Connect the AC power adapter to recharge the battery"](#) on page 83).

OVERHEATING. The printer is overheating. Service may be required.

PAPER OUT. To add printer paper, see ["Install printer paper"](#) on page 87.

Defibrillator alert message

DEFIB FAULT, CHECK INTERFACE CABLE. The monitor detects a problem with the interface cable. Check the cable and defibrillator.

Very low battery alert message

VERY LOW BATTERY, PLUG IN EXTERNAL POWER ADAPTER. The monitor battery needs to be recharged. Connect the ac power adapter to recharge the battery (["Connect the AC power adapter to recharge the battery"](#) on page 83).

If the battery is not recharged, the monitor will begin to disable monitor functions and eventually turn off completely.

Troubleshooting system error messages

If the monitor detects a system error, it will display a message similar to the following:

```
ERROR NUMBER: 1-123-4567  
REMOVE CUFF FROM PATIENT
```

```
PLEASE NOTIFY YOUR LOCAL SERVICE  
REPRESENTATIVE OR WELCH ALLYN PROTOCOL, INC.
```

This type of message indicates that the monitor detected an internal system problem which may require service for the monitor. If the monitor displays this message, disconnect the monitor from the patient. Write down the error number for the service department, and send the monitor to your facility's biomedical engineering department.

Note The message REMOVE CUFF FROM PATIENT as shown above does not imply that the detected error is related to NIBP. It is simply a reminder to disconnect the NIBP cuff from the patient if a cuff is connected.

5 Printing & trends


Print patient data

Print the displayed waveforms

Press **SNAPSHOT** or **START/STOP**. The Propaq CS monitor prints up to three of the displayed waveforms.

When four waveforms are displayed, the monitor prints the top three displayed waveforms (except for the ECG2 waveform which is never printed).

If you pressed **START/STOP**, the monitor continues to print until you press **START/STOP** again.

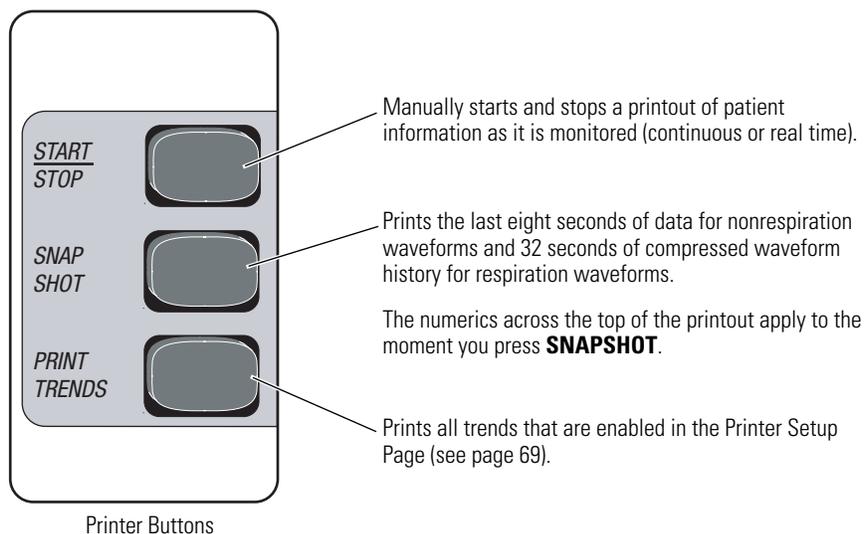

Note The symbol  indicates that the NIBP reading was taken in the presence of high artifact while monitoring ECG with the Smartcuf motion artifact filter on. Artifact can affect accuracy. To help reduce artifact, see [Step 6](#) on page 35.



Display or print trends

Display or print a single trend

- To display a patient data trend, press **SETUP, TRENDS** from the Main Menu. The monitor displays the Trend Menu:


The monitor stores trends every 2 minutes (except for NIBP) for up to 5 hours. After 5 hours, newly recorded trends replace the oldest trends.

NIBP trends are stored when measured. Up to 128 NIBP trends can be stored (up to 8 hours).

The TIME, HR/PR, and SpO2 are stored for all trends.

OFF indicates the vital sign was not being monitored.

ERR#x in an NIBP TREND display or printout indicates an NIBP equipment alert occurred. See "NIBP messages" on page 59 for NIBP alert error numbers and definitions.

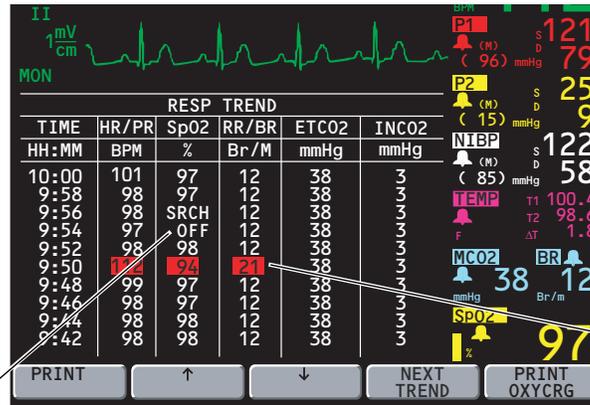

Numerics that have alarmed are highlighted.

Press to print the displayed trend.

Press to scroll through the recorded trends.


Press to scroll to the next vital sign trend (IBP, TEMP, and NIBP).

Press to print OxyCRG (see page 71).

Trends are also displayed on the Main Menu if all waveforms except ECG1 are turned off in the Wave Select Window.

- Press **NEXT TREND** as needed to display the desired trend.
- Press **PRINT** to print the displayed trend.

Note The symbol  indicates that the NIBP reading was taken in the presence of high artifact while monitoring ECG with the Smartcuf motion artifact filter on. Artifact can affect accuracy. To help reduce artifact, see [Step 6](#) on page 35.

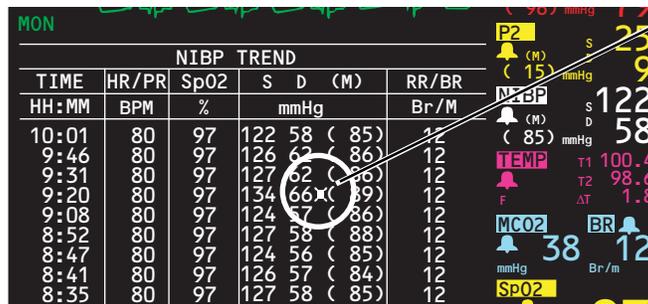

Print multiple trends manually or automatically

1. Press **SETUP, MORE, PRINTER** from the Main Menu to display the Printer Setup Page:

Specifies what time the trends automatically print. Trends are printed every 4 hours beginning at either 01, 02, 03, or 04 hours. (OFF=no printing.)

Select ON to include in trend printing.

Press to scroll to the next selection.

Press to change the displayed value.

Press to print all selected trends.

2. Press **NEXT** as needed to scroll down to the parameters listed below AUTO TREND (NIBP, RESP, etc.).
3. Press **NEXT** and **CHANGE** to set desired trends to ON.
4. To manually print all selected trends, press **PRINT TREND** on this page or press **PRINT TRENDS** on the bottom front panel of the monitor.
You can print all selected trends at any time by pressing the **PRINT TRENDS** button.
5. To program the monitor to automatically print selected trends every four hours, press **NEXT** as needed to highlight AUTO TREND, then press **CHANGE** to select the hours for printing.


For example, if you select 01 05 09 13 17 21 at 4:27, the printer will automatically print selected trends first at 5:00, then 9:00, etc.

Delete all patient trends

To delete all trends recorded for a patient, turn off the monitor.

Set printer options and automatic printing

1. Press **SETUP, MORE, PRINTER** from the Main Menu to display the Printer Setup Page:

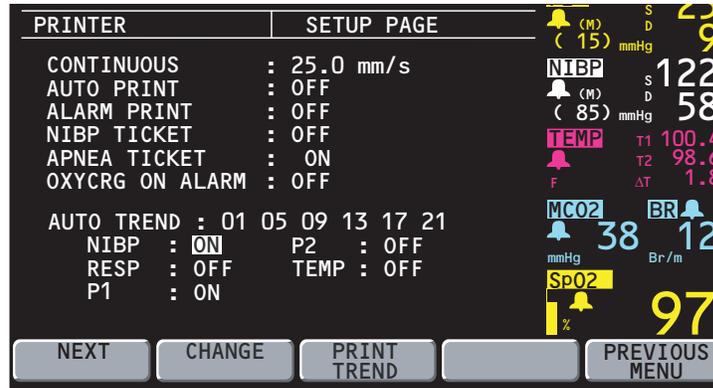

Press to scroll to the next selection.

Press to change the displayed value.

Press to print all selected trends.

2. Press **NEXT** and **CHANGE** as desired to set printer options.

CONTINUOUS Set the speed for continuous printing: 6.25, 12.5, or 25.0 mm/s.

AUTO PRINT Automatically print a waveform snapshot at the specified interval: 15 or 30 minutes, or 1, 2, or 4 hours (or OFF).

ALARM PRINT If ON, automatically prints patient data whenever a patient alarm occurs, beginning with 12 seconds of patient data history stored before the alarm occurred.

Printing continues for 20 seconds after you suspend the alarm. To immediately stop printing, press **START/STOP**.

Note Because the Alarm Print begins with the 12 seconds of patient data stored before the alarm occurred, the monitor stores and prints all Alarm Print data 12 seconds after the patient data appears on the display. The time annotated on the Alarm Print indicates the time the data was recorded.

NIBP TICKET If ON, automatically prints an NIBP TICKET with NIBP data whenever NIBP is measured.

APNEA TICKET If ON, automatically prints an APNEA TICKET with apnea data after the patient resumes breathing and/or every minute the apnea alarm continues.

OXYCRG ON ALARM If ON, automatically prints an OxyCRG whenever an SpO₂, HR/PR, RR/BR, or apnea patient alarm occurs (see page 71).

If an SpO₂ or HR/PR alarm occurs, the OxyCRG prints 60 seconds later. If an Apnea or RR/BR alarm occurs, the OxyCRG prints 75 seconds later. Highlighted labels in the printout indicate which alarms occurred.

AUTO
TREND


Automatically print trends at the selected hours. Only the parameters set to ON (for NIBP, RESP, P1, P2, or TEMP) are included in trend printouts.

Print OxyCRG

The OxyCRG is a printout of two minutes of continuous HR/PR and SpO₂ numerics, and a compressed respiratory waveform.

- To print OxyCRG, from the Main Menu press **SETUP, TRENDS, PRINT OXYCRG**.

If any of the parameters have been completely inactive for the two minutes prior to the printout, the associated band is empty.

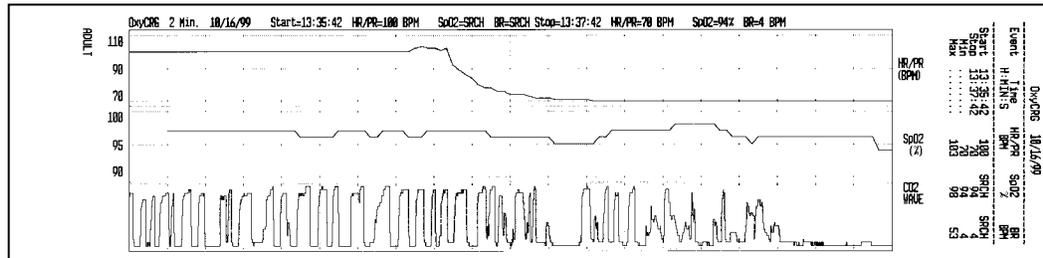

6

Acuity

Understanding the Propaq CS, Acuity, and FlexNet network

This section describes the operation of the Propaq CS monitor (software version 3.5X) with the Acuity FlexNet network. It describes both the Acuity hardwired and wireless communication options for the Propaq CS monitor.



WARNING When connecting the Acuity hardware connection, connect the Propaq CS monitor to an Acuity system only. Connecting to other networks could damage the monitor or injure the patient. If in doubt about the network jacks or devices, consult your facility's Biomedical Engineering Department.

WARNING Make sure the Acuity network cable is not damaged. If the wireless connection is not an option or not available, the Acuity network cable is the sole link between the Propaq CS monitor and the Acuity Central Station.

WARNING When considering a treatment protocol that involves wireless communication of patient data, be sure to recognize some limitations inherent in wireless communications. When the Propaq CS monitor is not connected to the network by wireless (or hardwired) connection:

- There are no patient alarms or alerts at the Acuity Central Station.
- Acuity does not perform arrhythmia and ST analysis on the patient data and does not generate related alarms.

If wireless communication is no longer available or is not functioning properly, consider using the hardware connection to Acuity.

WARNING If you don't set alarm limits, the Acuity system uses preset settings (for arrhythmia test limits), and the powerup default settings for the Propaq CS monitor.

WARNING The radio in this monitor has been authorized by the FCC for mobile use only. Mobile use as defined by the FCC is for operation 20 cm or more away from a person's head or torso. The distance does not apply to transient exposure due to incidental passage closer than the maximum permissible exposure (MPE) limit.

WARNING When the monitor moves out of range of the Acuity network, communication with Acuity is broken immediately, stopping the communication of patient vital-signs data. If the monitor is out of range of the Acuity network for several minutes, the radio enters a low-power state. When the monitor is again within range of the Acuity network, it can take as long as three minutes to restore communication with Acuity and resume the communication of patient vital-signs data.




WARNING Accessory equipment connected to the analog and digital interfaces must be certified according to the respective IEC standards (e.g., EN 60950 for data processing equipment and EN 60601-1 for medical equipment). Furthermore, all configurations shall comply with the system standard IEC 601-1-1. Anyone connecting additional equipment to the signal input or output connectors is configuring a medical system, and is therefore responsible that the system complies with the requirements of the system standard IEC 601-1-1. If in doubt, consult your Biomedical Engineering Department.

Note WHEN USED IN CANADA: To prevent radio interference to the licensed service, this device is intended to be operated indoors and away from windows to provide maximum shielding. Equipment (or its transmit antenna) that is installed outdoors is subject to licensing.

Pour empêcher que cet appareil cause du brouillage au service faisant l'objet d'une licence, il doit être utilisé à l'intérieur et devrait être placé loin des fenêtres afin de fournir un écran de blindage maximal. Si le matériel (ou son antenne d'émission) est installé à l'extérieur, il doit faire l'objet d'une licence.

The Propaq CS monitor can communicate with an Acuity Central Station as part of Welch Allyn's FlexNet network. FlexNet allows multiple devices to communicate through hardwired Ethernet networks and Wireless Local Area Networks (WLANs). The Acuity Central Station provides central patient monitoring for monitoring devices connected to the network.

As shown below, the Propaq CS can communicate through a hardwire Acuity connection. In addition, the Propaq CS can also be equipped with the Wireless option which allows two-way wireless communication with an Acuity Central Station through an access point in the FlexNet network. The access point is a digital radio transceiver that connects to the FlexNet network.

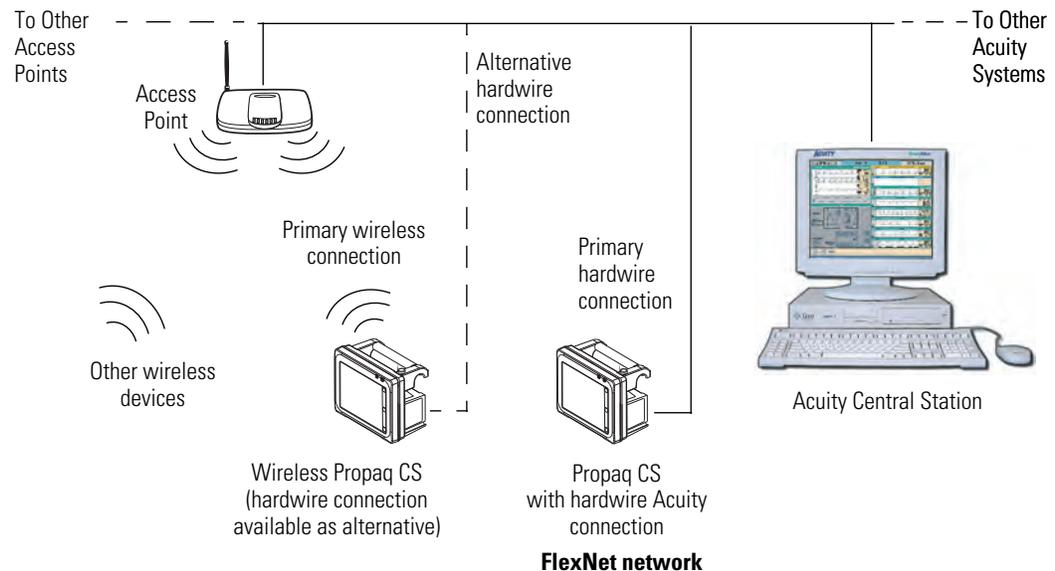


While connected to the network, the Propaq CS sends patient data to Acuity. Acuity continuously analyzes the data and provides appropriate alarm or alert messages at the Central Station and other network devices such as a hallway message panel or the Propaq CS itself. Acuity also stores the patient data for viewing or report printing.

If the Wireless Propaq CS is moved out of range or loses communication with the FlexNet network and Acuity, it continues to monitor the patient and display patient data. While not communicating with Acuity, the Propaq CS continues to generate local patient alarms or alert messages. Acuity does not perform waveform analysis or generate arrhythmia or ST analysis messages while the Propaq CS is not communicating with Acuity. When the Wireless Propaq CS is returned to within range of the FlexNet network, it automatically reconnects to Acuity and uploads trend information.

A Propaq CS with the Acuity hardwire communication option can be identified by the Acuity connector on the right side panel.

A Wireless Propaq CS can be identified by the yellow antenna cap on the top right corner of the monitor in front of the carrying handle. Every Wireless Propaq CS includes the Acuity hardwire communication option.

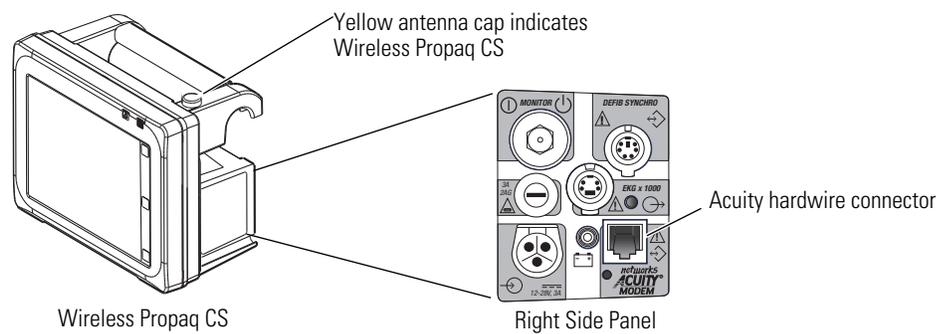

If a Wireless Propaq CS monitor is communicating with the network via the wireless interface and then the Acuity hardwire cable is used to connect the bedside Acuity network jack and the monitor, the monitor disconnects from the wireless connection and communicates through the hardwire connection. There may be a short interruption in the display of patient waveforms at Acuity while this change occurs.

Note The Wireless Propaq CS can not be used with the Modem-Propaq communication option, even when the Wireless Propaq CS is being used with a hardwire connection or is out-of-range of a FlexNet Network access point.

The Acuity Menu

The Acuity Menu on the Propaq CS allows you to control some of the interaction with the Acuity network. The Acuity Menu is only accessible while the monitor is connected to the Acuity network.

The Acuity Menu for the Wireless Propaq CS provides additional functions that allow you to manage the patient assignment and patient location from the monitor. These functions are described later in this document.

ACUITY MENU (HARDWIRED CONNECTION)

		NET OFF	SNAPSHOT	PREVIOUS MENU
--	--	------------	----------	------------------

Press to print a waveform displayed on the Propaq CS monitor screen at the Acuity printer.

Press to temporarily disconnect from the Acuity network.

ACUITY MENU (WIRELESS CONNECTION)

PATIENT		END TELE	SNAPSHOT	PREVIOUS MENU
---------	--	----------	----------	------------------

Press to access the Patient Menu with additional functions.

Press to print a waveform displayed on the Propaq CS monitor screen at the Acuity printer.

Press to disconnect from the Acuity network prior to discharging the patient.

PATIENT MENU

NEW ROOM	TRANSFER	NEW PAT	PAT INFO	PREV MENU
----------	----------	---------	----------	-----------

Press to display patient information such as ID, name, unit and room.

Press to assign the monitor to a new patient.

Press to transfer a patient to a new room in a new unit.

Press to reassign a patient to a new room in the same unit.

Hardwired (Serial) connection to Acuity


Connect the monitor to the Acuity System

1. If the Propaq CS monitor has already been connected to the patient, save the patient's Trends and Alarm Limit settings by keeping the monitor turned on.

The monitor transmits up to five hours of trend information when you connect it to the Acuity network.

If the monitor has not been connected to the patient, clear any prior patient's trends and alarm limit settings by turning off the monitor, then turning it on after a few seconds.

2. If the monitor is not already connected to the patient, attach leads and sensors to the patient as described in this manual.
3. Plug in the Acuity network cable to the Acuity network jack on the monitor side panel as shown. Plug in the other end of the cable to the bedside Acuity network jack.

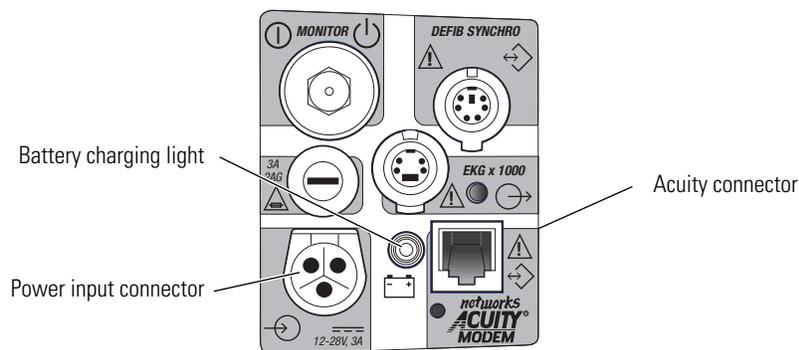

4. Connect the ac power adapter to the monitor and the wall outlet to charge the battery. Check to see that the green battery charging light on the monitor's right side panel is on.
5. When the monitor completes the connection to Acuity, it displays the message ON NETWORK (or alternating messages ON NETWORK and SERIAL if the monitor has wireless capability). Confirm the patient identification at the bedside or enter the patient information at the Acuity Central Station using the Patient ID Setup Window.
6. If alarm limits have not been set, do so at the monitor or at the Acuity Central Station using the Alarms Setup Window.

Note When a Propaq CS monitor in Adult or Pediatric Mode is connected to an Acuity System, the audible alarms at the bedside Propaq CS monitor can be delayed up to 4 minutes and 15 seconds. The delay time is selected in Acuity software at the time of Acuity installation. Visual alarm indications are not delayed.

Disconnect the hardwired monitor from the Acuity System



1. To disconnect the Propaq CS monitor from the Acuity network, press the **NET OFF** key on the upper left corner of the monitor display. (From the Main Menu you can also press **SETUP, ACUITY, NET OFF**.)
2. Within 15 seconds, disconnect the Acuity network cable from the Propaq CS monitor side panel and the bedside jack. If the patient will no longer be monitored with this monitor, turn off the monitor to erase trend information.

Note If the Propaq CS has the wireless option, be aware that after you disconnect the hardwire connection and disconnect the Acuity network cable, the Wireless Propaq CS will attempt to establish a wireless connection to Acuity as long as the power is on.

Switch from hardwired to wireless monitoring

1. Press the **NET OFF** key on the upper left corner of the monitor display. (From the Main Menu you can also press **SETUP, ACUITY, NET OFF**.)
2. Disconnect the Acuity network cable from the Wireless Propaq CS Acuity connector.
The Wireless Propaq CS automatically seeks to establish a wireless connection to the Acuity network.

Note If you disconnect the Acuity network cable without first pressing **NET OFF**, this will cause a momentary equipment alert at the monitor and Acuity.

Wireless connection to Acuity

Connect a new patient

1. Make sure the Acuity network cable is not plugged into the Acuity connector and an Acuity bedside network jack. Turn on power to the Wireless Propaq CS. The monitor displays the powerup screen for about 10 seconds, then displays the Main Menu. The monitor is in the powerup patient mode with the associated settings.

2. Confirm that within a few seconds the Wireless Propaq CS displays the flashing message WIRELESS in the upper left corner of the display, alternating with one of these messages:

SEARCHING indicates the monitor is searching for a connection with an access point.

CONNECTING indicates the monitor has associated with an access point, but is not fully connected to the Acuity network.

3. Confirm that within one minute the monitor displays these alternating messages:

ON NETWORK and WIRELESS

This indicates the Wireless Propaq CS is connected to the Acuity network.

4. After the network connection is established, the monitor may prompt you to select an Acuity Unit (if your facility has more than one Acuity unit). Scroll up or down to highlight the desired Acuity Unit, then press **Select**.
5. The Wireless Propaq CS displays a list of possible patients. If your patient has been pre-admitted to the selected Acuity unit, they will be included in the list.

If your patient is NOT in the list, highlight Select Patient at Central and press **Select**.

The patient name will need to be entered later at the Acuity Central Station.



WARNING If you do not select the patient name at the Wireless Propaq CS at this time, do not adjust any alarm limits until after the patient name and ID are confirmed at Acuity. When the patient name and ID are confirmed at Acuity, Acuity downloads the default settings and patient alarm limits for that Acuity unit to the Wireless Propaq CS, thereby overriding any previous settings and alarm limits.

If your patient is in the list, scroll to highlight your patient's name, then press **Select**.

6. Within a few seconds the Wireless Propaq CS displays a list of unassigned rooms.
 - If you want to assign the patient to a room, highlight the room and press **Select**.
 - If you do not want to assign a room at this time, highlight **Select Room at Central** and press **Select**.

The patient room will need to be entered later at the Wireless Propaq CS (see page 81) or at Acuity (see *Acuity Directions For Use*).

7. Customize alarm limits for your patient if needed.

Monitor a patient out of range of Acuity

While out of range of Acuity, the Wireless Propaq CS continues to monitor the patient and provide local alarms or alerts at the Propaq CS as needed.

When the patient with the Wireless Propaq CS goes out of range of Acuity, do the following:

1. A DROPOUT equipment alert occurs at the Acuity Central Station. Acknowledge the alert at Acuity.
2. An equipment alert occurs at the Wireless Propaq CS with this message:

NETWORK FAULT
CHECK ACUITY/NETWORK CONNECTION

Press a key on the Propaq CS to acknowledge the alert and silence the alert tones (if tones are enabled).

When the patient with the Wireless Propaq CS returns within range of Acuity, the Propaq CS automatically reconnects to Acuity.

Stop monitoring a patient with wireless prior to discharge

To discontinue monitoring the patient, follow these steps.

1. From the Main Menu, press **SETUP, ACUITY, END TELE**.
2. Disconnect the leads and sensors from the patient and turn off the power to the monitor.

If you do not turn off power to the monitor within about 30 seconds, the Wireless Propaq CS will automatically try to reconnect to the network.

If you do not use END TELE to disconnect from the network as described above, the Acuity Central Station generates a DROPOUT equipment alert at Acuity.

If you want to monitor this same patient at a later time, you will need to re-select the patient name from Wireless Propaq CS or confirm the patient ID at Acuity.

Reconnect a recently monitored patient

1. Turn on power to the Wireless Propaq CS and confirm that the monitor displays the powerup screen.
2. The Wireless Propaq CS will then present a series of menus and messages requesting you to provide information about the connection and patient. The actual screens presented depend on how long the patient has been disconnected.

Provide the information as requested. This may include:

- Select an Acuity unit.
- Select a patient from the patient list.
- Select a patient room from the room list.

If you do not select the patient name or room while connecting the patient, you will need to do that later at the Acuity Central Station. See the *Acuity Directions For Use* for more information.

Reassign a monitored patient to a new room in the same unit

If a patient is being monitored and you want to assign them to a new room in the same unit, follow these steps.

1. From the Main Menu, press **SETUP, ACUITY, PATIENT, NEW ROOM**.

Within a few seconds the Wireless Propaq CS displays a list of all available rooms, including the patient's current room.

- If you decide not to change the patient's current room assignment, press **Select** (the patient's current room is the default selection in the list).
- To assign the patient to a new room, scroll up or down to highlight the room and press **Select**.
- If you want to cancel the patient's current room assignment, but do not want to assign a new room at this time, you can highlight **Select Room at Acuity** and press **Select**. You can then assign the room later from the Acuity Central Station, or you can repeat this procedure and assign a new room from the Wireless Propaq CS.

Transfer a monitored patient to a new room in a different unit

If a patient is being monitored and you want to assign them to a new room in a different unit, follow these steps.

1. From the Main Menu, press **SETUP, ACUITY, PATIENT, TRANSFER**. Within a few seconds the Wireless Propaq CS displays a list of units.
2. Scroll up or down to highlight the new unit, then press **Select**.

The patient is not monitored at Acuity during the short time required by Acuity to process the transfer to the new unit (typically less than one minute). However, the patient continues to be monitored by the Wireless Propaq CS.

(If the selected unit is currently not available, the Wireless Propaq CS displays an appropriate message; press a key to acknowledge the message and cancel the transfer.)

3. After the patient is assigned to the new unit, the Wireless Propaq CS displays a list of unassigned rooms. (The patient's previous unit and room assignment is cancelled.)
 - To assign the patient to a new room, highlight the room and press **Select**.
 - If you decide not to assign the patient to a new room at this time, you can highlight **Select Room at Acuity** and press **Select**. You can then assign the room later from the Acuity Central Station, or you can assign a new room from the Wireless Propaq CS later using the procedure on page 81.

Switch from wireless to hardwired monitoring for the same patient

Plug the Acuity network cable into the Wireless Propaq CS Acuity connector and an Acuity bedside network jack.

The Wireless Propaq CS will switch to a hardwired Acuity network connection. No equipment alerts are generated.

Reassign the Wireless Propaq CS to a new patient

If you want to discontinue monitoring a patient and connect the Wireless Propaq CS to a new patient, follow these steps.

1. From the Main Menu, press **SETUP, ACUITY, END TELE**.
2. Turn off the monitor power, then turn it on again after a few seconds.

As an alternative to Steps 1 and 2, from the Main Menu you can press **SETUP, ACUITY, PATIENT, NEW PAT** for the same result.

The monitor then presents a series of menus and messages requesting you to provide information about the connection and patient. The actual screens presented depend on how the Acuity System is configured.

Provide the information as requested. This may include:

- Select an Acuity unit.
- Select a patient from the patient list. (After you select a new patient, all monitor operating settings are reset to the Acuity System default powerup settings.)
- Select a patient room from the room list.

If you do not select the patient name or room while connecting the patient, you will need to do that later at the Acuity Central Station. See *Acuity Directions For Use* for more information.

7 Maintenance

Avoid electrostatic discharge

When humidity in the working environment decreases, the human body and other insulators can become charged with static electricity due to friction.

To prevent unwanted electrostatic discharge (ESD), follow these standard guidelines:

- Maintain the recommended humidity of 40% to 60% in the work environment.
- Dissipate electrostatic charge before performing routine operator maintenance.

Connect the AC power adapter to recharge the battery



WARNING Use only accessories approved by Welch Allyn. Visit www.welchallyn.com. The use of any other accessories can result in inaccurate patient data, can damage the equipment, and can void your product warranty.

WARNING Use of other power adapters or power cords could cause a current leakage hazard or damage the Propaq CS monitor.

WARNING Place the power adapter where it cannot fall and harm someone.




Caution Leaving the monitor's lead-acid batteries in a completely discharged state may result in permanent battery damage. The batteries should be kept fully charged.

When the Propaq CS monitor battery voltage is low, the monitor displays the message LOW BATTERY at the top of the screen or the equipment alert message VERY LOW BATTERY, PLUG IN EXTERNAL POWER ADAPTER. You should connect an ac power adapter as soon as possible to recharge the battery.


If the battery is not recharged, the monitor will begin to disable monitor functions and eventually turn off completely.

1. Before connecting the ac power adapter, check the adapter power setting in the small window next to the power cord connector. Make sure the setting matches your ac power source (either 100V-120V or 200V-240V).

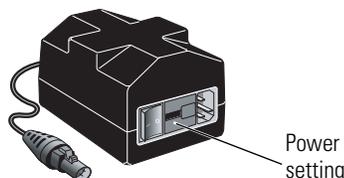

If it does not match, send it to your service department.

2. Plug the ac adapter power cord into the ac power adapter and the ac power source outlet.
3. Plug the ac adapter cord into the power input connector on the right side of the monitor.

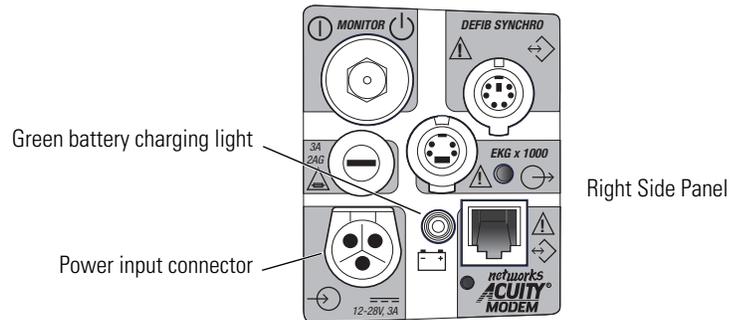

4. Confirm that the green battery charging light is on. The ac power adapter charges the battery even when monitor power is off. If monitor power is off, the battery charges to full capacity within 8 hours.


If the green light is not on, check all connections and make sure the ac power source is on.

If the green light is still not on, fuses may need replacement in the ac power adapter or the monitor. Contact your service department.

Replace power adapter fuses

If the green battery charging light is off and the ac power adapter does not provide power to the monitor even when all connections are intact, the adapter fuses may need to be replaced. This procedure must be performed by a qualified service person. To change fuses:

1. Unplug the removable power cord from the ac power source and adapter.

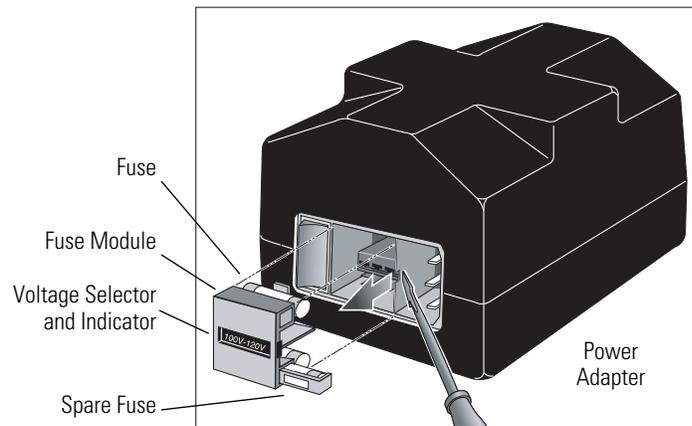

2. Using a small, flat-blade screwdriver, carefully pry the fuse module away from the adapter.
3. Remove and replace both fuses with the correct type specified on the adapter. The fuse module can contain spare fuses.



Caution Spare fuses are contained in housings next to the fuses in the fuse module as shown in the illustration. Between the fuses is a small printed-circuit board (PCB) that sets the power adapter to the desired ac mains voltage. When handling the fuse module, the PCB may slide out.

Caution Make sure the voltage selector indicates the proper ac input voltage. If you change the adapter voltage setting, you must replace all fuses to match the appropriate type specified on the bottom of the power adapter. The only fuses contained in the power adapter when shipped from the factory are fuses specified for the original adapter input voltage setting.

Caution Replace each fuse only with the specified type ("[Power Adapter Specifications](#)" on page 115).


Note Replace both fuses at the same time, even if only one fuse has opened due to an overcurrent situation. The unopened fuse may be damaged and unreliable.

If the small PCB between the fuses has slipped out of place, slide it back into place in the fuse module, and verify that the voltage setting indicated in the window on the fuse module is correct. If the voltage setting is incorrect, simply slide the PCB out of the fuse module, rotate it 180° and slide it back into place.

Replace monitor input power fuse

If the green battery charging light is off and the ac power adapter does not provide power to the monitor even when all connections are intact, the monitor's input power fuse may need to be replaced. This procedure must be performed by a qualified service person. To change fuses:

1. Disconnect the monitor from the patient and turn off the monitor.

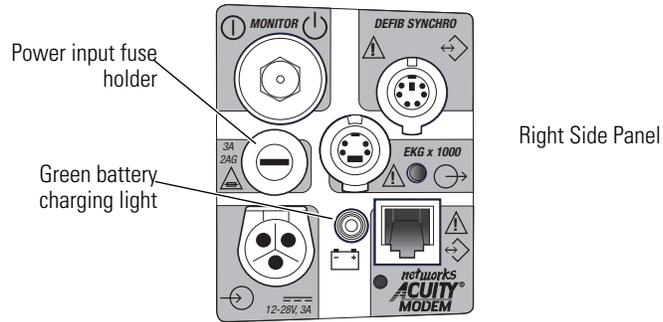

2. Disconnect the ac power adapter from the monitor.
3. Using a small, flat-blade screwdriver, turn the fuse carrier counterclockwise to release it.
4. Remove the fuse carrier and replace the fuse with the type 3A/250V, 2AG.

Install printer paper



WARNING Use only accessories approved by Welch Allyn. Visit www.welchallyn.com. The use of any other accessories can result in inaccurate patient data, can damage the equipment, and can void your product warranty.




Caution Use only low-debris printer paper approved by Welch Allyn. Use of other paper can cause unclear printing of patient data, printhead damage, and eventual printer failure. Store all paper (including a monitor loaded with paper) in compliance with paper storage specifications (“[Printer specifications](#)” on page 113).

1. Lay the monitor on its back to gain access to the bottom of the printer.
2. Squeeze the locks on the paper door and pull out to open it.
3. Lift the paper roll from the holder and pull out any paper remaining in the printer.
4. Place the new paper roll onto the spindle on the door as shown, and pull out several inches of paper.



5. Slide the end of the paper into the printer slot until it extends out the side.
6. Close the paper door and turn the monitor upright.
7. Simultaneously press the **START/STOP** and **PRINT TRENDS** button. Confirm the monitor prints a test print similar to the following:

VER. 2.02.00
52 5C 65

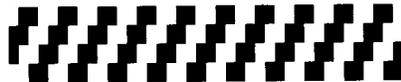

Inspect and clean the monitor and accessories

Before cleaning, thoroughly inspect the monitor and all accessories for any signs of damage, cracks, or improper mechanical function of keypads, switches, connectors, and printer paper door. While gently bending and flexing cables and tubing, inspect for damage, cracks, cuts, abrasions, extreme wear, exposed wires or bent connectors. Confirm connectors securely engage. Report damage or improper function to your service department.

Equipment	Cleaning Instructions	Approved Cleaning Solutions
Propaq CS Monitor ^{a b}	Wipe with a nearly-dry cloth moistened with cleaning solution. Thoroughly wipe off any excess cleaning solution. Do not let water or cleaning solution run into connector openings or crevices. ^c	Warm water, Coverage, Liquid soap, Wex-cide ^b , T.B.Q. ^b , Windex, Hydrogen peroxide solution, Ovation
NIBP cuff	Consult manufacturer's instructions.	Consult manufacturer's instructions.
Cables, tubing, CO ₂ sensor ^d	Wipe gently with cloth dampened with cleaning solution. Do not immerse the CO ₂ sensor in liquid.	Mild detergent solution; also consult manufacturer's instructions.
Masimo [®] SpO ₂ cables Nellcor [®] SpO ₂ cables	Consult manufacturer's instructions.	Consult manufacturer's instructions.
Other accessories	Consult manufacturer's instructions.	Consult manufacturer's instructions.

- Do not use these cleaning solutions (they may damage the monitor): Butyl alcohol, Denatured ethanol, Freon, Mild chlorine bleach solution, Isopropyl alcohol, Trichloroethane, Trichloroethylene, Acetone, Vesphene II, Enviroquat, Staphene, Misty, Glutaraldehyde, Fantastik, Formula 409, Cidex.
- The monitor may be disinfected to comply with OSHA requirements for cleaning and decontaminating spills of blood and other body fluids. (Federal OSHA Standard on bloodborne pathogens: 29 CFR 1910.1030, 12/6/91.) Wex-cide (Wexford Labs, Inc., Kirkwood, MO) and T.B.Q. (Calgon Vestal Lab., Calgon Corp., St. Louis, MO) are disinfectants that meet OSHA requirements, and are EPA approved. Wipe away disinfectants with a water-dampened cloth after the manufacturer's recommended period of time.
- If liquid gets into the right side panel connectors, it will drain out. If moisture gets into a left side panel connector, dry the connector with warm air, then check the monitoring functions for proper operation.
- The mainstream CO₂ sensor may also be disinfected with Wex-cide. Follow the disinfectant manufacturer's instructions. Do not leave Wex-cide on sensor longer than 30 minutes. Thoroughly clean off residue with water-dampened cloth. Prolonged exposure of the sensor to Wex-cide will damage the sensor.



Caution Do not autoclave the Propaq CS monitor or its accessories. Do not immerse the monitor in liquid when cleaning. Do not immerse accessories in liquid when cleaning unless the accessory manufacturer's cleaning instructions explicitly instruct you to do so.

Service interval recommendations

At the intervals recommended below, qualified biomedical service personnel should service the Propaq CS monitor. Service information is described in *Propaq CS Service Manual* (810-2708-XX).

Recommended Interval ^a	Service Action
Six months to two years	Complete functional verification; see <i>Propaq CS Service Manual</i> Inspect the monitor for mechanical and functional damage Inspect safety labels for legibility Inspect the side panel fuse for compliance to specified rating Verify that visual and acoustic alarms are functioning properly Test patient leakage current according to IEC 601-1/1988 Test patient leakage current with mains voltage on patient-applied parts according to IEC 601-1/1988: limit 50 μ A ^b
Minimum every three years	Check battery capacity

- More frequent service may be needed in extreme environments (heat, cold, dust, etc.).
- The leakage current should never exceed the 50 μ A limit. The data should be recorded in an equipment log. If the device is not functioning properly or fails any of the above tests, do not attempt to repair the device. Please return the device to the manufacturer or to your distributor for any required repairs.

Recycling monitor components

Within the European Union

Do not dispose of this product as 'unsorted municipal waste'. Prepare it for reuse or separate collection as specified by Directive 2002/96/EC of the European Parliament and the Council of the European Union on Waste Electronic and Electrical Equipment (WEEE).



Note If the monitor or the battery is contaminated, this directive does not apply.

For more specific disposal information, see www.welchallyn.com/weee, or contact Welch Allyn Customer Service.

Outside the European Union

When the monitor or the battery reaches end of life, recycle it locally according to national, state, and local regulations, or return it to Welch Allyn.

Extended storage precautions



Caution If a Propaq CS monitor has a battery installed or ac power connected and is stored for an extended period of time without use, the printer paper can cause damage to the printhead. Before storing a Propaq CS monitor for more than two months without use, remove the roll of printer paper.

Caution Storing the Propaq CS monitor for extended periods (more than three months) without being connected to the ac power adapter can cause damage to the battery. Even when the monitor is turned off, a very small amount of current is drawn from the battery. For long-term storage, remove the battery from the monitor. Battery removal is described in *Propaq CS Service Manual*.

Removing the battery will erase all stored Custom patient mode settings. See “[Customize Patient Mode settings](#)” on page 20 to reprogram Custom patient mode settings.

Change the Wireless Propaq CS network name


This procedure allows you to change the network name assigned to the Wireless Propaq CS (as long as the current network name is one of the pre-set names available in the Wireless Propaq CS Network Name Menu).

Note Changing the network name will cause the monitor to re-start and seek to connect with the FlexNet network corresponding to the new name. Do not attempt to change the network name unless you are a qualified biomedical service engineer or technician.

To change the network name:

1. From the Main Menu press **SETUP, MORE, MORE, SERVICE, YES, MORE, MORE, RADIO, CHANGE NET NAME** to access the Change Net Name screen.

The current network name is highlighted.

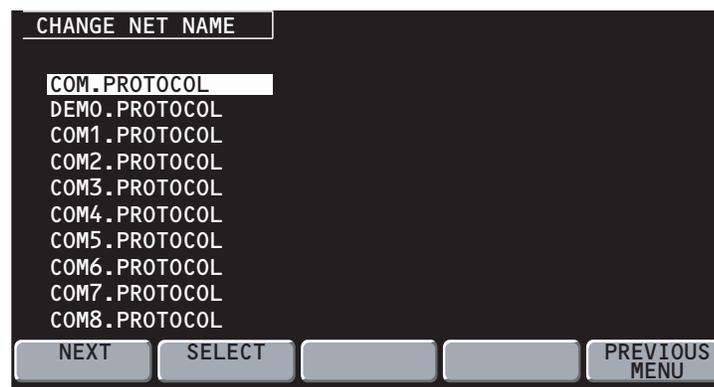

Note If the current Wireless Propaq CS network name is a custom (not pre-set) name, you cannot change the name from the Change Net Name screen (**CHANGE NET NAME** is not displayed). Contact Welch Allyn Technical Support for assistance.

2. Press **NEXT** as needed to scroll down and highlight the desired network name, then press **SELECT**.

The monitor displays a confirmation screen asking you to confirm that you want to change the network name.


- If you press **YES**, the monitor automatically turns itself off, then turns on and seeks to connect to a FlexNet network with the new network name.
- If you press **NO**, the monitor displays the Change Net Name screen again.

8

Reference

Set the time and date

1. From the Main Menu press **SETUP, MORE, MORE** to display the Time/Day window:

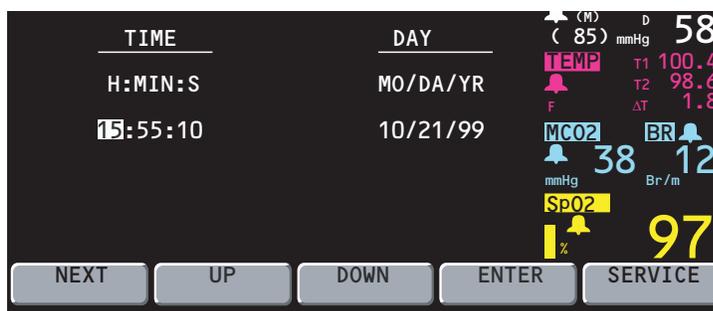

2. Press **NEXT, UP,** and **DOWN** as needed to set the time and date. Then press **ENTER** to store the new time and date.

Time/Day settings and trends



WARNING Changing the hour/minute/second setting for the monitor in the Time/Day window can cause the monitor to erase previously stored patient trend data.


When you change the hour/minute/second setting for the monitor in the Time/Day window, the monitor deletes any patient trend data that is older than five hours for non-NIBP trends or older than eight hours for NIBP trends according to the new clock setting.

However, if the monitor has not yet stored the full capacity of trends and you change the hour/minute/second setting to a time that is within the stored trend period, previously stored trends are not erased.

Changing the day, month, or year setting does not affect the stored patient trends.

Change the date format, ECG filter, and units

1. Make sure you are in the Adult patient mode (from the Main Menu press **SETUP, MORE, CHANGE, ADULT, YES**).
2. From the Main Menu press **SETUP, MORE, MORE, SERVICE, YES** (to access the Service Menu), **MORE, MORE, SETTINGS**. The monitor displays the Settings window:

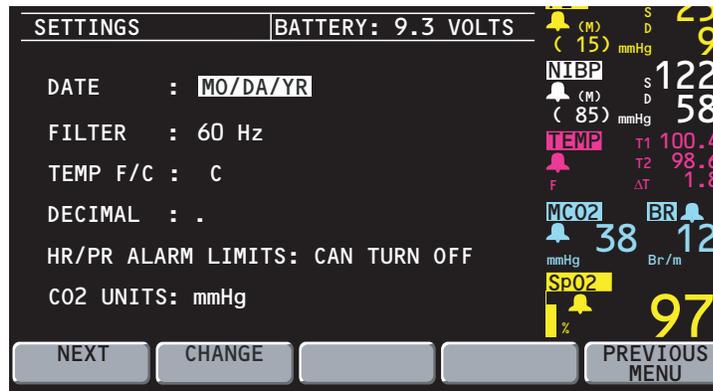

3. Press **NEXT** and **CHANGE** to select the desired settings.

DATE	Sets the date format: Month/Day/Year, Day.Month.Year, or Year/Month/Day.
FILTER	Sets the ECG filter frequency: 60 Hz, 50 Hz, or OFF. Make sure it is set to your ac mains frequency.
TEMP F/C	Sets the temperature display units: Fahrenheit or Celsius. Changing units does not erase the TEMP trends.
DECIMAL	Sets the decimal character as either a period (.) or a comma (,).
HR/PR ALARM LIMITS	Allows or prohibits turning off the HR/PR alarm limits. If CANNOT TURN OFF is selected, the ON/OFF key is not displayed for HR/PR in the Alarm Limits Menu.
CO ₂ UNITS	Sets the CO ₂ display units as mmHg, kPa, or percent (%). Changing units erases the CO ₂ trends and changes CO ₂ alarm limit settings to the factory default settings for the currently-used patient mode.

Note Any time you change the Date, Filter, Temp F/C, Decimal, HR/PR Alarm Limits (CAN or CANNOT TURN OFF), or CO₂ Units setting, the new setting also becomes the powerup default setting.

Factory default settings

The monitor is shipped from the factory with these preset default settings. For information about how to customize your monitor settings, see [“Change Powerup Patient Mode or store customized settings”](#) on page 19.

Setting	Factory default
Date ^a	MO/DA/YR. This setting is automatically updated whenever it is changed during use (continuously programmed).
Decimal ^d	. (Period) This setting is automatically updated whenever it is changed during use (continuously programmed).
HR/PR Sweep	25 mm/s
RR/BR Sweep	6.25 mm/s
Alarm Tone	MEDIUM
HR/PR TONE	LOW
HR/PR SOURCE	ECG
RR/BR Source	CO ₂ if available or ECG (not programmable)
Patient Mode	Adult
Display Brightness	Normal
ECG Bandwidth	Monitor
ECG Size	1 mV/cm
ECG1 Lead	II
ECG2 Lead	V
ECG Filter ^d	60 Hz. This setting is automatically updated whenever it is changed during use.
ECG Pacer	ON
RESP size	2X
RESP lead	Ld2
RESP sweep	6.25 mm/s
RESP monitoring	ON
RESP window	ON
IBP Range	0 to 180 mmHg
IBP Rescale	0 to 140 mmHg (not programmable)
IBP Mode	RESCALE
Invasive Pressure Formats	Label dependent
NIBP Mode	MANUAL
NIBP Auto Time	15 min
NIBP Smartcuf	ON
SpO ₂ SIZE	2x
SpO ₂ C-LOCK	OFF
SpO ₂ Response	NORMAL
TEMP F/C ^d	Celsius
CO ₂ Range	0 to 60 mmHg
CO ₂ Sweep	6.25 mm/s
CO ₂ Response	NORMAL
CO ₂ Units ^d	mmHg
CO ₂ Gas Compensation	OFF
Sidestream CO ₂ Flow Rate	Adult: 90 ml/minute Ped: 90 ml/minute Neonate: 90 ml/minute (The flow rate cannot be programmed to a different value in a Custom Patient Mode, see “Change Powerup Patient Mode or store customized settings” on page 19.)
Display Wave Select	Adult and Pediatric Patient Mode: ECG1, ECG2, P1, P2, and CO ₂ = ON, and large NIBP numerics are displayed (in order of priority); SpO ₂ and RESP = OFF. Neonatal Mode: all waveforms are ON and large NIBP numerics are displayed (in order of priority).
Trend Group	NIBP
Alarm Limits	All are ON except P2
HR/PR Alarm Limits ^d	CAN TURN OFF

Setting	Factory default
HR Limits	Adult: 50, 120 beats per minute Ped: 50, 150 beats per minute Neonate: 100, 200 beats per minute
NIBP Limits - Systolic	Adult: 75, 220 mmHg Ped: 75, 145 mmHg Neonate: 50, 100 mmHg
NIBP Limits - Diastolic	Adult: 35, 110 mmHg Ped: 35, 100 mmHg Neonate: 30, 70 mmHg
NIBP Limits - Mean	Adult: 50, 120 mmHg Ped: 50, 110 mmHg Neonate: 35, 80 mmHg
P1, P2 Limits - Systolic	Adult: 75, 220 mmHg Ped: 75, 145 mmHg Neonate: 50, 100 mmHg
P1, P2 Limits - Diastolic	Adult: 35, 110 mmHg Ped: 35, 100 mmHg Neonate: 30, 70 mmHg
P1, P2 Limits - Mean	Adult: 50, 120 mmHg Ped: 50, 110 mmHg Neonate: 35, 80 mmHg
SpO ₂ Limits	Adult: 90%, 100% Ped: 90%, 100% Neonate: 85%, 95%
RR/BR	Adult: 5, 30 Br/M Ped: 10, 45 Br/M Neonate: 10, 75 Br/M
TEMP Limits	35.0°, 37.8° C
ΔT Limits	0.0°, 2.8° C
ETCO ₂ Limits	25, 60 mmHg (3.0 and 8.0 for % and kPa)
INCO ₂ Limits	N/A, 5 mmHg (0.7 for % and kPa)
Apnea Delay	Adult/Ped: 20 seconds Neonate: 15 seconds
Printer Settings	
Printer Alarm Print	OFF
Printer Auto Print	OFF
Printer NIBP Ticket	OFF
Printer Apnea Ticket	ON
Printer Print Speed	25 mm/s
Printer Auto Trend	OFF
Printer Trend Selections	NIBP and P1 = ON; all others = OFF
Printer OxyCRG on Alarm	OFF

- a. Any time you change the Date, Filter, Temp F/C, Decimal, HR/PR Alarm Limits (Can or Cannot Turn Off) or CO₂ Units setting, the new setting also becomes the powerup default setting.

Specifications

ECG specifications

The ECG channel meets all the requirements for Cardiac Monitors Heart Rate Meters and Alarms specified ANSI/AAMI EC13-1992, except for Standardizing Voltage (section 3.2.9.9). The channel also meets the American National Standard, Safe Current Limits for Electromedical Apparatus (ANSI/AAMI ES1-1993).

Characteristic	Specification
Connector	AAMI 6 pin or Hewlett-Packard compatible 12-pin style connector (optional).
Selectable Leads	I, II, III, aVR, aVL, aVF, V
Lead Fault Indicator	LA, LL, RA, RL, C, multiple
ECG Size (sensitivity) in mV/cm	4, 2, 1, 0.5, 0.2
Display Sweep Speeds	12.5, 25, and 50 mm/s
QRS Tone Volume	High, Low, Medium, Off
QRS Tone Frequency	900 Hz for Propaq CS monitor without Expansion Module, 665 Hertz when equipped with SpO ₂ but SpO ₂ not being monitored; variable pitch with SpO ₂ option and SpO ₂ being monitored
Bandwidth: MONITOR	Adult Mode:0.5 to 40 Hz Pediatric Mode:0.5 to 120 Hz Neonatal Mode:0.5 to 120 Hz
EXTENDED	Adult Mode:0.05 to 40 Hz Pediatric Mode:0.05 to 120 Hz Neonatal Mode:0.05 to 120 Hz (see Real-Time ECG Analog/Defib Sync specification)
Sample Rate	364 Hz
Input Protection	Electrosurgery and defibrillator protected when used with specified ECG cables. All models also include electrosurgery interference suppression.
Lead Fail Sense Current	50 nA dc for active leads 100 to 200 nA dc for driven lead, depending on number of electrodes attached
Tall T-wave Rejection	Meets AAMI (USA) EC13-1992, section 3.1.2.1.c, for 1.2 mV T-wave and 1 mV QRS using AAMI test waveform.
Common Mode Rejection	<1 mV p-p RTI for 10V rms, 50/60 Hz input, 200 pF source impedance, input unbalanced, FILTER function OFF <0.1 mV p-p RTI for 10V rms, 50/60 Hz input, 200 pF source impedance, input unbalanced, FILTER function ON
Input Impedance	>2.5 M Ω differential @ 60 Hz
Input Range (ac)	10 mV peak to peak
Input Range (dc)	Up to \pm 300 mV
System Noise	\leq 30 μ V peak-to-peak, RTI, with all inputs = 47K in parallel with 0.047 μ F.

Characteristic	Specification
QRS Detector	Adult or Pediatric Amplitude Range: 0.22 to 5.0 mV (RTI) Neonatal Amplitude Range: 0.1 to 5.0 mV (RTI) Neonatal and Pediatric Width Range (Duration): 40 to 120 ms Adult Width Range (Duration): 70 to 120 ms
Heart Rate Range	25 to 350 beats per minute (measurement) 25 to 300 beats per minute (display)
HR/PR Alarm Limits Range	25 to 250 beats per minute
Heart Rate Meter Response Time	Responds to change in heart rate within 5 to 9 seconds depending on physiological waveform. (As measured per AAMI standard EC 13-1992 clause 4.1.2.1 (f), including 3.1.2.1 parts f. and g. waveforms.) Includes 1 second readout update interval.
HR Accuracy	±3 beats per minute or 3%, whichever is greater NOTE: AAMI Test 4.1.4 part f: Accuracy is affected (i.e., rate drops) when QRS and pacer spikes are nearly simultaneous as occasionally is the case during this AAMI test.
Heart Rate Averaging Method	Heart rate = 60 / latest average interval in seconds. For higher heart rates, latest average interval = 7/8 of previous average interval + 1/8 of latest interval. For lower heart rates, latest average interval = 3/4 (previous average interval) + 1/4 latest interval. Transition rates for choice of formula include hysteresis and are 70 and 80 beats per minute.
Drift Tolerance (AAMI Specification EC13-1992, 3.2.6.3)	80 beats per minute indicated for 80 beats per minute ECG plus drift waveform
Pacer Display	Pacer indicator shown on screen if PACER function turned on; pacer spike always shown if of sufficient amplitude.
Pacer Pulse Rejection	Pacer detection range (i.e., will show the dashed vertical marker) for 0.1 ms pulses is ±3 mV to ±700 mV, and drops linearly to ±2 mV to ±700 mV for 0.2 to 2 ms pulses. Will not count as heartbeats approximately 95% of pacemaker pulses within pacer detection range, with or without AAMI (EC13 1992) tails of 4, 25, 50, 75, or 100 ms decay time constant, whose tail amplitudes are 2.5% or 25%, 2mV maximum, whether ventricular only, or A-V sequential pulses, all per AAMI tests 3.1.4.1 and 3.1.4.2
Response to Irregular Rhythm (AAMI specification EC13-1992, 3.1.2.1. Part e.)	
Ventricular Bigeminy (VB)	78 to 81 bpm (80 bpm expected)
Slow Alternating VB	57 to 65 bpm (60 bpm expected)
Rapid Alternating VB	118 to 123 bpm (120 bpm expected)
Bidirectional Systole	88 to 93 bpm (90 bpm expected)
1mV Ventricular Tachycardia	197 to 198 bpm (206 bpm expected)
2mV Ventricular Tachycardia	193 to 197 bpm (206 bpm expected)

Real-Time ECG analog/defib sync specifications

Special cables are required to interface the defib sync connector to a Physio-Control LIFEPAK 5 or LIFEPAK 6s defibrillator. The sync and real-time ECG outputs do not operate during in-service mode.

Signal	Specification
Sync Output	0 to 5 V pulse, 100 ±5 ms wide, starts within 35 ms after peak of R-wave. 15 mA short circuit current.
Real-time ECG Output	Range = ±6 V minimum, centered about 0 V, Gain = 1000X, noninverting for lead II, inverting for all other leads, delay <3 ms, 0.05-100 Hz, going to -5.9 V ±5% during ECG lead fail. V lead has no Real-Time analog output.
Marker Input (Defib Sync only)	Normally 0 V in, a pulse either ±3 to ±15 V for 10-70 ms puts a marker in ECG trace. ~ 5 kΩ input resistance.
Shield	Common terminal for other signals

Impedance pneumography (RESP) specifications

Characteristic	Specification
Sweep speed	3.13, 6.25, 12.5 mm/s; user-selectable
Amplitude range	1x, 2x, 4x, 8x, 16x
Excitation signal characteristics	65 μA RMS ±5% at 63.0 kHz pseudo sine wave
Sensing electrodes	User selectable RA-LA or RA-LL
Base impedance (in addition to 1k $\frac{3}{4}$ resistors in ECG cables)	100 to 1200 ohms is normal monitoring range, approx. 1200-1500 ohms range produces a "NOISY SIGNAL, CHECK ELECTRODES" equipment alert. Above approx. 1500 ohms produces a "RESP FAULT, LEAD FAIL" equipment alert. Thresholds are dependent on ECG cable type.
Impedance dynamic range	20 ohms
Signal bandwidth after detection	0.06 Hz (single pole) to 3.2 Hz (2 pole)
Breath detection threshold	140 milliohms or 2x CVA, whichever is greater
Respiration rate range	Adult/Ped: 0 (apnea), 2 to 150 breaths/min Neonate: 0 (apnea), 3 to 150 breaths/min
Respiration rate accuracy	±2 breaths/min or ±2%, whichever is greater
Respiration rate source (RR)	When CO ₂ is active, CO ₂ is the BR source. Otherwise, RESP from ECG is the RR source.
Apnea alarm delay accuracy	+1 second
Resolution	5 seconds
Apnea alarm delay settings	Central apnea only - alarm delay is set by the user Adult/Ped = 6, 10, 15, 20, 25, 30 seconds Neonate = 6, 10, 15, 20 seconds
Cardiovascular artifact rejection (CVA)	Presence of CVA is detected automatically. Breaths will be picked in the presence of CVA unless the Breath Rate is within 5% of the Heart Rate or a sub-multiple of the heart rate.
Motion artifact rejection	not rejected
Obstructive apnea	not detected

Invasive pressure specifications

Characteristic	Specification
Transducer Type	Strain-gauge resistive bridge, or HP quartz (with HP Option). ^a
Transducer Excitation Impedance Range	200 to 2000 Ω
Transducer sensitivity	5 $\mu\text{V}/\text{V}/\text{mmHg}$
Excitation Voltage	4.85 V Pulsed dc @ 181 Hz ^b
Connector	ITT-Cannon plug MS3106F-14S-6P Std. Hewlett-Packard compatible 12-pin connector (optional).
Bandwidth	Digital filtered, dc to 20 Hz
Zero Drift	± 1 mmHg without transducer drift
Zero Adjustment	± 200 mmHg including transducer offset
Numeric Accuracy	± 2 mmHg or 2% of reading, whichever is greater, plus transducer error
Pressure range	-30 to 300 mmHg
Pulse range	25 to 250 beats per minute ^c
IBP Alarm Limit Ranges	All patient modes Systolic, Diastolic, Mean -30 to 300 mmHg
Leakage Current	Meets ANSI/AAMI risk (leakage) requirements
Electrosurgery interference suppression	Included in all models

a. Transducers with 40 $\mu\text{V}/\text{V}/\text{mmHg}$ sensitivity are not compatible.

b. Duty factor depends on transducer impedance. For 200 to $\sim 900 \Omega$, duty factor is $\cong 11\%$. Above $\sim 900 \Omega$, the duty factor increases to $\cong 91\%$.

c. At pulse rates exceeding 250 beats per minute, refer to the IBP waveforms on the display or printout to determine systolic and diastolic pressures.

NIBP specifications

Characteristic	Specification
Method	Oscillometric
Control	Automatic and manual measurement control
Auto Intervals	1, 2, 3, 5, 10, 15, 30, and 60 minutes
Turbocuf	Maximum measurements allowable in a 5-minute period
Displayed Pressures	Systolic, Diastolic, and Mean plus on-screen manometer
Systolic Range	Adult: 30 to 260 mmHg Ped: 30 to 160 mmHg Neonate: 25 to 120 mmHg
Diastolic Range	Adult: 20 to 235 mmHg Ped: 15 to 130 mmHg Neonate: 10 to 105 mmHg
Mean Range	Adult: 20 to 255 mmHg Ped: 15 to 140 mmHg Neonate: 10 to 110 mmHg
Static Manometer Accuracy	±3 mmHg
Minimum Inflation Pressure	Adult: 100 mmHg Ped: 80 mmHg Neonate: 50 mmHg
Maximum Allowable Pressure	Adult: 270 mmHg Ped: 170 mmHg Neonate: 132 mmHg
Default Inflation Pressure	Adult: 160 mmHg Ped: 120 mmHg Neonate: 90 mmHg
Normal Overpressure Limit (results in up to 2 retries)	Adult: 280 mmHg Ped: 200 mmHg Neonate: 141 mmHg
Single Fault Overpressure Limit	Adult: 308 mmHg Ped: 220 mmHg Neonate: 154 mmHg
Leak Rate	After a 1 minute settling period, leak rate is ≤4 mmHg over a 3-minute period at 270 mmHg.
Pulse Rate Range	30 to 220 beats per minute
NIBP Alarm Limit Ranges	Neonate: Systolic 25 to 120 mmHg Diastolic 10 to 105 mmHg Mean 10 to 110 mmHg Pediatric: Systolic 30 to 160 mmHg Diastolic 15 to 130 mmHg Mean 15 to 140 mmHg Adult: Systolic 30 to 260 mmHg Diastolic 20 to 235 mmHg Mean 20 to 255 mmHg
Maximum Determination Time (with retries)	Adult: 4.5 minutes Ped: 4 minutes Neonate: 3 minutes
Maximum Determination Time (no retries)	Adult: 3 minutes Ped: 2 minutes Neonate: 1.5 minutes
Typical Determination Time without Artifact	30 to 45 seconds
Minimum Time between automatic measurements	30 seconds (Auto Mode) 2 seconds (Turbo Mode)
Artifact Filtering	Smartcuf software algorithm (may be enabled or disabled; requires ECG monitoring). NIBP measurements can still be taken if Smartcuf is disabled.
Electrosurgery Interference Suppression	Included in all models.
NIBP Performance	Per EN 1060-1, EN 1060-3 and ANSI/AAMI SP10-1992
NIBP Safety	Per EN 60601-2-30

Temperature specifications

Characteristic	Specification
Range	0° to +50°C; 32° to +122°F
Displays	T1, T2, and ΔT
Probes	Compatible with YSI Series 400 and 700 probes. HP side panel only compatible with YSI 400 and has HP connector.
Units	°C and °F selectable
Channel Accuracy	Temperature RangeTolerance 0° to +10°C±0.2°C >10° to +50°C±0.1°C +32° to +50°F±0.4°F >50° to +122°F±0.2°F
Resolution	0.1°C or °F
Temperature Alarm Limit Range (T1, T2)	32.0° to 122.0°F 0° to +50.0°C
Electrosurgery interference suppression	Included in all models.

Pulse oximetry (SpO₂) specifications

Pulse Oximetry (SpO₂) Specifications for Masimo SpO₂

Characteristic	Specification
Saturation (% SpO ₂)	
Range	1% to 100%
Resolution	1%
Alarm Limits	52% to 100% (upper) 50% to 98% (lower)
Probe Accuracy (25° to 41° C)	
Adults, Pediatrics: No motion	70% to 100% ±2 counts 0% to 69% unspecified
Neonates: No motion	70% to 100% ±3 counts 0% to 69% unspecified
Adults, Pediatrics, Neonates: During Motion ^{a,b}	70% to 100% ±3 counts 0% to 69% unspecified
Pulse Rate	
Range: No motion	26 to 239 beats per minute, ±3 counts
Range: During motion ^{a,b}	26 to 239 beats per minute, ±5 counts
Resolution	1 beat per minute
Alarm Limits	27 to 250 beats per minute (upper) 25 to 248 beats per minute (lower) Note: Any pulse rate above 239 will activate the pulse rate alarm, even if the upper alarm limit is set above 239. If the lower alarm limit is set to 25, a pulse rate of 25 will activate the pulse rate alarm due to the limitation of the displayable numeric range.
Pulse Rate Accuracy	
No Motion	±3 beats per minute
During Motion ^{a,b}	±5 beats per minute
Measurement averaging time	8 seconds
Alarm Hold-Off Time Period	10 seconds; resets if the sensor reports levels within limits before 10 seconds elapses
Circuitry	Microprocessor controlled Automatic self-test of oximeter when powered on Automatic setting of default parameters Automatic alarm messages
Electrosurgery interference suppression	Yes
Sensor Compatibility	Visit www.welchallyn.com
Sensor LEDs	
RED Wavelength	660 nm (nominal)
INFRARED Wavelength	905 nm (nominal)
Sensor Energies (Radiant Power)	0.13 mW to 0.79 mW at 50 mA pulsed

- a. Motion for adults and pediatrics is defined as rubbing and tapping motions at 2 to 4 Hz at an amplitude of 1 to 2 cm and a non-repetitive motion between 1 to 5 Hz at an amplitude of 2 to 3 cm in induced hypoxia studies in the range of 70-100% SpO₂ against a laboratory co-oximeter and ECG monitor. This variation equals ±1 standard deviation which encompasses 68% of the population.
- b. Motion for neonates is defined as foot motions at 2 to 4 Hz at an amplitude of 1 to 2 cm against a laboratory co-oximeter and ECG monitor. This variation equals ±1 standard deviation which encompasses 68% of the population.

Pulse oximetry (SpO₂) specifications for Nellcor SpO₂

Characteristic	Specification^a
Saturation (% SpO ₂)	
Range	0% to 100%
Resolution	1%
Alarm Limits ^b	52% to 100% (upper) 50% to 98% (lower)
Probe Accuracy ^c (saturation levels between 70% and 100%, 28° to 42°C)	
Adult/Pediatric	Digit accuracy: ±2 counts
Neonatal	Digit accuracy: ±3 counts
Pulse Rate	
Range	25 to 250 beats per minute
Alarm Limits	27 to 250 beats per minute (upper) 25 to 248 beats per minute (lower)
Pulse Rate Accuracy	±3 beats per minute
Alarm Hold-Off Time Period	10 seconds; resets if the sensor reports levels within limits before 10 seconds elapses
Circuitry	Microprocessor controlled Automatic self-test of oximeter when powered on Automatic setting of default parameters Automatic alarm messages
Electrosurgery interference suppression	Yes
Sensor Compatibility	Visit www.welchallyn.com
Sensor LEDs	
RED Wavelength	660 nm (nominal)
INFRARED (IR) Wavelength	890 nm (nominal)
Sensor Energies (Radiant Power)	
Electrical Power	52.5 mW max.
Optical Power	15 mW max.

- a. When performing SpO₂ monitoring, the monitor SpO₂ channel may not (in rare instances) initially detect a damaged SpO₂ sensor or extension cable (as described in EN865:1997, Section 51.109), but may simply display a flat line and no numerics. If this occurs, try connecting a new SpO₂ sensor (and extension cable, if needed) and restart the monitor to resume normal monitoring.
- b. Minimum difference between upper and lower alarm limits is 2%.
- c. Refer to www.welchallyn.com for accuracy specifications for all Nellcor SpO₂ probes recommended for use.

Capnography (CO₂) specifications

General CO₂ Specifications (mainstream CO₂ and sidestream CO₂)

Characteristic	Specification
CO₂ Display	
Screen Display	CO ₂ waveform and ET/CO ₂ and INCO ₂ (when in alarm) numerics
Numeric Display Ranges	ET/CO ₂ : 0-99 mmHg, 0-13.2 kPa, 0-23.1% INCO ₂ : 8 ^a -25 mmHg, 1.1 ¹ -5 kPa, 1.1 ¹ -5%
Waveform Scale (Maximum)	0-100 mmHg, 0-14 kPa, 0-14%
Units	mmHg, kPa, %; user-selectable
Sweep Speed	3.13, 6.25, 12.5 mm/s; user-selectable
Response Modes	Fast: 15 s sampling time period Normal: 30 s sampling time period Slow: 45 s sampling time period
Gas Compensation	OFF: CO ₂ value = calculated CO ₂ value; O ₂ > 50%, No N ₂ O: CO ₂ value = calculated CO ₂ value x 1.03; N ₂ O > 50%: CO ₂ value = calculated CO ₂ value x 0.952
Alarm Limit Ranges	ET/CO ₂ : 0-99 mmHg, 0-13.2 kPa, 0-13.2% INCO ₂ : 2-25 mmHg, 0.2-5 kPa, % (no lower limit)
Resolution	1 mmHg
Accuracy	Mainstream ^b : 0-30 mmHg, ±3 mmHg 31-99 mmHg, ± 10% of value Sidestream ^c : 0-30 mmHg, ±3 mmHg 31-99 mmHg, ± 10% of value
Altitude Error	±0.4%/1,000 ft (304.8 m)
Breath Rate Display	
Screen Display	Numeric
Breath rate (BR) source	When CO ₂ is active, CO ₂ is BR source. Otherwise, RESP from ECG is RR source.
Units	Breaths/Minute
Range	Adult/Ped: 0 (apnea), 2 to 150 breaths/min Neonate: 0 (apnea), 3 to 150 breaths/min
Resolution	±1 breaths/min
Accuracy	±1 breaths/min or ±5%, whichever is greater ^d
Alarm Limits Range	Adult/Ped: 2 to 150 breaths/min Neonate: 3 to 150 breaths/min
Apnea Alarms and Tickets	
Apnea Ticket	Set to auto print after apnea event and after 1 minute continued apnea
Apnea Alarm Accuracy	± 2 s
Apnea delay setting	Adult/Ped = 6, 10, 15, 20, 25, 30 seconds Neonate = 6, 10, 15, 20 seconds
Barometric Pressure	
Pressure Compensation	Automatic
Operating Range	-2,000 to 15,000 ft (-610 to 4572 m) 817 to 429 mmHg
Screen Display	Numeric (CO ₂ Status Window)
Units	mmHg, kPa, or %
Accuracy	±3 mmHg or 2.5% of difference from calibration pressure, whichever is greater
CO₂ Performance	
Specification	Per ISO 9918:1993 (E) / EN 864:1996

a. Lower if in alarm.

b. Based on these airway conditions: sensor temperature = 42°C, airway adapter temperature = 33°C, water vapor pressure = 38 mmHg; standard gas mixture = CO₂ in balance air, fully hydrated at 33°C; barometric pressure = 760 mmHg and flow = 60 ml/min.

c. Based on the following additional airway conditions: Sample line = 7 ft, 0.055 in ID (2.13 m, 1.4 mm ID); Sample flow rate = 175 ml/min; Welch Allyn watertrap (new/unused); Respiratory rate ≥50 bpm, stable to ±3 breaths/min; Inspired/Expired time ratio = 1:2; Barometric pressure = 760 mmHg.

d. For sidestream CO₂, this applies only for BR≥50.

Mainstream CO₂ specifications

Characteristic	Specification
Mainstream CO₂ sensor	
Sensor Type	Mainstream
Principle of Operation	Non-dispersive, infrared, single-beam, single path/wavelength, ratiometric
Warm-up time (CO ₂ sensor and monitor)	45 s typical, 3 min maximum
Response Time	30 ms typical, 60 ms maximum
Waveform Rise Time	<120 ms to 90% after step change
Calibration	Verify semi-annually, calibrate only as required
Sensor Housing Temperature	42°C nominal
Mainstream CO₂ sensor and cable dimensions and weight	
Sensor Height ^a	1.003 in (2.548 cm)
Sensor Width ¹	1.036 in (2.631 cm)
Sensor Depth ¹	0.78 in (1.981 cm)
Sensor Weight ¹	< 0.53 oz (15.03 g)
Cable Length	10 ft (3.05 m) nominal
Mainstream CO₂ airway adapter	
Type	Per ISO 3040, single-use
Size	15 mm ID, (meets ISO specifications)
Material	clear polycarbonate, with sapphire windows
Added Deadspace	< 6cc (0.37 cubic inches) for adult model, <0.6 cc (0.037 cubic inches) for low deadspace model
Mainstream CO₂ sensor environmental specifications	
Operating Ambient Temperature	10° to 40°C
Storage Temperature	-20° to 60°C
Operating Altitude	-2,000 to 15,000 ft (-610 to 4,572 m), 817 to 429 mmHg
Storage Altitude	-2,000 to 40,000 ft (-610 to 12,192 m), 817 to 141 mmHg
Operating and Storage Humidity	0% to 95%, noncondensing
Shock	100 g for 4 ms
Vibration	5-35 Hz, 0.015 in (0.038 cm) peak-to-peak, 35-100 Hz, 1 g acceleration
Drop	36 inches free fall to floor (tile over concrete, one drop each face, one drop each edge/corner)

a. Not including cable

Sidestream CO₂ specifications

Characteristic	Specification
Sensor Type	Sidestream, internal
Principle of Operation	Non-dispersive, infrared, single-beam, single path/wavelength, ratiometric
Operating Ambient Temperature	5° to 40°C
Startup Time	30 seconds typical, 3 minutes maximum
Rise Time	240 ms (10% to 90%) at 175 ml/min
Delay Time	1.12 seconds maximum ^a
Total System Response Time	1.36 seconds maximum (Rise Time and Delay Time)
Calibration	Verify semi-annually, calibrate only as required
Sampling Chamber	Internal (replaceable by service technician)
Pneumatic and Exhaust System	Integral
Barometric Pressure Compensation	Automatic
BTPS, ATPS, STPD ^b	CO ₂ value = calculated CO ₂ value x 0.977
Sampling Line	7-foot sampling line, ID 0.055 in (1.4 mm), for use with disposable single-use cannula (CO ₂ only or CO ₂ sampling/O ₂ delivery)
Watertrap	Disposable single-use
Flow Rate	90 or 175 ml/min, user-selectable

- a. Based on the following additional airway conditions: Sample line = 7 ft, 0.055 in ID (2.13 m, 1.4 mm ID); Sample flow rate = 175 ml/min; Welch Allyn watertrap (new/unused).
- b. BTPS (Body Temperature and Pressure, Saturated), ATPS (Ambient Temperature and Pressure, Saturated), STPD (Standard Temperature and Pressure, Dry).

Alarms specifications

Characteristic	Specification
Indicators	Flashing red alarm bell light: patient alarm(s) Continuously on red alarm bell light: patient alarm silenced or suspended Flashing yellow crossed-alarm light: equipment alert Continuously on yellow crossed-alarm light: one or more patient alarm limit(s) off
Tone Frequency	900 Hertz
Tone Patterns	Apnea: 1 second on, 1 second off Patient alarm: 1 second on, 2 seconds off Equipment alert: 1 second on, 4 seconds off
Selectable Tone Volume	Low, Medium, High
Limits	Settable on all parameters
Control	Automatic preset or manual settings
Alarm Priority	Highest priority: Apnea, then patient alarms Lowest priority: Equipment alerts
Alarm on Tachycardias	Most tachycardias will alarm in less than 8 seconds. These include AAMI 3.1.2.1 part f. waveforms. Certain multifocal tachycardias may initially alarm as "low rate."
Apnea delay setting	Adult/Ped = 6, 10, 15, 20, 25, 30 seconds Neonate = 6, 10, 15, 20 seconds
Alarm Holdoff Time Period ^a	HR/PR = 3 seconds (except NIBP PR) SpO ₂ = 10 seconds RR/BR = 5 seconds
Audio Alarm Holdoff with Acuity	When a Propaq CS monitor in Adult or Pediatric Mode is connected to an Acuity System, the audio alarms at the bedside monitor can be delayed up to 4 minutes and 15 seconds. The delay time is selected in Acuity software at the time of Acuity installation. Visual alarm indications and Nurse Call alarm are not delayed.

- a. To help minimize false alarms, the monitor briefly delays or "holds off" triggering alarms for limit violations for these vital signs. After the alarm holdoff period begins, if the monitor detects that the patient's vital sign has returned to acceptable limits, the monitor cancels the alarm holdoff. The next time a vital sign limit is violated, the monitor starts a new holdoff period.

Nurse Call specifications

Characteristic	Specification
Maximum switch current	1 A
Maximum switch voltage	30 V ac/dc
Isolation	1500 Vrms
Alarm relay	Energized during apnea alarm or patient alarm ^a
Customized cable (008-0634-XX)	One end is a 4-pin plug compatible with the monitor Nurse Call connector; the other end must be customized to connect to the local Nurse Call system.

- a. Pressing the **Silence/Reset Alarm** key or **SILENCE** silences the Nurse Call alarm for 90 seconds.

Nurse Call cable specifications

This cable (008-0634-XX) must be customized by a biomedical technician to connect to the local Nurse Call system.

Trends specifications

Characteristic	Specification ^a
Model 242 Parameters	NIBP, T1, T2, ΔT , HR (heart rate/pulse rate), SpO ₂ , End-tidal CO ₂ , Inspired CO ₂ , Breath Rate/Resp Rate
Model 244 Parameters	NIBP, P1, T1, T2, ΔT , HR (heart rate/pulse rate), SpO ₂ , End-tidal CO ₂ , Inspired CO ₂ , Breath Rate/Resp Rate
Model 246 Parameters	NIBP, P1, P2, T1, T2, ΔT , HR (heart rate/pulse rate), SpO ₂ , End-tidal CO ₂ , Inspired CO ₂ , Breath Rate/Resp Rate
Duration	5 hours for non-NIBP trends (up to 150 readings) A maximum of 128 readings (up to 8 hours) for NIBP trends
Resolution	All channels except NIBP sample data at 2-minute intervals. For NIBP trends, a new entry is placed in the table each time an NIBP determination is made.

a. Assumes SpO₂ and CO₂ functions are present.

Display specifications

Characteristic	Specification
Type	Color active matrix; TFT (Thin Film Transistor) LCD module
Resolution	640 x 480 pixels; 1 pixel = R + G + B dots
Active Viewing Area	6.73 x 5.10 inches (170.9 x 129.6 mm)
Pixel Pitch	0.0105 inches (0.267 mm)
Viewing Angle	U/D 40°, R/L 60° (typical), S 10:1 contrast ratio
Contrast Ratio	150:1 (typical); measured in dark room at center of screen
Display Color	18-bit (6 bits per primary color)
Luminance	200 cd/m ² (typical); measured at saturation point
Response Time	40 ms (maximum); "white to black"

Wireless Propaq CS Radio specifications (5 GHz)

Characteristic	Specification
FlexNet™ Network	5 GHz orthogonal frequency division multiplexing (OFDM) wireless local area network (WLAN) and 10/100/1000 base-T Ethernet network
Modulation	OFDM
Output power	40 mW maximum; country-dependent
IEEE standards	802.11a, 802.11e, 802.11h, 802.1X
Monitors per access point	20 (max.)



Caution Some countries restrict the use of 5-GHz bandwidths. The 802.11a radio in the Propaq CS monitor uses only the channels indicated by the access point with which the radio associates. The hospital IT department must therefore configure all associated access points to operate within approved domains.

Channel restrictions in the 5-GHz band, by country, are as follows:

Restrictions for use in the 5 GHz bands ^a		
Allowed frequency bands ^b	Allowed channel numbers ^c	Countries
5.15 to 5.25 GHz	36, 40, 44, 48	Austria
5.15 to 5.35 GHz	36, 40, 44, 48, 52, 56, 60, 64	Cyprus, Czech Republic, France, Hungary, Slovakia
5.15 to 5.35 GHz and 5.470 to 5.725 GHz	36, 40, 44, 48, 52, 56, 60, 64, 100, 104, 108, 112, 116, 120, 124, 128, 132, 136, 140	Belgium, Bulgaria, Denmark, Estonia, Finland, Germany, Greece, Iceland, Ireland, Italy ^d , Latvia, Liechtenstein, Lithuania, Luxembourg, Malta, Netherlands, Norway, Poland, Portugal, Slovenia, Spain, Sweden, Switzerland, U.K.

- This device may not be operated outdoors when using the bands 5150 to 5350 MHz (Channels 36, 40, 44, 48, 52, 56, 60, 64).
- This device must be used with Access Points that have employed and activated a radar detection feature required for European Community operation in the 5GHz bands. This device will operate under the control of the Access Point in order to avoid operating on a channel occupied by any radar system in the area. The presence of nearby radar operation may result in temporary interruption in communications of this device. The Access Point's radar detection feature will automatically restart operation on a channel free of radar. You may consult with the local technical support staff responsible for the wireless network to ensure the Access Point device(s) are properly configured for European Community operation.
- To remain in conformance with European spectrum usage laws for Wireless LAN operation, the above 5 GHz channel limitations apply. The user should check the current channel of operation. If operation is occurring outside of the allowable frequencies as listed above, the user must cease operating the device at that location and consult the local technical support staff responsible for the wireless network.
- In Italy the end-user must apply for a license from the national spectrum authority to operate this device outdoors.

Wireless Propaq CS Radio specifications (2.4 GHz)

Characteristic	Specification
FlexNet Network	2.4 GHz Wireless Local Area Network (WLAN) and 10/100 Base-T Ethernet network
Frequency ^a	2.402 to 2.480 GHz
Modulation	GFSK, Frequency Hopping Spread Spectrum (FHSS)
Output Power	112 mW (maximum)
IEEE 802.11 compliant	Yes
Wireless Propaq CS monitors per Access Point	10 (max.)

- a. When used within certain countries, authorization for use is restricted as follows:
- France The equipment is internally restricted to the 2.448-2.482 GHz frequency range.
 - Spain The equipment is internally restricted to the 2.447-2.473 GHz frequency range.
 - Japan The equipment is internally restricted to the 2.473-2.495 GHz frequency range.
 - Italy Operation requires a user license.

Note: The frequency ranges specified above are subject to geographic-specific regulatory authorities.

Monitor (environmental) specifications

Characteristic	Specification
Operating Temperature	0° to 40° C
Shipping and Storage Temperature	-20° to 60° C
Operating Altitude	-2,000 to 15,000 ft (-610 to 4,572 m)
Shipping and Storage Altitude	-2,000 to 40,000 ft (-610 to 12,192 m)
Operating Relative Humidity	15% to 95%, noncondensing per MIL STD 810E, Procedure 1-natural
Shipping and Storage Relative Humidity	15% to 95%, noncondensing per MIL STD 810E, Procedure 1-natural
Shock	50 g
Vibration, Random	0.02 g ² /Hz from 10 to 500 Hz, ramping down to 0.002 g ² /Hz at 2000 Hz. Operating 1 hour per axis, 3 hours per test. Designed to meet RTCA DO-160D, Category C.
Degree of protection against ingress for monitors <i>without</i> CO ₂ or printer options	IPX1 rating, drip proof per EN60529: 1991



Caution The monitor may not meet performance specifications if it is not used or stored within these environmental specifications.

Monitor (physical) specifications

Characteristic	Specification
Protection classifications, all configurations^a	
Type of Protection against Electric Shock—Power Adapter	Power adapter class 1
Type of Protection against Electric Shock—Monitor (connected to power adapter or internal battery)	Protective earth not available in monitor. Monitor designed and tested to meet Double Insulation Requirement.
Degree of Protection Against Electric Shock, for Parts Applied to Patients	See monitor labels
Recovery time following defibrillator discharge	Less than or equal to 10 seconds
Electrosurgery interference suppression	Suitable for use in the presence of electrosurgery
Method of Disinfection	Not suitable for autoclaving (see cleaning instructions, “Inspect and clean the monitor and accessories” on page 88)
Flammable Anesthetics	Not suitable for use with flammable anesthetics
Monitor only	
Height	8.2 in (20.8 cm) with handle
Width	9.6 in (24.4 cm)
Depth	5.6 in (14.1 cm)
Weight	7.6 lb (3.4 kg)
Monitor with SpO₂ module	
Height	8.2 in (20.8 cm) with handle
Width	9.6 in (24.4 cm)
Depth	7.7 in (19.7 cm)
Weight	10.8 lb (4.9 kg)
Monitor with expansion module (Printer / SpO₂ / MCO₂)	
Height	11.4 in (28.8 cm)
Width	9.6 in (24.4 cm)
Depth	7.7 in (19.7 cm) with back feet
Weight with Printer, SpO ₂ , and MCO ₂	14.4 lb (6.5 kg)

a. Per EN 60601-1 unless otherwise stated.

Printer specifications

Characteristic	Specification
Operation	
Operating Modes	Continuous, Snapshot, Auto Print, Auto Trend, Tabular Trend, Alarm Print, NIBP Ticket, Apnea Ticket, OxyCRG, OxyCRG on Alarm
Auto Print Intervals	15 min, 30 min, 1 hour, 2 hours, 4 hours
Auto Trend Shifts	Once every 4 hours
Number of Waveforms	Up to three: ECG1, P1, P2, SpO ₂ , CO ₂ , RESP
Grid	5 mm and 1 mm gradations
Annotation	Date, Time, Print Mode, Speed, Heart Rate, Systolic, Diastolic, Mean, SpO ₂ , Breath Rate, ETCO ₂ , INCO ₂ , Temperature, ΔT, Pacer Status, Company Logo, ECG Bandwidth, Patient Mode, scale factors for all traces and, if Acuity is connected, patient name and identification.
Printing Speeds	6.25, 12.5, 25.0 mm/s, simulated 6.25 mm/s for CO ₂ and RESP in Snapshot mode
Printer mechanism	
Printing Method	Thermally sensitive dot method
Dot structure	320 dots per line
Printing width	53 mm
Horizontal Dot Pitch	0.165 mm, 6 dots/mm
Vertical Dot Pitch	0.165 mm
Paper Feed Method	Friction Feed
Paper Feed Precision	±2% @ 25° C and 60% Relative Humidity
Paper Width	60 mm
Reliability	30 million pulses/dot
Environmental	
Monitor/expansion module	
Operating Temperature	+5° to 40° C
Shipping and Storage Temperature	-20° to 60° C
Operating Relative Humidity	35% to 85% noncondensing
Shipping, Storage Relative Humidity	15% to 90% noncondensing
Operating Altitude	-2,000 to 15,000 ft (-610 to 4,572 m)
Shipping and Storage Altitude	-2,000 to 40,000 ft (-610 to 12,192 m)
Shock	30 g
Vibration, Random	0.02 g ² /Hz from 10 to 500 Hz, ramping down to 0.002 g ² /Hz at 2000 Hz. Operating 1 hour per axis, 3 hours per test.
Electromagnetic Compatibility (EMC)	Per IEC/EN 60601-1-2, which is a collateral standard of IEC/EN 60601-1, for electromagnetic compatibility.
Paper storage	
Short-term Storage Environment (up to 7 days)	-20 to 40°C; 5% to 80% noncondensing
Long-term Storage Environment (up to 5 years)	25°C (optimal), 65% noncondensing

Power Specifications

Characteristic	Specification
Mode of Operation	Continuous
Battery Pack Type	Sealed, gel-type lead acid
Battery Pack Capacity	Monitor only: 8 V, 2.7 Ampere-Hours; Monitor with Expansion Modules: 8 V, 5.4 Ampere-Hours
Battery Recharger Circuitry	Internal, powered by external power adapter
DC Input Power Required	12 to 28 V, 25 Watts
Input Fuse Rating	3A/250V, Type 2AG (0.57x 0.177 in)
Battery Recharge Time with instrument on	Range of 8 hours to 12 hours typical, depending upon product configuration
Battery Recharge Time with instrument off	Range of 6 hours to 8 hours depending upon product configuration
Recharge time until monitor is usable, starting with discharged but non-faulty battery	≤2 minutes typically (longer time required before NIBP, printer, and CO ₂ are available)
Low Battery Voltage and Operation	< 7.8 V: Caution message LOW BATTERY. < 7.6 V: Caution messages LOW BATTERY, PRINTER DISABLED and LOW BATTERY, NIBP DISABLED. < 7.4 V: Equipment alert VERY LOW BATTERY < 7.3 V: Equipment alert LOW BATTERY, HEATER DISABLED (MCO ₂) < 7.0 V: Monitor automatically turns off.
Typical operating times on battery	
Monitor only	2 hours
Monitor and SpO ₂ (Baqpaq)	5 hours
Monitor with Expansion Module with printer, SpO ₂ and CO ₂ Options	3 hours

Power Adapter Specifications

Characteristic	Specification
Protection classifications, all adapters^a	
Type of Protection Against Electric Shock	Class I, (Protectively Earthed)
Degree of Protection Against Harmful Ingress of Water	For ordinary, indoor locations only.
Method of Disinfection	Not suitable for autoclaving
Flammable Anesthetics	Not suitable for use with flammable anesthetics
Environmental specifications, all adapters	
Operating Temperature	0° to 50° C
Shipping and Storage Temperature	-20° to 60° C
Operating Altitude	-2,000 to 15,000 feet (-610 to 4,572 m)
Shipping and Storage Altitude	-2,000 to 40,000 feet (-610 to 12,192 m)
Operating Relative Humidity	15% to 95%, noncondensing
Shipping, Storage Relative Humidity	15% to 95%, noncondensing
Shock	50 g
Vibration	Random Vibration, 0.02 g ² /Hz from 10 to 300 Hz, ramping down to 0.002 g ² /Hz at 500 Hz. Operating 1 hour per axis, 3 hours/ test.
Physical specifications	
Length	5.0 in (12.7 cm)
Width	3.6 in (9.1 cm)
Height	3.1 in (7.9 cm)
Weight	3.1 lb (1.4 kg)
Universal power adapter, part No. 503-0054-00	
Rated Input	100-120 V ac, 500 mA, 50/60 Hz
Rated Fuses	T800 mA/250 V, Time-Delay, 5x20mm
Rated Output (Continuous)	16-24 V dc, 25 VA
Additional Features	Detachable power cord, pilot light
Universal power adapter, part No. 503-0054-01	
Rated Input	200-240 V ac, 250 mA, 50/60 Hz
Rated Fuses	T400 mA/250 V, Time-Delay, 5 x 20mm
Rated Output (Continuous)	16-24 V dc, 25 VA
Additional Features	Detachable power cord, pilot light

a. Per EN 60601-1 unless otherwise stated.

9

Compliance

General

The 802.11a Wireless PC Card must be installed and used in strict accordance with the manufacturer's instructions as described in the user documentation that comes with the product.

This product contains encryption. It is unlawful to export out of the U.S. without obtaining a U.S. Export License.

Federal Communications Commission (FCC)

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

- This device may not cause harmful interference.
- This device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy. If not installed and used in accordance with the instructions, it may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by tuning the equipment off and on, the user is encouraged to try and correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the distance between the equipment and the receiver
- Connect the equipment to outlet on a circuit different from that to which the receiver is connected
- Consult the dealer or an experienced radio/TV technician for help

The user may find the following booklet prepared by the Federal Communications Commission helpful:

The Interference Handbook

This booklet is available from the U.S. Government Printing Office, Washington, D.C. 20402. Stock No. 004-000-0034504.

Welch Allyn is not responsible for any radio or television interference caused by unauthorized modification of the devices included with this Welch Allyn product, or the

substitution or attachment of connecting cables and equipment other than specified by Welch Allyn.

The correction of interference caused by such unauthorized modification, substitution or attachment will be the responsibility of the user.

Industry Canada (IC) emissions

This device complies with RSS 210 of Industry Canada.

Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of this device.

L'utilisation de ce dispositif est autorisée seulement aux conditions suivantes: (1) il ne doit pas produire de brouillage et (2) l'utilisateur du dispositif doit être prêt à accepter tout brouillage radioélectrique reçu, même si ce brouillage est susceptible de compromettre le fonctionnement du dispositif.

This Class B digital apparatus complies with Canadian ICES-003.

Cet appareil numérique de la classe B est conforme à la norme NMB-003 du Canada.

European Union

Czech	Welch Allyn tímto prohlašuje, že tento <i>RLAN device</i> je ve shodě se základními požadavky a dalšími příslušnými ustanoveními směrnice 1999/5/ES.
Danish	Undertegnede Welch Allyn erklærer herved, at følgende udstyr <i>RLAN device</i> overholder de væsentlige krav og øvrige relevante krav i direktiv 1999/5/EF
Dutch	Bij deze verklaart Welch Allyn dat deze <i>RLAN device</i> voldoet aan de essentiële eisen en aan de overige relevante bepalingen van Richtlijn 1999/5/EC.
English	Hereby, Welch Allyn, declares that this <i>RLAN device</i> is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EC.
Estonian	Käesolevaga kinnitab Welch Allyn seadme <i>RLAN device</i> vastavust direktiivi 1999/5/EÜ põhinõuetele ja nimetatud direktiivist tulenevatele teistele asjakohastele sätetele.
Finnish	Welch Allyn vakuuttaa täten että <i>RLAN device</i> tyyppinen laite on direktiivin 1999/5/EY oleellisten vaatimusten ja sitä koskevien direktiivin muiden ehtojen mukainen.
French	Par la présente, Welch Allyn déclare que ce <i>RLAN device</i> est conforme aux exigences essentielles et aux autres dispositions de la directive 1999/5/CE qui lui sont applicables
German	Hiermit erklärt Welch Allyn die Übereinstimmung des Gerätes <i>RLAN device</i> mit den grundlegenden Anforderungen und den anderen relevanten Festlegungen der Richtlinie 1999/5/EG. (Wien)
Greek	ΜΕ ΤΗΝ ΠΑΡΟΥΣΑ Welch Allyn ΔΗΛΩΝΕΙ ΟΤΙ <i>RLAN device</i> ΣΥΜΜΟΡΦΩΝΕΤΑΙ ΠΡΟΣ ΤΙΣ ΟΥΣΙΩΔΕΙΣ ΑΠΑΙΤΗΣΕΙΣ ΚΑΙ ΤΙΣ ΛΟΙΠΕΣ ΣΧΕΤΙΚΕΣ ΔΙΑΤΑΞΕΙΣ ΤΗΣ ΟΔΗΓΙΑΣ 1999/5/ΕΚ
Hungarian	Alulírott, Welch Allyn nyilatkozom, hogy a <i>RLAN device</i> megfelel a vonatkozó alapvető követelményeknek és az 1999/5/EC irányelv egyéb előírásainak.
Italian	Con la presente Welch Allyn dichiara che questo <i>RLAN device</i> è conforme ai requisiti essenziali ed alle altre disposizioni pertinenti stabilite dalla direttiva 1999/5/CE.
Latvian	Ar šo Welch Allyn deklarā, ka <i>RLAN device</i> atbilst Direktīvas 1999/5/EK būtiskajām prasībām un citiem ar to saistītajiem noteikumiem.
Lithuanian	Šiuo Welch Allyn deklaruoją, kad šis <i>RLAN device</i> atitinka esminius reikalavimus ir kitas 1999/5/EB Direktyvos nuostatas.
Malti	Hawnhekk, Welch Allyn, jiddikjara li dan <i>RLAN device</i> jikkonforma mal-htigijiet essenzjali u ma provvedimenti oħrajn relevanti li hemm fid-Direttiva 1999/5/EC
Portuguese	Welch Allyn declara que este <i>RLAN device</i> está conforme com os requisitos essenciais e outras disposições da Directiva 1999/5/CE.
Slovak	Welch Allyn týmto vyhlasuje, že <i>RLAN device</i> spĺňa základné požiadavky a všetky príslušné ustanovenia Smernice 1999/5/ES.
Slovene	Šiuo Welch Allyn deklaruoją, kad šis <i>RLAN device</i> atitinka esminius reikalavimus ir kitas 1999/5/EB Direktyvos nuostatas.
Spanish	Por medio de la presente Welch Allyn declara que el <i>RLAN device</i> cumple con los requisitos esenciales y cualesquiera otras disposiciones aplicables o exigibles de la Directiva 1999/5/CE
Swedish	Härmed intygar Welch Allyn att denna <i>RLAN device</i> står i överensstämmelse med de väsentliga egenskapskrav och övriga relevanta bestämmelser som framgår av direktiv 1999/5/EG.

Electromagnetic compatibility

Special precautions concerning electromagnetic compatibility (EMC) must be taken for all medical electrical equipment. The Propaq CS monitor complies with IEC EN 60601-1-2:2001.

- All medical electrical equipment must be installed and put into service in accordance with the EMC information provided in this document and in *Propaq Directions for Use*.
- Portable and mobile RF communications equipment can affect the behavior of medical electrical equipment.

Propaq CS monitors comply with all applicable and required standards for electromagnetic interference.

- It does not normally affect nearby equipment and devices.
- It is not normally affected by nearby equipment and devices.
- It is safe to operate the monitor in the presence of high-frequency surgical equipment.
- However, it is good practice to avoid using the monitor in extremely close proximity to other equipment.

Propaq CS monitor: Guidance and manufacturer's declaration—Electromagnetic emissions

The Propaq CS monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the monitor should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment—guidance
RF emissions CISPR 11	Group 2	The Monitor must emit electromagnetic energy in order to perform its intended function. Nearby electronic equipment may be affected.
RF emissions CISPR 11	Class B	The monitor is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

Propaq CS Monitor: Guidance and manufacturer's declaration—Electromagnetic immunity

The Propaq CS monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the monitor should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment—guidance
EN ISO9919	20 V/m. 1 KHz AM modulation	20 V/m	Intended for use during patient transport outside the healthcare facility
EN ISO21647	20 V/m. 1 KHz AM modulation	3 V/m	Intended for use during patient transport outside the healthcare facility
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV for power supply lines ±1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	±1 kV differential mode ±2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions, and voltage variations on power-supply input lines IEC 61000-4-11	<5% U_t (>95% dip in U_t) for 0.5 cycle 40% U_t (60% dip in U_t) for 5 cycles 70% U_t (30% dip in U_t) for 25 cycles <5% U_t (>95% dip in U_t) for 5 sec	<5% U_t (>95% dip in U_t) for 0.5 cycle 40% U_t (60% dip in U_t) for 5 cycles 70% U_t (30% dip in U_t) for 25 cycles <5% U_t (>95% dip in U_t) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the monitor requires continued operation during power mains interruption, it is recommended that the monitor be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Note	U_t is the AC mains voltage prior to application of the test level.		

Propaq CS Monitor: Guidance and manufacturer’s declaration—Electromagnetic immunity

The Propaq CS monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the monitor should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment—guidance
			Portable and mobile RF communications equipment should be used no closer to any part of the monitor, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance
Conducted RF IEC 61000-4-6	3 V _{rms} 150 kHz to 80 MHz	2 V _{rms}	$d = 1.75 \sqrt{P}$
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	2 V/m	$d = 1.2 \sqrt{P}$ 80 MHz to 650 MHz $d = 1.75 \sqrt{P}$ 650 MHz to 800 MHz $d = 2.3 \sqrt{P}$ 800 MHz to 2.5 GHz where P is the maximum output power rating of the transmitter in watts according to the transmitter manufacturer and d is the recommended separation distance in meters. Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a , should be less than the compliance level in each frequency range ^b . Interference might occur in the vicinity of equipment marked with the following symbol: 
Note 1	At 80 MHz and 800 MHz, the higher frequency range applies.		
Note 2	These guidelines might not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.		
a	Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the monitor is used exceeds the applicable RF compliance level above, the monitor should be observed to verify normal operation. If abnormal performance is observed, additional measures might be necessary, such as reorienting or relocating the monitor.		
b	Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 2 V/m.		

Recommended separation distances between portable and mobile RF communications equipment and the Propaq CS Monitor

The Propaq CS monitor is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the monitor can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the monitor as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter (m)			
	150 kHz to 80 MHz $d = 1.75 \sqrt{P}$	80 MHz to 650 MHz $d = 1.2 \sqrt{P}$	650 MHz to 800 MHz $d = 1.75 \sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3 \sqrt{P}$
0.01	0.175	0.12	0.175	0.23
0.1	0.554	0.38	0.554	0.73
1	1.75	1.2	1.75	2.3
10	5.54	3.8	5.54	7.3
100	17.5	12	17.5	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (w) according to the transmitter manufacturer.

Note 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2 These guidelines might not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Index

A

- Access points, 74
- AC Power Adapter
 - Change fuses, 85
 - Change voltage selection, 85
 - Connect to monitor, 83
- AC power adapter, specifications, 115
- AC power filter for ECG, 26
- Acuity, 74
 - Connector, 7
 - Connect the monitor to Acuity, 77
 - Disconnect the monitor, 78
 - Warnings, 77
- Acuity network equipment alert message, 64
- Airway adapter, Mainstream CO₂, 43
- Alarm Holdoffs, 108
- Alarm Limits
 - CO₂, 49
 - Customize alarm limits (STAT SET), 56
 - Setup, 17
- Alarm Limits Menu, 17
- Alarms
 - Automatic printing on alarm, 70
 - Nurse Call Alarm, 57
 - Setup alarm limits, 17
 - Setup alarms, 17
 - Specifications, 108
- Alarms Status Menu, 17
- Alarm Tone volume, 16
- Apnea
 - Automatic printing on apnea alarm, 70
- Apnea alarm, 18
- Apnea measurement, 43, 45
- Apnea ticket (printout), 70
- Artifact interference and waveform display, 23
- Artifact marker on display, prints (NIBP), 33
- Artifact reduction with C-LOCK (SpO₂), 41
- Artifact reduction with SMARTCUF (NIBP), 32
- Artifact with NIBP, 1, 34, 35, 67, 68
- Automatic NIBP measurements, 35

B

- Battery
 - Long-term storage precautions, 90
 - Low battery message, 65, 83
 - Recharging, 83
- Beeper loudness, 16

C

- Capnography (see Mainstream CO₂ and Sidestream CO₂)
- Centigrade temperature units, 37
- Change alarm limits, 17
- Charge battery, 83
- Cleaning instructions, 88
- C-LOCK, SpO₂ measurement, 41
- Clock setting, 93
- CO₂
 - Alarm limits, 49
 - Equipment alert messages, 61, 62
 - Flowrate, Sidestream CO₂, 49
 - Gas compensation, 48
 - Mainstream CO₂, 43
 - Response time, 48
 - Select measurement units, 94
 - Set up CO₂ display, 47
 - Sidestream CO₂, 45
 - Source selection, 49
 - Specifications, general, 105
 - Sweep speed selection, 48
- Connector locations, 7
- Connectors, patient, 8
- Continuous printing, 70
- Controls, 7
- Cuff inflation target (NIBP), 32
- Cuff selection (NIBP), 33
- Current patient mode, 18
- Customize alarm limits, 56
- Custom patient modes, 19

D

- Data communication equipment alert message, 64
- Date and time setting, 93

- Date format setting, 94
- Decimal character setting, 94
- Default inflation pressure (NIBP), 32
- Default settings, 95
- DEFIB FAULT message, 65
- Defibrillator Synchronization
 - Connector, 7
 - Equipment alert messages, 65
 - Specifications, 99
- Delete all patient trends, 69
- Disconnect monitor from Acuity, 78
- Disinfection instructions, 88
- Display
 - Artifact interference, 23
 - CO₂, 47
 - ECG/RESP, 24
 - Equipment alert messages, 58
 - General description, 10
 - IBP waveforms, 29
 - NIBP, 34
 - Pacemaker indicators, 27
 - Specifications, 109
 - SpO₂, 39, 40
 - Sweep speed selection, 16
 - Sweep speed selection, CO₂ and RESP, 48
 - Temperature, 37
 - Trends, 68
 - Waveform selection, 16

E

ECG

- Display, 24
- Electrode selection and placement, 23
- Electrosurgery and interference, 24
- Equipment alert messages, 58
- Filter to reduce interference, 26
- Lead selection, 25
- Pacemaker display indicators, 27
- Specifications, 97
- Waveform size adjustment, 25
- ECG/RESP menu 1, 24
- ECG/RESP menu 2, 25
- ECG filter setting, 94
- ECG LEAD, 25
- ECG SIZE, 25
- Electrocautery and IBP monitoring, 28
- Electrodes and ECG (see ECG)
- Electrosurgery and ECG interference, 24
- Environmental specifications, 111
- Equipment Alert
 - Acuity Network Message, 64
 - Defibrillator Messages, 65
 - ECG Messages, 58
 - IBP Messages, 59

- Low battery messages, 65
- Mainstream CO₂ Messages, 61
- Messages, 58
- NIBP Messages, 59
- Printer Messages, 65
- PROGRAM FAULT Message, 64
- RESP Messages, 58
- Sidestream CO₂ Messages, 62
- SpO₂ Messages, 61
- Temperature Messages, 61
- ERR message for NIBP, 68
- ETCO₂ (End-Tidal CO₂), 43, 45
- Extended storage precautions, 90

F

- Factory default settings, 95
- Factory patient modes
 - Setup, 19
- Fahrenheit temperature units, 37
- False alarms with SpO₂, 39, 41
- Filter to reduce ECG interference, 26, 94
- FlexNet, 74
 - Access points, 74
 - Automatic reconnection, 75
 - Operation out of range, 74
- Flowrate, Sidestream CO₂, 49
- Format adjustment for IBP numerics, 31
- Fuse replacement
 - AC power adapter, 85
 - Monitor input power, 86

G

- Gas compensation, CO₂, 48

H

- Heart Tone volume, 16
- Hewlett-Packard patient connectors, 8
- High frequency ventilation and RESP, 22
- HP (Hewlett-Packard) patient connectors, 8
- HR/PR Alarm Limits, on/off enable, 94
- HR/PR source, 16
- HR/PR tone volume adjustment, 16

I

IBP

- Adjust waveform scale, 30
- Automatic display adjustment, 30
- Change waveform display labels, 31
- Display, 29
- Electrocautery warning, 28

- Equipment alert messages, 59
- Monitoring, 28
- Numeric format adjustment, 31
- Range Mode, 30
- Rescale Mode, 30
- Specifications, 100
- Transducer, 28
- Zero a transducer, 28

Impedance Pneumography (see RESP)

INCO₂ (Inspired CO₂), 43, 45

Input power fuse replacement, 86

INSERV, 13

In-Service mode, 13

Inspect the monitor, 88

Interference filter for ECG, 26

Interference with electrosurgery, ECG, 24

Invasive Blood Pressure (see IBP)

L

- Label selection for IBP, 31
- Lead selection, RESP, 26
- Learn to use Encore, 13
- Line power adapter, 83
- Loudness selection, 16
- Low battery message, 65, 83

M

Main Menu, General description, 11

Mainstream CO₂

- Airway adapter, 43
- Alarm limits, 49
- Display, 47
- Equipment alert messages, 61
- Gas compensation, 48
- Menus, 48
- Monitoring, 43
- Response time, 48
- Specifications, 106
- SpO₂ option and CO₂, 43
- Switch to Sidestream CO₂, 49
- Turn on/off, 49
- Warnings, 43

Maintenance instructions, 89

Manometer bar (NIBP display), 34

Marriot Configuration for ECG, 23

MCO₂(see Mainstream CO₂)

Menus

- Alarm Limits, 17
- Alarms Status, 17
- CO₂, 48
- ECG/RESP Menu 1, 24
- ECG/RESP Menu 2, 25
- General description, 11

- IBP, 29
- Main Menu, 11
- NIBP, 34
- Printer Setup Page, 70
- Setup Menus, 12
- SpO₂, 39, 40

Messages, equipment alerts, 58

Micropaq

- Automatic reconnection, 75
- Communication with network, 74
- Out of range operation, 74

Model numbers, 8

Modem-Propaq connector, 7

Mode Setup window, 19

Monitoring

- ECG/RESP, 21
- IBP, 28
- Mainstream CO₂, 43
- NIBP, 32
- Sidestream CO₂, 45
- SpO₂, 38
- Temperature, 37

Motion artifact and NIBP, 1, 34, 35, 67, 68

N

NET OFF, disconnect from Acuity, 78

Network

- Access points, 74
- Automatic reconnection, 75
- FlexNet, 74
- Operation out of range, 74

Network fault equipment alert message, 64

New patient setup, 15

NIBP

- Artifact, 1, 34, 35, 67, 68
- Artifact filtering message, 60
- Automatic Mode, 35
- Automatic printing on NIBP, 70
- Cuff and hose selection, 33
- Cuff placement, 34
- Default inflation pressure, 32
- Display information, 34
- Equipment alert messages, 59
- Interval for automatic measurements, 35
- Manual Mode, 35
- Monitoring, 32
- Motion artifact, 1, 34, 35, 67, 68
- NIBP IN PROGRESS message, 60
- NIBP Menu, 34
- Printout symbol with artifact, 67
- Retries, 35
- SMARTCUF artifact filter, 32
- Specifications, 101
- Start/stop measurement, 35

- Symbol in display, 34, 68
- Symbol in printout, 67
- TURBOCUFF repeated measurements, 35
- Warnings, 32
- NIBP IN PROGRESS message, 60
- NIBP ticket (printout), 70
- NIBP Trend, 68
- Non-Invasive Blood Pressure (see NIBP)
- Nurse Call Alarm
 - Description, 57
 - Specifications, 108

O

- Options, 8
- Out of range, 74
- OxyCRG
 - Automatic printing on alarm, 70
 - Description, 71
 - How to print, 71
- Oxygen saturation display, 39, 40
- Oxygen saturation measurement(see SpO2)

P

- Pacemaker display indicators, 27
- Pacemaker patients and monitoring, 27
- Paper replacement, printer, 87
- Patient mode
 - Change current patient mode, 18
 - Change powerup patient mode, 19
 - Check at powerup, 15
 - Custom, 19
 - Factory, 19
- Patient Mode window, 15
- Periodic maintenance, 89
- Physical inspection, 88
- Physical specifications, 112
- Power
 - Specifications, 114
- Power Adapter
 - Specifications, 115
- Power adapter, 83
- Power fuse replacement, 86
- Powerup patient mode, 19
- Powerup procedure, 15
- Practice using Encore, 13
- Print
 - Automatic alarm printing, 70
 - Automatic apnea ticket printing, 70
 - Automatic NIBP ticket printing, 70
 - Automatic OxyCRG printing, 70
 - Automatic printing, 70
 - Automatic trend printing, 69
 - Continuous printing, 70

- Displayed trend, 68
- Displayed waveforms, 67
- Multiple trends, 69
- OxyCRG, 71
- Set printer options, 70
- Printer
 - Equipment alert messages, 65
 - Generate a test print, 87
 - Long-term storage precautions, 90
 - Paper replacement, 87
 - Set options, 70
 - Specifications, 113
- Printer Setup Page, 70
- Printing
 - Artifact symbol with NIBP, 67
- PROGRAM FAULT equipment alert message, 64
- Programmable patient modes, 19
- Propaq CS
 - Models and options, 8
 - Physical inspection, 88
- Pulse Oximetry(see SpO2)

R

- Rebreathing, INCO2, 48
- Recharge battery, 83
- Reconnection to network, 75
- REMOVE CUFF FROM PATIENT message, 65
- Replace fuses
 - AC power adapter, 85
 - Monitor input power, 86
- Replace printer paper, 87
- Reset clock, 93
- RESP
 - Display, 24
 - Equipment alert messages, 58
 - High frequency ventilation, 22
 - Lead selection, 26
 - Specifications, 99
 - SpO2 monitoring as backup method, 22
 - Turn on or off, 26
 - Waveform size adjustment, 25
- Response time, CO2, 48
- RESPONSE time (SpO2), 40
- Retries (NIBP), 35
- Right side panel, 7

S

- SCO2(see Sidestream CO2)
- Service
 - Recommendations, 6
 - Service Manual, 6
- Service interval recommendations, 89
- Service Menus, 12

- Set time and date, 93
- Settings, factory default, 95
- Setup
 - Alarm limits, 17
 - Alarms, 17
 - Custom patient modes, 19
 - New patient, 15
 - Patient modes, 19
 - Powerup patient mode, 19
- Setup Menus, 12
- Side panel
 - Left side panel, 8
 - Right side panel, 7
- Sidestream CO2
 - Alarm limits, 49
 - Display, 47
 - Equipment alert messages, 62
 - Flow rate, 49
 - Gas compensation, 48
 - Menus, 48
 - Monitoring, 45
 - Rapid breath rate and ETCO2 values, 45
 - Rapid respiratory rate for children, 45
 - Response time, 48
 - Specifications, 107
 - SpO2 option and CO2, 45
 - Switch to Mainstream CO2, 49
 - Turn on/off, 49
 - Warnings, 45
 - Watertrap insertion, 45
- Skin preparation for ECG, 23
- SMARTCUF NIBP artifact filter, 32
- SMARTCUF symbol on display, prints, 33
- Snapshot printing, 67
- Sound volume selection, 16
- Source selection, CO2, 49
- Specifications
 - AC power adapter, 115
 - CO2, general, 105
 - Display, 109
 - ECG, 97
 - Environmental, 111
 - IBP, 100
 - Mainstream CO2, 106
 - Monitor, physical, 112
 - NIBP, 101
 - Nurse Call Alarm, 108
 - Power, 114
 - Printer, 113
 - Real-Time ECG Analog/Defib Sync, 99
 - RESP, 99
 - Sidestream CO2, 107
 - SpO2, 103
 - Temperature, 102
 - Trends, 109

- SpO2
 - Adjust waveform size, 39, 40
 - C-LOCK, 41
 - Display, 39, 40
 - Equipment alert messages, 61
 - Monitoring, 38
 - NIBP and SpO2, 39, 40
 - Reducing false alarms, 39, 41
 - Response time selection, 40
 - Sensor selection, 39, 40
 - Specifications, 103
 - SpO2 Menus, 39, 40
 - Spot-Check monitoring, 42
 - Standby Mode, 42
 - Warnings, 38
- Spot-Check SpO2 monitoring, 42
- STANDBY and SpO2, 42
- Standby Mode, SpO2, 42
- STARTUP message, Sidestream CO2, 47
- STATSCALE, 10
- STAT SET, customize alarm limits, 56
- STBY message, SpO2 STANDBY, 42
- Storage precautions, 90
- Sweep speed selection, 16
- Sweep speed selection, CO2 and RESP, 48
- Symbol
 - In NIBP display, 34, 68
 - In NIBP printout, 67
- Synchronizing R-waves, C-LOCK, 41

T

- Temperature
 - Display, 37
 - Equipment alert messages, 61
 - Monitoring, 37
 - Select measurement units, 94
 - Specifications, 102
 - Warnings, 37
- Temperature units, change, 37
- Time and date setting, 93
- Tone volumes, 16
- Transducer for IBP, 28
- Trends
 - Automatic printing, 69
 - Delete all trends, 69
 - Display, 68
 - NIBP and symbol, 68
 - Print all selected trends, 69
 - Select trends for printing, 69
 - Specifications, 109
- TURBOCUF (automatic NIBP), 35
- Turn on waveform display, 16

U

Units of measure, CO2, 94

Units of measure, temperature, 37, 94

V

Ventilation, high frequency and RESP, 22

VERY LOW BATTERY message, 65

Vital sign waveform display, 16

Volume tone selection, 16

W

WARMUP message, Mainstream CO2, 47

Warnings

Acuity, 77

ECG/RESP, 21

IBP, 28

Mainstream CO2, 43

NIBP, 32

Pacemaker patients, 27

Sidestream CO2, 45

SpO2, 38

Temperature, 37

Warranty period service, 6

Watertrap for Sidestream CO2, 45

Waveform

CO2, 47

ECG size adjustment, 25

IBP, 30

RESP size adjustment, 25

SpO2, 39, 40

Waveform display, 10

Waveform display selection, 16

Wave Select window, 16

Windows, general description, 10

Z

Zero a transducer, 28