Document Description:	CSB-CVSM 2.20 62 Character Passphrase Limitation	Document Number: Version:	80021548 A
	Welch \ Allyn ™	Page 1	of 1
Printed or electronic versions	of this document not accessed directly from the designated Welch Allyn Control	lled Quality Information System as	re For Reference Only

Customer Service Bulletin				
Product:	CVSM –Connex Vital Signs Monitor		Date: 2016-06-03	
Subject:	CSB-CVSM 2.20 62 Character Passphrase Limitation			
HW Version(s	, All WITAIAGE MANAGE	W Version(s) ffected:	CVSM 2.20.00, 2.20.01	
Serial Number Affected:	NIΔ	ot or Date ode Affected:	NA	
Classification: Informational Only				
Distribution:	□ Customer Care □ F	Product Service	⊠ Field Service	
		Customers	☐ Company Confidential	
Training Req	uired: □ Yes ⊠ No			
Summary:	CVSM software versions 2.20.00 and 2.20.01 only support passphrases up to 62 characters in length. To enable support for 63 character passphrases update the CVSM software to version 2.20.03.			
Issue:	CVSM host firmware version 2.20.00 and 2.20.01 is unable to configure the radio with a 63 character passphrase. Configuring the radio using authentication type WPA2-PSK with a 63 character passphrase will prevent the radio from connecting, regardless of the security protocol CCMP or TKIP.			
Materials:	Contact Welch Allyn Technical Support to receive a software update to version 2.20.03 or later.			
Required Tools	Welch Allyn Service Tool version 1.8 or PC running Windows 7 or 8 USB cable Type A to Mini B	or later (available	from WelchAllyn.com)	
References:	80021512 CVSM 2.20.03 Software Re 20014045 TN-CVSM 2.20 62 Characte		nitation	

Version	Sec, Pg, Para Changed	Change Made	Date Version Created	Version Created By (initials)
---------	-----------------------------	-------------	-------------------------	-------------------------------------

THIS INFORMATION IS THE PROPERTY OF WELCH ALLYN, INC. AND AS SUCH SHALL NOT BE REPRODUCED, COPIED, OR USED AS A BASIS FOR THE MANUFACTURE OR SALE OF EQUIPMENT OR DEVICES WITHOUT THE EXPRESS WRITTEN PERMISSION OF WELCH ALLYN, INC.

Document Description: CSB-CVSM 2.20 62 Character Passphrase Limitation	Document Number: 80021548 Version: A
Welch Allyn™	Page 2 of 2
Printed or electronic versions of this document not accessed directly from the designated Welch Allyn Controlled Quality Information System are For Reference Only.	

A N/A	Initial Release	2016-05-31	GCS
-------	-----------------	------------	-----

THIS INFORMATION IS THE PROPERTY OF WELCH ALLYN, INC. AND AS SUCH SHALL NOT BE REPRODUCED, COPIED, OR USED AS A BASIS FOR THE MANUFACTURE OR SALE OF EQUIPMENT OR DEVICES WITHOUT THE EXPRESS WRITTEN PERMISSION OF WELCH ALLYN, INC.