


## Scale model 6002 deck installation instructions

# 725547

DIR 80021621 Ver. A Revised 2016-07

© 2016 Welch Allyn, Inc.

## Weighing deck mounting instructions

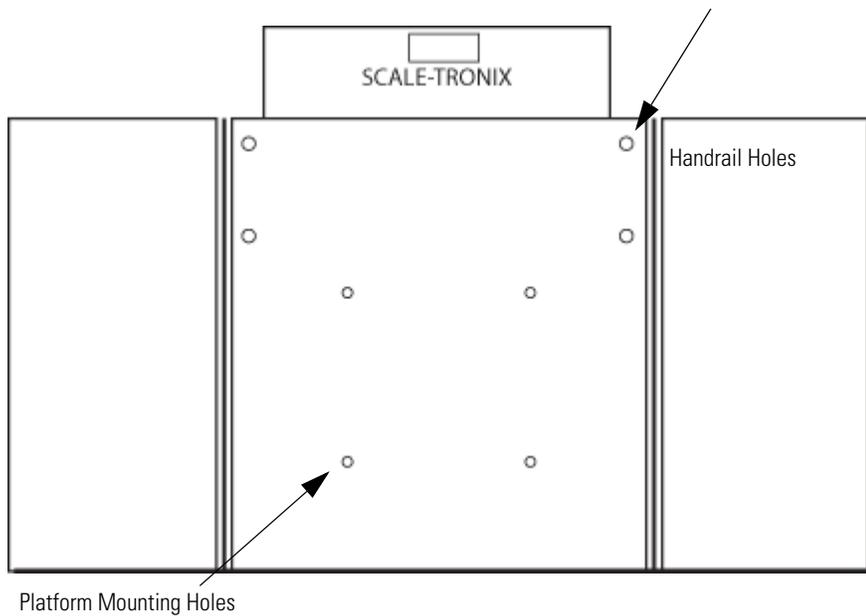

1. Position the weighing deck so that the two sets of holes under the flaps are closest to the mast.
2. Line the weighing deck up with the mounting holes in the aluminum plate.
3. Install the four flat head screws through the weighing deck.
4. Securely tighten the supplied mounting screws with the provided hex key.

For information about any Welch Allyn product, contact your local Welch Allyn representative at [www.welchallyn.com/about/company/locations.htm](http://www.welchallyn.com/about/company/locations.htm).



Welch Allyn, Inc.  
4341 State Street Road  
Skaneateles Falls, NY 13153  
USA  
[www.welchallyn.com](http://www.welchallyn.com)



(91)725547