<b>Document Description:</b>	CSB CVSM Bar Code Scanner Unresponsive	<b>Document Number:</b> 80021792 <b>Version:</b> B	
	<b>Welch Allyn</b> ™	Page <b>1</b> of <b>2</b>	
Printed or electronic versions of this document not accessed directly from the designated Welch Allyn Controlled Quality Information System are For Reference Only			

## **Customer Service Bulletin**

Product:	6000-916HS KIT, 2D SCANNER, HS-1M, GE 7000-916HS SCANNER KIT HS	•	2017-08-17 <b>Date:</b>	
Subject:	CSB CVSM BAR CODE SCANNER UNRESPONSIVE			
HW Version(s) Affected:	WA 409398 (JDK-1967) WA 411464 (JDK-2249) WA 411697 (JDK-2450) WA 411698 (JDK-2449)	SW Version(s) Affected:	FW-HS1G6-01-A4, FW- HHG6-05-A1, FW-HHG6-05- A2	
Serial Numbers Affected:	N/A	Lot or Date Code Affected:	N/A	

Classification: As Needed

Distribution:		ner Care		
	⊠ ASPs	□ Distributors	□ Customers	☐ Company Confidential
Training Requi	red: ⊠`	∕es □ No		
Summary:	when conn HHG6-05-/ may go into vendor (Ja- to address available fr	ected to a powered A2 addressed issue o suspend mode and dak) has provided a these issues. The s	on device. The previous where the bar code so do no longer transmit data software update and a software update, installarver. See Procedure be	cause the scanner to lockup sly released version FW-canner connected to a CVSM at the case of the case
Issue:	to enumera as hot swa In earlier fi	ate when connected pping since the hos rmware versions the	to a powered USB port t device is already power bar code scanner may	become unresponsive in
	"read" butte	on will not activate t	he reader and no data o	into the scanner. Pressing the can be sent to the CVSM host. coted or the barcode scanner

Document Description: CSB CVSM Bar Code Scanner Unresponsive	Document Number: 80021792 Version: B	
Welch Allyn™	Page <b>2</b> of <b>4</b>	
Printed or electronic versions of this document not accessed directly from the designated Welch Allyn Controlled Quality Information System are For Reference Only.		

disconnected from one USB port and connected to another in the same power cycle to resolve.

Updating the bar code scanner's software and/or replacing the cable will reduce the risk of the scanner becoming unresponsive. In the event the scanner does not respond after updating the software, pressing the scanner "read" button will reenumerate the scanner to the CVSM host. Press the button a second time to scan.

The cable is replaced following the simple installation instructions included with the kit. The cable utilizes a snap connector to connect to the scanner. A paper clip is required to remove the cable. The replacement cable snaps into the scanner handle.

## Action:

The latest Jadak bar codes scanners sold by Welch Allyn (material 412449 and 412450) are already provisioned with firmware version FW-HHG6-05-A3 this service bulletin does not apply.

This service bulletin applies to only those materials referenced in the hardware versions affected section above. Customers may download and install the software update from Jadak. The download includes instructions to identify the affected scanners and install the new firmware.

Customers with Bar code scanners under warranty are entitled to receive a warranty replacement cable. For devices, out of warranty the replacement cable service kit is available for purchase.

## Required materials:

Download from Jadak's FTP site:

Jadak Scanner Configuration Update Guide

HSM USB Serial Driver ver 3.4.8

JadakBarcodeScannerUpdate2.06\_A3.zip

(Includes Jadak Update Tool version 2014.12.145, Software update: FW-HHG6-05-A3.moc, and JadakUpdateTool.bat)

Purchase from Welch Allyn:

106594 KIT, HS-1, A1 - SHIELDED CABLE

Contains: Coiled UB cable, cable gasket, sticker for release point, and installation instructions

## Required Tools:

To update the software:

PC with Windows 7, 8, 8.1 or 10

FTP Program or Web Browser with FTP support i.e. Google Chrome, Mozilla Firefox, or Microsoft Internet Explorer

WinZip or other compression utility to expand zip files.

To replace the cable:

<b>Document Description:</b> CSB CVSM Bar Code Scanner Unresponsive	Document Number: 80021792 Version: B	
Welch Allyn™	Page <b>3</b> of <b>4</b>	
Printed or electronic versions of this document not accessed directly from the designated Welch Allyn Controlled Quality Information System are For Reference Only.		

**Procedure:** Go to the Jadak FTP site: <a href="ftp://ftp.jadaktech.com/">ftp://ftp.jadaktech.com/</a>

The FTP logon window will open. Logon credentials are case sensitive.

A paper clip is needed to remove the existing cable before replacing.

Username: JadakService Password: hs1update Click the Log on button.

Download:

HSM USB Serial Driver ver 3.4.8.zip
 Open JadakBarcodeScannerUpdate v2.06 folder
 Download:

- Jadak Scanner Configuration Update Guide.pdf
- JadakBarcodeScannerUpdate2.06\_A3.zip

Follow the procedure as detailed in Jadak Scanner Configuration Update Guide. This version of the Jadak Update Tool only supports updates to software version FW-HS1G6-01-A4, FW-HHG6-05-A1, FW-HHG6-05-A2

For any issues with the process call Jadak support; contact information is provided in the Jadak Scanner Configuration Update Guide.

When complete reconnect the barcode scanner to the CVSM. Power cycle the CVSM after connecting the barcode scanner to insure the scanner properly enumerates.

Version	Sec, Pg, Para Changed	Change Made	Date Version Created	Version Created By (initials)
A	N/A	No changes to content. Update to CSB format to remain consistent with intent to be able to distribute to customers upon request. DIR 20014180_C had the "customers" option selected in the Distribution section, but was structured in SAP as a TSB – conflicting with Customer Care policy against distributing TSB documents to customers (only CSBs can be distributed externally, upon request).	D*	КМС
В	Product	Add 7000-916HS	D*	CCS
	HW version affected	Add WA materials 411464, 411697, 411698	D*	GCS

FCD-1482 Rev J	PARENT: WI-17206803	Page 3 of 4
----------------	---------------------	-------------

<b>Document Description:</b> CSB CVSM Bar Code Scanner Unresponsive	Document Number: 80021792 Version: B	
<b>Welch Allyn</b> ™	Page <b>4</b> of <b>4</b>	
Printed or electronic versions of this document not accessed directly from the designated Welch Allyn Controlled Quality Information System are For Reference Only.		

	SW Version Affected	Add HHG6-05-A1, FW-HHG6-05-A2	
	Summary	Update to include, "scanner may lock up when connected to a powered-on device"	
	Issue	Updated to include, "Version FW-HHG6- 05-A2 could miss USB interrupts resulting in the scanners failure to enumerate when connected to a powered USB port. This is commonly referred to as hot swapping since the host device is already powered on".	
	Action	Update to add, "The latest Jadak bar codes scanners sold by Welch Allyn (material 412449 and 412450) are already provisioned with firmware version FW-HHG6-05-A3 this service bulletin does not apply".	
	Materials	Replace: "JadakBarcodeScannerUpdate2.05.zip" with "JadakBarcodeScannerUpdate2.06_A3.zip" "FW-HHG6-05-A2.moc" with, "FW- HHG6-05-A3.moc"	
	Procedure	Download instructions changed to conform to new directory structure Added update supports versions FW- HHG6-05-A1, FW-HHG6-05-A2	