ENGLISH

	WARNING The warning statements in this manual identify conditions or practices that could lead to illness, injury, or death. Warning symbols will appear with a grey background in a black and white document.
	Caution The caution statements in this manual identify conditions or practices that could result in damage to the equipment or other property, or loss of data.
	Consult Directions for Use
	Do not re-use, Single use device
	Product Identifier
	Reorder Number
	Global Trade Item Number
	Lot Code
	This way up
	Fragile
	Temperature limit
	Humidity limitation
	Defibrillation-proof Type BF applied part
	Use by Date
	Separate collection of batteries. Do not dispose as unsorted municipal waste. Contains Li-Metal batteries.

Indications for Use
The disposable vaginal speculum is used to dilate the vagina and expose the interior of the vagina and exterior of the cervix during pelvic examinations and other gynecological procedures.

Warnings and Cautions
WARNING: Inspect each KleenSpec® box for shipping damage.

WARNING: Inspect speculum prior to use. Do not use if damaged.

WARNING: If a speculum is damaged, do not use any specula from the same case.

WARNING: Specula, when in transit or storage, may be subject to damage beyond the control of the manufacturer or supplier.

WARNING: Never use the speculum with laser equipment.

WARNING: Treat used vaginal specula as bio hazardous infectious material. Dispose of used vaginal specula in suitable disposal unit or in accordance with local regulations.

WARNING: The speculum is not compatible with gamma irradiation or autoclave sterilization.

WARNING: No modification of this equipment is allowed.

CAUTION: Used by trained personnel only.

CAUTION: Rx Only. U.S. federal law restricts this device to sale by or on the order of a physician.

Warranty
Welch Allyn warrants the speculum to meet Welch Allyn's specifications for the Product at the time of purchase and to be free from original defects in material and workmanship under normal use and service. Specula are intended for single use only.

Welch Allyn should be notified via telephone of any defective product and the item should be immediately returned, securely packaged and postage prepaid to Welch Allyn. Loss or damage in shipment shall be at the purchaser's risk.

Welch Allyn will not be responsible for loss associated with the use of any Welch Allyn product that has been altered.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY IMPLIED WARRANTY OR MERCHANTABILITY, FITNESS FOR PARTICULAR PURPOSE, OR OTHER WARRANTY OF QUALITY, WHETHER EXPRESSED OR IMPLIED. WELCH ALLYN WILL NOT BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES.

For information about electromagnetnic compatibility (EMC), see the Welch Allyn website: www.welchallyn.com/emc-590LED.

A printed copy of the Emissions and immunity information can be ordered from Welch Allyn for delivery within 7 calendar days.

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