

Question or Item	Answer
Did you verify that you are using the most recent version of this form?	<input checked="" type="checkbox"/> Yes <input type="checkbox"/> No
Date	2023-11-16
Individual creating the TSB	Matthew Bell
Product(s) affected:	CSM and CVSM (Main Housing Material Only)
Model(s) affected:	All models
Lot or Date Code Affected:	All
<p>Summary:</p> <p>Refer to the following compatibility chart for results of chemical-resistance testing Welch Allyn has performed on the Connex® Spot Monitor (CSM) and Connex® Vital Signs Monitor (CVSM). Unless otherwise noted, those that are compatible are compatible for the main housings only. The LCD displays and any accessory components (including Braun thermometer handles) should follow instructions as provided in the respective Directions For Use.</p>	

MANUFACTURER	CLEANING AGENT	CSM	CVSM
Bode	Bacillo® AF Wipes	C ¹	NC
Clinell®	Universal Wipes	C	NC
Clinitex®	Detergent Wipes	C ¹	NC
Clorox Healthcare®	Bleach Germicidal Wipes	C	NC
Clorox Healthcare®	Dispatch® Wipes	C ¹	C ³
Clorox Healthcare®	Fuzion™ Solution	C ¹	NC
Clorox Healthcare®	Hydrogen Peroxide Wipes	NT	C
Clorox Healthcare®	Disinfecting Wipe	NT	NC
Diversey™	Accel® INTERvention™ Wipes	C	NC
Diversey™	Accel® PREvention™	NT	NC
Diversey™	Accel® TB Wipes	C	NC
Diversey™	Oxivir® 1 Wipes	C ¹	NC
Diversey™	Oxivir® Plus 1:40 Solution	C ¹	NC
Diversey™	Oxivir® TB Wipes	C	NC
Diversey™	Oxivir® Excel Wipes	NT	C
Diversey™	Rescue™ Wipes	NT	NC
Diversey™	Virex® II (256) Solution	C	NT
Diversey™	Virex® TB	NC	NC
Ecolab	Sekusept™ Plus - 1.5% Solution	C ¹	NC
Metrex™	CaviWipes™	C	NC
Metrex™	CaviWipes 1™	NC	NT
MG Chemicals®	8241 Alcohol Wipes	NT	NC
PDI®	Sani-Cloth® Prime Wipes	C ¹	NT
PDI®	Sani-Cloth® Active Wipes	C ¹	NT
PDI®	Sani-Cloth® Bleach Wipes	C ¹	NC
PDI®	Super Sani-Cloth® Wipes	C	NC
PDI®	Sani-Cloth® AF3 Wipes	NC	NC
PDI®	Sani-Cloth® Plus Wipes	C	NC
Reynard	Neutral Detergent Wipes	C ¹	NT
Reynard	Premier Disinfectant Wipes	C ¹	NT
Schuelke	Mikrozyd® AF Wipes	C ¹	NC
Spartan Chemical	Super HDQ®L 10 Solution	C	NT
Vernacare	Tuffie 5 Wipes	C	NC
Veridien	Viraguard Wipes	C ¹	NC
Wexford Labs	CleanCide Wipes	C	C ³
N/A	70% IPA Solution	C	C ²
N/A	50% Bleach Solution	C ¹	NT
N/A	10% Bleach Solution	NT	C ³

Legend	
C	Compatible
NC	Not Compatible
NT	Not Tested

¹ Not tested or not compatible with display.
² Applied to a clean cloth.
³ Not compatible with the Braun dock.

Version	Sec, Pg, Para Changed	Change Made	Date Version Created	Version Created By (initials)
A	N/A	Initial Release	2018-09-15	MKP
B	Page 1 Page 2	<p>Classification section: Un-checked “Company” box and fixed so “Confidential” is showing.</p> <p>Summary: replaced the word “screens” with “displays” added the words “handles” & “respective”. Changed “direction for use” to “Direction For Use”</p> <p>Reword note #¹ to state: “Not tested or not compatible with display”</p> <p>Amended column “CSM” & “CVSM” to reflect the deletion of note #² therefore moving up #'s³ &⁴ making them #'s² &³. Removed “or handle” from note #³.</p>	2018-10-15	MKP
C	Page 2	Add Reynard Neutral Detergent Wipes and Premier Disinfectant Wipes	2020-07-28	AJP
D	Page 2	Add Clorox Disinfecting Wipes	2022-04-11	AJP
E	Page 2 All	Added Diversey Oxivir Excel Wipes Formatting	2023-11-16	KMJ MDB