	CSB - Connex® Vital Signs Monitor (CVSM)	Document Number:	80027695	
Document Description:	Emulate Spot Lxi Setting with the Welch Allyn	Version:	A	
_	Service Hub			
	Welch Allyn™			
Printed or electronic versions of this document not accessed directly from the decignated Walch Allyn Controlled Quality Information System are For Deference Only				

Customer Service Bulletin

Product:	CVSM, and Welch Allyn® Service Hub Date: 2020-12-23					
Subject:	CVSM Spot Lxi Setting with the Welch Allyn Service Hub					
HW Version(s) Affected:	All	SW Version(s) Affected:	All			
Serial Numbers Affected:	All	Lot or Date Code Affected:	All			
Classification:	As Needed					
Distribution:	□ Customer Care □		□ Field Service			
		⊠ Customers	□ Company			
Training Require	ed: ⊠ Yes □ No					
Summary: When Connex® Vital Signs Monitor (CVSM) is connecting to the Welch Allyn® Service Hub, the Emulate Spot Vital Sign LXi setting in the Advanced Menu needs to be disabled for the device to work with a component piece of software to the Welch Allyn® Service Hub, Service Monitor. Emulate Spot Vital Sign LXi settings feature: This option specifies that data sent to the network appears as Spot Vital Signs Lxi data. When this setting is enabled on CVSM, the device communicates in the same language as Spot Lxi, which creates an incompatibility to the Welch Allyn® Service Hub. When the Emulate Spot Vital Sign Lxi setting is disabled, the CVSM is able to communicate its service data to the Welch Allyn Service Hub in the correct communication protocol language. If the setting is not disabled, the CVSM device data file will not be created nor readable by Service Monitor with the outcome that the CVSM device will not appear on Service Hub.						

THIS INFORMATION IS THE PROPERTY OF WELCH ALLYN, INC. AND AS SUCH SHALL NOT BE REPRODUCED, COPIED, OR USED AS A BASIS FOR THE MANUFACTURE OR SALE OF EQUIPMENT OR DEVICES WITHOUT THE EXPRESS WRITTEN PERMISSION OF WELCH ALLYN, INC.

Document Description: CSB - Connex® Vital Signs Monitor (CVSM)	Document Number: 80027695	
Emulate Spot Lxi Setting with the Welch Allyn Service Hub	Version: A	
Welch Allyn ™		
Printed or electronic versions of this document not accessed directly from the designated Welch Allyn Controlled Quality Information System are For Reference Only.		

To Disable Emulate Spot Vital Sign LXi settings on CVSM:

- 1. Access the Advanced settings.
 - a. Touch the Settings tab.
 - b. Touch the Advanced tab.
 - c. Touch Enter password.
 - d. Enter your password and touch OK. The General tab appears.
- 2. Touch the Data Management tab.
- 3. Touch the Clinical Data tab. Directions for use Advanced settings
- 4. Specify settings
- 5. Uncheck Emulate Spot Vital Sign LXi

Version	Sec, Pg, Para Changed	Change Made	Date Version Created	Version Created By (initials)
A	N/A	Initial Release	2020-12-23	KMG

THIS INFORMATION IS THE PROPERTY OF WELCH ALLYN, INC. AND AS SUCH SHALL NOT BE REPRODUCED, COPIED, OR USED AS A BASIS FOR THE MANUFACTURE OR SALE OF EQUIPMENT OR DEVICES WITHOUT THE EXPRESS WRITTEN PERMISSION OF WELCH ALLYN, INC.

FCD-1482 Rev K Parent: WI-17206803