**Hillrom™**

# Hug-U-Vac® Large Steep Trend Positioner

Instructions for Use  
Product No. A-60005

80028183  
Version C

# INSTRUCTIONS FOR USE

---

ENGLISH .....	3
中文简体 .....	26
中文繁體 .....	50
DANSK .....	74
FRANÇAIS .....	97
DEUTSCH .....	121
BAHASA INDONESIA .....	146
ITALIANO .....	170
日本語版 .....	194
한국어 .....	217
LATVISKI .....	240
ROMÂNESC .....	263
SRPSKI .....	290
SLOVENSKY .....	314
SLOVENŠČINA .....	338
ESPAÑOL .....	363
TIẾNG VIỆT .....	387

# INSTRUCTIONS FOR USE

---

## IMPORTANT NOTICES

**Prior to using this or any other type of medical apparatus with a patient, it is recommended that you read the Instructions for Use and familiarize yourself with the product.**

- Read and understand all warnings in this manual and on the device itself prior to use with a patient.
- The  symbol is intended to alert the user to important procedures or safety instructions regarding the use of this device.

The  symbol on the labels is intended to show when the IFU should be referenced for use.

- The techniques detailed in this manual are only manufacturer's suggestions. The final responsibility for patient care with respect to this device remains with the attending physician.
- Device function should be checked prior to each usage.
- This device should only be operated by trained personnel.
- All modifications, upgrades, or repairs must be performed by an authorized specialist.
- Keep this manual available for future reference.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority listed in this document.

# INSTRUCTIONS FOR USE

---

## Table of Contents

### Hug-U-Vac® Large Steep Trend Positioner (A-60005)

1	General Information: .....	6
1.1	Copyright Notice: .....	6
1.2	Trademarks: .....	6
1.3	Contact Details: .....	7
1.4	Safety Considerations: .....	7
1.4.1	Safety hazard symbol notice: .....	7
1.4.2	Equipment misuse notice: .....	7
1.4.3	Notice to users and/or patients: .....	7
1.4.4	Safe disposal: .....	8
1.5	Operating the system: .....	8
1.5.1	Applicable Symbols: .....	8
1.5.2	Intended User and Patient Population: .....	9
1.5.3	Compliance with medical device regulations: .....	10
1.6	EMC considerations: .....	10
1.7	EC authorized representative: .....	10
1.8	Manufacturing Information: .....	10
1.9	EU Importer Information: .....	10
1.10	Authorised Australian sponsor: .....	10
2	System .....	11
2.1	System components Identification: .....	11
2.2	Product Code and Description: .....	11
2.3	List of Accessories and Consumable Components Table: .....	11
2.4	Indication for use: .....	12
2.5	Intended use: .....	12
2.6	Residual Risk: .....	12
3	Equipment Setup and Use: .....	13
3.1	Prior to use: .....	13
3.2	Setup: .....	13
3.3	Device controls and indicators: .....	20
3.4	Storage, Handling and Removal Instructions: .....	21

# INSTRUCTIONS FOR USE

---

3.4.1	Storage and Handling:.....	21
3.4.2	Removal Instruction: .....	21
3.5	Troubleshooting Guide: .....	21
3.6	Device Maintenance:.....	21
4	Safety Precautions and General Information: .....	22
4.1	General Safety Warnings and Cautions:.....	22
4.2	Product Specifications:.....	23
4.3	Sterilization Instruction: .....	23
4.4	Cleaning and Disinfection Instruction: .....	23
5	List of Applicable Standards:.....	24

# INSTRUCTIONS FOR USE

---

## 1 General Information:

Allen Medical Systems, Inc. is a subsidiary of Hill-Rom, Inc., (NYSE: HRC), a leading worldwide manufacturer and provider of medical technologies and related services for the health care industry. As an industry leader in patient positioning, our passion is improving patient outcomes and caregiver safety, while enhancing our customers' efficiency. Our inspiration comes from providing innovative solutions to address our customers' most pressing needs. We immerse ourselves in our customers' world, to better address these needs and the daily challenges of their environment. Whether developing a solution to address patient positioning challenges or creating a system to offer safe and effective surgical site access for the surgical team, we are committed to providing products of exceptional value and quality.

Allen products are backed by responsive and reliable service and complimentary on-site product demonstrations.

### 1.1 Copyright Notice:

#### Revision

© 2019 Allen Medical Systems Inc. ALL RIGHTS RESERVED.

No part of this text shall be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopying, recording, or by any information or retrieval system without written permission from Allen Medical Systems, Inc. (Allen Medical).

The information in this manual is confidential and may not be disclosed to third parties without the prior written consent of Allen Medical.

### 1.2 Trademarks:

Trademark information can be found at [Allenmedical.com/pages/terms-conditions](https://www.allenmedical.com/pages/terms-conditions).

Products may be covered by one or more patents. Please consult listing at [Hill-rom.com/patents](https://www.hill-rom.com/patents) for any patent(s).

# INSTRUCTIONS FOR USE

---

## 1.3 Contact Details:

For ordering information please see catalog.

Allen Customer Service Contact Information:

 <b>North America</b>	 <b>International</b>
 (800) 433-5774	 +1 978 266 4200 ext.4286
 (978) 263-8846	 +1 978 266 4426

## 1.4 Safety Considerations:

### 1.4.1 Safety hazard symbol notice:

	<b>DO NOT USE IF PRODUCT SHOWS VISIBLE DAMAGE OR MATERIAL DEGRADATION.</b>
--	--

### 1.4.2 Equipment misuse notice:

Do not use the product if package is damaged or unintentionally opened before use. All modifications, upgrades, or repairs must be performed by an authorized specialist.

### 1.4.3 Notice to users and/or patients:

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**Note:** Refer to the surgical table manufacturer's user guide for instructions on use. Always refer to the surgical table manufacturer's weight limits.

	<b>NEVER EXCEED THE WEIGHT CAPACITY OF THE OPERATING ROOM TABLE</b>
---	---

# INSTRUCTIONS FOR USE

---

## 1.4.4 Safe disposal:


Customers should adhere to all federal, state, regional, and/or local laws and regulations as it pertains to the safe disposal of medical devices and accessories. If in doubt, the user of the device shall first contact Hill-Rom Technical Support for guidance on safe disposal protocols.

## 1.5 Operating the system:

### 1.5.1 Applicable Symbols:

Symbol used	Description	Reference
	Indicates the device is a medical device	MDR 2017/745
	Indicates the medical device manufacturer	EN ISO 15223-1
	Indicates the manufacturer's serial number. The device serial number is encoded as <b>1YYWWSSSSSS</b> . <ul style="list-style-type: none"><li>• <b>YY</b> indicates the year of manufacture. i.e. <b>118</b>WWSSSSSS where 18 represents the year 2018.</li><li>• <b>WW</b> indicates the number of the manufacturing week per a standard shop calendar. (Leading zeros included.)</li><li>• <b>SSSSSS</b> is a sequential unique number.</li></ul>	EN ISO 15223-1
	Indicates the medical device Global Trade Item Number	21 CFR 830 MDR 2017/745
	Indicates the manufacturer's lot code using the Julian Date yyddd, where <b>yy</b> indicates the last two digits of the year and <b>ddd</b> indicates the day of the year. i.e. April 4 <sup>th</sup> , 2019 would be represented as 19094.	EN ISO 15223-1
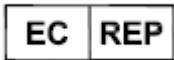
## INSTRUCTIONS FOR USE

	Indicates the date when the medical device was manufactured	EN ISO 15223-1
	Indicates the manufacturer's catalogue number	EN ISO 15223-1
	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions.	EN ISO 15223-1
	Indicates the device do not contain natural rubber or dry natural rubber latex	EN ISO 15223-1
	Indicates the authorized representative in the European Community	EN ISO 15223-1
	Indicates the Medical Device complies to REGULATION (EU) 2017/745	MDR 2017/745
	Indicates a Warning	IEC 60601-1
	Indicates the need for the user to consult the instruction for use	EN ISO 15223-1

### 1.5.2 Intended User and Patient Population:

**Intended User:** Surgeons, Nurses, Doctors, Physicians and OR healthcare professionals involved in the device intended procedure. Not intended for Lay persons.

**Intended Populations:**

This device is intended to be used with patients that do not exceed the weight in the safe working load field specified in the product specification section 4.2

# INSTRUCTIONS FOR USE

---

## 1.5.3 Compliance with medical device regulations:

This Product is a non-invasive, Class I Medical Device. This system is CE-marked according to Annex VIII, Rule 1, of the Medical Device Regulations (REGULATION (EU) 2017/745)

## 1.6 EMC considerations:

This is not an electromechanical device. Therefore, EMC Declarations are not applicable.

## 1.7 EC authorized representative:

HILL-ROM SAS  
B.P. 14 - Z.I. DU TALHOUE  
56330 PLUVIGNER  
FRANCE  
TEL: +33 (0)2 97 50 92 12

## 1.8 Manufacturing Information:

ALLEN MEDICAL SYSTEMS, INC.  
100 DISCOVERY WAY  
ACTON, MA 01720 USA  
800-433-5774 (NORTH AMERICA)  
978-266-4200 (INTERNATIONAL)

## 1.9 EU Importer Information:

Baxter Medical Systems GmbH + Co. KG  
Carl-Zeiss Straße 7-9  
07318 Saalfeld/Saale  
Germany


## 1.10 Authorised Australian sponsor:

Welch Allyn Australia Pty. Ltd.  
1 Baxter Drive  
Old Toongabbie, NSW 2146  
Australia

# INSTRUCTIONS FOR USE

## 2 System

### 2.1 System components Identification: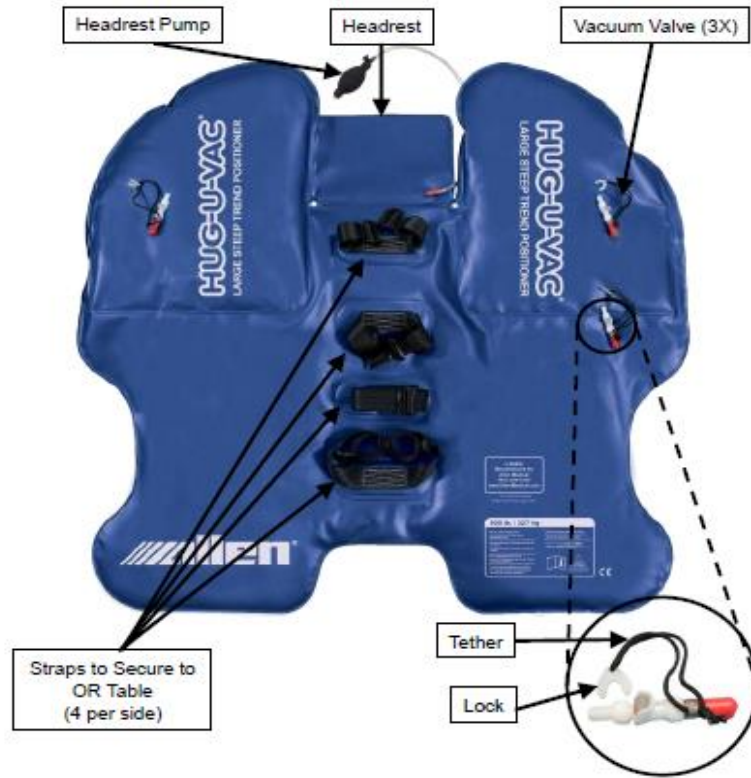

### 2.2 Product Code and Description:

A-60005 - Hug-U-Vac Large Steep Trend Positioner

### 2.3 List of Accessories and Consumable Components Table:

The following list are accessories and components that may be used with this device.

Name of Accessory	Product Number
Allen Steep Trend Secondary Supports	A-63600
Allen Large Security Strap	O-RSX2

Name of Consumable	Product Number
--------------------	----------------

# INSTRUCTIONS FOR USE

---

Large Steep Trend Positioner Disposable covers	A-60105
---	---------

**Note:** Consult the corresponding IFU for the products mentioned in the above table.

## 2.4 Indication for use:

The Hug-U-Vac Trend Positioner is used in a variety of surgical procedures including, but not limited to gynecology, urology, laparoscopy, general and robotic surgery. These devices are capable of being used with a broad patient population as determined appropriate by the caregiver or institution.

## 2.5 Intended use:

The Hug-U-Vac Trend Positioner is designed to Stabilize patient on OR table in a variety of surgical procedures including, but not limited to gynecology, urology, laparoscopy, general and robotic surgery. These devices are intended to be used by healthcare professionals within the Operating Room setting.

## 2.6 Residual Risk:

This product complies with relevant performance, safety standards. However, device harm from misuse, device damage, malfunction, device or function hazards cannot be completely excluded.

# INSTRUCTIONS FOR USE

---

## 3 Equipment Setup and Use:

### 3.1 Prior to use:

- a. Inspect the product looking for any visible damage or sharp edges that could be caused by a drop or impact during storage.
- b. Make sure product has been properly cleaned and disinfected and wiped dry prior to each use.
- c. Inspect the product, looking for any visible punctures, tears or splits in the material.
- d. Inspect the valves, and test their operation to ensure proper function prior to placing the patient on the positioner.
- e. Inspect all straps and buckles, looking for any visible tears or fraying of the straps and any damage to the buckles.
- f. Ensure a valve lock is tethered to each valve prior to use.
- g. Ensure there are no sharp objects on the surface of the OR table that may puncture the positioner once it is in position.

### 3.2 Setup:


1. Place the Large Steep Trend positioner on the operating room (OR) table for final patient position, aligning the perineal cutout at the bottom of the positioner with the cutout on the OR table, and center the positioner longitudinally on the OR table.

## INSTRUCTIONS FOR USE

---

2. Secure the Hug-U-Vac® straps to the side rails of the OR table ensuring the positioner remains centered on the OR table.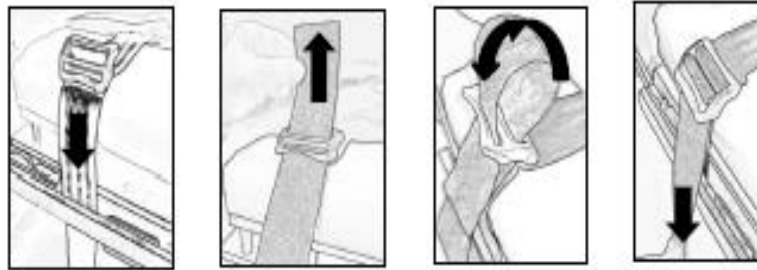

3. Smooth out the Hug-U-Vac® so that the internal microbeads are evenly distributed throughout the positioner.

4. Evacuate the main chamber until the internal beads no longer move freely. Then close the valve (this prevents the beads from shifting when patient is moved onto the positioner).

## INSTRUCTIONS FOR USE

---

5. Fit the disposable slipcover onto the Hug-U-Vac® following the disposable instructions.

6. Transfer the patient onto the positioner, be sure the positioner doesn't get pinched between stretcher and OR table.
7. Position the patient so that their torso is centered longitudinally on the positioner and their buttocks is aligned with the edge of the perineal cutout, ensuring their head rests on the inflatable pillow.

## INSTRUCTIONS FOR USE

---

8. Open the valves and allow the positioner to relax.
9. Position one person on either side of the patient to facilitate the activation of the positioner.


10. OR staff will use 1 hand to mold the support around the patient's shoulders, while taking care not to press the Hug-U-Vac® against the brachial plexus area or the patients' face. They will use their waist to contour the Hug-U-Vac® to the patients' upper arms and their free hand to fold the hand section up to secure the patients' hands. Make certain the positioner does not impinge on the surgical field.

## INSTRUCTIONS FOR USE

11. Once the Hug-U-Vac® has been properly positioned, check that the valve is open, attach surgical suction to the valve and begin evacuating the positioner. First evacuating the main chamber, then evacuating the auxiliary chambers at each shoulder of the patient.

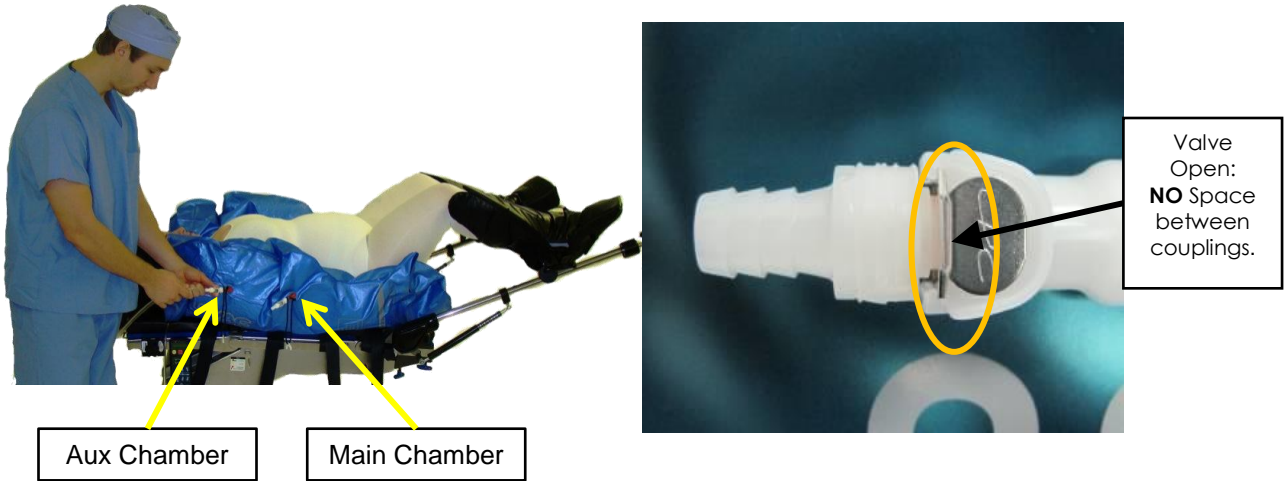
12. Continue pulling vacuum until the positioner becomes firm.

# INSTRUCTIONS FOR USE

13. Close and lock each valve after evacuating the chambers and detach suction hose.

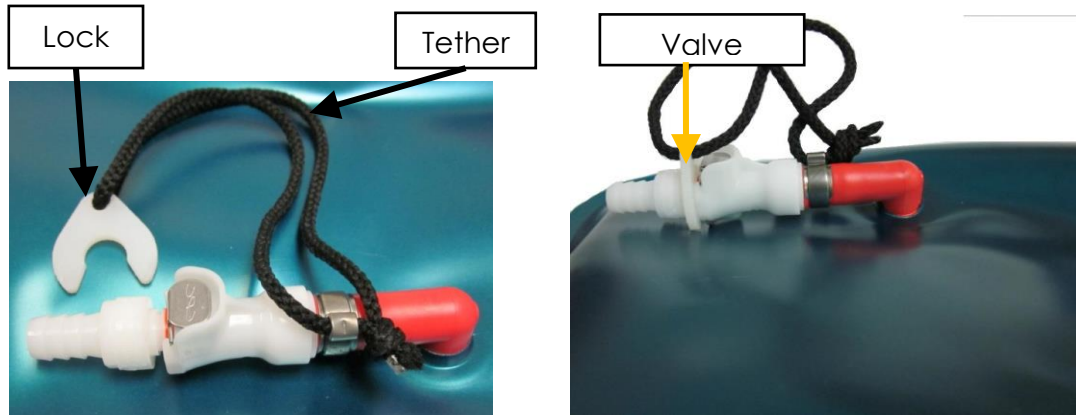Ensure that the valve is held securely by the nozzle when removing suction tubing or valve damage may occur.

14. Attach Allen Steep Trend Secondary Supports (STSS) or equivalent to the head of the OR table. If using STSS see IFU for A-63600\*.

- Position support approximately in line with the patient's acromion
- Ensure there is at least a 1" gap between the support and the Hug-U-Vac®.
- Place the OR table in Steep Trend and allow the patient to settle into the positioner.


## INSTRUCTIONS FOR USE

---

15. Return the table to the horizontal position, then readjust the secondary supports as necessary to ensure they are not pressing on the Hug-U-Vac®.

**\*NOTE:** The A-63600 Allen Steep Trend Secondary are only for backup in the unanticipated event the Hug-U-Vac® positioner loses suction during a case. They should not otherwise influence the positioning of the patient.

16. Prior to draping the patient, check for clearance of the brachial plexus area and for excess pressure on the shoulders. If the positioner is applying pressure to the brachial plexus area or shoulder, reposition the shoulder section of the Hug-U-Vac®.

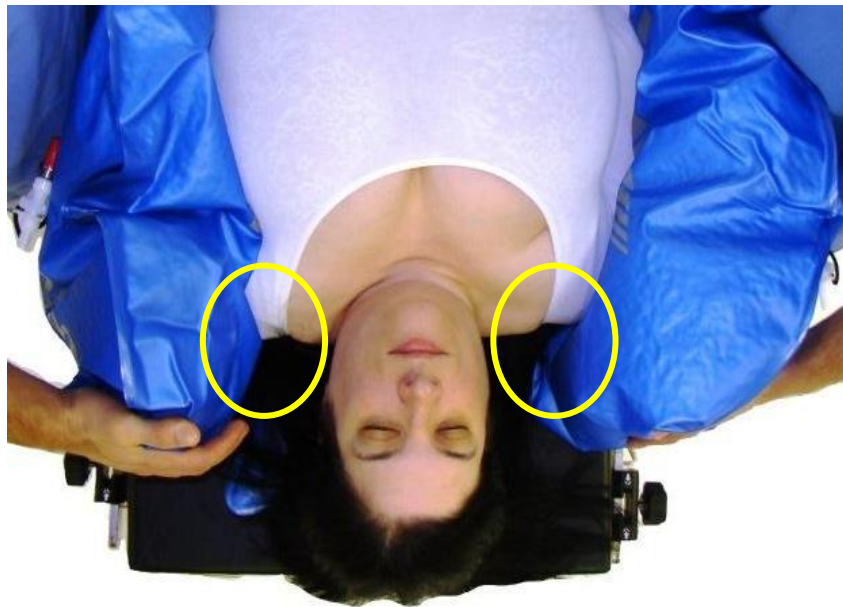**\*NOTE:** An Allen Security strap (O-RSX-2) may be strapped across the patient's chest, per the IFU for O-RSX-2 for added security.


**WARNING!** Once the desired form is obtained around the patient, DO NOT leave suction applied to the device. Close the Hug-U-Vac valve and stop suction during the procedure. After the procedure is completed, release the Hug-U-Vac valve and allow air to return into the device.

# INSTRUCTIONS FOR USE


## 3.3 Device controls and indicators:

### VALVE OPERATION:

**To Open Valve**

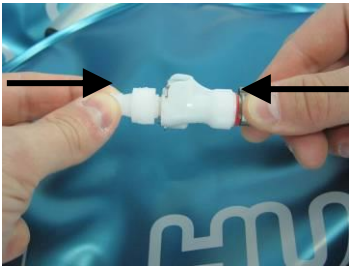
Push two couplings of valve together until an audible click is heard.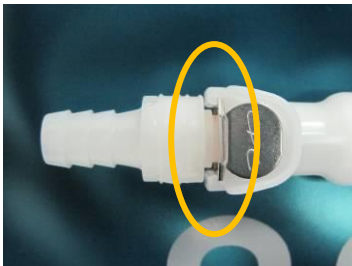


When **NO** space is visible between the couplings, the valve is in the **Open** position.

**To Close Valve**

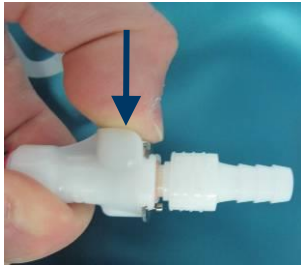Press the metal thumb latch on top of valve to **CLOSE** the valve, an audible click should be heard.

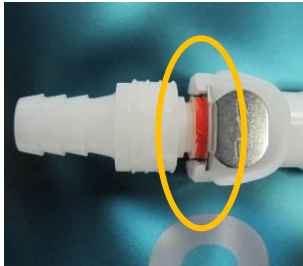
When space is visible between couplings, the valve is in the **Closed** position.

**To Lock Valve**

Press the plastic lock clip into the gap of the closed valve until it clicks into place.

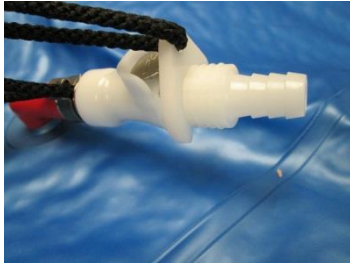
Never exceed the weight capacity of the O.R table.  
Always lock the valve close and remove suction tube

# INSTRUCTIONS FOR USE

---

## BUCKLE OPERATION:

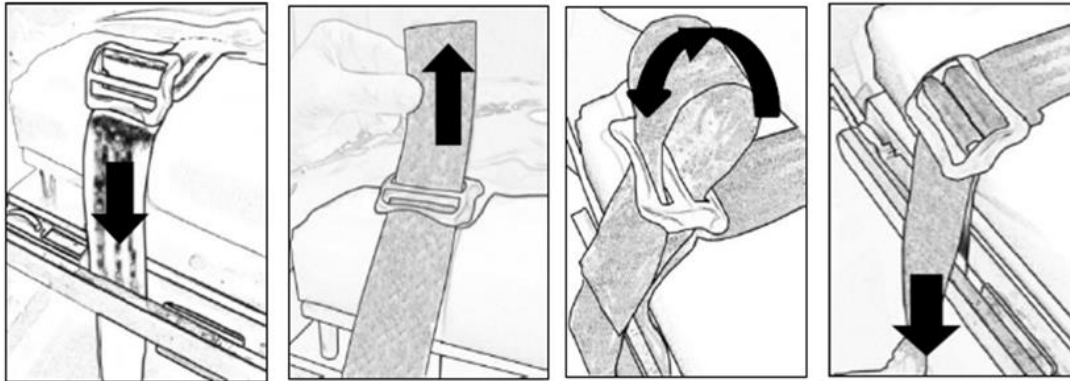## 3.4 Storage, Handling and Removal Instructions:

### 3.4.1 Storage and Handling:

The product should be stored in a clean and safe environment to prevent product damage. See storage Specifications under Product Specification section.

### 3.4.2 Removal Instruction:

- Position the table at a 0° angle.
- Remove the secondary support.
- Remove the vacuum from Hug-U-Vac.
- Safely remove the patient from the Hug-U-Vac (and O.R Table).
- Release the Hug-U-Vac straps from the side rails of the OR table and pick up and store the Hug-U-Vac. See storage specifications (Section 4.2)

**DO NOT** leave the Hug-U-Vac valve closed with suction still applied in between procedures or in storage. Leaving the suction connected will lead to collapsing and damage of the internal beads and thus, greatly reduce the life of the product.

## 3.5 Troubleshooting Guide:

This device does not have a troubleshooting guide. For technical support user of the device shall first contact Hill-Rom Technical Support.

## 3.6 Device Maintenance:

Make sure that all labels are installed and can be read. Replace labels as necessary by using a plastic scraper to remove the label. Use an alcohol wipe to remove any adhesive residue.

Contact Allen Medical Systems, Inc. if you need to repair or replace the device contact us using the information from the contact details section (1.3).

# INSTRUCTIONS FOR USE

---

## 4 Safety Precautions and General Information:

### 4.1 General Safety Warnings and Cautions:

#### **WARNING:**

- a. Do not use if product shows visible damage.
- b. Prior to using this device, please read the instructions for equipment set up and use. Familiarize yourself with the product before application on a patient.
- c. To prevent patient and/or user injury and/or equipment damage, examine the device and surgical-table side rails for potential damage or wear prior to use. Do not use the device if damage is visible, if parts are missing or if it does not function as expected.
- d. Product must be used in conjunction with the Allen Secondary Supports (A-63600) or equivalent, to ensure patient safety in the unlikely event of vacuum loss while in the Steep Trendelenburg position.
- e. Patient injury or death may occur if the Hug-U-Vac® positioner fails while the patient is in the steep Trendelenburg position, and a secondary support system is not in use.
- f. This Product should be used in conjunction with an Allen large security strap (O-RSX-2), secured across the patient's chest for added security.
- g. Always ensure product is properly and securely fastened to the operating table prior to use.
- h. Secure positioners straps only to the operating table's accessory rails.
- i. Never secure positioners straps to table width extenders or any secondary accessory rails.
- j. All modifications, upgrades, or repairs must be performed by an authorized specialist.
- k. Always ensure the vacuum valves are securely closed and locked, and that the positioner maintains vacuum prior to draping the patient or beginning any surgical procedure (3.3.1)
- l. Do not exceed the maximum patient weight limit or width as specified by the manufacturer.
- m. Do not store near extreme hot or cold.
- n. Not for use with K-Thermia heating pads or similar.
- o. DO NOT leave the Hug-U-Vac valve closed with suction still applied in between procedures or in storage. Leaving the suction connected will lead to collapsing and damage of the internal beads and thus, greatly reduce the life of the product.

#### **CAUTION:**

Do not exceed safe working load shown in the product specification table

# INSTRUCTIONS FOR USE

---

## 4.2 Product Specifications:

Mechanical Specifications	Description
Product Dimensions	51" x 54" (129 cm x 137 cm)
Material	Urethane, Polyurethane, Polystyrene beads, Black Webbing.
Safe Working Load on the device	500 lbs. patient (226 kg)
Overall Weight of Complete Device	10 lbs. (4.5 kg)
Storage Specifications	Description
Storage temperature	-29° C to +60° C
Storage Relative humidity range	15% to 85%
Operating temperature	This device is intended to be used in a controlled Operating Room environment.
Operating Relative humidity range	
Electrical Specifications	Description
Not Applicable.	Not Applicable.
Software Specifications	Description
Not Applicable.	Not Applicable.
Compatibility Specifications	Description
Operating Room Table Compatibility	The Hug-U-Vac Large Steep Trend Positioner is compatible any rail size using the Hug-U-Vac Straps

**Note:** Consult the corresponding IFU for the products mentioned in the above table.

## 4.3 Sterilization Instruction:

This device is not intended to be sterilized. Equipment damage may occur.

## 4.4 Cleaning and Disinfection Instruction:

### WARNING:

- Do not use bleach or products that contain bleach to clean the device. Injury or equipment damage can occur.
- After each use, clean the device with alcohol-based wipes.
- Do not put the device into water. Equipment damage can occur.
- Use a cloth and a quaternary ammonium disinfecting/cleaning solution to clean and disinfect the device.
- Read and follow the manufacturer's recommendation for low-level disinfection.

## INSTRUCTIONS FOR USE

---

- Read and follow the cleaning product's instructions. Use caution in areas where liquid can get into the mechanism.
- Wipe the device with a clean, dry cloth.
- Make sure that the device is dry before you store it or use it again.

### 5 List of Applicable Standards:

Sl. no	Standards	Description
1	EN 62366-1	Medical devices - Part 1: Application of usability engineering to medical devices
2	EN ISO 14971	Medical devices- Application of risk management to medical devices.
3	EN 1041	Information supplied by the manufacturer of medical devices
4	EN ISO 15223-1	Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements
5	EN ISO 10993-1	Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management process
6	IEC 60601-2-46	Medical electrical equipment - Part 2-46: Particular requirements for the basic safety and essential performance of operating tables
7	ISTA	International Safe Transit Association standards for package testing

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