WELCH ALLYN HARVEY DLX double and triple head stethoscopes

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WELCH ALLYN HARVEY DLX double y triple head stethoscopes

WELCH ALLYN HARVEY DLX double e triple head stethoscopes

WELCH ALLYN HARVEY DLX double ja triple head stethoscopes -stetoskoopit

WELCH ALLYN HARVEY DLX double i triple head stethoscopes

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Welch Allyn, Inc. ("Welch Allyn") assumes no responsibility for any injury to anyone that may result from (i) failure to properly use the product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual, or (ii) any illegal or improper use of the product.

Baxter, Hillrom, and Welch Allyn are trademarks of Baxter International, Inc., or its subsidiaries. Any other trademarks, product names, or brand images appearing herein are the property of their respective owners.

	Welch Allyn, Inc. 4341 State Street Road Skaneateles Falls, NY 13153 USA hillrom.com
	Welch Allyn, Inc. is a subsidiary of Hill-Rom Holdings, Inc.

EC REP	and EU IMPORTER Welch Allyn Limited Navan Business Park, Dublin Road Navan, Co. Meath C15 AW22 Ireland
CE	Authorized Australian Sponsor Welch Allyn Australia Pty Limited 1 Baxter Drive Old Toongabbie NSW 2146 Australia

ENGLISH

Instructions for use

Intended use

Welch Allyn stethoscopes are used for diagnostic purposes by projecting the sounds associated with the heart, arteries, and veins and other internal organs.

Indications for use

Auscultation during physical assessment.

Symbol descriptions

For information on the origin of these symbols, see the Welch Allyn symbols glossary: welchallyn.com/symbolsglossary

MD	#
Medical device	Product Identifier
EC REP	EC REP
Manufacturer	Authorized Representative in the European Community
LOT	i
Lot code	Consult Instructions for use
REF	GTIN
Reorder number	Global Trade Item Number
	LATEX
Keep dry	Not made with natural rubber latex

	Recyclable
Fragile	
!	CAUTION The caution statements in this manual identify conditions or practices that could result in damage to the equipment or other property, or loss of data.
!	WARNING The warning statements in this manual identify conditions or practices that could lead to illness, injury, or death. Warning symbols will appear with a grey background in a black and white document.

Separate collection of Electrical and Electronic Equipment. Do not dispose as unsorted municipal waste.

Warnings

- !** **WARNING** Cross-contamination of stethoscope could cause patient infection. Clean and disinfect stethoscope between uses as described in the Instructions for use.
- !** **WARNING** Eartips can be swallowed and cause a choking hazard. Ensure all parts are properly attached and stored.
- !** **WARNING** Stethoscope hoses can be a strangulation hazard. Keep away from children.

Residual risk

This product complies with relevant electro-magnetic interference, mechanical safety, performance, and biocompatibility standards. However, the product cannot completely eliminate potential patient or user harm from the following:

- Harm or device damage associated with electro-magnetic hazards,
- Harm from mechanical hazards,
- Harm from device, function, or parameter unavailability,
- Harm from misuse error, such as inadequate cleaning, and/or
- Harm from device exposure to biological triggers that may result in a severe systemic allergic reaction.

Chestpieces

Both the double head and triple head offer a flat diaphragm and bell. The triple head also utilizes a corrugated diaphragm for additional auscultatory capabilities. Selection is accomplished by holding the chestpiece stem in one hand and rotating the chestpiece until a click is heard or felt.

Flat diaphragm

"The workhorse." Excellent for all sounds and murmurs. Best for higher frequency heart and lung sounds. Also ideal to identify splitting of sounds, systolic clicks and ejection sounds (4.76 cm diameter).

Corrugated diaphragm

Amplifying quality. Excellent for an overview of heart sounds. Especially good for low-frequency gallop sounds and murmurs (4.76 cm diameter).

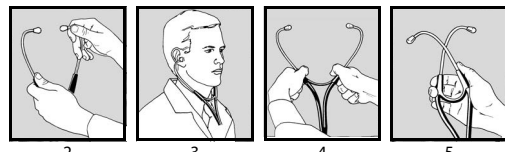
Bell

Provides excellent detection and clarity for low and medium frequency heart sounds and murmurs such as a faint gallop or diastolic rumble (2.79 cm diameter).

Headset adjustment

Adjust the headset as follows:

- Select eartips that provide the maximum acoustic seal and comfort.

- To adjust the angle of the binaurals, hold the spring in one hand and rotate each binaural to the correct angle.
- Make sure that the binaural tubes are pointed slightly forward going into the ear canal.
- To reduce spring tension, grasp each eartube at its bend and gently pull outwards.
- To increase spring tension, squeeze the eartubes together.

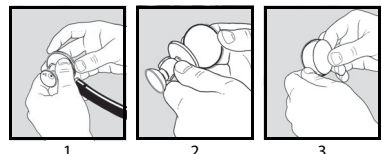
Cleaning, disinfection, and maintenance

Clean and disinfect the stethoscope to prevent possible cross-contamination.

- Use 70% isopropyl alcohol or soapy water to wipe down:
- Eartubes and tubing
- Eartips (remove for thorough cleaning)
- Exterior of the diaphragm

Under normal use the rim and diaphragm should rarely need to be removed for cleaning.

To remove the diaphragm, follow the instructions below.

- Hold the bell and turret section in one hand, and exert pressure on the back of the diaphragm. Grip the threaded ring of the diaphragm with your other hand and turn the ring counterclockwise to loosen.
- Separate the threaded ring from the turret section.
- Press the diaphragm disc out of the metal ring and replace with new disc. Thread the ring back on to chestpiece. Routinely inspect the stethoscope for wear and damage. Do not use if you see signs of damage, if the instrument malfunctions, appears not to be working properly, or if you notice a change in performance. Contact Hillrom Technical Support for assistance.

Disposal

Users must adhere to all federal, state, regional, and/or local laws and regulations as it pertains to the safe disposal of medical devices and accessories. If in doubt, the user of the device shall first contact Hillrom Technical Support for guidance on safe disposal protocols.

PATENT/PATENTS

hillrom.com/patents

May be covered by one or more patents. See above Internet address. The Hill-Rom companies are the proprietors of European, US, and other patents and pending patent applications.

Hillrom Technical Support

For information about any Hillrom product, contact Hillrom Technical Support at hillrom.com/en-us/about-us/locations

Notice to users and/or patients in the EU

Any serious incident that has occurred in relation to the device, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Specifications

Head diameter 1.9in (48.3 mm)
Overall length 28in (711 mm)
Tubing length 17in (431.8 mm)

Lot code

Lot decoder YY-JJJ where YY = last 2 digits of the year, JJJ = consecutive day of the Julian year.

Accessories

5079-315 HARVEY DLX ACCESSORY KIT

Warranty

The HARVEY DLX stethoscope comes with a 10-year warranty against parts and manufacturing defects. Should a material or workmanship defect be discovered, Welch Allyn will repair or replace the product at no charge.

This warranty does not cover damage in shipment or failure caused by tampering, obvious carelessness or abuse. Hillrom reserves the right to make changes without notice in design, specifications and models. The only warranty Hillrom makes is the express written warranty extended on the sale or rental of its products.

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AB'deki Kullanıcılara ve/veya Hastalara Yönelik Bildirim