<b>Document Description:</b>	CSB-Modality Manager SSO Security Vulnerability	<b>Document Number:</b>	80029474
		Version:	A
	<b>Welch Allyn</b> ™		
Printed or electronic versions of this document not accessed directly from the designated Welch Allyn Controlled Quality Information System are For Reference Only.			

Customer Service Bulletin						
Products:	Q-Stress <sup>®</sup> System, XScribe <sup>™</sup> System, HScribe <sup>™</sup> System, Vision Express <sup>™</sup> System, Diagnostic Cardiology Suite <sup>™</sup> (DCS) ECG, Connex <sup>®</sup> Cardio ECG, RScribe <sup>™</sup> System  Date: 29-NOV-2021					
Subject:	Unauthorized access to application functions and patient data					
HW Version(s) Affected:	N/A	SW Version(s) Affected:	,			
Serial Numbers Affected:	N/A	Lot or Date Code Affected:	N/A			

Disposition:	Reactive Service Use			
Distribution:	⊠ Custon	ner Care		⊠ Field Service
	⊠ ASPs	□ Distributors	□ Customers	□ Company
Training Required:	⊠ Yes	□ No		
Summary:	Q-Stress, XScribe, HScribe, Vision Express, DCS (Connex Cardio) and RScribe, systems configured to use Single Sign-on via their modality manager configuration settings, are affected by an improper authentication defect. This vulnerability allows the application to accept manual entry of any Active Directory (AD) account that is provisioned in the application without supplying a password and then will grant access to the application as the supplied AD account, with all associated privileges.			

THIS INFORMATION IS THE PROPERTY OF WELCH ALLYN, INC. AND AS SUCH SHALL NOT BE REPRODUCED, COPIED, OR USED AS A BASIS FOR THE MANUFACTURE OR SALE OF EQUIPMENT OR DEVICES WITHOUT THE EXPRESS WRITTEN PERMISSION OF WELCH ALLYN, INC.

Workaround:	Disable the Single Sign-on (SSO) feature in the respective modality manager configuration settings. Please refer to the IFU and/or Service Manual for instructions on how to disable SSO. Once SSO is disabled, the software applications will require AD user credentials (username and password) upon startup.
Resolution:	This issue has been documented into our feedback system (B104648) for correction in future software releases for each modality.

THIS INFORMATION IS THE PROPERTY OF WELCH ALLYN, INC. AND AS SUCH SHALL NOT BE REPRODUCED, COPIED, OR USED AS A BASIS FOR THE MANUFACTURE OR SALE OF EQUIPMENT OR DEVICES WITHOUT THE EXPRESS WRITTEN PERMISSION OF WELCH ALLYN, INC.