



### Products affected


- Q-Stress® 6.x.x (all versions from 6.0.0 to 6.3.0) and XScribe 5.xx to 6.x.x (all versions from 5.01 to 6.3.0)
- HScribe™ 5.xx and 6.x.x (all versions from 5.01 to 6.4.0) and Vision Express 6.x.x (all versions from 6.1.0 to 6.4.0)
- RScribe™ 5.xx, 6.x.x, and 7.x.x (all versions from 5.01 to 7.0.0)

### Notes

- This instruction is applicable for systems currently configured with the logon authentication method set to **Active Directory** and with **Single Sign On** (SSO) turned on.
- This process will briefly interrupt service. Perform these steps at an appropriate time.
- After turning off SSO, users will then need to enter their account name and password when logging into the system.

### Steps to turn off SSO

1. Log into Windows as an administrator.
2. In the Start menu go to **Mortara Modality Manager** and then select **Modality Manager Configuration Tool**.
3. When prompted that services will be stopped, click **OK**.
4. The **Modality Manager Configuration Utility** window will be displayed.

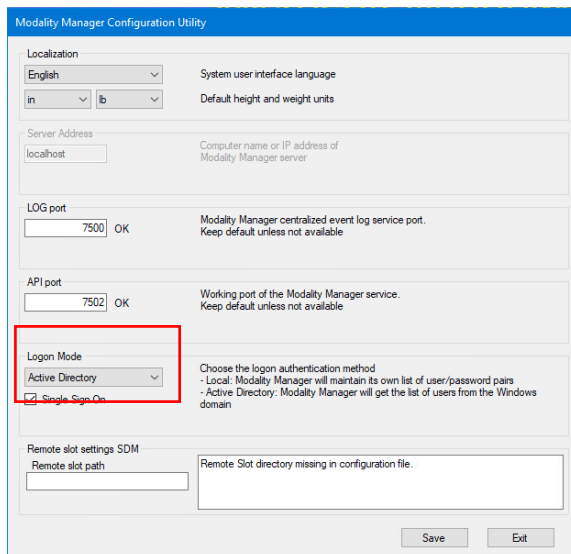

5. Make sure **Single Sign On** is off.
6. Click **Save** then **Exit**.