

Document Number: 80029856

Document Version: A

Printed or electronic versions of this document not accessed directly from the designated Hillrom Controlled Quality Information System are For Reference Only.

Customer Service Bulletin

Product:	Reusable Ear/Nasal Specula					
Subject:	Cleaning and Disinfection					
HW Version(s) Affected:	N/A	SW Version(s) Affected:	N/A			
Serial Numbers Affected:	N/A	Lot or Date Code Affected:	N/A			
Classification:	As Needed					
Distribution:	⊠ Customer Care	⊠ Product Service	☐ Field Service			
	☐ ASPs ⊠ Distributors	□ Customers	☐ Company Confidential			
Cleaning and Disinfection Instructions provided in the Otoscope IFU applies equally to the reusable ear specula. Reusable Specula REF Numbers: 24302-U, 24303-U, 24304-U, 24305-U, 22002, 22003, 22004, 22005, 22009 used with Welch Allyn Diagnostic, MacroView, Operating, Consulting and Pneumatic Otoscopes and 3.5V Nasal Illuminator. Clean with mild soap and water, disinfect with 70% - 90% isopropyl alcohol (IPA). Justification for use of 70% - 90% IPA ≥1 minute contact time for Low-Level and Intermediate-						
Level Disinfection on smooth, hard surfaces can be found in Table 1A, Page 106 of CDC Guideline for Disinfection and Sterilization in Healthcare Facilities (2008)						
https://www.cdc.gov/infectioncontrol/pdf/guidelines/disinfection-guidelines-H.pdf						

END OF BULLETIN

Version	Sec, Pg, Para Changed	Change Made	Date Version Created	Version Created By (initials)
---------	-----------------------------	-------------	-------------------------	-------------------------------------

THIS INFORMATION IS THE PROPERTY OF HILLROM HOLDINGS AND AS SUCH SHALL NOT BE REPRODUCED, COPIED, OR USED AS A BASIS FOR THE MANUFACTURE OR SALE OF EQUIPMENT, DEVICES, OR SOFTWARE WITHOUT THE EXPRESS WRITTEN PERMISSION OF HILLROM.



Document Number: 80029856

Document Version: A

Α	N/A	Initial Release	2022-05-23	DJT