

## Welch Allyn Connex 360 Mobile Stand

### Assembly instructions

REF 775652, 80030030 Ver. F  
Revision date: 2025-01

Baxter, Welch Allyn and Connex 360 are trademarks  
of Baxter International Inc. or its subsidiaries.

[baxter.com](https://baxter.com)

**Baxter Technical Support**  
[baxter.com/contact-us](https://baxter.com/contact-us)





[bax.to/getstarted](https://bax.to/getstarted)




(91)775652


REF 9000-C360

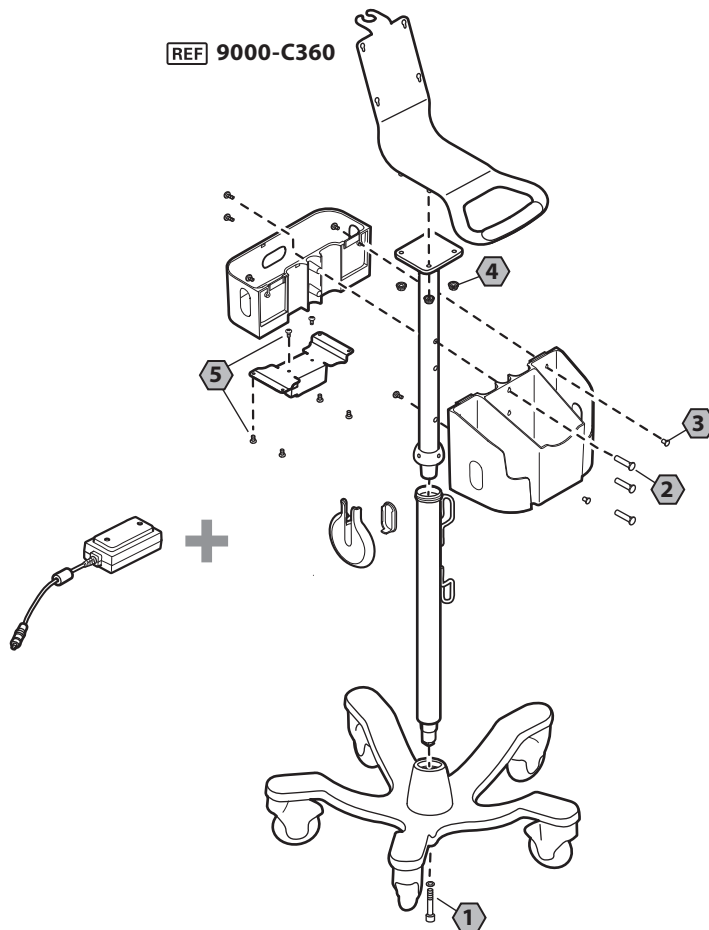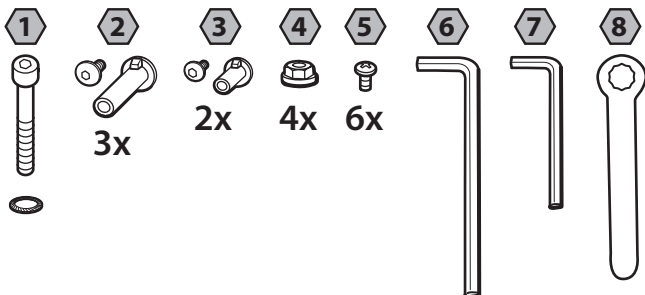

1

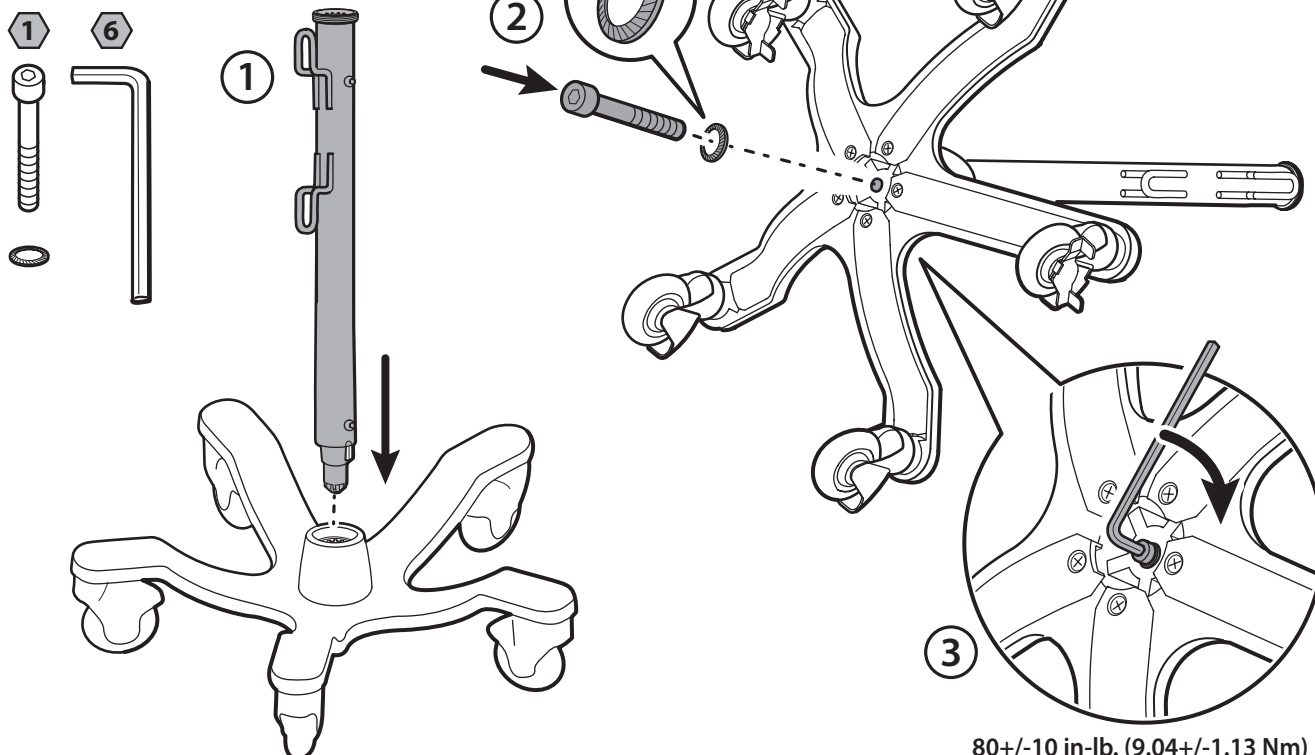


80+/-10 in.-lb. (9.04+/-1.13 Nm)

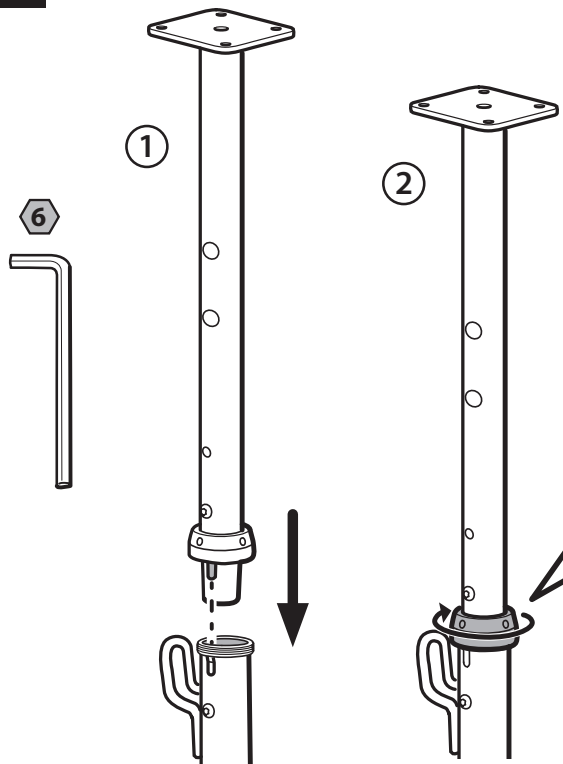


(EN) Insert the hex wrench as shown and tighten the collar clockwise.  
DO NOT turn the wrench.


(FR) Insérez la clé à six pans comme indiqué et serrez le système de verrouillage dans le sens des aiguilles d'une montre. Ne tournez PAS la clé.

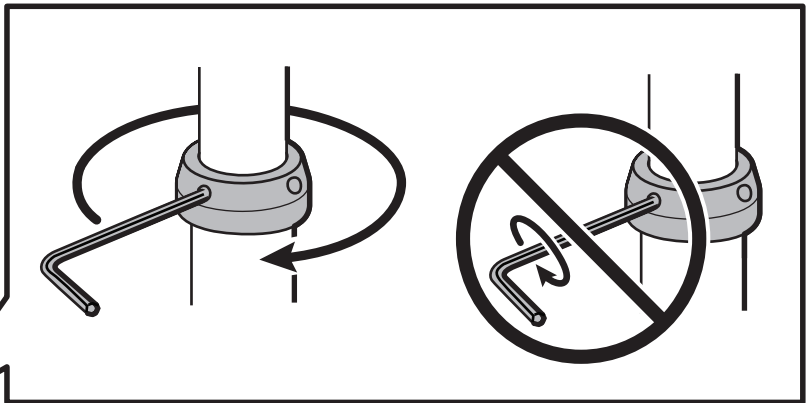


20+/-5 lb. (89+/-22 N)


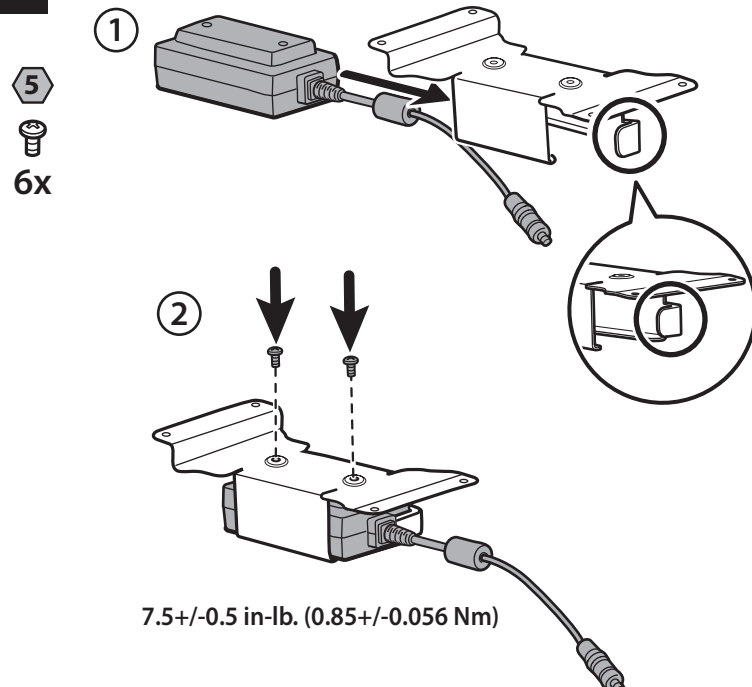


7.5+/-0.5 in-lb. (0.85+/-0.056 Nm)

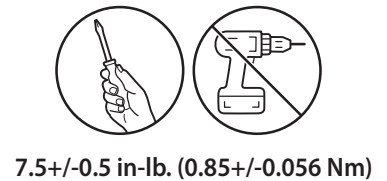


7.5+/-0.5 in-lb. (0.85+/-0.056 Nm)

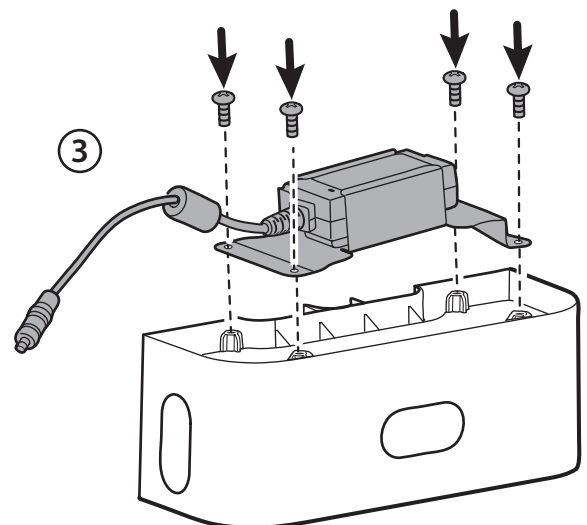

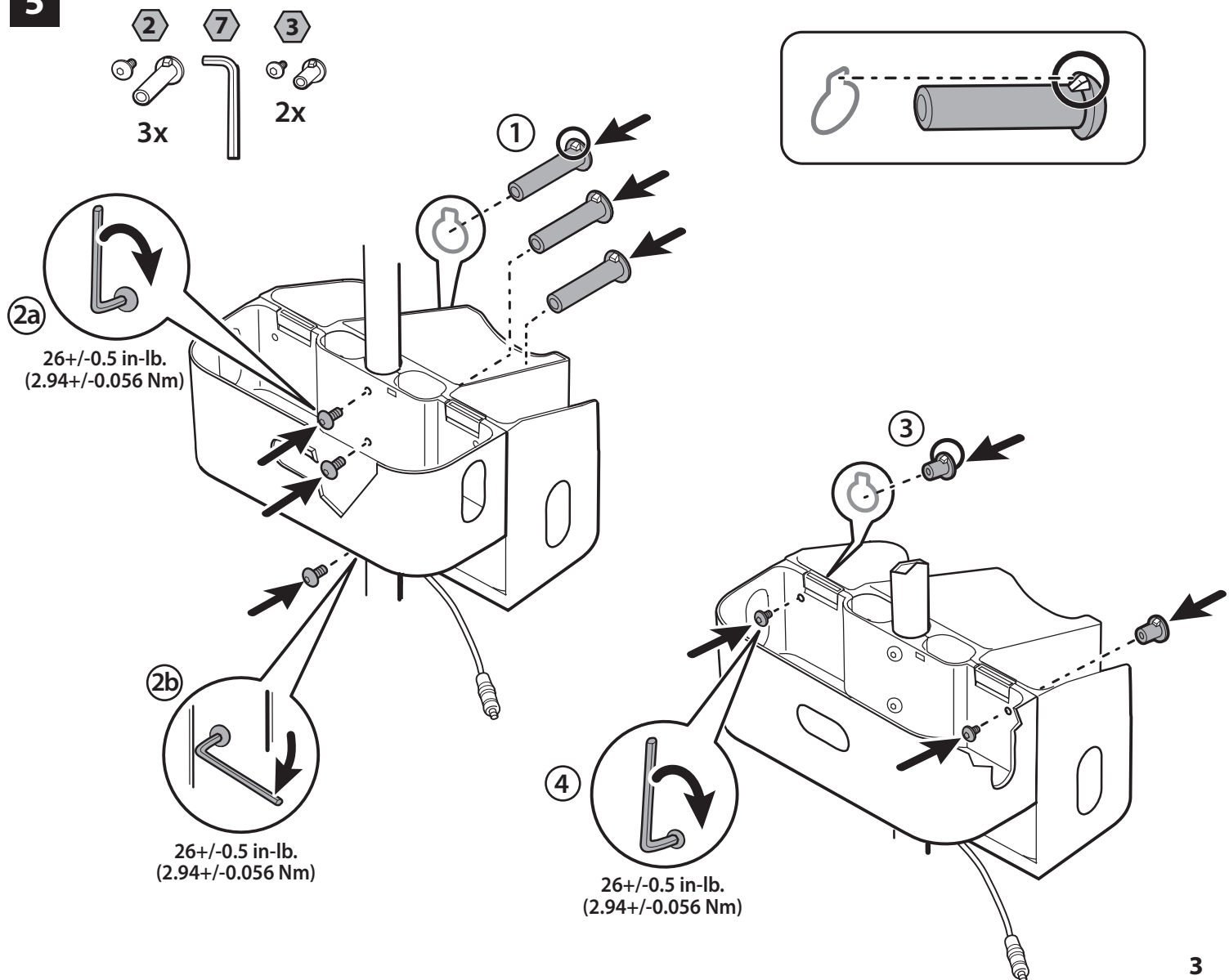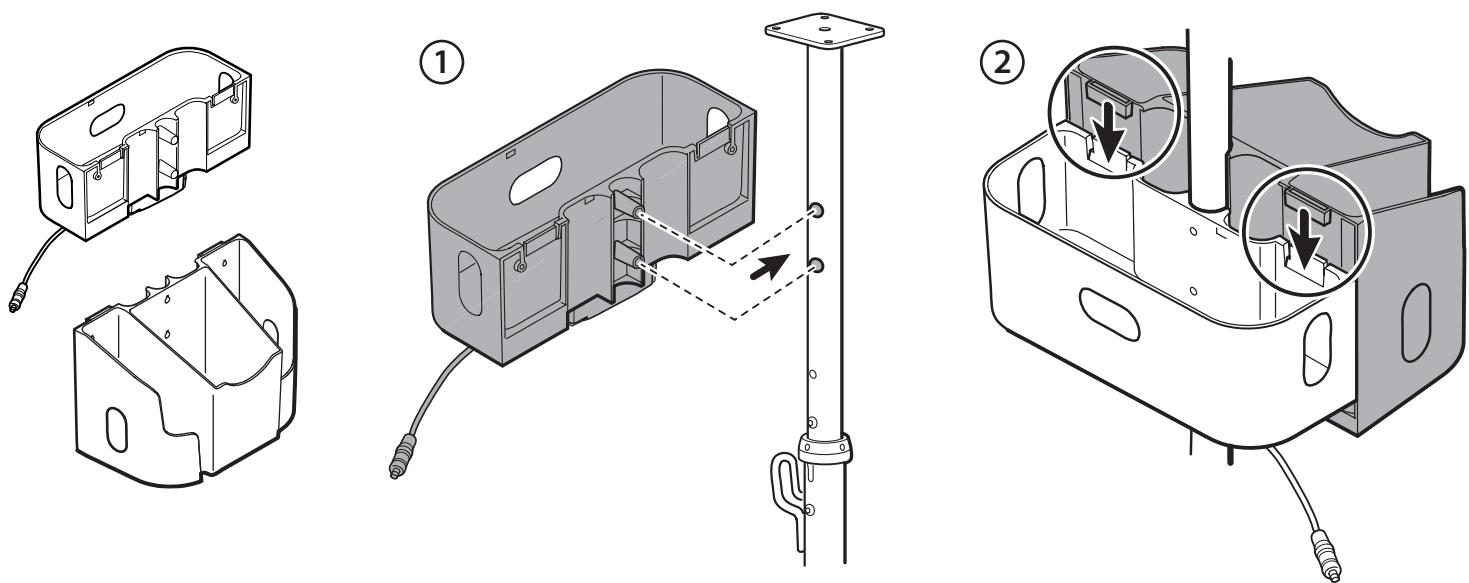

6

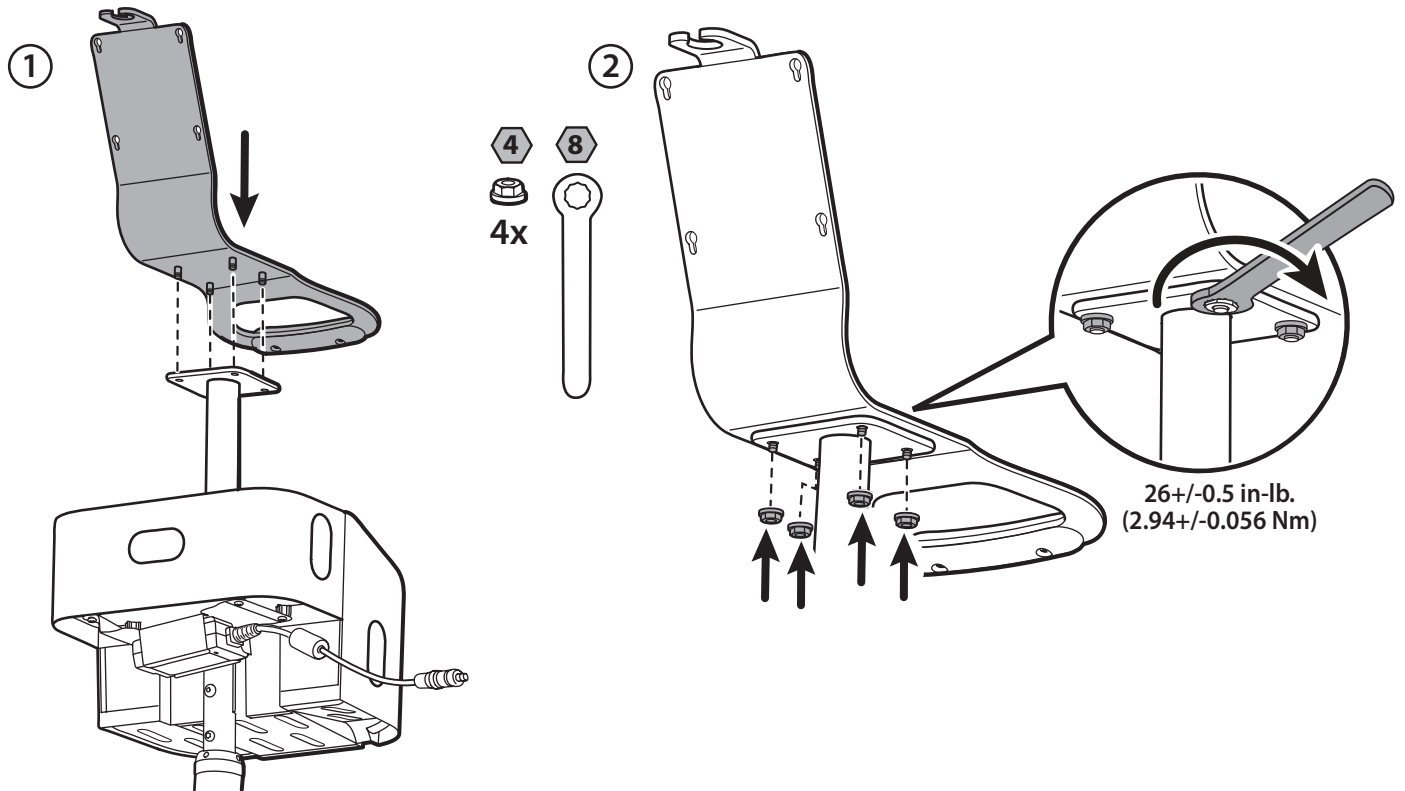

7

