# ***Baxter***

**Welch Allyn**

**KleenSpec**

800 Series Cordless Illumination System

Instructions for use

Baxter, KleenSpec, and Welch Allyn are trademarks of Baxter International Inc. or its subsidiaries.

The Bluetooth word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. Any use of such marks by Baxter is under license.

For information about any Baxter product, contact Baxter Technical Support: [baxter.com/contact-us](http://baxter.com/contact-us).

Notice to Users and/or Patients in EU:

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

777186, 80031298E

This manual applies to 901070 VAGINAL SPECULUM LIGHTING SYSTEM.

Revised 2026-02

Welch Allyn, Inc.  
4341 State Street Road  
Skaneateles Falls, NY 13153 USA

[baxter.com](http://baxter.com)

Welch Allyn Limited  
Navan Business Park  
Dublin Road  
Navan, Co Meath  
C15 AW22, Ireland

Authorized Australian Sponsor  
Welch Allyn Australia Pty. Ltd.  
1 Baxter Drive  
Old Toongabbie NSW 2146  
Australia  
1800 445 576

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## Introduction

### Intended use

When used with the **Welch Allyn KleenSpec** Disposable Vaginal Speculum (vaginal speculum), **Welch Allyn KleenSpec** 800 Series Cordless Illumination System (the illuminator) provides illumination during pelvic examinations and other gynecological procedures, such as pap smears, dilation and curettage (D&C) biopsy, and electrosurgery.

The intended users of the device are clinicians who are qualified to perform a pelvic examination. The intended environment is any location where a pelvic examination is conducted (hospital, clinic, office, long term care facility, etc.). The intended patients are all female patients who are eligible for a pelvic examination and who the clinician determines will fit with the size specula that are available (extra small, small, medium, large).

### Contraindications

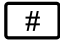
The illuminator (either by itself or in conjunction with a **KleenSpec** vaginal speculum) is not intended to be used for eye examinations or to provide a diagnosis.

### Symbols

The symbols illustrated on the following pages may appear in this Instructions for use or on the 800 Series Cordless Illuminator (illuminator) or charging station.

For information on the origin of these symbols, visit [bax.to/docs-wa-symbols](http://bax.to/docs-wa-symbols) for the Welch Allyn symbols glossary.

| Symbol   | Description   | Symbol   | Description                            |
|--|---|--|--|
|  <b>WARNING</b> | The warning statements in this manual identify conditions or practices that could lead to illness, injury, or death.<br><br>Warning symbols will appear with a gray background in a black and white document. |  | Keep dry                               |
|  <b>CAUTION</b> | The caution statements in this manual identify conditions or practices that could result in damage to the equipment or other property, or loss of data.   |  | Non-ionizing electromagnetic radiation |
|                 | Global Trade Item Number  |  | Consult Instructions for use           |
| <b>R<sub>x</sub> ONLY</b>  | For use by or on the order of a licensed medical professional   |  | This way up                            |
|                 | Manufacturer  |   | Fragile                                |

| Symbol  | Description   | Symbol   | Description   |
|---|---|--|---|
|    | Date of manufacture   |    | Unique device identifier  |
|    | Separate collection of Electrical and Electronic Equipment. Do not dispose as unsorted municipal waste. |     | Temperature limit   |
|    | Product identifier  |    | Humidity limitation   |
|    | Reorder number  |    | Recyclable  |
|    | Authorized Representative   |    | Medical device  |
|    | Lot code  |    | Rechargeable battery  |
|   |   | Li-ion   |   |
|   | Approved for use in Japan   |     | Type BF applied part when used with the KleenSpec Disposable Vaginal Speculum   |
|  | Regulatory Compliance Mark (Australia)  |  | Importer  |
| R-NZ  | Regulatory Compliance Mark (New Zealand)  |  | Follow instructions for use (IFU) -- mandatory action. A copy of the IFU is available on this website. A printed copy of the IFU can be requested and will be provided free of charge within 7 calendar days. |
|   |   | <a href="http://bax.to/docs">bax.to/docs</a>   |   |
|  | Atmospheric pressure limitation   |  |   |

## Warnings and Cautions

-  **WARNING** Patient injury risk. No modifications to the illuminator is allowed. Modification of the illuminator could be hazardous to patients and personnel.
-  **WARNING** Patient injury risk. This device comes with a power supply and/or power cord that is intended for use only with this device. The power supply or power cord have not been tested and approved for use with other devices that may have the same power connectors. If you cannot locate the original power supply and/or power cord, contact Baxter Technical Support: [baxter.com/contact-us](http://baxter.com/contact-us) to obtain replacement parts.
-  **WARNING** Patient injury risk. **KleenSpec** Cordless Illumination System is not suitable for use in the presence of a flammable anesthetic mixture with air, oxygen, or nitrous oxide. An explosion may result.

-  **WARNING** Inspect illuminator prior to use. Do not use if damaged.
-  **WARNING** To avoid eye discomfort, do not look into the beam.
-  **WARNING** Use only the cleaning methods and disinfection solutions described in this manual. Clean the device prior to disinfection.
-  **WARNING** The illuminator is a biohazard after use. Follow good hospital practices and the instructions for cleaning and disinfection of the illuminator within this manual prior to the next usage.
-  **CAUTION** Disconnect the AC power cord from the charger to the mains outlet to remove power to the device.
-  **CAUTION** The charging station and power cord are not protected from the ingress of liquid.
-  **CAUTION** Do not position any part of the illumination system directly against a wall. Do not position in any way that makes it difficult to disconnect AC power from the device. To remove all AC power from the illumination system, unplug the external power supply from the mains outlet.
-  **CAUTION** Check low battery indicator for battery charge. Ensure battery is in good condition, not bulging or cracking before charging.

## Residual Risk

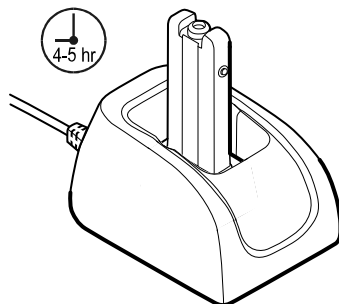
This product complies with relevant electro-magnetic interference, mechanical safety, performance, and biocompatibility standards. However, the product cannot completely eliminate potential patient or user harm from the following:

- Harm or device damage associated with electro-magnetic hazards,
- Harm from mechanical hazards,
- Harm from device, function, or parameter unavailability,
- Harm from misuse error, such as inadequate cleaning, and/or
- Harm from device exposure to biological triggers that may result in a severe systemic allergic reaction.

## Use and maintain the illuminator

### Charge the illuminator

Charge the illuminator before using it the first time.

1. Connect the charging base to AC power.
2. Place the illuminator (either direction) into the charging station.
3. Remove the charged illuminator when you are ready to use it.

Full charging takes 4 to 5 hours.


**NOTE** It is safe to leave the illuminator in the charging station after it is charged.

The charging base has the following status indicators.

| Status | Description   |
|--------|---|
| Off    | No AC power   |
| Green  | AC power / Full charge                                  |
| Amber  | Illuminator inserted into charging base and is charging |

## Use the illuminator for a pelvic examination

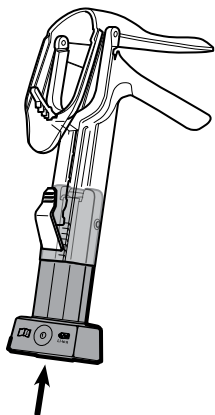1. Fully insert the illuminator into a **KleenSpec** Disposable Vaginal Speculum (in either direction).
2. Press the power button on the illuminator.

3. Complete the examination.
4. When the examination is completed, remove the speculum and press the power button to turn off the illuminator.
5. Remove the illuminator from the speculum.

## Clean and disinfect

The following are approved wipes for cleaning and disinfecting.

- **CaviWipes**
- Super **Sani-Cloth**
- 70% isopropyl alcohol

## Clean and disinfect the illuminator

Following are directions for cleaning and intermediate disinfecting of the illuminator.

1. Clean the illuminator.
  - a. Following the wipe manufacturer's instructions, wipe the entire illuminator to remove any visible debris.
  - b. Discard used wipe appropriately.
2. Disinfect the illuminator (intermediate level).
  - a. Follow wipe label instructions for appropriate contact times.
  - b. Discard used wipe appropriately.

## Clean the charging base

1. Unplug the power cord.
2. Use an approved wipe to remove visible debris.
3. Discard used wipe appropriately.

## Inspect the system

1. Examine all components of the cordless illumination system regularly. Components include the illuminator and charging base.
2. If any component is worn or damaged, replace it with a Baxter-approved part. For ordering information, contact your local Baxter representative: [baxter.com/contact-us](http://baxter.com/contact-us).

## Disposal

Disposal must be in accordance with the following steps:

1. Follow the cleaning and disinfection instructions presented in this instructions for use.
2. Segregate material in preparation for the recycling process.
3. Disassemble and recycle components based on the type of material:
  - Plastic to be recycled as plastic waste
  - Metal to be recycled as metal
    - Includes loose components containing more than 90% metal by weight
    - Includes screws and fasteners
  - Electronic components, including the power cord, to be disassembled and recycled as Waste of Electrical and Electronic Equipment (WEEE)
  - Batteries to be dismantled from the device and recycled as per WEEE

Users must adhere to all federal, state, regional, and/or local laws and regulations as it pertains to the safe disposal of medical devices and accessories. If in doubt, the user of the device shall first contact Baxter Technical Support for guidance on safe disposal protocols.

## Appendices

### Appendix A: Specifications

Charging station power supply classification: US, Canada, & International; Class I and internally powered

| Characteristic | Specification      |
|----------------|--------------------|
| Input          | 100-240v / 50-60Hz |
|                | 160-80 mA          |
| Output         | 5v DC              |
|                | 1400 mA            |

| Characteristic | Specification        |
|----------------|----------------------|
| Category       | Not AP/APG Equipment |

## Physical specifications

| Characteristic                          | Specification   |
|---|---|
| Illuminator                             | 1.96 W x 1.37 D x 3.74 H in.; 50 W x 35 D X 95 H mm   |
| Charger                                 | 3.14 x 4.33 x 2.55-4.60 in.; 80 W x 110 D X 65-117 H mm   |
| Power supply                            | 1.24 W x 2.16 L x 1.61 D in.; 31.5 W x 55 L x 41 D mm   |
| LED radiation output                    | <6.67 mW at 400-750 wavelengths   |
| Battery cell                            | Capacity 400 mAh<br>Voltage 3.7 V<br>Chemistry Li-Ion Polymer<br>Rechargeable Li-Ion Polymer<br>Battery Charge time 4 hours<br>On-Time use 80 minutes |
| Intentional radiator (charging station) | Frequency 112-205 kHz<br>Maximum output power 1.5 W<br>Type: inductive magnetic field   |

## Environment [temperature and humidity]

| Characteristic    | Specification  |
|-------------------|--|
| Operating         | +10 °C (50 °F) and +35 °C (95 °F)<br>700 hPa - 1060 hPa<br>30% - 75% non-condensing  |
| Transport/Storage | -20 °C (-4 °F) and +49 °C (120 °F)<br>500 hPa - 1060 hPa<br>15% - 95% non-condensing |

## Operation

When used with the vaginal speculum, the cordless illuminator provides illumination during pelvic examinations and other gynecological procedures, such as pap smears, dilation and curettage (D&C), biopsy, and electro-surgery.

## Safety, EMC and regulatory compliance

UL 60601-1 Medical Electrical Equipment, Part 1: General Requirements for Safety

CAN/CSA C22.2 No. 601.1-M90 Medical Electrical Equipment, Part 1: General Requirements for Safety

IEC 60601-1 Medical Electrical Equipment, Part 1: General Requirements for Safety, and associated CB Scheme Report and Certificate.

EN 60601-1 Medical Electrical Equipment, Part 1: General Requirements for Safety

EN 60601-1-2 Medical Electrical Equipment, Part 1-2: General Requirements for Safety – Collateral Standard: Electromagnetic Compatibility – Requirements and Tests

CISPR 11/EN 55011/AS\_NZS CISPR 11, RF Emissions

CISPR 11, Conducted Emissions

47 CFR Part 18

This device complies with Part 18 of the FCC rules.

CAN ICES-001 (A) / NMB-001 (A)

Hereby, Baxter declares that the radio equipment type Wireless Power Transfer (WPT), Qi Standard, 112KHz, 1.2Watts is in compliance with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at the following internet address: [bax.to/kleenspec-cordless-illumination-system](http://bax.to/kleenspec-cordless-illumination-system).

Country-specific standards are included in the applicable Declaration of Conformity.

Industry Canada (IC) Emissions

This device complies with Industry Canada license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

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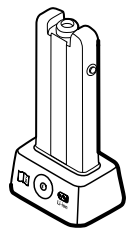
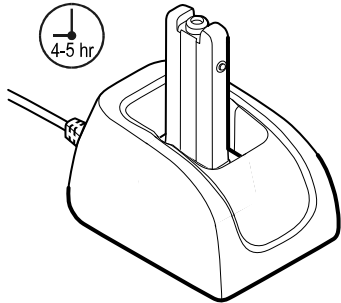
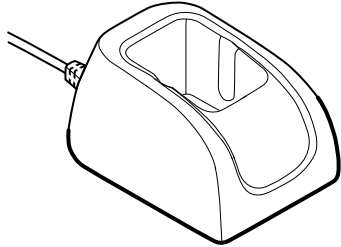
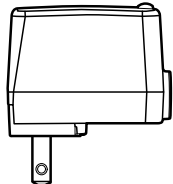
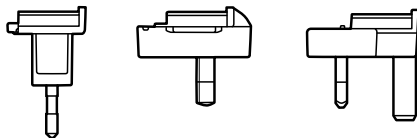
DD=Day of manufacture

## Appendix B: Accessories

| Part number | Description  | Illustration  |
|-------------|--|---|
| 80000       | <b>KleenSpec</b> Vaginal Speculum Cordless Illuminator |  |

| Part number     | Description  | Illustration  |
|-----------------|--|---|
| 80010           | <b>KleenSpec</b> Vaginal Speculum Cordless Illuminator with Charging Station/Domestic  |    |
| 80015           | <b>KleenSpec</b> Vaginal Speculum Cordless Illuminator with Charging Station/International   |   |
| 74010           | <b>KleenSpec</b> Charging Station/Domestic   |    |
| 74015           | <b>KleenSpec</b> Charging Station/International  |   |
| FW8002.1MUSB/05 | Power Supply<br> <b>NOTE</b> Available only with the 74010, 74015, 80010, and 80015 Charging Stations              |   |
| 1899414         | International Power Supply Adaptor Kit<br> <b>NOTE</b> Available only with the 74015 and 80015 Charging Stations. |  |
|                 |  <b>NOTE</b> USB Cable is not sold separately.  |   |

## Appendix C: Compatible devices

### **KleenSpec** Vaginal Specula—Premium 590 Series

- 590XS **KleenSpec** 590 Series Premium Disposable Vaginal Specula, X-Small
- 59000 **KleenSpec** 590 Series Premium Disposable Vaginal Specula, Small
- 59001 **KleenSpec** 590 Series Premium Disposable Vaginal Specula, Medium
- 59004 **KleenSpec** 590 Series Premium Disposable Vaginal Specula, Large

### **KleenSpec** Vaginal Specula—Premium 590 Series with Smoke Tube

- 59005 **KleenSpec** 590 Series Premium Disposable Vaginal Specula with Smoke Tube, Small
- 59006 **KleenSpec** 590 Series Premium Disposable Vaginal Specula with Smoke Tube, Medium

## Appendix D: EMC guidance and manufacturer's declarations

### EMC compliance

Special precautions concerning electromagnetic compatibility (EMC) must be taken for all medical electrical equipment. This device complies with IEC/EN 60601-1-2.

- All medical electrical equipment must be installed and put into service in accordance with the EMC information provided in the device's instructions for use.
- Portable and mobile RF communications equipment can affect the behavior of medical electrical equipment.

The device complies with all applicable and required standards for electromagnetic interference.

- It does not normally affect nearby equipment and devices.
- It is not normally affected by nearby equipment and devices.
- It is not safe to operate the device in the presence of high-frequency surgical equipment.
- It is good practice to avoid using the device in extremely close proximity to other equipment.

**WARNING** The use of the 800 Series adjacent to or stacked with other equipment or medical electrical systems should be avoided because it could result in improper operation. If such use is necessary, the 800 Series and other equipment should be observed to verify that they are operating normally.

**WARNING** Use only accessories recommended by Baxter for use with the 800 Series. Accessories not recommended by Baxter may affect the EMC emissions or immunity.

**WARNING** Maintain minimum separation distance between the 800 Series and portable RF communication equipment. Performance of the 800 Series may be degraded if proper distance is not maintained.

**NOTE** The 800 Series has no essential performance (patient safety) requirements.

The 800 Series is intended for use in the electromagnetic environment specified below. The customer or user of the 800 Series should assure that it is used in such an environment.

#### Electromagnetic emissions

| Emissions test  | Compliance | Electromagnetic environment - guidance   |
|---|------------|--|
| RF emissions<br>CISPR 11                                | Group 1    | The 800 Series uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.  |
| RF emissions<br>CISPR 11                                | Class B    | The 800 Series is suitable for use in all establishments, including domestic establishments and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.  |
| Harmonic emissions<br>IEC 61000-3-2                     | Class A    |  <b>WARNING</b> This equipment/system is intended for use by healthcare professionals only. This equipment/system may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures, such as re-orienting or relocating the 800 Illumination System or shielding the location. |
| Voltage fluctuations/flicker emissions<br>IEC 61000-3-3 | Complies   |  |

**Electromagnetic immunity**

| <b>Immunity test</b>   | <b>IEC 60601 test level</b>   | <b>Compliance level</b>  | <b>Electromagnetic environment—<br/>guidance</b>   |
|--|---|--|--|
| Electrostatic discharge (ESD)<br>IEC 61000-4-2   | ±8 kV contact<br>±2 kV, ±4 kV, ±8 kV,<br>±15 kV air   | ±8 kV contact<br>±2 kV, ±4 kV, ± 8 kV,<br>±15 kV air   | Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.  |
| Electrical fast transient/burst<br>IEC 61000-4-4   | ±2 kV for power supply lines<br>±1 kV for input/output lines  | ±2 kV<br>±1 kV   | Mains power quality should be that of a typical commercial or hospital environment.  |
| Surge<br>IEC 61000-4-5   | ±0.5 kV, ±1 kV line-to-line<br>±0.5 kV, ±1 kV, ±2 kV line-to-ground   | ±1 kV<br>±2 kV   | Mains power quality should be that of a typical commercial or hospital environment.  |
| Voltage dips, short interruptions and voltage variations on power supply input lines<br>IEC 61000-4-11 | 0 % U <sub>T</sub> , 0.5 cycle<br>At 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315°<br>0 % U <sub>T</sub> , 1 cycle<br>70 % U <sub>T</sub> , 25/30 cycles, single phase at 0°<br>0 % U <sub>T</sub> , 250/300 cycle | 0 % U <sub>T</sub> , 0.5 cycle<br>0 % U <sub>T</sub> , 1 cycle<br>70 % U <sub>T</sub> , 25/30 cycles<br>0 % U <sub>T</sub> , 250/300 cycle | Mains power quality should be that of a typical commercial or hospital environment. If the user of the 800 Series requires continued operation during power mains interruptions, we recommend powering them from an uninterruptible power supply or a battery. |
| Power frequency (50/60 Hz) magnetic field<br>IEC 61000-4-8   | 30 A/m  | 30 A/m   | Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.  |
| Conducted RF<br>IEC 61000-4-6  | 3 V <sub>rms</sub><br>150 kHz–80 MHz<br>6 V <sub>rms</sub> in ISM and amateur radio bands 150 kHz–80 MHz<br>80% AM at 1 kHz   | 3 V <sub>rms</sub><br>150 kHz–80 MHz<br>6 V <sub>rms</sub> in ISM and amateur radio bands 150 kHz–80 MHz<br>80% AM at 1 kHz                | Recommended separation distance:<br>1<br>$d = \left[ \frac{3.5}{V_1} \right] \sqrt{P}$<br>$d = \left[ \frac{12}{V_2} \right] \sqrt{P}$   |

| Immunity test | IEC 60601 test level              | Compliance level                  | Electromagnetic environment—<br>guidance        |
|---------------|-----------------------------------|-----------------------------------|---|
| Radiated RF   | 10 V/m                            | 10 V/m                            | $d = [\frac{23}{E_1}] \sqrt{P}$ 800 MHz–2.7 GHz |
| IEC 61000-4-3 | 80 MHz–2.7 GHz<br>80% AM at 1 kHz | 80 MHz–2.7 GHz<br>80% AM at 1 kHz | $d = [\frac{12}{E_1}] \sqrt{P}$ 80–800 MHz      |

**NOTE**  $U_T$  is the AC mains voltage prior to application of the test level.

**NOTE** At 800 MHz, the higher frequency range applies.

**NOTE** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

**NOTE** In separation-distance equations,  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and  $d$  is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol:

Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast, and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, consider an electromagnetic site survey. If the measured field strength in the location in which the 800 Series are used exceeds the applicable RF compliance level in this table, observe the 800 Series to verify normal operation. If you observe abnormal performance, additional measures may be necessary, such as reorienting or relocating the 800 Series.

- <sup>1</sup> Portable and mobile RF communications equipment should be used no closer to any part of the 800 Series, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.

### Recommended separation distances between portable and mobile RF communications equipment and the 800 Series

The 800 Series is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or user of the 800 Series can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the 800 Series as recommended below, according to the maximum output power of the communications equipment.

| Rated max.<br>output power of<br>transmitter (W) | Separation distance according to frequency of transmitter (m) |                                   |                                 |                                 |
|--|---|-----------------------------------|---------------------------------|---------------------------------|
|  | 150 kHz to 80 MHz<br>outside ISM bands                        | 150 kHz to 80 MHz<br>in ISM bands | 80 MHz to 800 MHz               | 800 MHz to 2.7 GHz              |
|  | $d = [\frac{3.5}{V_1}] \sqrt{P}$                              | $d = [\frac{12}{V_2}] \sqrt{P}$   | $d = [\frac{12}{E_1}] \sqrt{P}$ | $d = [\frac{23}{E_1}] \sqrt{P}$ |
| 0.01   | 0.12  | 0.20                              | 0.12                            | 0.23                            |
| 0.1  | 0.37  | 0.63                              | 0.38                            | 0.73                            |

### Separation distance according to frequency of transmitter (m)

| Rated max.<br>output power of<br>transmitter (W) | 150 kHz to 80 MHz<br>outside ISM bands | 150 kHz to 80 MHz<br>in ISM bands | 80 MHz to 800 MHz               | 800 MHz to 2.7 GHz              |
|--|--|-----------------------------------|---------------------------------|---------------------------------|
|  | $d = [\frac{3.5}{V_1}] \sqrt{P}$       | $d = [\frac{12}{V_2}] \sqrt{P}$   | $d = [\frac{12}{E_1}] \sqrt{P}$ | $d = [\frac{23}{E_1}] \sqrt{P}$ |
| 1  | 1.17                                   | 2.00                              | 1.20                            | 2.30                            |
| 10   | 3.69                                   | 6.32                              | 3.79                            | 7.27                            |
| 100  | 11.67                                  | 20.00                             | 12.00                           | 23.00                           |

**NOTE** For transmitters rated at a maximum output power not listed above, the recommended separation distance in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

**NOTE** At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

**NOTE** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.

### Test specifications for enclosure port immunity to proximity magnetic fields

| Test frequency | Modulation                            | Immunity test level (A/m)              |
|----------------|---------------------------------------|--|
| 30 kHz         | Continuous wave                       | 8                                      |
| 134.2 kHz      | Pulse modulation <sup>1</sup> 2.1 kHz | 65 (rms before modulation is applied)  |
| 13.56 MHz      | Pulse modulation <sup>1</sup> 50 kHz  | 7.5 (rms before modulation is applied) |

<sup>1</sup> The carrier shall be modulated using a 50% duty cycle square wave signal.

### Test specifications for enclosure port immunity to RF wireless communications equipment

| Test frequency (MHz) | Band <sup>1</sup> (MHz) | Service <sup>1</sup> | Modulation <sup>2</sup>                        | Maximum power (W) | Distance (m) | Immunity test level (V/m) |
|----------------------|-------------------------|----------------------|--|-------------------|--------------|---------------------------|
| 385                  | 380-390                 | TETRA 400            | Pulse modulation <sup>2</sup> 18 Hz            | 1.8               | 0.3          | 27                        |
| 450                  | 430-470                 | GMRS 460, FRS 460    | FM <sup>3</sup> ±5 kHz deviation<br>1 kHz sine | 2                 | 0.3          | 28                        |
| 710                  | 704-787                 | LTE band 13, 17      | Pulse modulation <sup>2</sup> 217 Hz           | 0.2               | 0.3          | 9                         |
| 745                  |                         |                      |  |                   |              |                           |
| 780                  |                         |                      |  |                   |              |                           |

|      |           |  |   |     |     |    |
|------|-----------|--|---|-----|-----|----|
| 810  | 800-960   | GSM 800/900,<br>TETRA 800, iDEN<br>820, CDMA 850,<br>LTE Band 5          | Pulse modulation <sup>2</sup><br>18 Hz  | 2   | 0.3 | 28 |
| 870  |           |  |   |     |     |    |
| 930  |           |  |   |     |     |    |
| 1720 | 1700-1990 | GSM 1800; CDMA<br>1900; GSM 1900;<br>DECT; LTE Band 1,<br>3, 4, 25; UMTS | Pulse modulation <sup>2</sup><br>217 Hz | 2   | 0.3 | 28 |
| 1845 |           |  |   |     |     |    |
| 1970 |           |  |   |     |     |    |
| 2450 | 2400-2570 | Bluetooth, WLAN,<br>802.11 b/g/n, RFID<br>2450,<br>LTE Band 7            | Pulse modulation <sup>2</sup><br>217 Hz | 2   | 0.3 | 28 |
|      |           |  |   |     |     |    |
|      |           |  |   |     |     |    |
| 5240 | 5100-5800 | WLAN 802.11 a/n  | Pulse modulation <sup>2</sup><br>217 Hz | 0.2 | 0.3 | 9  |
| 5500 |           |  |   |     |     |    |
| 5785 |           |  |   |     |     |    |

<sup>1</sup> For some services, only the uplink frequencies are included.

<sup>2</sup> The carrier shall be modulated using a 50% duty cycle square wave signal.

<sup>3</sup> As an alternative to FM modulation, 50% pulse modulation at 18 Hz may be used because while it does not represent actual modulation, it would be worst case.

## Appendix E: Limited warranty

Baxter warrants the product to be free of defects in material and workmanship and to perform in accordance with manufacturer's specifications for the period of one year from the date of purchase from Baxter or its authorized distributors or agents.

The warranty period shall start on the date of purchase. The date of purchase is: 1) the invoiced ship date if the device was purchased directly from Baxter, 2) the date specified during product registration, 3) the date of purchase of the product from a Baxter authorized distributor as documented from a receipt from said distributor.

This warranty does not cover damage caused by: 1) handling during shipping, 2) use or maintenance contrary to labeled instructions, 3) alteration or repair by anyone not authorized by Baxter, and 4) accidents.

The product warranty is also subject to the following terms and limitations: Accessories are not covered by the warranty. Refer to the instructions for use provided with individual accessories for warranty information.

Shipping cost to return a device to a Baxter service center is not included.

A service notification number must be obtained from Baxter prior to returning any products or accessories to Baxter's designated service centers for repair. To obtain a service notification number, contact Baxter Technical Support at [baxter.com/contact-us](http://baxter.com/contact-us).

THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. BAXTER'S OBLIGATION UNDER THIS WARRANTY IS LIMITED TO REPAIR OR REPLACEMENT OF PRODUCTS CONTAINING A DEFECT. BAXTER IS NOT RESPONSIBLE FOR ANY INDIRECT OR CONSEQUENTIAL DAMAGES RESULTING FROM A PRODUCT DEFECT COVERED BY THE WARRANTY.

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