

January 15, 2026

Dear Valued Customer,

We are pleased to announce that Baxter is releasing a new scanner that is able to read High Frequency and Low Frequency RFID tags in addition to 1D and 2D barcodes. The new Dual RFID scanner is the “Diamond Barcode/Dual Frequency RFID Scanner and Holster Kit”, part number 7000-FHR4.

The Diamond Dual RFID scanner is only compatible with the **Welch Allyn Connex Spot Monitor (CSM)** at this time and requires CSM Firmware 1.30.00 or greater. The scanner requires CSM Firmware 1.54.01 or greater for Single Sign-On (SSO) functionality.

The scanner will ship with a coiled cord and scanner holster. Replacement coiled cords and replacement scanner holsters will also be available for individual purchase.

Additionally, Baxter is discontinuing the “**JADAK Barcode/High Frequency RFID Scanner**”, part number 7000-916HSR, but will continue selling the scanner until inventory is depleted. Baxter recommends the new Diamond Dual RFID scanner as the replacement for the 7000-916HSR scanner.

New Dual RFID Scanner

Diamond Barcode/Dual Frequency RFID Scanner and Holster Kit, Part Number: 7000-FHR4



Scanner Replacement Parts

Universal Scanner Holster Kit, Part Number: 108781

- Note: Part 108781 is compatible with the 7000-FHR4, 7000-916HS, 7000-916HSR, 6000-916HS, 6000-916HSR, 7000-916HS1RS, and 7000-CS6080 Scanners

Diamond Scanner Coiled Cord Replacement Kit, Part Number: 109122

- Note: Part 109122 is only compatible with the 7000-FHR4 Scanner

WELCH ALLYN Multi-Parameter Vital Signs Devices – Scanner Offerings

Welch Allyn Multi-Parameter Vital Signs Devices

Scanner Offerings



SCANNER FUNCTIONALITY	CONNEX SPOT MONITOR		CONNEX VITAL SIGNS MONITOR with 2.42 firmware or later	CONNEX INTEGRATED WALL SYSTEM with 2.42 firmware or later
	with 1.30.00 firmware or later	with 1.40.10 A0002 firmware or later		
Barcode Only	7000-916HS (not compatible with Single Sign-On)	7000-CS6080 (not compatible with Single Sign-On)	6000-916HS (not compatible with Single Sign-On)	6000-916HS (not compatible with Single Sign-On)
High Frequency RFID + Barcode	7000-916HSR***		6000-916HSR**	6000-916HSR**
Low Frequency RFID + High Frequency RFID + Barcode	7000-916HS1RS*** 7000-FHR4*		7000-916HS1RS**	7000-916HS1RS**

* Requires CSMs with 1.54.01 firmware or greater for Single Sign-on (SSO) **Requires CVSMs/CIWSs with 2.40 firmware or greater for Single Sign-On (SSO) ***Requires CSMs with 1.32 firmware or greater for Single Sign-on (SSO)

COIL CORD REPLACEMENT KIT COMPATIBILITY	6000-916HS	7000-916HS	7000-CS6080	6000-916HSR	7000-916HSR	7000-916HS1RS	7000-FHR4
JADAK Scanner Coiled Cord Replacement Kit (106594)	✓	✓	✗	✓	✓	✓	✗
Zebra Scanner Coiled Cord Replacement Kit (108782)	✗	✗	✓	✗	✗	✗	✗
Diamond Scanner Coiled Cord Replacement Kit (109122)	✗	✗	✗	✗	✗	✗	✓

SCANNER HOLSTER REPLACEMENT KIT COMPATIBILITY	6000-916HS	7000-916HS	7000-CS6080	6000-916HSR	7000-916HSR	7000-916HS1RS	7000-FHR4
Universal Scanner Holster Kit (108781)	✓	✓	✓	✓	✓	✓	✓

Note: Not all scanners and replacement kits are available in every country. Please consult your local Baxter sales representative or customer care with any questions.

Reading RFID Tags with the 7000-FHR4 Scanner

To read an RFID tag with the 7000-FHR4 scanner, bring the RFID tag within 3 inches (7.62 cm) or into contact with the back of the scanner opposite the trigger button. When a tag is detected, the scanner trigger LED will begin flashing and the tag will automatically be read.



If you desire the Diamond Dual RFID scanner to be configured, please contact the Baxter technical support team at www.baxter.com/contact-us.

To obtain more information about the 7000-FHR4 scanner, please visit Hillrom.com or contact your local Baxter sales representative or customer care for additional details.

Sincerely,

James Bennett
Product Manager, Global Marketing
Patient Monitoring
Baxter Healthcare Corporation

Baxter, Connex and Welch Allyn are trademarks of Baxter International Inc. or its subsidiaries.
JADAK is a trademark of Novanta Corporation. Zebra is a trademark of Zebra Technologies Corp.

Baxter Healthcare Corporation
4341 State Street Road Skaneateles, NY 13153
T 800.535.6663

DIR 80031329 VER B GBU-FLC45-250001 (V1.0) 01/2025